inCoris TZI C Shade Guide

Gebrauchsanweisung
Operating instructions
Notice d'utilisation

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The inCoris TZI C Shade Guide helps facilitate shade matching and choosing the suitable inCoris TZI C color for an esthetically pleasing result. The inCoris TZI C Shade Guide represents the full color selection of all available inCoris TZI C blocks. inCoris TZI C blocks comprise the colors A1-D3 from the VITA classical A1 - D4[®]* color system.

inCoris TZI C blanks are offered in the colors A1, A2, A3 and A3.5.

Note on the colors:

The colors of the restorations from inCoris TZI C blocks and blanks appear significantly lighter in the patient's mouth than when not in the mouth. For this reason, in principle, using the "inCoris TZI C Shade Guide" from inCoris TZI C is recommended, to always ensure the best possible result. The inCoris TZI C Shade Guide reflects the colors that are created with the "CEREC SpeedGlaze 6570142" and when sintering in the inFire HTC speed. More light-colored results are only achieved in the Super speed program with darker colors (A3.5, A4, B3, and D3) than when using the Classic and Speed sintering.

Even in the case of sintering in the CEREC SpeedFire furnace, the colors match those of the inCoris TZI C Shade Guide.

Shade matching

Daylight is recommended for shade matching.

For shade matching, hold a small color sample plate of the inCoris TZI C Shade Guide to the unprepared tooth to be defined (outside the mouth) and compare the small color sample plates until you find the matching color. This color then corresponds to the block/blank of the same name. The small color sample plate corresponds approximately to the thickness of a restoration. For this reason, shade matching using the small color sample plates of the inCoris TZI C Shade Guide is strongly recommended.

Notes on care

For disinfection of the Sirona color sample holders and the respective small Sirona color sample plates, we recommend **commercially available mild spray disinfectants or disinfection cloths**.

According to the disinfectant manufacturers these disinfectants are effective against HIV and HBV pathogens and are available through dental stores. Observe the directions for use provided by the manufacturers of these disinfectants.

The Sirona color sample holders and the respective small Sirona color sample plates may not be disinfected using a hot water sterilizer or aggressive immersion baths.

* VITA classical A1 - D4[®] is a registered trade mark belonging to the company VITA Zahnfabrik, D-Bad Säckingen.

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