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ez adjust
aluminum crutches
Push-Button/Adjustable Fully Assembled

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drive
MEDICAL DESIGN & MANUFACTURING

● 10430-8
Adult
Patient Height
5'2" to 5'10"

● 10431-8
Youth
Patient Height
4'6" to 5'2"

● 10432-1
Tall
Patient Height
5'10" to 6'6"

CAUTIONS:

- Use after proper training under the direction of your Doctor or Physical Therapist.
- Replace Crutch Tips if worn.
- Keep Hand Grip tightened.
- Use extra caution on wet and slippery surfaces.

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ez adjust aluminum crutches

Push-Button/Adjustable Fully Assembled

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For maximum fit and assembly,
see instructions below:

1. **Height:** Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired patient height is obtained. To help in doing this, the foot-piece has a circular ring marked to indicate location of push-button. Rotate foot-piece again until both push buttons snap back into place.
2. **Handgrip:** Adjust handgrip to fit an approximate 10° bend at patient elbow by pulling out EZ adjust buttons, setting to proper height and replacing.
3. **Safety Check:** With crutches positioned per diagram below, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure both push-buttons are firmly in place and handgrip nuts are tightened securely.

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