GUARDIAN®

Easy Care™ Commodes

User

Model 30210-1, 30210-4 –

Aluminum Commode with Back

Models 30211-4, 30213-4, 30213XD –

Steel Commode with Back

Model 30214-2 –

Steel Commode with Back, Extra Wide

Includes pail, lid and splash guard.

Instructions

& Warranty

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Instructions

Modèle 30210-1, 30210-4 –

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Garantía

IMPORTANT SAFEGUARDS

Read all instructions before using. Important information is highlighted by these terms:

WARNING—Important safety information for hazards that might cause serious injury.

CAUTION— Information for preventing damage to the product.

NOTE— Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

Please follow these instructions carefully whether commode is used bedside or over the toilet. Commode's height is adjustable for use in an elevated position. Back of commode is removable for use over toilet.

WARNING

Please read carefully before use. Failure to comply may result in severe injury.

WARNING

Ensure all detent buttons are properly engaged in position prior to use.

WARNING

Do not exceed the maximum weight capacity listed on your commode.

FAILURE TO FOLLOW THE ABOVE INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

SAVE THESE INSTRUCTIONS.

HEIGHT ADJUSTMENT

Commode's height is adjustable for use in an elevated position.

1. Depress detent button on leg piece and move up or down to the desired position.
2. Ensure that the detent button is fully engaged and leg does not move.
3. Repeat steps 1 and 2 for the remaining three legs.
4. Make sure the commode sits level and all of the legs are at the same height.

USER INSTRUCTIONS

Use only as instructed by your healthcare professional.

Sitting and Standing

1. Back up to the commode until you feel the back of your knees touching it.
2. Reach back for the armrests. Place both hands firmly on the armrests before sitting on the commode.
3. Slowly lower yourself onto the commode by bending your elbows and, if possible, your hips and knees.
4. Reverse steps 2 through 4 for getting up. Push on the armrests as much as comfortably possible. Make sure you have your balance before releasing your grasp from the commode armrests.

Over the Toilet Use

1. Lift seat of commode and remove bucket.
2. Replace with splash guard.
3. Lift lid of toilet. Position commode over toilet with splash guard in bowl and lower commode seat.

NOTE—Check to ensure that commode is level and stable before sitting.

CLEANING

Your commode can be easily cleaned with regular household cleaning products. DO NOT use abrasive powders or scouring pads.

NOTE— Tips on this product contain dry natural rubber.

LIFETIME LIMITED WARRANTY

Your Guardian product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian brand product or this warranty, please call us toll-free at 800.333.4000.

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Sunrise Medical Inc.