No. 1 choice
of hospitals
& mothers

Steam Steriliser

- EN** Instructions for use
- DE** Gebrauchsanweisung
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- ES** Manual de instrucciones
- PT** Manual de instruções
- EL** Οδηγίες χρήσης
- HE** הוראות שימוש
- SV** Bruksanvisning
- DA** Brugsanvisning
- FI** Käyttöohjeet
- NO** Bruksanvisning
- PL** Instrukcja obsługi
- RU** Инструкция по применению
- HU** Használati utasítás
- CS** Návod na použití
- TR** Kullanma talimatları
- JA** 使用説明書
- KO** 사용 설명서
- TW** 使用說明書
- ZH** 使用说明书

EN Parts List IX
DE Teileliste IX
FR Liste des pièces IX
IT Elenco componenti IX
NL Onderdelenlijst IX
ES Lista de piezas IX
PT Lista de peças IX
EL Κατάλογος εξαρτημάτων IX
HE IX מילון חלקי
SV Lista över delar IX
DA Reservedelsliste IX
FI Osaluettelo IX
NO Reservedelsliste IX
PL Wykaz części zamiennych IX
RU Список деталей IX
HU Alkatrészek listája IX
CS Seznam dílů IX
TR Parça Listesi IX
JA 部品リスト IX
KO 부품 명칭 IX
TW 配件清單 IX
ZH 零件列表 IX

EN	Instructions for use	5
DE	Gebrauchsanweisung	14
FR	Mode d'emploi	23
IT	Istruzioni per l'uso	32
NL	Gebruiksaanwijzing	41
ES	Manual de instrucciones	50
PT	Manual de instruções	59
EL	Οδηγίες χρήσης	68
HE	הוראות שימוש	85
SV	Bruksanvisning	86
DA	Brugsanvisning	95
FI	Käyttöohjeet	104
NO	Bruksanvisning	113
PL	Instrukcja obsługi	122

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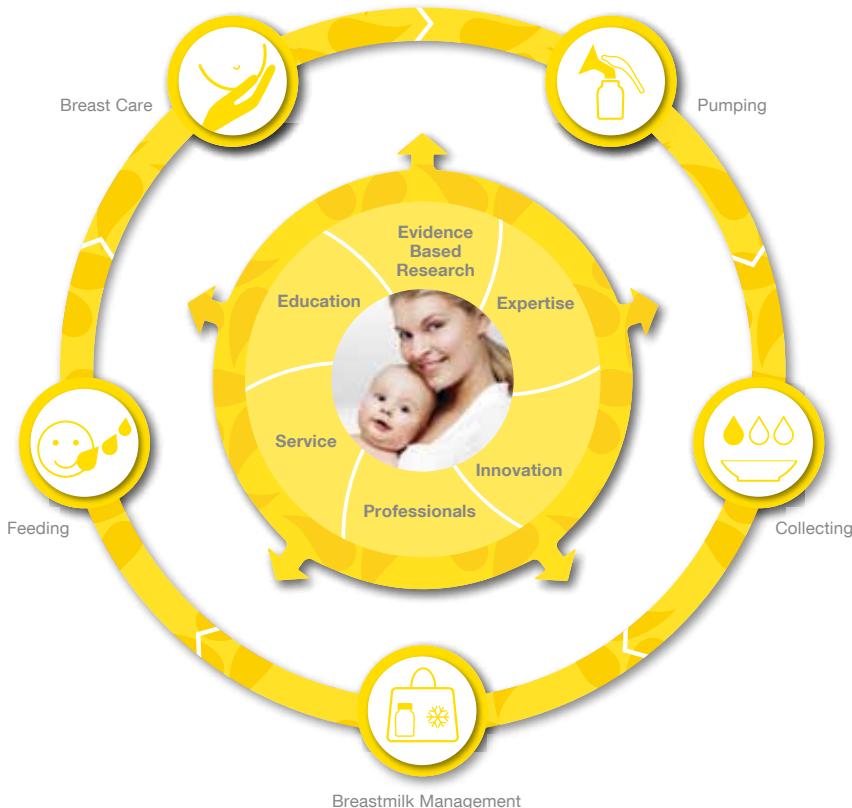
Breastfeeding Solution Circle

Dear Parents,

Congratulations on your decision to purchase a Medela product. For over 40 years, Medela has helped to enhance babies' health through the life-giving benefits of breast milk and supported mothers during their breastfeeding experience – making it as safe and as easy as possible. Medela strives to understand nature and over the years has become one of the world's leading experts in breastfeeding.

Breastfeeding is one of the most beautiful and natural experiences you can share with your baby. It's a unique time when you feel intensely close and intimate. Medela provides products for your various needs during breastfeeding. The Medela Steam Steriliser supports the wellbeing of your baby by hygienically cleaning your pumping and feeding equipment.

Medela wishes you and your baby all the best!

Product Overview

I Steam Steriliser

The new Medela Steam Steriliser has been designed to help busy mothers: It disinfects bottles and accessories, kills household germs, and automatically switches off, when the task is done, keeping the bottles disinfected for several hours.

II Indications for Use

The Medela Steam Steriliser

- disinfects breastmilk bottles and accessories
- components need to withstand boiling water (100 °C)
- is intended for home use only

III Important Safeguards

When using electrical devices, basic safety precautions should always be followed.

Read all instructions carefully before use.

Product

- Use the product only for its intended use as described in this instruction manual.
- Only use the power cord that comes with the Steam Steriliser.
- Keep appliance dry and do not place in direct sunlight.
- Never immerse the Steam Steriliser in water.
- Check components for wear or damage.
- Only use with the following Medela components: PersonalFit™ breastshield, all bottles, caps, lids and teats.

Electrical

- Make sure the voltage of the device is compatible with the power source.
- Plug the power cord into the Steam Steriliser first and then into wall socket.
- Always unplug electrical devices immediately after use.
- Separation from the mains is only assured through the disconnection of the power cord from the wall socket outlet.
- Keep the power cord away from heated surfaces.
- Never operate an electrical device if it has a damaged power cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
- Do not reach for any electrical device if it has fallen into water. Unplug immediately.
- If the supply cord is damaged, it must be replaced by a new cord set immediately to avoid a hazard.

Using/Feeding

- This appliance is intended for use by responsible persons, keep out of the reach of children.
- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience or knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure they do not play with the appliance.
- There are no user-serviceable parts inside the Steam Steriliser. Do not attempt to repair the Steriliser yourself.
- Always use on a flat, stable and heat-resistant surface.
- Whilst pressing the button, please hold the Steam Steriliser base to prevent from slipping.
- Always ensure that the correct volume of water has been added before switching on the Steam Steriliser.
- When in use, the steriliser contains boiling water and steam escapes from the vent in the lid. Always use and handle the equipment with care. Danger of scalding!
- Never place items on top of the steriliser.
- During the disinfecting cycle the lid and the steriliser body are very hot.
- Never use the steriliser without water.
- Take care when opening the cap/cover after a completed cycle as some residual steam may escape. Danger of scalding!
- Use a cloth when removing bottles, breast shields and all other accessories from the Steam Steriliser

III Important Safeguards

Cleaning

- Failure to follow de-scaling instructions may cause irreparable damage to the product.
- Do not touch the hot plate in the Steam Steriliser. Danger of scalding.
- Always let the Steam Steriliser cool down completely before starting a new disinfecting cycle.
Waiting time for next use 15 min.

SAVE THIS INSTRUCTION MANUAL

IV Operation

Sterilising

Place the Steam Steriliser on a flat, stable surface. Pour in 100 ml of tap water.

Note: Using cool distilled or filtered water may reduce limescale build-up.

Disassemble and wash thoroughly all parts that need to be disinfected in warm and soapy water. Rinse in clear water.

Load the bottles neck downwards and bigger accessories onto the bottom of the Steam Steriliser.

Place the tray (4) into the Steam Steriliser. Put teats, lids, discs and other small parts in the tray (4).

Close the Steam Steriliser with the lid (2).

Plug the power cord into the Steam Steriliser and into wall socket as well.

Press **@**-Button
LED turns constant orange and steaming process continues for 8 Minutes (approx. 100 °C).

@-Button LED blinks green when heating process is finished.
Buzzer activates five acoustic signals (slow).

@-Button LED turns constant green

Do not remove the lid (2) straight after the finished cycle. Wait approximately 15 minutes to allow the Steam Steriliser and the contents to cool down. Buzzer activates five acoustic signals (fast). Steam Steriliser will switch off automatically after the disinfecting cycle is finished. The control light will then turn off.

Take the bottles and all other accessories out of the Steam Steriliser.

The Medela Bottle Warmer can be stored inside the Steam Steriliser.

V Cleaning and Hygiene

- After each use**
- Always unplug the appliance from the mains before cleaning it.
 - Do not clean the appliance when hot.
 - Wait 15 minutes after use to allow the Steam Steriliser to cool down.
 - Ensure that the plug does not come into contact with water.
 - Wipe the inside and outside of the Steam Steriliser with a damp (not wet) cloth.

Note: Use water of drinking quality for any described cleaning procedures

Do not clean with abrasive cleaners

Never place the Steam Steriliser directly under tap water

Never clean the Steam Steriliser in a dishwasher

Cleaning After a while in use lime scale will build-up around the hotplate of the Steam Steriliser. This should be removed at regular intervals.

Every month (De-scale)

Version 1

- Mix **100 ml/3.3 oz** of water with citric acid* according to manufacturer's instructions and pour solution into empty Steam Steriliser.
- DO NOT place bottles or accessories into the Steam Steriliser when de-scaling.
- DO NOT put lid (2) on.
- Press On-/Off-Button (6). The Cleaning process has started.
- After 1-2 minutes unplug and leave the solution to cool for 30 minutes before emptying the Steam Steriliser and rinsing with tap water.
- After this run a full sterilising cycle, without content, with the lid (2) on and using exactly 100 ml of water.
- Wipe the inside and outside of the Steam Steriliser with a damp (not wet) cloth only.

Version 2

- Pour 50 ml/1.8 oz of vinegar mixed with 50 ml/1.8 oz of cold water into the Steam Steriliser.
- DO NOT place bottles or accessories into the Steam Steriliser when de-scaling
- DO NOT start the sterilising cycle.
- Leave liquid in the unit until lime scale has dissolved.
- Empty the unit and rinse thoroughly with clear water.
- Wipe the inside and outside of the Steam Steriliser with a damp (not wet) cloth only.

*Citric acid is available from most drugstores and pharmacies

Note: Do not use anything other than citric acid or vinegar as this may cause irreparable damage

VI Troubleshooting

Steam Steriliser does not switch on

Check that the plug is in the Steam Steriliser and in the wall socket also. Press ⓧ-Button -> check with other appliances as to whether the socket has power supply.

Bottles are very hot after disinfection

Allow bottles to cool down.

VII Warranty

Medela AG ("Medela"), warrants to the original end user ("Customer") that this product will be free from defects in workmanship and materials, under normal use, for one (1) year.

Instructions are strictly respected, in particular in case of extreme and/or continuous applications/use of the product.

The warranty period begins on the proved purchase date (conclusion of purchase contract). Products purchased from non-authorised distributors, e.g. internet market places, auctions or other types of public sale (including but not limited to eBay) are not covered by this warranty. For hygiene reasons, Medela Products are single user products; use by more than one person voids the warranty. Medela's sole obligation under this express warranty shall be, at Medela's sole option and expense, to replace or repair the Product or part thereof (in case the product is not available, to deliver a similar type of the Product) or to refund the purchase price paid for the Product. Medela warrants any repaired or replaced Product or part for a period of ninety (90) days from delivery, or through the end of the original warranty, whichever is longer. All Products or parts that are replaced become the property of Medela. This express warranty does neither cover consumables, nor third party products.

Obtaining Warranty Service. Customer must contact the retailer where the product was purchased from, the authorised distributor of Medela or Medela's Service Centre indicated by Medela within the applicable warranty period to obtain warranty service authorisation. Dated proof of original purchase from Medela, its authorised distributor or its authorised retailer and a description of the defect will be required. Medela is not responsible for Products or parts received without a warranty service authorisation. Repaired or replacement Products will be delivered to Customer free of charge. The repaired product or part will be delivered as soon as reasonably possible. Medela shall not be responsible for any damages occurring during such delivery. Medela shall, at its sole discretion, decide on the place of performance for work under warranty. Medela shall have the right to charge for additional costs for services under warranty if the site of the Product is other than where the Product was originally delivered.

Warranty Exclusive. Customer's sole remedy for breach of the warranty shall be the express warranty. The foregoing warranty is exclusive and is in lieu of all other warranties, terms or conditions, expressed or implied, either in fact or by operation of law, statutory or otherwise, including warranties, terms or conditions of merchantability, fitness for a particular purpose, satisfactory quality and non-infringement, all of which are expressly disclaimed. Medela shall not be liable if the alleged defect or malfunction was caused by Customer's or any other person's misuse, neglect, improper handling, unauthorised attempts to open, repair or modify the Product, inadequate maintenance, disregard of operating instructions, excessive load or stress, normal wear and tear, incorrect voltage or any other cause beyond the range of its intended use, by accident, fire, or other hazards, or other cause not due or attributable to Medela. This warranty does not cover physical damage to the Product or malfunctions resulting from the use of the Product in conjunction with any sort of ancillary or peripheral equipment and Medela determines that there is no fault with the Product itself.

VII Warranty

Limitation of Liability. Medela also excludes any liability, whether based in contract or tort (including negligence), for incidental, consequential, indirect, special, or punitive damages of any kind, or costs of procurement of substitute products by Customer, or for the loss of revenue or profits, loss of business, loss of information or data, or other information of financial loss arising out of or in consequence with the sale, maintenance, use, performance, failure, or interruption of this product, even if Medela or its distributors have been advised of the possibility of such damages, and limits its liability to replacement, repair, or refund of the purchase price paid, at Medela's option. This Limitation of Liability for damages will not be affected if any remedy provided herein shall fail of its essential purpose.

Disclaimer. Should a court of jurisdiction not allow the entire exclusion or limitation of implied warranties or the limitation of incidental or consequential damages for certain products supplied to consumers or the limitation of liability for personal injury, such implied warranties and such liabilities will be limited to the duration of the applicable express warranty.

Governing Law and Place of Jurisdiction. This Limited Warranty shall be governed by the laws of Switzerland, excluding all conflict of laws principles and excluding the United Nations Convention on the International Sale of Goods dated 11 April 1980. The ordinary courts at the Medela registered office in Baar, Switzerland shall be competent. Medela shall, at its sole discretion, also be entitled to take legal action in the competent courts at the Customer's place of business or domicile. With this Limited Warranty, Medela grants Customer specific legal rights which do not restrict any statutory consumer rights.

VIII Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment. Improper disposal may have harmful effects in the environment and on public health.

IX Parts List

- 1 Lid Handle
- 2 Lid
- 3 Tray Handle
- 4 Tray
- 5 Outerhousing
- 6 On/Off-Button

Note: Whenever you press "On/Off"-Button, Steriliser will switch off automatically.

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Medela AG, Medical Technology