

- 66079#
- 66079#
- 66079#
- 66079#
- 66079#
- 66079#

spanish
english
breastpumps
for symphony
pumping kit
double
symphony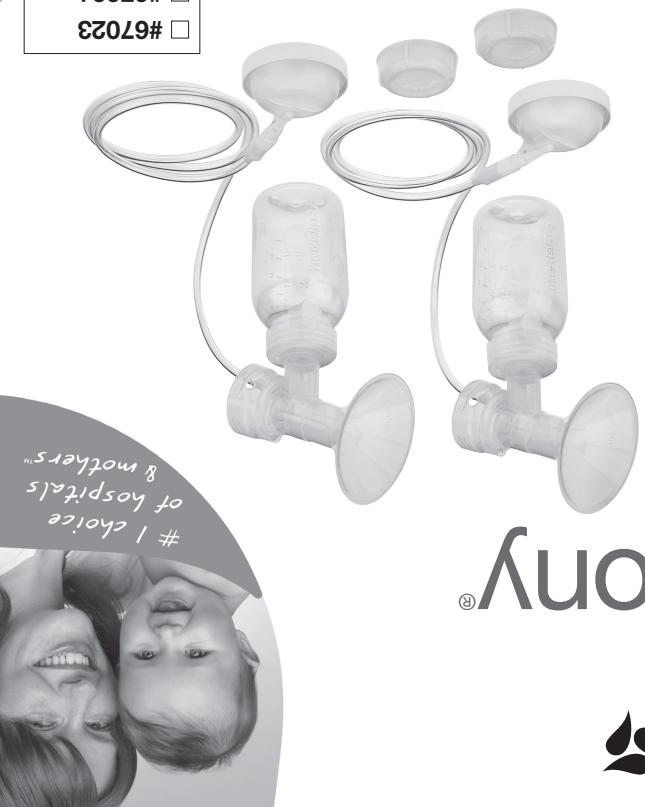

Symphony® Breastpump Kit Parts

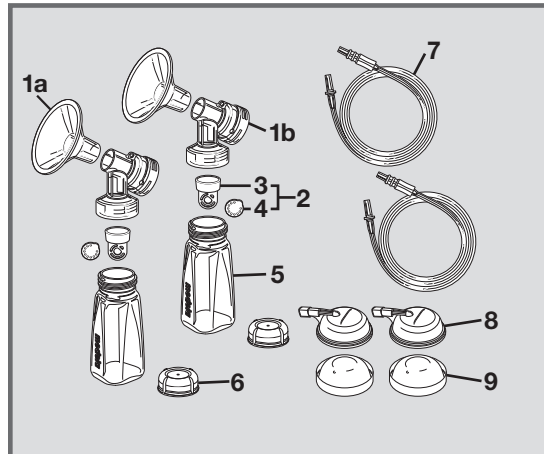Item	Order Number
1a. PersonalFit™ Breastshield, Standard (24mm)	87073
1b. PersonalFit Breastshield Connector.....	87071
2. Valve and White Membrane	8000001
3. Valve	8100475
4. Membrane.....	8207013
5. Collection Connector.....	8117009
6. Solid Container Lid	8107183
7. Tubing	8007213
8. Membrane Cap	8100667
9. Protective Membrane	8200094

Medela, Inc., P.O. Box 660, 1101 Corporate Drive,
McHenry, IL 60051-0660
Phone: (800) 435-8316 or (815) 363-1166
Fax: (815) 363-1246
Email: customer.service@medela.com

Medela and Symphony are registered trademarks
and PersonalFit, Quick Clean, Micro-Steam and
#1 Choice of Hospitals & Mothers are trademarks
of Medela, Inc.

1908095 C 0210 © 2010 Medela, Inc.
Printed in the U.S.A.

Cleaning and Sterilization
Follow these directions unless instructed otherwise by
your healthcare provider.

IMPORTANT
This product's kit is a single-user item unless
properly sterilized between users. Use by more than
one user without adequate sterilization may pose
a health risk and voids the warranty.

Hygiene
Wash hands before touching
breasts or containers, and avoid
touching the inside of containers
or lids.

First Use
Before first use, disassemble all kit parts and
sanitize by boiling for 10 minutes.

***67099S** - has been sterilized and does not require
cleaning before first use.

After First Use
Follow these cleaning instructions unless you are
told otherwise by your healthcare professional. Prior
to cleaning, take apart all parts of your breastpump
accessory kit. Be especially sure to remove the valve
from the breastshield and separate the membrane from
the valve prior to cleaning.

To Sterilize in the Hospital:
All disassembled parts can be autoclaved to a
maximum of 272° F for 3 minutes at 29 PSIG or 250° F
for 15-20 minutes at 15 PSIG.

To avoid damage during and immediately after
sterilization:

- Sterilize parts without applying external pressure to
them.

- Allow parts to cool before applying external pressure.
If gas sterilization is used, residual testing is
recommended to determine the proper length
of aeration.

To Sanitize at Home:
Disassemble all parts that come in contact with breast
and milk.

- After each use:**
- Disassemble and wash all parts that come in contact
with the breast and milk in warm soapy water.
 - Rinse in clear water.
 - Air-dry on clean towel and cover parts when not in
use.

All parts may also be washed in top rack of dishwasher.

For Quicker Cleaning:
Medela offers Quick Clean™ Micro-Steam™ Bags for
daily kit disinfection.

Tubing Care
Inspect tubing after each pumping session for
condensation and/or milk.

If condensation appears in tubing:

To eliminate condensation after you have completed
pumping, continue running the pump with the tubing
attached for another 1-2 minutes or until dry.

If milk appears in tubing:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing, pump
connector, piston, cylinder, membranes and membrane
caps (parts vary by pump used). Wash in soapy water,
then rinse in clear cold water.
3. Shake water droplets out of tubing and hang to air-
dry. For faster drying, attach tubing to pump and run
pump 1-2 minutes or until dry, or pour a small amount
of isopropyl alcohol through the tubing to dry.

Symphony® Double Pumping Kit Instructions

- 1 Make sure breast-
shield and connector are
securely attached.
- 2 Snap white
membrane onto yellow
valve.
- 3 Push yellow valve
into place on breastshield
connector.
- 4 Screw collection
container onto
breastshield.
- 5 Assemble protective
membrane and membrane
cap.
- 6 Insert the yellow
end of tubing into port on
membrane cap.
- 7 Insert pale yellow
end of tubing into opening
in connector.
- 8 To unlock lid, press
the oval release button
on top of the Symphony®
Breastpump. Keeping
button depressed, lift up
on the handle to open
case and access pumping
mechanisms.
- 9 Push membrane/
cap into place on pumping
mechanism. A "click" will
be heard once properly
seated. To ensure proper
fit, be sure to center the
membrane caps' tube
ports directly over the
crevice on the pump
housing.
- 10 Close lid securely
before pumping. If lid
does not close properly
membrane caps may not
be completely seated or
may require adjustment.
- 11 Switch the pump on
to begin Stimulation/
let-down phase by
pressing the power button.
- 12 Center breastshield
over your nipple and
start pumping.
- 13 If your let-down
occurs before the
Expression phase begins,
you may pump more
efficiently by pressing the
button to begin the
Expression phase.

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Medela, Inc., P.O. Box 660, 1101 Corporate Drive, McHenry, IL 60051-0660