URGENT – Voluntary Medical Device Correction HeartStart Vehicle Wall Mount (Model M5528A) and DC Power Module (Model M5529A)

Incorrect wiring during installation by user may result in a smoke or fire hazard

Dear Customer,

This letter is to inform you that a Philips investigation has found that a small number of users have modified the wiring of the HeartStart MRx Vehicle Wall Mount or the DC Power Module for the Philips HeartStart MRx defibrillator when connecting the Vehicle Wall Mount (Model M5528A) to the DC Power Module (Model M5529A). These user modifications involve cutting and splicing or soldering the DC output cable (that comes with the DC Power Module) and/or the electrical connector on the Vehicle Wall Mount.

Such improper connections can result in reversed electrical polarity from the DC Power Module to the Vehicle Wall Mount, and pose a smoke or fire hazard to patients or caregivers in proximity to the Vehicle Wall Mount. This Field Safety Notice is intended to inform you about:

- What the problem is and under what circumstances it can occur
- Actions that should be taken by the customer/user in order to mitigate risks for patients/caregivers
- Actions planned by Philips to address the problem

This document contains important information for the continued safe and proper use of your equipment

Please review the following information with all members of your staff who need to be aware of the contents of this communication. It is important to understand the implications of this communication.

Please retain a copy with the equipment Instructions for Use.

This issue can only arise when **both** the Vehicle Wall Mount and DC Power Module are installed together **and** modifications have been made to the DC Power Module power cable or Vehicle Wall Mount electrical connector. Vehicle Wall Mounts installed without a DC Power Module cannot be affected.

The HeartStart MRx monitor/defibrillator, which may be attached to the Vehicle Wall Mount, does not contribute to this issue under any circumstances and is not the subject of this correction.

Philips Healthcare

Cardiac Care Emergency Care Solutions

-2/3-

FSN86100085 2009 NOVEMBER

Philips is updating the Instructions for Use for both the DC Power Module and the Vehicle Wall Mount with warnings to not make any unauthorized modifications to the equipment (wall mount, cables, mounting brackets, or M5529A DC Power Module). The updated Instructions for Use are included with this letter, and should be implemented to ensure the Vehicle Wall Mount and DC Power Module are installed and operated properly.

Please see the attached Field Safety Notice which provides information on how to identify affected devices and instructions on actions to be taken. Please follow the "ACTION TO BE TAKEN BY CUSTOMER / USER" section of the notice.

Should you have any questions or concerns about the Device Correction, please contact your local Philips representative:

German: 0800 80 3000

French: 0800 80 3001

Ensuring that you have the highest quality medical devices, accessories and supporting documentation is our top priority. Your satisfaction with Philips products as well as with our response to this problem is very important to us.

Sincerely,

Michael Miller Jr.
Senior VP and GM
Emergency Care Solutions
Cardiac Resuscitation

Attachments

Field Safety Notice

AFFECTED PRODUCTS	Product: Philips HeartStart Wall Mount, Model M5528A (if used with DC Power Module, Model M5529A). Units Affected: Units distributed by Philips and shipped between September 1, 2004 and October 1, 2009. Distributed by: Philips Healthcare, 3000 Minuteman Road, Andover, MA, 01810.
PROBLEM DESCRIPTION	A small number of users have modified the wiring of the HeartStart Vehicle Wall Mount for the Philips HeartStart MRx monitor/defibrillator when connecting the Vehicle Wall Mount to the DC Power Module (Model M5528A), as part of installation. These modifications involve cutting and splicing or soldering the DC output cable from the DC Power Module and/or the electrical connector on the Wall Mount. Such incorrect installation may result in reversing the electrical polarity of the connection, which can lead to excessive heating at the lower mounting bracket of the Vehicle Wall Mount, where it interfaces with the monitor/defibrillator.
HAZARD INVOLVED	Elevated temperatures at the lower mounting bracket on the Vehicle Wall Mount can melt the plastic mounting bracket and smoke. If not promptly detected, combustion of the assembly may result. These events could result in exposure of patients or caregivers to smoke or fire, particularly if they are in an enclosed space, such as an ambulance. Furthermore, resulting damage to the ambulance or to the monitor/defibrillator could disrupt patient care.
HOW TO IDENTIFY AFFECTED PRODUCTS	See affected products above.
ACTION TO BE TAKEN BY CUSTOMER / USER	Immediately check all Vehicle Wall Mounts to verify that either (1) they are not connected to a DC Power Module or (2) that there is a proper connection between the two. The latter can be verified by checking whether the green external power indicator on a HeartStart MRx mounted on the Vehicle Wall Mount is lit when power is supplied to the DC Power Module. If you cannot verify either of the two conditions above, immediately disconnect the power to DC Power Module, and contact your local representative. Note: In any case, the associated HeartStart MRx is not affected and can remain in service.
ACTIONS PLANNED BY PHILIPS	Philips is voluntarily initiating a corrective action consisting of: • Issuance of a procedure for identification of incorrect installations as described in this Field Safety Notice • Issuance of revised Instructions for Use for Vehicle Wall Mount for HeartStart MRx and DC Power Module for the HeartStart MRx including a revised description of correct installation and verification of performance
FURTHER INFORMATION AND SUPPORT	If you need any further information or support concerning this issue, please contact your local Philips representative at: German: 0800 80 3000 French: 0800 80 3001

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HeartStart MRx Vehicle Wall Mount

The M5528A HeartStart MRx Vehicle Wall Mount (VWM) is designed to hold the HeartStart MRx Monitor/Defibrillator (M3536A) during transport. The typical use is within an ambulance.

Unpacking

Table 1 **Package Content**

Item	Quantity
HeartStart MRx Vehicle Wall Mount	1
Instructions for Use	1
Vehicle Mounting template	1
Upper Mounting Bracket	1
screws for Upper Mounting Bracket (12 mm)	2
Lower Mounting Bracket	1
screws for Lower Mounting Bracket (14 mm)	2

Installation

Installation of the VWM should be performed by a qualified mechanical and electrical technician.

Selecting the VWM Location

The VWM is typically mounted on the left or right interior panels of the vehicle, or on the interior panel directly behind the driver's seat. The VWM must be screwed to the chassis of the vehicle. This prevents it from becoming detached during normal use and in extreme situations such as in a collision.


CAUTION: When the VWM is fastened to the wall, the wall and screws must withstand a force of at least 330 lb (150 kg) in all directions. Do not attach the VWM to the vehicle doors.

When identifying a suitable location for the VWM to be mounted:

- ☐ Select a flat surface to support the VWM.
- ☐ Allow sufficient space at each side of the VWM to accommodate the use of side accessory pouches on the HeartStart MRx (see [Figure 1](#)).
- ☐ Allow sufficient space at the top of the unit to accommodate operation of the release handle.

- NOTE:** The HeartStart MRx cannot be inserted into the VWM when any one of the following is attached:
- the rear accessory pouch,
 - the bedrail hook,
 - the M3539A AC power module.

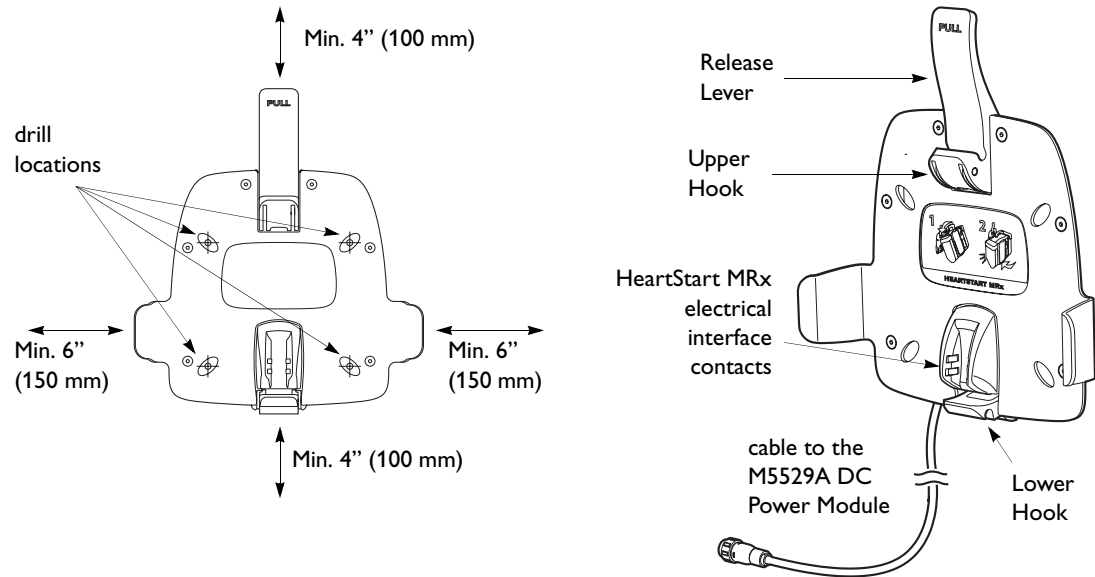
Figure 1 **Positioning of the Mounting Frame and Wall Mount Elements**

Preparing the HeartStart MRx

- © To prepare the HeartStart MRx for wall mount:
- 1 Remove the rear accessory pouch and bedrail hook (if installed).
 - 2 Using a Torx T-20 screwdriver, screw the two mounting brackets into the MRx (see [Figure 2](#)). Note that the 14-mm screws are for the lower bracket, and the 12-mm screws are for the upper bracket. Apply 1.1 Nm (10 inch-lb) of torque when inserting the screws.

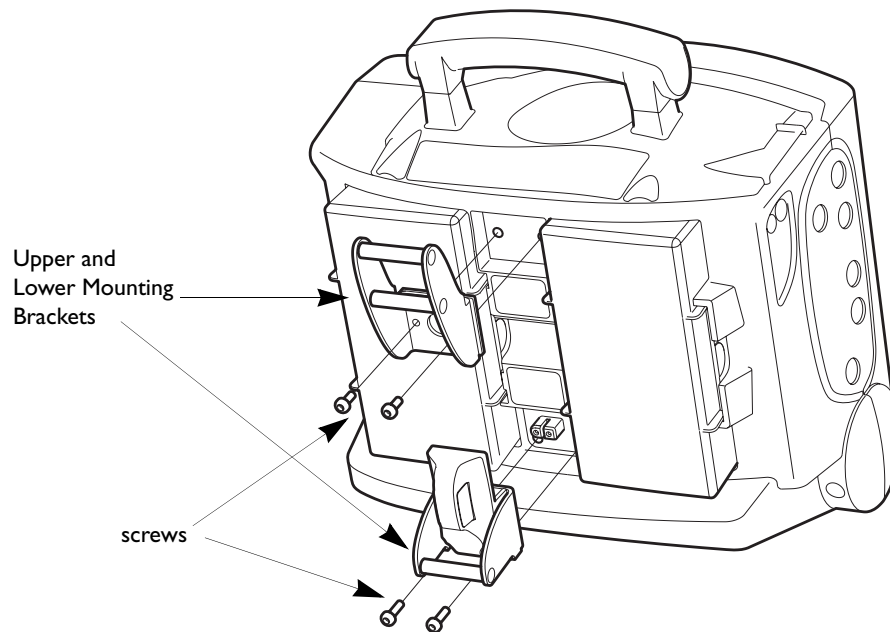Figure 2 **HeartStart MRx Vehicle Wall Mount Preparation**

Operation

© To Insert the HeartStart MRx into the VWM:

- 1 Place the Upper Mounting Bracket over the hook of the release handle (see [Figure 3](#)).
- 2 Push the lower part of the MRx into the VWM.
You should feel the lower mounting bracket lock into place.


Figure 3 **HeartStart MRx Operation**

TIP: If the device does not click into place, check that no objects interfere with the locking mechanism.

Use the DC Power Supply module (M5529A, see [Figure 4](#)) or a Philips-approved battery support system to charge the HeartStart MRx batteries in transport. If you do not use the DC Power Supply module, then tuck the DC Power Supply output cable away.

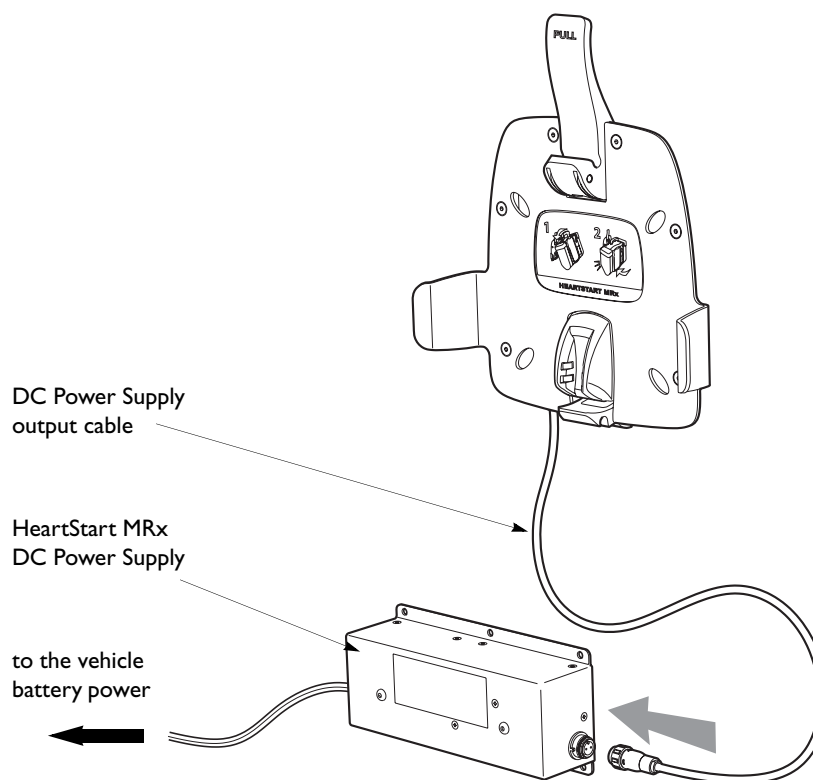Figure 4 Connecting the VWM to the DC Power Supply (optional)

Performance Verification

© To verify installation:

- ☐ Check that the VWM is securely mounted to the interior vehicle panel.
- ☐ Check that the Upper and Lower Mounting Brackets are securely fastened to the HeartStart MRx.
- ☐ Check that the HeartStart MRx can easily be installed into the VWM, as described in “Operation” on page 4.
- ☐ Pull forward the front bottom portion of the MRx to ensure that it is securely mounted in position.
- ☐ Check that you can remove the MRx from the bracket by pulling on the release handle.
- ☐ If you use the M5529A HeartStart MRx DC Power Module, then:
 - a** Insert one or two batteries into the HeartStart MRx.
 - b** Install the HeartStart MRx into the VWM.
 - c** Connect the DC Power Module to the vehicle battery.
 - d** Verify that the green External Power indicator at the upper right corner of the HeartStart MRx screen is on.

If any of the verification steps fail, then **stop using the VWM and correct the installation.**

Safety Considerations

- WARNINGS:**
- Electrical shock hazard exists. Do not touch metal contacts.
 - Do not perform any unauthorized modification to the equipment (wall mount, cables, mounting brackets, or M5529A DC Power Module, if used in installation).
 - Do not mount the VWM directly below infusion pumps or other similar equipment where there is a possibility of liquid spill.

Maintenance

The VWM has no internal electronics that require maintenance.

Verify the installation every month. In addition to the performance verification (see “[Performance Verification](#)” on page 5), watch for:

- frayed cables,
- worn and oxidized contacts,
- loose mechanical joints.

Clean the VWM with a damp cloth using mild soap and water.

Repair

The VWM has no serviceable parts. The repair philosophy for the VWM is unit replacement. For information on ordering, call 1-800-225-0230 or visit our website at:

<http://philips.com/healthcarestore>.

Specifications

Standards

IEC 60601-1 (2001), IEC 60601-1-2 (2001), EN 1789 (1999)

Physical

Weight (excluding cables): 1.5 kg (3.3 lb)

Dimensions (L x W x H): 75 x 300 x 320 mm (3.0" x 11.2" x 12.6")

Environmental

Temperature: Operating: 0°—40°C (32—104°F); Storage: –30°—70°C (–22—158°F)

Humidity: 5%—95% non-condensing

Altitude: –500—7500 m (–1,500—25,000 ft)

Symbol Definitions

Table 2 **VWM Packaging Symbols**

Symbol	Definition
	Attention – See operating instructions in <i>HeartStart MRx Instructions for Use</i> .
	Recyclable material.
	This device complies with the requirements of the Medical Device Directive 93/42/EEC.
WEEE 	This appliance is marked according to the European directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). Dispose of in accordance with local regulations.

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Philips Healthcare is part
of Royal Philips Electronics

On the web

www.philips.com/heartstart

By e-mail

healthcare@philips.com

By fax

+31 40 27 64 887

By postal service

Philips Healthcare
3000 Minuteman Road
Andover, MA 01810-1085

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All rights are reserved.

Reproduction in whole or in part
is prohibited without the prior
written consent of the copyright
holder.

Philips Healthcare reserves the right
to make changes in specifications or
to discontinue any product at any
time without notice or obligation
and will not be liable for any
consequences resulting from the use
of this publication.

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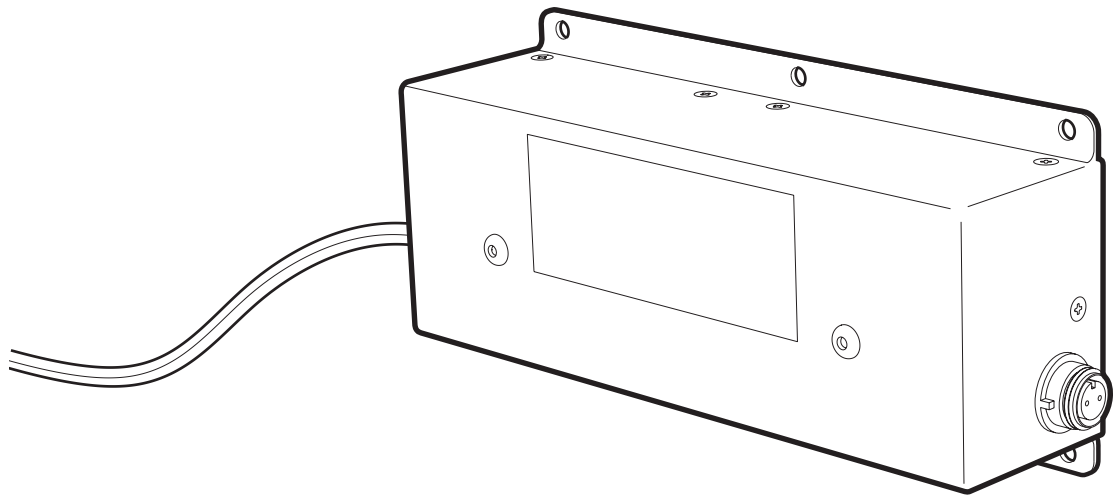
HeartStart MRx DC Power Module

The M5529A HeartStart MRx DC Power Module (Power Module) is intended to provide a secondary power source to the HeartStart MRx monitor/defibrillator (M3536A) during transport and to charge the HeartStart MRx batteries (M3538A) from an external DC power source. The typical use environment is within an ambulance. The external power source is normally the ambulance vehicle battery.

WARNING: Always use the DC Power Module with a battery installed in the HeartStart MRx to ensure continuous operation of the device.

Unpacking

Table 1 **Package Content**

Item	Quantity
HeartStart MRx DC Power Module	1
Instructions for Use	1
Vehicle Mounting Template	1
DC output cable	1

Installation

Installation of the DC Power Module should be performed by a qualified mechanical and electrical technician.

Selecting the DC Power Module Location

The DC Power Module must be securely screwed to the interior side panels of the vehicle. Select a suitable location where the DC Power Module can be mounted.

CAUTION: The DC Power Module mounting location should be a well-ventilated area to avoid overheating. When mounted, the indicator lights must be visible.

When identifying a suitable location for the DC Power Module to be mounted:

- ☐ Select a flat surface to support the DC Power Module.
- ☐ Allow sufficient space at each side of the DC Power Module to accommodate the use of input and output cables.

The Vehicle Mounting Template indicates the drill hole dimension and drilling location.

Connecting the DC Power Module Cables

Input connections must be made by a qualified electrician. Connect the input power cable to the vehicle battery power. The red wire connects to the battery + (positive), and the black wire connects to the battery – (negative).

After the DC Power Module is securely mounted, connect the DC output to the HeartStart MRx using the DC output cable (see [Figure 1](#)) or to the M5528A Vehicle Wall Mount (see [Figure 2](#)).

Figure 1 **DC Power Module Connected to the HeartStart MRx**


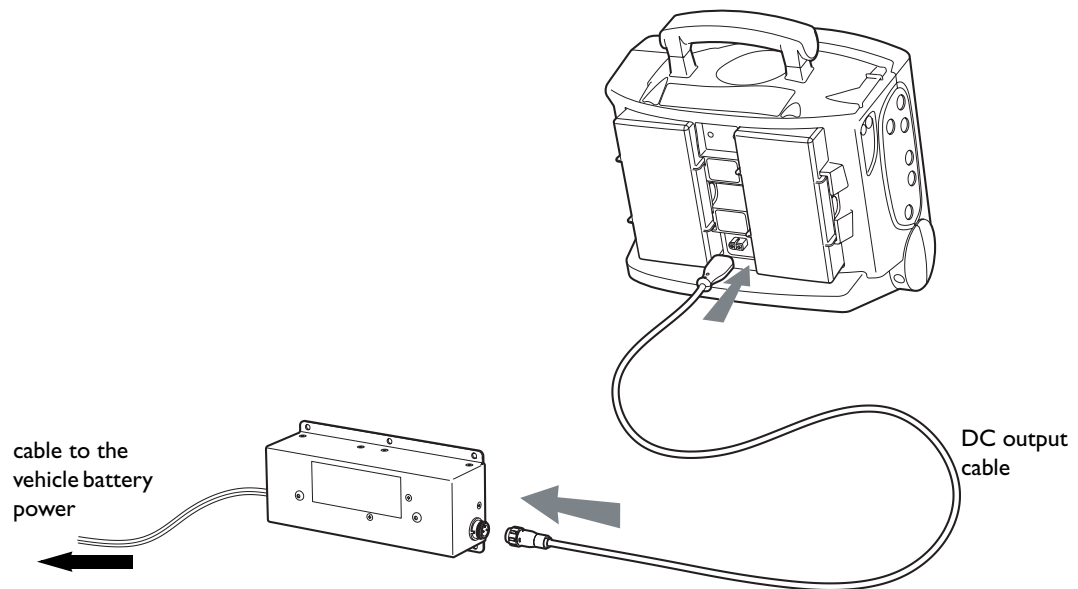
Figure 2 **DC Power Module Connected to the Vehicle Wall Mount (optional)**

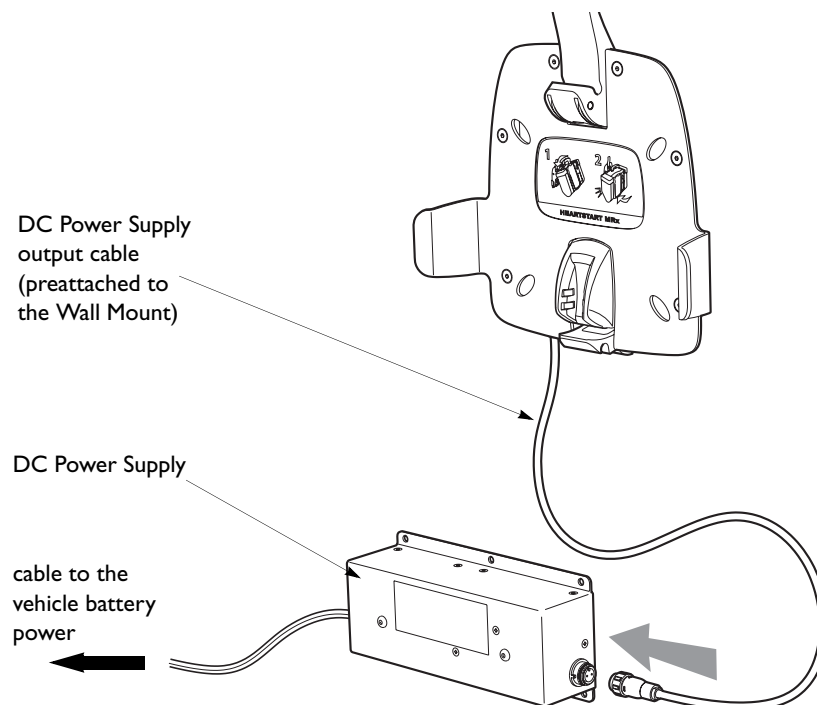
Performance Verification

- ☉ To verify the HeartStart MRx DC Power Module installation.

When the DC Power Module is connected to the HeartStart MRx (directly or through the Vehicle Wall Mount):

- ☐ Check that the input power light is on (see Figure 3).
- ☐ Check that the output power light is on (see Figure 3).
- ☐ Verify that the green External Power indicator at the upper right corner of the HeartStart MRx screen is on.

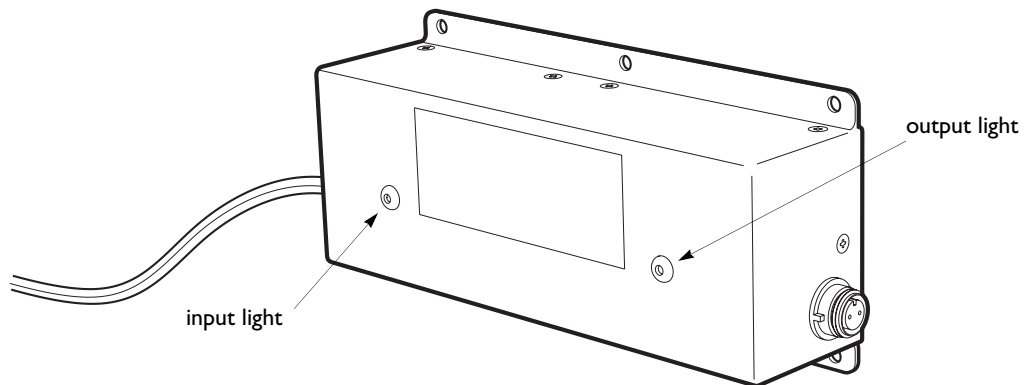Figure 3 HeartStart MRx DC Power Module, Input and Output Lights

If any of the verification steps fail, then **stop using the power module and correct the installation.**

Safety Considerations

-
- WARNINGS:**
- Electrical shock hazard exists. Do not touch metal contacts.
 - Do not perform any unauthorized modification to the equipment (DC Power Module, cables, mounting brackets, or M5529A Vehicle Wall Mount, if used in installation).
 - Do not mount the DC Power Module directly below infusion pumps or other similar equipment where there is a possibility of liquid spill.
 - Mounting the DC Power Module onto surfaces without sufficient hold will render the total installation unsafe and therefore unsuitable for vehicle use.
 - When the DC Power Module is supplying power to the HeartStart MRx, electromagnetic compatibility with surrounding devices should be assessed. Fixed, portable, and mobile radio frequency communications equipment can affect the performance of medical equipment. Refer to the *HeartStart MRx Instructions for Use* for detailed electromagnetic interference information and guidance.
 - The DC Power Module must always be used with a battery installed in the HeartStart MRx to ensure continuous operation of the MRx.
 - Do not remove the cover of the DC Power Module.
-

Maintenance

The DC Power Module has no internal electronics that require maintenance.

Verify the installation every month. In addition to the performance verification (see “[Performance Verification](#)” on page 4), watch for frayed cables.

Clean the DC Power Module with a damp cloth using mild soap and water.

Repair

The DC Power Module has no serviceable parts. The repair philosophy for the DC Power Module is unit replacement. For information on ordering, call 1-800-225-0230 or visit our website at: <http://philips.com/healthcarestore>.

Specifications

Standards

IEC 60601-1 (2001); IEC 60601-1-2 (2001); EN 1789 (1999)

Physical

Weight (excluding cables): 1.2 kg (2.64 lb).

Dimensions (L x W x H): 220 x 105 x 60 mm (8.6" x 4.1" x 2.4")

Electrical

Input Voltage: 10 — 32 V DC

Maximum Input Current: 11 A

Output Voltage: 18 V DC $\pm$ 3%

Maximum Output Current: 5 A

Environmental

Temperature: Operating: 0°—40°C (32—104°F); Storage: -30°—70°C (-22—158°F)

Humidity: 5%—95% non-condensing

Ingress Protection Rating: IPX1 (IEC 60529)

Symbol Definitions

Table 2 **DC Power Module Packaging Symbols**

Symbol	Definition
	Attention – See operating instructions in the <i>HeartStart MRx Instructions for Use</i> .
	Input
	Output
	Recyclable material.
	This device complies with the requirements of the Medical Device Directive 93/42/EEC.
	Class II equipment.
IPX1	Protected against vertically falling water drops.
WEEE 	This appliance is marked according to the European directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). Dispose of in accordance with local regulations

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