



THE FOLLOWING IS INCLUDED WITH YOUR ALARMED WALL BRACKET:

- (A) Alarmed Wall Bracket (AWB)
- (B) Four (4) wall mollies 2.
- (C) Four (4) Phillips-head screws 3.
- (D) Two (2) reset keys and one (1) magnetic fob 4









RECOMMENDATIONS:

- Follow ADA height and placement guidelines. To access ADA guidelines go to https://www.access-board.gov/guidelines-and-standards.
- *(E) Use hearing protection when testing the alarm; the alarm sounds at 105 dB.

With the alarm lock in the OFF position, mount the alarm bracket to a wall.

- Using the included hardware affix the bracket in a safe but visible place.
- Don't forget the Comprehensive Unit dimensions extend beyond the bracket.





Turn the alarm lock to ON with the included key.

- You have 30 seconds to properly seat the Unit before the alarm arms.
- You may want to practice seating the Unit to avoid accidentally activating the alarm.



Seat the Comprehensive Unit on the bracket.

- Center the Comprehensive Unit on the bracket arms.
- Align the nub on the bracket with the receiver on the Comprehensive Unit.
- The Comprehensive Unit will snap into place and the two matching red magnets will be in close proximity.











Affix the Theft Deterrent Cord.

- a. With the Comprehensive Unit seated, pass the Theft Deterrent Cord through a locking hole.
- b. Pass the cord through the Comprehensive Unit handle and insert the end into the hole on the alarm face. It will fit snugly.











Alarmed Wall Bracket is ready for use.

a. When the Comprehensive Unit is disturbed, alarm will sound immediately and continue for 30 seconds.





Reset alarm after activation:

- a. Set alarm lock to OFF with key.
- b. Remove the Comprehensive Unit from bracket.
- c. Set alarm lock to ON.
- d. Replace the Comprehensive Unit on the bracket arms, ensuring nub snaps into Unit within 30 seconds.
- e. Reinsert Theft Deterrent Cord.



Deactivate alarm:

- a. Using key, set alarm lock to OFF.
- b. Remove the Comprehensive Unit.
- c. When ready to replace the Comprehensive Unit, follow steps in "Reset alarm after activation."



Replacing the AWB alarm module battery:

The AWB Alarm Module is powered by a standard 9 volt battery.

- a. Remove the 3/32 hex screw.
- b. Remove the alarm face plate.
- c. Replace the 9 volt battery.
- d. Replace alarm face plate and hex screw.



