



Avoximeter[®] 1000E

Whole Blood Oximeter



Operator's Manual

Manufacturing Company Location



Accriva Diagnostics, Inc.

6260 Sequence Drive, San Diego, CA USA

Phone: 1-858-263-2300

Fax: 1-858-314-6700

web site: www.accriva.com

Copyright and Trademarks

Copyright© 2015 Accriva Diagnostics, Inc. All rights reserved. This material may not be reproduced or copied, in whole or in part, without the written permission of Accriva.

Accriva and AVOXimeter are registered trademarks of Accriva Diagnostics, Inc. in the United States and other jurisdictions.

U.S. Patents. 5,430,542 and 6,262,798. Euro/UK Patent 0663070. Other patents pending.

Technical Support

Contact Technical Support at (800) 579-2255 or (858) 263-2502, or by e-mail at techsupport@accriva.com.

Table of Contents

1	INTRODUCTION.....	1
	Intended Use of the <i>AVOXimeter 1000E</i>	1
	Summary and Explanation of the Test.....	1
	Hemodynamic Calculations	2
	Saturation Step-Ups	3
	Operating Precautions and Warnings	5
	Limitations	5
2	DESCRIPTION.....	6
	Front Panel.....	6
	Keypad	7
	Menus	8
	Test Cuvettes	9
	Connections	10
	Automatic Standby and Shutdown.....	10
	Instrument Lockouts	10
	Instrument Specifications	11
	Reportable Range	11
	Accuracy	11
	Precision	11
	Interference	11
	Calibration	12
3	GETTING STARTED	13
	Unpacking and Inspection	13
	Materials Provided.....	13
	Materials Required But Not Provided.....	13
	Optional Materials	14
	Charging the Batteries.....	14
	Setting Up the Instrument.....	15
	Setting Display Backlighting	15
	Specifying Units for Total Hemoglobin (THb).....	15
	Changing the Date and Time.....	16
	Setting the Standby Delay	17
	Specifying Mandatory Entry of User ID and/or Patient ID	18
	Changing the QA User ID.....	21
	Requiring Entry of a Patient ID	22
	Specifying Oximetry Site Prompts	23
	Specifying Use of Oximetry Sites	24
	Entering a Different Value for Hüfner's Number.....	25

4	OPERATION	27
	Startup	27
	Entering a User ID (Optional)	27
	Entering a Patient ID (Optional)	27
	Sample Collection and Preparation.....	28
	Sample Collection.....	28
	Sample Preparation	28
	Running a Test	29
	Using the Printer	30
	Printing the Current Test Results	30
	Specifying Automatic Printing of Results.....	30
	Changing the Serial Port Baud Rate and Parity	30
	Data Management	31
	Printing all Stored Data.....	31
	Printing All Optical Quality Control Data.....	31
	Reviewing and/or Printing the Last Sample	32
	Locating, Reviewing, and/or Printing any Sample	32
	Aborting Printing of Results	33
	Purging all Test Records.....	33
	Quality Control.....	34
	Performing Optical Quality Control.....	34
	Running Liquid Controls.....	35
	Entering Liquid Control Lot Numbers	36
	Tagging a Liquid Control Test with a Lot Number	37
	Enabling QC Lockout.....	38
	Calibration	39
	Cuvette Pathlength	39
	Re-Calibration.....	39
	Shutdown	40
	Hemodynamic Calculations	41
	Entering the Patient Age, Height, Weight, and Sex.....	41
	Oxygen Uptake Rate	43
	Body Surface Area.....	44
	Saturation Step-Ups	45
	Flow Calculations.....	46
	Systemic and Pulmonary Resistances.....	50
	Stroke Volume and Stroke Index	52
	Printing Hemodynamic Values	54
	Troubleshooting.....	55
5	MAINTENANCE	59
	Cleaning the Optical Detector	59
	Replacing the Battery	63

6	QUALITY CONTROL LOGS	65
7	WARRANTY	68
	Certification, Warranty and Service Warranty, and Service	68
8	SAFETY STANDARDS	70
	Guidance and Manufacturer's Declaration – Electromagnetic Emissions.....	70
	Guidance and Manufacturer's Declaration – Electromagnetic Immunity	72
INDEX	73

Important Labels and Symbols

Before using the *AVOXimeter 1000E*, it is essential that the contents of this *Operator's Manual*, any labels on the instrument or its packaging, and instructions accompanying *AVOXimeter 1000E* cuvettes are read and understood by the operator. These materials make reference to additional symbols that are explained below:



Product Conforms to Directive 98/79/EC, 27 October 1998 on In-Vitro Diagnostic Medical Devices



Expiration Date of Cuvettes



Serial Number of Instrument



Lot Number of Cuvettes



Catalogue Number of Devices



Do Not Reuse— Single Use Only



Upper and Lower Temperature Limitations (For Storage or Use)




For *in vitro* Diagnostic Use



Attention - Read Accompanying Documentation or Instructions



Consult Instructions for Use

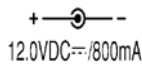

Input Port for DC Power Cord from AC/DC Power Module - Polarity, VDC and A Input



Serial Output Port for Data Transfer – RS232C



Temperature Probe Input (*AVOXimeter 4000* Only)



Name and Address of Manufacturer



Warning - Biohazard



Medical Equipment per Annex 1A, Item 8 Directive 2002/96/EC For Electronic Equipment Waste – Contact Technical Support @ 1-800-579-2255

1 Introduction

Intended Use of the *AVOXimeter 1000E*

The *AVOXimeter 1000E* is a battery-operated bedside whole blood oximeter that performs individual point-of-care measurements of oxyhemoglobin saturation (%HbO₂) and total hemoglobin concentration [THb] on lithium/sodium or heparin or EDTA anticoagulated whole blood samples. Oxygen content [O₂] of the blood sample is automatically calculated from the %HbO₂ and THb measurements.

No sample preparation is required, and analysis is quickly accomplished by injecting the sample into a disposable cuvette and inserting the cuvette into the instrument. The *AVOXimeter 1000E* then illuminates the sample with multiple wavelengths, records the optical density of the sample at each of the wavelengths, and computes the results. In less than 10 seconds, the oxyhemoglobin fraction, the total hemoglobin concentration, and the oxygen content of the sample are shown in appropriate units on the liquid-crystal display on the front panel.

Data management capabilities are included with the instrument. These capabilities include a combined storage of up to 500 patient, and/or QC results, designation of quality control levels and lot numbers, tagging of test results with date, time, Patient ID and/or Operator ID, entry of oximetry site and subsite, and printing of results.

The instrument also can calculate ten hemodynamic variables (such as body surface area, estimated oxygen consumption rate, and cardiac output using the Fick principle) from data that are entered by the operator. It can also calculate differences in oxygen saturation between adjacent anatomical sites from which blood samples were taken (saturation step-ups), to aid in diagnosing intracardiac and great vessel shunts.

In vitro diagnostic use, For Professional Use, Rx Only

Summary and Explanation of the Test

The *AVOXimeter 1000E* measures whole blood %HbO₂ and [THb] using disposable single-use cuvettes. The operator inserts a whole blood sample into a cuvette, the cuvette is inserted into the test chamber on the instrument, and the results are displayed. The results will remain on the display while the cuvette remains in the instrument.

The result can be automatically printed along with the time and date the test was run, the Patient ID, Operator ID, and other information entered. The result is also saved in an internal database, which has the capability to store up to 500 results.

Individual *AVOXimeter 1000E* instruments can be customized so that optical quality control tests must be performed whenever a specified period of time has elapsed. In addition, up to three liquid control lot numbers for each level of liquid QC can be stored in the *AVOXimeter 1000E* and can be tagged to the stored or printed liquid QC records. The instruments can also be configured so that only authorized operators can operate the system and that patient IDs are required for each test run.

The *AVOXimeter 1000E* measures oxygenated hemoglobin [HbO₂], deoxyhemoglobin [HHb], methemoglobin [MetHb], and carboxyhemoglobin [COHb] directly, using novel optics and multiple

wavelengths. This reduces interference from dyshemoglobins and other interfering substances such as fetal hemoglobin and bilirubin and minimizes the effects of hemolysis.

The measured values are used to calculate total hemoglobin [THb] and percent oxyhemoglobin saturation [%HbO₂] of the sample, using the fractional method described below:

$$[\text{THb}] = [\text{HbO}_2] + [\text{HHb}] + [\text{MetHb}] + [\text{COHb}]$$

$$[\% \text{HbO}_2] = \frac{[\text{HbO}_2] \times 100}{[\text{HbO}_2] + [\text{HHb}] + [\text{MetHb}] + [\text{COHb}]}$$

Oxygen content [O₂] of the sample is then calculated:

$$[\text{O}_2] = \frac{1.39 \times [\text{THb}] \times [\% \text{HbO}_2]}{100}$$

where 1.39 is the ml of oxygen assumed to be carried by one gram of oxygenated hemoglobin (Hüfner's Number). Depending on your facility protocols, the Hüfner's Number stored in the AVOXimeter 1000E can be set at any value in the range of 1.30 to 1.39 (see page 25).

Note: Below is a legend for oxygen saturation terminology used throughout this document:

Abbreviation	Term	Units Displayed
THb	Total Hemoglobin	g/dl
HbO ₂	Oxyhemoglobin	g/dl
MetHb	Methemoglobin	g/dl
COHb	Carboxyhemoglobin	g/dl
HHb	Reduced Hemoglobin	g/dl
%HbO ₂	Percent Oxyhemoglobin Saturation (Fractional Oxygen Saturation)	n/a
[O ₂]	Oxygen Content	ml/dl

Hemodynamic Calculations

Based on the results, hemodynamic calculations are performed as described below.

$$\text{Cardiac Output} \frac{\text{ml}}{\text{min}} = \frac{\text{Oxygen Uptake}}{[\text{O}_2]_{\text{arterial}} - [\text{O}_2]_{\text{venous}}}$$

Note: Oxygen uptake rate can either be measured from expired gases, or it can be estimated from the patient's age, height, weight, and sex (see below).

$$\begin{aligned}
 \text{Body Surface Area} &= 0.007185 (\text{Weight kg})^{0.425} (\text{Height cm})^{0.725} \\
 \text{ml/min} & \\
 \text{Oxygen Uptake}_{\text{males}} &= 157.3 \text{ BSA} + 10 - (10.5 * \text{age} + 4.8) \\
 \text{ml/min} & \\
 \text{Oxygen Uptake}_{\text{females}} &= 157.3 \times \text{BSA} - (10.5 * \text{age} + 4.8) \\
 \text{ml/min} & \\
 \text{Cardiac Index} &= \frac{\text{Cardiac Output}}{\text{BSA}} \\
 \text{L/min/ m}^2 & \\
 \text{Stroke Volume} &= \frac{\text{Cardiac Output}}{\text{Heart rate}} \\
 \text{ml/beat} & \\
 \text{Stroke Index} &= \frac{\text{Stroke Volume}}{\text{BSA}} \\
 \text{ml/beat/m}^2 & \\
 \text{Pulmonary Blood Flow} &= \frac{\text{Oxygen Uptake}}{[\text{O}_2] \text{ Pulmonary Arterial} - [\text{O}_2] \text{ pulmonary venous}} \\
 \text{ml/min} & \\
 \text{Total Systemic Resistance} &= \frac{\text{Arterial Pressure} - \text{Right Atrial Pressure}}{\text{Cardiac Output}} \\
 \text{mmHg/ml/min} & \\
 \text{Total Pulmonary Resistance} &= \frac{\text{Pulmonary Arterial Pressure} - \text{Left Atrial Pressure}}{\text{Cardiac Output}} \\
 \text{mmHg/ml/min} & \\
 \text{Pulmonary to Systemic Flow Ratio} &= \frac{\text{Pulmonary Flow}}{\text{Systemic Flow}} \\
 \text{Pulmonary to Systemic Flow Ratio} &= \frac{\% \text{HbO}_2 \text{ Arterial} - \% \text{HbO}_2 \text{ Venous}}{\% \text{HbO}_2 \text{ Pulmonary Venous} - \% \text{HbO}_2 \text{ Pulmonary Arterial}}
 \end{aligned}$$

Saturation Step-Ups

Saturation step-ups are the differences in oxyhemoglobin saturation [%HbO₂] between adjacent anatomical sites. These values are used as an aid in diagnosing intracardiac and great vessel shunts. The AVOXimeter 1000E calculates saturation step-ups between the following anatomical sites:

- Right atrium and superior vena cava
- Right ventricle and right atrium
- Pulmonary artery and right ventricle
- Pulmonary vein and left atrium
- Left atrium and left ventricle

- Left ventricle and aorta

When calculating saturation step-ups, the *AVOXimeter 1000E* searches the database for all results for each main oximetry site for that patient (sub-sites are ignored), averages the results for each site, and then calculates the saturation step-ups from the average result for each main site. This information can then be printed from the *AVOXimeter 1000E*.

Note: *Use of average readings rather than individual values improves the precision of saturation step-up calculations.*

Operating Precautions and Warnings



- For *in vitro* Diagnostic use.
- Do not allow blood, water, or other liquids to enter the instrument.
- The *AVOXimeter 1000E* instrument is designed for use only with *AVOXimeter 1000E* cuvettes.
- Do not re-use test cuvettes.
- Always keep cuvettes in sealed bag with desiccant.
- When filling cuvette, do not use excessive pressure on the syringe or cause the vent patch to bulge outward by overfilling the cuvette.
- For proper calibration and calibration verification, use only the controls recommended in this manual. Controls from other sources may yield erroneous results.
- The *AVOXimeter 1000E* instrument is designed to be used for testing in a stationary position. DO NOT perform testing while carrying or holding the instrument.
- In order to charge the *AVOXimeter 1000E* instrument, the AC power cord should be plugged into an electrical service outlet and the AC/DC power module while the DC power cord from the AC/DC power module is plugged into the DC port in the back of the instrument.
- DO NOT expose the *AVOXimeter 1000E* instrument to extreme temperature (above 35°C, 95°F). Such exposure could affect the performance of any type of electronic instrumentation.
- DO NOT drop the *AVOXimeter 1000E* instrument, and do not use the results if the instrument is dropped during a test.
- Only properly qualified personnel should attempt to open and perform work on the *AVOXimeter 1000E* instrument as identified in this manual.
- DO NOT remove the AC/DC power module from the *AVOXimeter 1000E* instrument by pulling on the cord.
- The use of accessory equipment (e.g., printers, etc.) not identified in this manual either in the patient vicinity, or that does not comply with either the equivalent safety requirements of this equipment or IEC/EN 61010-1:2010 or IEC/EN 61010-2-101:2002, may lead to a reduced level of safety with the resulting system.



Any items exposed to human blood, plasma or serum must be handled cautiously as a biohazardous material in accordance with laboratory safety practices and federal and local regulations. Federal, state and local laws and regulations require that hazardous waste be disposed of in a specific manner. Waste material from the *AVOXimeter 1000E* which may be classified as biohazardous include used cuvettes. It is important that steps be taken to dispose of these materials in accordance with the prevailing regulations in your location.

Limitations

Do not disturb the instrument while a test is in progress. As with all diagnostic tests, *AVOXimeter 1000E* test results should be scrutinized in light of a specific patient's condition and therapy. Any results exhibiting inconsistency with the patient's clinical status should be repeated or supplemented with additional test data.

2 Description

The *AVOXimeter 1000E* is a Point of Care device for use at the bedside. It contains a test chamber which performs all operations to measure the oxyhemoglobin saturation (%HbO₂), total hemoglobin (THb), and oxygen content (O₂) of a whole blood sample after the operator inserts a test cuvette containing the sample into the test chamber.

Each *AVOXimeter 1000E* is calibrated at the factory. The *AVOXimeter 1000E* can be operated either from its internal batteries or from the AC adapter. The batteries are charged when the AC Adapter is connected.



Figure 1. *AVOXimeter 1000E* Oximeter

Front Panel

The front panel (Figure 2) contains the test chamber, a keypad with the **Enter/On** key, action and menu keys, number keys, and a display panel. Operator instructions are shown on the display panel, and the operator enters commands and information using the keypad.

When the test is completed, the results are shown on the display panel and stored in system memory.

The display panel is illuminated to enhance visibility in low light conditions. The illumination can be adjusted (or turned off) to conserve power during battery operation.

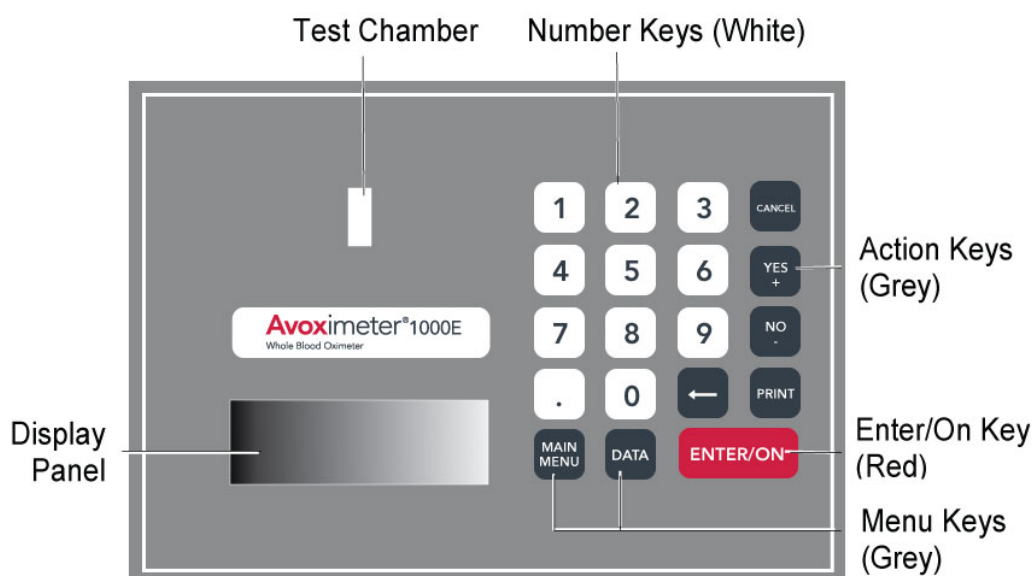


Figure 2. Front Panel

Keypad


The routine analysis of blood samples does not require the use of menus or the numeric keypad. However, these enable the user to take advantage of many useful features.




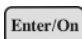

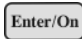



The purpose of each key is summarized below:

Key	Purpose
	Switch the instrument on. Select a command.
	Display a menu of commands for calibration, printing, stored data, and shutdown.
	Display a menu of commands for entering hemodynamic variables, entering device settings, entering the time and date, viewing battery status, and managing data.
	Print the results that are displayed.
	Respond to questions that are displayed.
	Backspace over a numerical entry (such as a QC lot number) so that it can be corrected.
	Return to the previous menu.
	Enter characters for Operator IDs or Patient IDs. Enter a character for selection of a command.

Menus

The principal menus, their commands, and the procedure to access each menu are summarized below:

Note: Press the  button at any time to return to the previous menu.

Menu	Commands	Access
<u>Main Menu</u>	<div>1. Calibration 2. Printer Mode 3. Stored Data 4. Turn Off</div>	Press the  key when a test is not running or another menu is not active.
Calibration Submenu	<div>1.High HbO2 CalPt.L2 2.Low HbO2 CalPt.L1 3.Cuvette Pathlength 4.Hüfners# 1.30-1.39</div>	Press 1 followed by  while the main menu is displayed.
Printer Mode Submenu	<div>1.Auto Print ON/OFF 2.Print Stored Data 3.Printer Parameters 4.Print Opt. QC Only</div>	Press 2 followed by  while the main menu is displayed.
Stored Data Submenu	<div>1. Select Sample # 2. Newest Sample 3. Purge All Data (+=Next; -=Previous)</div>	Press 3 followed by  while the main menu is displayed.
<u>Data Menu</u>	<div>1. Hemodynamics Menu 2. Device Settings 3. Time,Date,Battery 4. Data Management</div>	Press the  key when a test is not running or another menu is not active.
Hemodynamics Submenu (Page 1)	<div>1.Pt Age, Ht, Wt, Sex 2.Oxygen Uptake Rate 3.Body Surface Area 4Show Sat Step-UPs↓↑</div>	Press 1 followed by  while the data menu is displayed to display Page 1.
Hemodynamics Submenu (Page 2)	<div>5Flow Calculations↑+ 6.Resistances 7.StrokeVolume, Index 8.Print Hemodynamics</div>	Press + or - while the hemodynamics submenu is displayed to toggle between Page 1 and Page 2.
Device Settings Submenu	<div>1.Display Brightness 2.Units 3.Standby Delay</div>	Press 2 followed by  while the data menu is displayed.
Time, Date, and Battery Submenu	<div>1. Set & View Time 2. Set & View Date 3. Battery Status</div>	Press 3 followed by  while the data menu is displayed.
Data Management Submenu	<div>1.Oxim. Sites ON/OFF 2.User & Patient ID 3.QC Lockout 4.Liquid QC Lot Nos.</div>	Press 4 followed by  while the data menu is displayed.

Test Cuvettes

Tests are performed with single-use disposable test cuvettes (Figure 3). Each test cuvette contains a finger grip, filling port, optical window, and a vent patch.

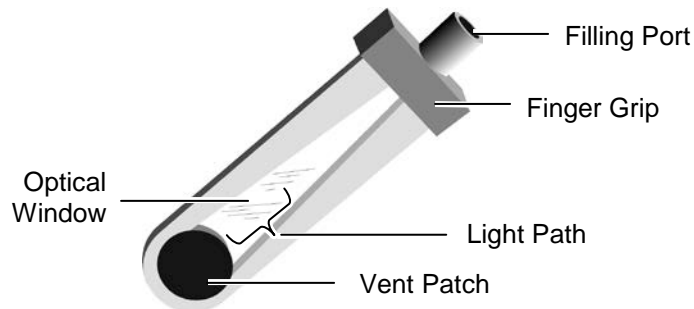


Figure 3. Test Cuvette

A whole blood sample is inserted into a test cuvette by connecting a luer lock syringe, slip syringe or capillary tube containing the whole blood sample to the filling port and then gently pressing the syringe plunger to dispense approximately 50 μ L of whole blood into the test cuvette. Air escapes from the vent patch at the end of the test cuvette while the whole blood sample is being inserted. The test cuvette (with the syringe still attached) is then inserted into the test chamber of the instrument (see page 28 for details).

Note: *Be sure to handle the cuvette either by the edges or by the finger grip. Refer to the package insert accompanying the test cuvettes for storage and handling instructions.*



- Remove any blood or debris from the exterior of the test cuvette before inserting it into the test chamber.
- After filling the cuvette with blood, inspect the vent patches to ensure they are not bulging out. If a vent patch protrudes, discard the cuvette. **Do not insert a cuvette with a protruding vent patch into the test chamber.**



BIOHAZARD WARNING: Any items exposed to human blood, plasma or serum must be handled cautiously as a biohazardous material in accordance with laboratory safety practices and federal and local regulations. Federal, state and local laws and regulations require that hazardous waste be disposed of in a specific manner. Waste material from the AVOXimeter 1000E which may be classified as biohazardous include used cuvettes. It is important that steps be taken to dispose of these materials in accordance with the prevailing regulations in your location.

Connections

Connections to the AC Adapter and an optional printer (or a computer) are made at the rear of the instrument (Figure 4).



Use only the AC Adapter provided with the instrument.



Figure 4. Power and Printer Connections

Automatic Standby and Shutdown

The AVOXimeter 1000E enters a low-power standby mode after the instrument has been idle for a specified period of time (the **standby delay**). The instrument is factory preset for a standby delay time of 60 minutes, but a time of **10 to 180** minutes can be specified (see page 17). To resume normal operation when the instrument is in standby, press and hold down any key for one second.

Note: *The AVOXimeter 1000E also enters standby if the battery charge becomes critically low.*

The AVOXimeter 1000E shuts down after it has been in standby for 4 hours.

Instrument Lockouts

The instrument can be configured to allow use only by authorized operators and/or to allow use only if Optical Quality Control (OQC) has been performed.

Instrument Specifications

<i>Size</i>	20.3 cm (8.0 in) x 25.4 cm (10.0 in) x 9.5 cm (3.8 in)
<i>Weight</i>	1.8 kg (4 lbs)
<i>Operating Temperature</i>	Room temperature (15°C to 30°C, 59°F to 86°F)
<i>Battery Type</i>	Nickel Cadmium (NiCad)
<i>Operating Time On Battery</i>	Approximately 8 hours (constant run at medium brightness). Tests may also be run while the AVOXimeter 1000E is plugged into the AC/DC power module.
<i>Anticipated Battery Life</i>	Approximately 500 charge / discharge cycles
<i>Power Supply/Chargers</i>	Input: 100 / 240 VAC, 50 / 60 Hz Output: 12 VDC, 830 mA
<i>Serial Data Port</i>	RS232C
<i>Sample Type</i>	Whole blood
<i>Sample Volume</i>	50 µL
<i>Analysis Time</i>	7 to 10 seconds per sample
<i>Analysis Wavelengths</i>	5

Reportable Range

<i>%HbO₂</i>	0 to 100%
<i>THb</i>	4 to 25 g/dL
<i>[O₂ Content]</i>	0 to 35 mL O ₂ /dL

Accuracy

<i>%HbO₂</i>	±1 %HbO ₂
<i>THb (>10 g/dL)</i>	±0.45 g/dL
<i>THb (<10 g/dL)</i>	±0.35 g/dL

Precision

<i>%HbO₂</i>	0.5 %HbO ₂
<i>THb</i>	0.3 g/dL

Interference

<i>Bilirubin</i>	None
<i>Hemolysis</i>	None
<i>Carboxyhemoglobin</i>	None
<i>Methemoglobin</i>	< 1% %HbO ₂ , < 0.2 g/dL THb (THb = 16 g/dL, MetHb < 10%, 7.1 < pH < 7.8)
<i>Fetal Hemoglobin</i>	< 1% %HbO ₂ , < 0.45 g/dL THb (THb = 16 g/dL, HbF < 100%)

Calibration

The AVOXimeter 1000E is factory-calibrated and employs highly stable state-of-the-art light sources. Factory tests indicate that, when used in accordance with this Operator's Manual and other instructions, the AVOXimeter 1000E is capable of maintaining its calibration for at least two years.

Should recalibration be required contact a Technical Support representative for assistance (see page 39).

Proper calibration also requires entry of the correct cuvette pathlength by the user (see page 39) and use of a customary value for Hüfner's number (see page 25).



CAUTION: *If quality control results are not acceptable, erroneous results are encountered, or error messages are displayed, the most likely cause is contamination of the optical detector by blood or debris, which cannot be resolved by re-calibration. Consult the Troubleshooting section for additional information.*

3 Getting Started


Unpacking and Inspection

Note: *Inspect each component for damage when unpacking. If damage is observed, contact your shipping representative immediately.*

1. Remove any protective packaging that may be present around the instrument.
2. Examine the packaging material to be sure that the AC adapter, connecting cables, or other components have been removed. The materials that are provided are listed below.

Note: *Do not discard the packaging material.*

Materials Provided

Item	Quantity
AVOXimeter 1000E Instrument	1
AC adapter (Part No. AP5300X)	1
Power Cord (Part No. HR1235)	1
 Operator's Manual	1
Optical Quality Control Filters (Part No. E-QCYO)	2

Note: *An AC power cord is supplied only for the 110VAC version of the US/Canada/Japan instrument. For all others, the customer must obtain a 3 conductor AC power cord that is compatible with an IEC 320 connection at the power supply AC inlet and any other local requirements.*

Materials Required But Not Provided

Item	Quantity
AVOXimeter 1000E Cuvettes (Part No. C100B)	As Needed
Liquid Controls (RNA) See page 34 for additional information.	As Needed

Optional Materials

Item	Quantity
Dymo Printer	1
• 110 V	
• 220 V	
Printer Paper	As Needed

Charging the Batteries

Charge the batteries before the system is used for the first time.

1. Plug the AC adapter into an electrical service outlet.
2. Connect the AC adapter cord to the power connector on the rear of the instrument.
3. Allow the battery to charge for at least eight hours.

Note: *The AC Adapter can remain connected all the time.*

4. To ensure adequate charge, leave the instrument connected to the AC adapter for a minimum of eight hours. This eliminates the risk of the instrument powering down during a test.

Fully charged batteries will allow the AVOXimeter 1000E to analyze blood samples continuously for up to 8 hours when the display is set at medium brightness.




Battery power can be conserved by:


- Reducing (or turning off) display backlighting (see page 15).
- Reducing the standby delay (see page 17).

Note: *The batteries can suffer from a “memory effect” if they are charged before being completely discharged. For optimal battery performance, discharge completely when possible before charging them. The message “**Battery Critical – Connect Charger**” will be displayed when the battery is completely discharged.*



The message “**Battery Critical – Connect Charger**” is displayed and the instrument reverts to the standby mode if the battery power is insufficient to complete the test. The AC Adapter must be used for additional tests until the battery is recharged.

Checking the Battery:

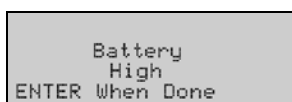1. Display the “Time, Date, and Battery” menu (a submenu of  , see page 8).
2. Press  followed by  to display the battery status:



```
Battery
High
ENTER When Done
```




3. Press  to display the time, date, and battery menu again.
4. Press  to return to the previous menu, if desired.


Setting Up the Instrument

The user can specify the display brightness, specify the units that are used for reporting total hemoglobin (THb), change the date and time, and specify the length of time that the instrument is idle before it enters the standby mode.



Setting Display Backlighting

Lighting of the display can be reduced to conserve battery power or increased to improve visibility.

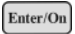1. Display the "Device Settings" menu (a submenu of , see page 8).
2. Press  followed by  to display the screen for changing the backlighting:

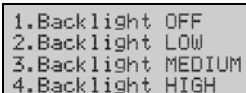

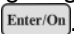



```
1.Backlight OFF
2.Backlight LOW
3.Backlight MEDIUM
4.Backlight HIGH
```

3. Select the backlighting level (or turn it off) by pressing the corresponding number key followed by . The backlighting changes accordingly.
4. Press  to display the "Device Settings" menu again.


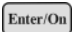

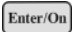

Specifying Units for Total Hemoglobin (THb)

Measured values of total hemoglobin can be expressed in units of mmol/L or g/dL.

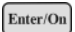1. Display the "Device Settings" menu (a submenu of , see page 8).
2. Press  followed by  to display the screen for changing the units for total hemoglobin:

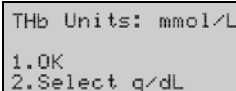




```
THb Units: mmol/L
1.OK
2.Select g/dL
```

3. Press  followed by  to change the units.
4. Press  followed by  when the desired units are displayed. The "Device Settings" menu is again displayed.
5. Press  to return to the previous menu, if desired.

Changing the Date and Time

Changing the Time:


Important: *The User ID for the “QA User” must be entered in order to change the time. See page 21 for details.*

1. Display the “Time, Date, and Battery” menu (a submenu of , see page 8).
2. Press  followed by  to display the current time:

```
(HHMM)
Time: 0911    (24 hr)
1. OK
2. Enter New Value
```




3. Press  followed by  to change the time:

```
(HHMM)
Time: █      (24 hr)
```

4. Use the number keys to enter the new time, then press . The new time is displayed.




```
(HHMM)
Time: 0811    (24 hr)
1. OK
2. Enter New Value
```

Note: *The time is displayed in 24-hour format. For example, 3:30 pm is displayed as 1530.*

5. Press  followed by . The “Time, Date, and Battery” menu is again displayed.
6. Press  to return to the previous menu, if desired.

Changing the Date:

Important: *The User ID for the “QA User” must be entered in order to change the date. See page 21 for details.*

1. Display the “Time, Date, and Battery” menu (a submenu of , see page 8).
2. Press  followed by  to display the current date:

```
(MMDDYY)
Date: 012107
1. OK
2. Enter New Value
```

3. Press **2** followed by **Enter/On** to change the date:

```
(MMDDYY)
Date: █
```

4. Use the number keys to enter the new date, then press **Enter/On**. The new date is displayed.

```
(MMDDYY)
Date: 012207
1. OK
2. Enter New Value
```

5. Press **1** followed by **Enter/On**. The “Time, Date, and Battery” menu is again displayed.
6. Press **Cancel** to return to the previous menu, if desired.

Setting the Standby Delay

The instrument enters a low-power Standby mode after being inactive for a specified length of time (the Standby Delay). The instrument is factory preset for a standby delay time of 60 minutes, but a time of **10** to **180** minutes can be specified.

1. Display the “Device Settings” menu (a submenu of **DATA**, see page 8).
2. Press **3** followed by **Enter/On** to display the standby delay (**60** minutes is the factory default value):

```
Standby Delay in
Minutes: **** 60****
1.OK
2.Enter New Value
```

3. Press **2** followed by **Enter/On** to display the screen for changing the standby delay:

```
Set Standby Delay in
Minutes: █ ****
Range:10-180 minutes
0 Disables Delay
```

4. Use the number keys to enter the new standby delay (or disable the standby delay), then press **Enter/On**. The new standby delay is displayed.

```
Standby Delay in
Minutes: **** 30****
1.OK
2.Enter New Value
```

5. Press **1** followed by **Enter/On**. The “Device Settings” menu is again displayed.
6. Press **Cancel** to return to the previous menu, if desired.




Specifying Mandatory Entry of User ID and/or Patient ID


A User ID and/or Patient ID can be entered for any test. The User ID and/or Patient ID will be included in the results record for tests that are run.

The instrument can be configured so that a valid numeric User ID must be entered before a test result will be displayed. If this option is enabled, the user is prompted to enter (or verify, if it was previously entered) the User ID after the test has started. If a valid User ID is not entered, the instrument locks until a valid User ID is entered.



Enabling the Requirement to Enter a User ID:


Important: *The User ID for the “QA User” must be entered in order to enable the requirement to enter a User ID. See page 21 for details.*

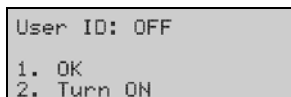1. Display the “User and Patient ID” menu (a submenu of , “Data Management,” see page 8).
2. Press  followed by  to display the current User ID setting:




```
User ID: OFF
1. OK
2. Turn ON
```

3. Press  followed by . A confirmation prompt is displayed, and the “User ID” menu is displayed:

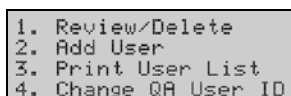




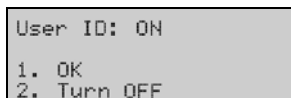
```
1. Review/Delete
2. Add User
3. Print User List
4. Change QA User ID
```

4. If needed, review the list of authorized users, add or delete authorized users, or print the user list (see below).
5. Press  to display the “Data Management” menu.

Disabling the Requirement to Enter a User ID:


Important: *The User ID for the “QA User” must be entered in order to disable the requirement to enter a User ID. See page 21 for details.*

1. Display the “User and Patient ID” menu (a submenu of , “Data Management,” see page 8).
2. Press  followed by  to display the current User ID setting:






```
User ID: ON
1. OK
2. Turn OFF
```

3. Press  followed by . A confirmation prompt is displayed, and the “User and Patient ID” menu is again displayed.

4. Press  to return to the previous menu, if desired.

Note: *The current User ID and/or Patient ID must be re-entered if the instrument is idle for longer than 15 minutes or is turned off.*

Adding a User ID:

1. Display the "User and Patient ID" menu (a submenu of , "Data Management," see page 8).
2. Press  followed by  to display the current User ID setting:

```
User ID: ON
1. OK
2. Turn OFF
```





Note: *User ID: must be **ON** to proceed.*

3. Press  followed by  to display the "User ID" menu:




```
1. Review/Delete
2. Add User
3. Print User List
4. Change QA User ID
```

4. Press  followed by  to display the screen for entering a new User ID:
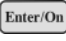
```
Enter User ID:
*****
<Up to 9 digits;
no leading zeros>
```

5. Enter the new User ID, using the number keys, then press . A confirmation screen is displayed.
6. Press  followed by  to confirm the new User ID. The "User ID" menu is again displayed.
7. Press  to return to the previous menu, if desired.

Reviewing Users /Deleting User IDs:

1. Display the "User and Patient ID" menu (a submenu of , "Data Management," see page 8).
2. Press  followed by  to display the current User ID setting:

```
User ID: ON
1. OK
2. Turn OFF
```

Note: *User ID: must be **ON** to proceed.*

3. Press **1** followed by **Enter/On** to display the "User ID" menu:

```
1. Review/Delete
2. Add User
3. Print User List
4. Change QA User ID
```

4. Press **1** followed by **Enter/On** to display the first User ID in the list:

```
User ID: 4733
+/- to Scroll
1.Return to PrevMenu
2.Delete This ID
```

5. Press **Yes** or **No** to scroll through the list of users. To delete a user, display that user, press **2** followed by **Enter/On**, and respond **Yes** to the confirmation prompt.
6. Press **1** followed by **Enter/On** to display the "User ID" menu.
7. Press **Cancel** to return to the previous menu, if desired.

Printing a List of Users:

1. Prepare the printer (see page 30).
2. Display the "User and Patient ID" menu (a submenu of **DATA**, "Data Management," see page 8).
3. Press **3** followed by **Enter/On** to display the current User ID setting:

```
User ID: ON
1. OK
2. Turn OFF
```

Note: *User ID: must be **ON** to proceed.*

4. Press **1** followed by **Enter/On** to display the "User ID" menu:

```
1. Review/Delete
2. Add User
3. Print User List
4. Change QA User ID
```

5. Press **3** followed by **Enter/On** to print the list. A message is displayed after the list has printed:




```
Printing Complete.
Press ENTER.
```

6. Press **Enter/On** to display the "User ID" menu.
7. Press **Cancel** to return to the previous menu, if desired.

Changing the QA User ID

The default QA User ID is **123456**. The default QA User ID can be changed so that only a user with access to the new QA user ID can change the time and date or change User ID and Patient requirements.

Important: *The User ID for the “QA User” must be entered in order to change the QA User ID. If the QA User ID is changed, be sure to record the new QA User ID. If the QA User ID is lost, contact Technical Support for assistance (see page ii).*


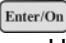
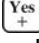
1. Display the “User and Patient ID” menu (a submenu of , “Data Management,” see page 8).
2. Press  followed by  to display the current User ID setting:

```
User ID: ON
1. OK
2. Turn OFF
```


Note: *User ID: must be **ON** to proceed.*

3. Press  followed by  to display the “User ID” menu:


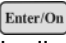
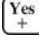

```
1. Review/Delete
2. Add User
3. Print User List
4. Change QA User ID
```

4. Press  followed by , then respond  to the confirmation prompt. The screen for entering a new User ID is displayed:

```
Enter User ID:
*****
<Up to 9 digits;
no leading zeros>
```

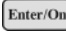5. Enter the new User ID, using the number keys, then press . A confirmation screen is displayed:




```
User ID:
***** 911*****
1. OK
2. Enter New Value
```


6. Press  followed by , then respond  to the confirmation prompt. The “User ID” menu is again displayed.
7. Press  to return to the previous menu, if desired.

Requiring Entry of a Patient ID



The instrument can be configured so that a Patient ID must be entered before a test can be run.

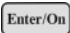
Important: *The User ID for the “QA User” must be entered in order to enable the requirement to enter a Patient ID. See page 21 for details.*

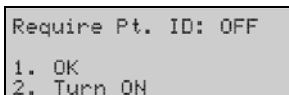1. Display the “User and Patient ID” menu (a submenu of , “Data Management,” see page 8).
2. Press  followed by  to display the current Patient ID setting:





```
Require Pt. ID: OFF
1. OK
2. Turn ON
```



3. Press  followed by . A screen is displayed for specifying the minimum and maximum number of digits:

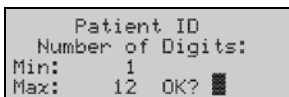

```
Patient ID
Number of Digits:
Min: 1
Max: 12 OK? █
```

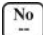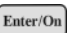4. If the minimum and/or maximum number of digits are to be changed, press  and enter a new value (followed by ) for the minimum and maximum number of digits.

Note: *If the minimum and/or maximum number of digits is changed, the instrument will continue to prompt that up to twelve digits is acceptable when entering a Patient ID. Operators should be instructed to ignore this prompt.*

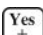5. Respond  to the confirmation prompt. The “User and Patient ID” menu is again displayed.
6. Press  to return to the previous menu, if desired.


Specifying Oximetry Site Prompts

The results from each test can be tagged with the anatomical site from which the blood sample was drawn. There are 15 main oximetry sites, some of which are divided into optional subsites:

Main Oximetry Site	Code	Subsite Codes	
No Label	0		
AO: Aorta	1	1: Ascending 2: Descending	
APV: Anomalous pulmonary vein	2		
ART: Artery, systemic	3	1: Brachial 2: Femoral 3: Iliac	4: Renal
IVC: Inferior vena cava	4	1: High 2: Low	
CS: Coronary sinus	5		
LA: Left atrium	6		
LV: Left ventricle	7	1: Outflow 2: Inflow 3: Apex	
PA: Pulmonary artery	8	1: Right 2: Main 3: Left	
PAW: Pulmonary artery wedge	9	1: Right 2: Left	
PV: Pulmonary vein	10	1: Right 2: Right Upper 3: Right Lower	4: Left 5: Left Upper 6: Left Lower
RA: Right atrium	11	1: High 2: Mid 3: Low	4: Lateral Wall 5: Tricuspid
RV: Right ventricle	12	1: Outflow 2: Inflow 3: Apex	
SVC: Superior vena cava	13	1: High 2: Low 3: PLSVC	4: Innominate 5: Jugular
UA: Umbilical artery	14		
UV: Umbilical vein	15		




Specifying Use of Oximetry Sites


If use of oximetry sites is enabled, the instrument prompts the user to specify whether an oximetry site code and subsite code are to be entered whenever a specimen is run.

If  is selected, the operator must then enter the numerical code for the main oximetry site from a list displayed on the screen.


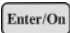



If  is selected, the result will display in the usual manner.

Note: If the Patient ID is also entered when testing, the AVOXimeter 1000E can calculate Saturation Step-Ups for a patient using the average oxyhemoglobin saturation (%HbO₂) values that were obtained at each site during the catheterization.

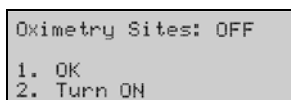1. Display the "Data Management" menu (a submenu of , see page 8).
2. Press  followed by  to display the oximetry screen:






```
Oximetry Sites: OFF
1. OK
2. Turn ON
```


3. Press  followed by . A confirmation prompt is displayed and the instrument will prompt the user if they would like to turn on oximetry subsites as well. Press  to turn them on or  to leave them turned off. The "Data Management" menu is again displayed.
4. Press  to return to the previous menu, if desired.

Note: *When use of oximetry sites is enabled, the instrument prompts for entry of the oximetry site code and subsite code during the test procedure (page 29).*

Entering a Different Value for Hüfner's Number


Hüfner's number (the volume of oxygen that can be carried by one gram of hemoglobin) is used to calculate the oxygen content of a sample (see page 1). Although Hüfner's number is generally assumed to be 1.39 mL/g of Hb, facilities may use a different value and may wish to set the AVOXimeter 1000E to match other instruments in the facility.

1. Display the "Calibration" menu (a submenu of , see page 8).
2. Press  followed by  to display the screen for accepting or changing the value used for Hüfner's number:

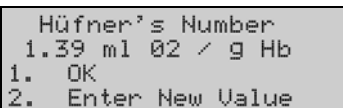

```
Hüfner's Number
1.39 ml 02 / g Hb
1. OK
2. Enter New Value
```


3. Press  followed by  to display the screen for entering a different value:

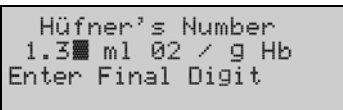


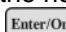

```
Hüfner's Number
1.3 ml 02 / g Hb
Enter Final Digit
```

Note: A default value of **1.39** is entered at the factory. Values ranging from **1.30** to **1.39** can be entered.

4. Press the number key (e.g., ) corresponding to the new value:

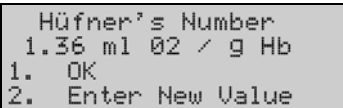

```
Hüfner's Number
1.36 ml 02 / g Hb
1. OK
2. Enter New Value
```

5. Check that the new Hüfner's number is correctly displayed, then press  followed by . The "Calibration" menu is again displayed.
6. Press  to return to the previous menu, if desired.

This page was left intentionally blank.

4 Operation

Startup

1. Press **Enter/On**. The instrument starts and performs a series of self-tests.
2. “READY” and “Insert Cuvette” are displayed when a test can be run:

```

---READY---
Insert Cuvette
Pt. ID:      None
UserID:      None
  
```

3. Run the quality control test(s) for the day (see page 31).

Entering a User ID (Optional)

1. Display the “User and Patient ID” menu (a submenu of **DATA**, “Data Management,” see page 8).
2. Press **1** followed by **Enter/On** to display the screen for entering the User ID:

```

Enter User ID:
*****
<Up to 9 digits;
no leading zeros>
  
```

3. Enter the User ID, then press **Enter/On**. A confirmation screen is displayed.
4. Press **1** followed by **Enter/On**, then press **Cancel** three times to return to the “READY” and “Insert Cuvette” screen.

Entering a Patient ID (Optional)

Note: Although entering a Patient ID is optional, it is strongly recommended. If Patient ID is not entered, Saturation Step-Ups cannot be computed.

1. Display the “User and Patient ID” menu (a submenu of **DATA**, “Data Management,” see page 8).
2. Press **2** followed by **Enter/On** to display the screen for entering the Patient ID:

```

Enter Patient ID:
*****
<Up to 12 digits;
leading zeros OK>
  
```

3. Enter the Patient ID, then press **Enter/On**.
4. Press **Cancel** three times to return to the “READY” - “Insert Cuvette” screen.

Sample Collection and Preparation

Sample Collection

Collect freshly-drawn whole blood samples in a heparin- or EDTA-anticoagulated syringe. Do not use samples that contain excessive volumes of anticoagulant or are diluted with saline.

Note: Refer to CLSI document H18-A3, entitled "Procedures for the Handling and Processing of Blood Specimens – Approved Guideline, Third Edition" for additional information on sample collection.



CAUTION: Universal safety precautions should be taken when handling and processing samples. Spills should be immediately disinfected with an appropriate disinfectant solution to avoid spreading contamination to laboratory personnel or equipment.

Sample Preparation

1. If the sample was not infused into the cuvette immediately after blood draw, mix the whole blood sample by rolling the syringe between the palms of your hands.
2. Connect a syringe containing the sample to an unused cuvette. Hold the cuvette by means of the finger grip on the black cap (see below).
3. Firmly holding the syringe and cuvette at a 45 degree angle, fill the cuvette by gently pressing the syringe plunger.




CAUTION: Never force sample into the cuvette. If a cuvette does not fill easily, discard it and use a new one.

4. Fill cuvette until the sample fills the cuvette to the opposite end. Do not continue to fill the cuvette and cause the vent patch to bulge.



CAUTION: Carefully review the additional precautions on page 9.

5. Verify that the light path area is free of bubbles.
6. Remove any sample from the exterior of the cuvette before placing the cuvette (with syringe still attached) into the test chamber.

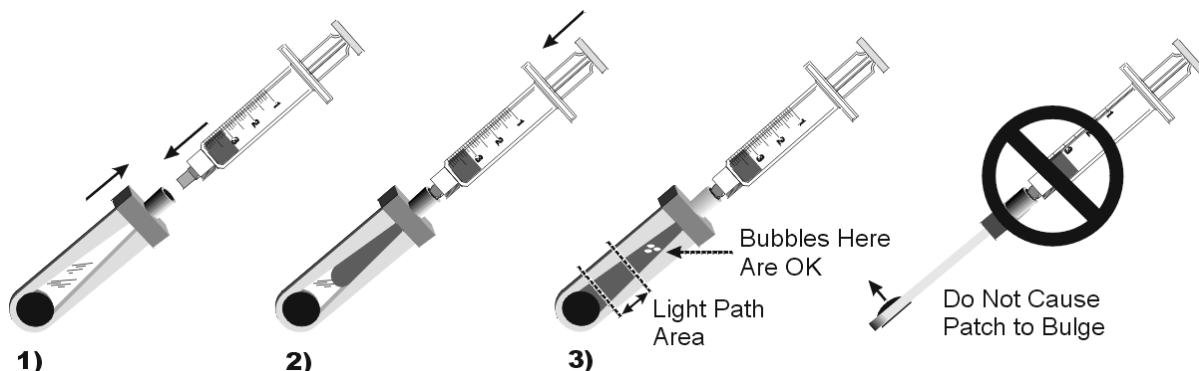

Running a Test


1. Enter the User ID and/or Patient ID, if desired (see page 27).
2. Verify that the instrument is ready to run a test and that the “READY” - “Insert Cuvette” screen is displayed:

```

---READY---
Insert Cuvette
Pt. ID:      None
UserID:     None
  
```

3. Holding the cuvette by the finger grip on the black cap, insert the cuvette (with the syringe still attached) into the test chamber, as shown below.

Important: *Always keep the syringe attached when inserting the cuvette into the test chamber. Removing the syringe may cause inaccurate results.*

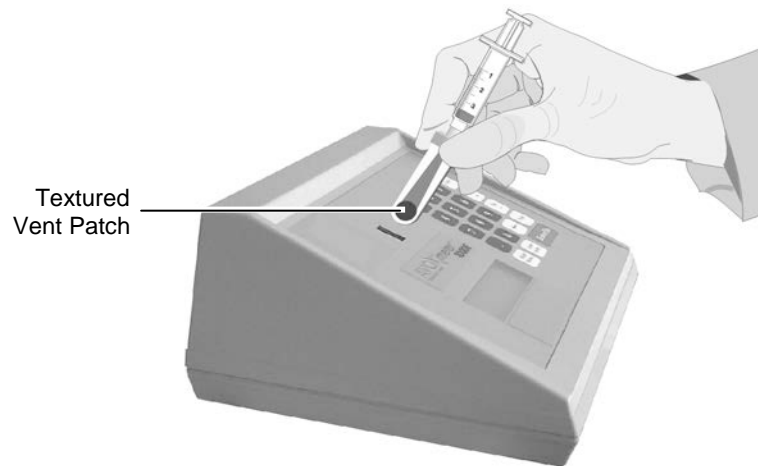

Note: If an optical quality control test is being run, insert the yellow or orange filter instead of the cuvette. If use of oximetry sites is enabled (page 24), enter the site information when prompted.



CAUTION: *Never inject sample directly into the test chamber.*



CAUTION: *Carefully review the additional precautions on page 9.*

4. Test results are displayed within ten seconds:

```

Sample # 49
%HbO2 = 94.8 %
[O2] = 18.6 ml/dl
[THb] = 14.1 g/dl
  
```

5. Holding the cuvette by the finger grip on the black cap, remove the cuvette from the test chamber.

Using the Printer

An optional printer can be connected by means of the serial port at the rear of the instrument (see page 10).




Note: Contact your Accriva representative to purchase the optional printer.


Printing the Current Test Results

1. When the test results are displayed on the screen, press .




Specifying Automatic Printing of Results

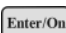The instrument can be configured so that results are automatically printed at the end of a test.

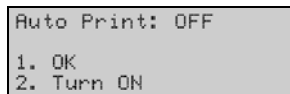1. Display the "Printer Mode" menu (a submenu of , see page 8).
2. Press  followed by  to display the automatic printing screen:







```
Auto Print: OFF
1. OK
2. Turn ON
```

3. Press  followed by . A confirmation prompt is displayed, and the "Printer Mode" menu is again displayed.
4. Press  to return to the previous menu, if desired.



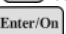
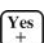

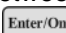


Changing the Serial Port Baud Rate and Parity

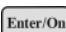If a computer is being connected to the AVOXimeter 1000E, the baud rate and parity of the serial port on the instrument may need to be changed to optimize communication.

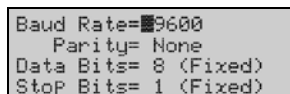1. Display the "Printer Mode" menu (a submenu of , see page 8).
2. Press  followed by  to display the port parameters:



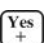


```
Baud Rate=9600
Parity= None
Data Bits= 8 (Fixed)
Stop Bits= 1 (Fixed)
```


3. Press  and/or  to change the baud rate. When the correct value is displayed, press  to advance the cursor to the "Parity" field.
4. Press  and/or  to toggle between "NONE," "ODD," and "EVEN." When the correct value is displayed, press .
5. Press  to confirm the new values. The "Printer Mode" menu is again displayed.
6. Press  to return to the previous menu, if desired.

Data Management


Stored results can be reviewed, printed, and purged (if desired).

Printing all Stored Data

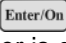

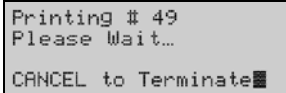
1. (If needed) Turn on the printer.
2. Display the "Printer Mode" menu (a submenu of , see page 8).
3. Press  followed by . All test records are printed, beginning with the latest test. The record number is displayed as each record is printed:




```
Printing # 49
Please Wait...
CANCEL to Terminate
```


4. When printing is completed, the "Printer Mode" menu is again displayed.
5. Press  to return to the previous menu, if desired.

Printing All Optical Quality Control Data

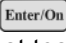1. (If needed) Turn on the printer.
2. Display the "Printer Mode" menu (a submenu of , see page 8).
3. Press  followed by . All optical quality control test records are printed, beginning with the latest test. The record number is displayed as each record is printed:

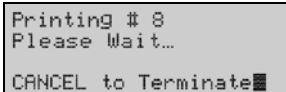

```
Printing # 8
Please Wait...
CANCEL to Terminate
```

4. When printing is completed, the "Printer Mode" menu is again displayed.
5. Press  to return to the previous menu, if desired.

Reviewing and/or Printing the Last Sample

1. (If needed) Turn on the printer.
2. Display the "Stored Data" menu (a submenu of **Main Menu**, see page 8).
3. Press **2** followed by **Enter/On**. The last test record is displayed:

```

Sample # 49
%HbO2 = 94.8 %
[O2] = 18.6 ml/dl
[THb] = 14.1 g/dl
    
```

4. Press **Enter/On** to view additional information, if desired.
5. Press **Print** to print the test record.
6. Press **Cancel** to return to the previous menu, if desired.

Locating, Reviewing, and/or Printing any Sample

1. (If needed) Turn on the printer.
2. Display the "Stored Data" menu (a submenu of **Main Menu**, see page 8).
3. Press **1** followed by **Enter/On**. A screen to specify the test record number is displayed:

```

Show Sample #
Valid Range:
1 to 394
    
```

4. Enter the desired record number and press **Enter/On** to view the first page of the sample record:

```

Sample # 24
%HbO2 = 94.1 %
[O2] = 11.6 ml/dl
[THb] = 8.9 g/dl
    
```


5. Press **Enter/On** again to display the second page of results for that sample.
 - The second page of a Patient sample record contains the User ID, Patient ID, and Oximetry Site (if entered).
 - The second page of a QC sample record contains the QC Level and Lot Number (if entered).
6. If needed, press **Yes** (to view the next record) or **No** (to view the previous record).
7. Press **Print** to print a test record while it is displayed.
8. Press **Cancel** to return to the previous menu, if desired.

Aborting Printing of Results

1. Press while results are printing to discontinue printing of additional results. A confirmation prompt is displayed, indicating that all samples were not printed.
2. Press to return to the previous menu.

Purging all Test Records

1. Display the "Stored Data" menu (a submenu of , see page 8).
2. Press followed by . A confirmation prompt is displayed:

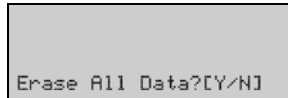

Erase All Data?[Y/N]

3. Press to confirm.

Note: Purging test records may take up to two minutes. "Purging Data..." is displayed on the screen while the purge is being performed.

4. When the purge is completed, "All Data Purged" is displayed on the screen, then the "Stored Data" menu is again displayed.
5. Press to return to the previous menu, if desired.

Quality Control

Good laboratory practice recommends that routine quality control testing should be part of a comprehensive quality assurance program. Quality control testing of the AVOXimeter 1000E consists of the following operations:

- Daily optical quality control.
- Weekly testing of one level of liquid controls.

In addition, the AVOXimeter 1000E performs a “self-check” to verify that the light source is operating properly every time it is turned on.

Note: *If quality control results are out of range, refer to the Troubleshooting section for instructions. If the “self-check” fails, contact Technical Support (see page ii).*

Performing Optical Quality Control

The yellow and orange optical filters supplied with the AVOXimeter 1000E provide a convenient means of verifying that the optics are not obscured by blood or debris and that the instrument is properly calibrated. Each filter simulates a cuvette containing a blood sample of known composition and has a serial number that matches the serial number of the instrument.



CAUTION: *Each filter is labeled with a serial number that matches the serial number of the instrument. Verify that the serial number on each filter matches the serial number of the instrument when performing optical quality control. Each set of yellow and orange filters can only be used only with the AVOXimeter 1000E of the same serial number and cannot be shared between different instruments.*

1. Obtain the yellow and orange optical filters.
2. Run a test (see page 29), using the yellow optical filter in place of a test cuvette.
3. Record the THb and %HbO₂ results, or print the results of the test (see page 30).
4. Repeat steps 2 and 3 using the orange filter.
5. Verify that the results for each filter are within the expected ranges listed below:

Optical Filter	%HbO ₂ Expected Range	THb Expected Range
Yellow	93.5 to 96.5%	7.7 to 8.3 g/dL
Orange	37.2 to 40.8%	16.4 to 17.6 g/dL

Note: *The Expected Range is also shown on the label on the side of each filter.*

6. If desired, plot the optical filter results on daily logs such as the ones provided in the *Quality Control Logs* section on page 65.

Running Liquid Controls




It is recommended to use the following liquid controls.

- RNA Medical CO-Oximeter controls, available from RNA Medical, Devens, MA., phone 978 772-9070 or toll-free 800-533-6162.
 1. Fill a test cuvette with a liquid control (see page 28).
 2. Run a test (see page 29).
 3. Record the THb and %HbO₂ results, or print the results of the test (see page 30).
 4. Verify that the results are within the expected range of values for the liquid control.


Entering Liquid Control Lot Numbers

Three lot numbers for each of three levels of liquid control can be entered into the instrument for later reference.

Note: Lot numbers of liquid controls can be manually tagged to each liquid QC test as it is run (see page 37).

1. Display the “Data Management” menu (a submenu of , see page 8).
2. Press  followed by  to display a menu for selection of a control level:







```
Select Level:
1. Level 1
2. Level 2
3. Level 3
```

3. Select the level for which a lot number is to be added and press . A menu for selection of a lot number is displayed:

```
L2 Select Lot:
1. 943255
2. 122547
3. UNDEFINED
```

4. Select the lot to be changed and press . A screen for entry of the lot number is displayed:

```
QC Lot:
*****
1. OK
2. Enter New Value
```

5. Press  followed by  to enter a new value.
6. Enter the new lot number, using the number keys, then press . A confirmation screen is displayed.
7. Press  followed by  to confirm the new lot number. The “Data Management” menu is again displayed.
8. Press  to return to the previous menu, if desired.

Tagging a Liquid Control Test with a Lot Number

The lot number of the control being tested can be included in the results for the test.

Note: Lot numbers of liquid controls must be entered before they can be tagged to a liquid QC test (see page 36).

1. Insert a cuvette filled with liquid quality control material (see page 35). A prompt to confirm liquid QC is displayed:

```
Is this liquid QC?  
[Y/N]
```

2. Press to confirm. A prompt to confirm that the level and lot numbers are being entered is displayed:

```
Do you wish to  
enter level and lot  
numbers? [Y/N]
```

3. Press to confirm. A menu for selection of a control level is displayed:

```
Select Level:  
1. Level 1  
2. Level 2  
3. Level 3
```

4. Select the level that is being run and press . A menu for selection of a lot number is displayed:




```
L2 Select Lot:  
1. 943255  
2. 122547  
3. UNDEFINED
```


5. Select the previously entered lot number that is being run and press .
6. The sample results will be displayed on the screen, and the lot number and LQC level will be added to the record for that sample.

Enabling QC Lockout


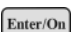
If QC Lockout is enabled, the operator cannot run a test if optical quality control is not successfully run at the specified intervals.


Note: The User ID for the “QA User” must be entered in order to enable QC Lockout and specify the interval for running optical quality control. See page 21 for details.

1. Display the “Data Management” menu (a submenu of , see page 8).
2. Press  followed by  to display the current QC Lockout setting:

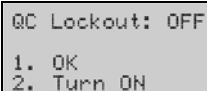




```
QC Lockout: OFF
1. OK
2. Turn ON
```

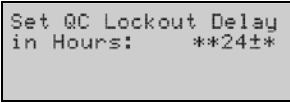3. Press  followed by . A screen is displayed for specifying the interval between optical quality control tests:



```
Set QC Lockout Delay
in Hours:      **24±**
```

4. To change the interval, press  and enter a new value (followed by ). The “Data Management” menu is again displayed.
5. Press  to return to the previous menu, if desired.

Calibration



The AVOXimeter 1000E is factory-calibrated and employs highly stable state-of-the-art light sources.


Cuvette Pathlength

Accuracy of total hemoglobin measurements depends on using the correct cuvette pathlength (see page 1). The user must check the pathlength whenever using a different lot number of cuvettes.

Note: The cuvette lot number and pathlength are included on the carton label and bag for each lot of cuvettes.

Entering a Different Cuvette Pathlength:


1. Display the "Calibration" menu (a submenu of , see page 8).
2. Press **3** followed by  to display the screen for entering the cuvette pathlength:

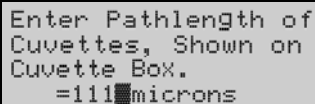

```

Enter Pathlength of
Cuvettes, Shown on
Cuvette Box.
=111microns
  
```

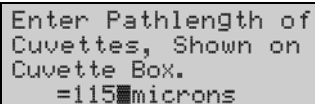


3. To change the interval, press  and enter a new value:



```

Enter Pathlength of
Cuvettes, Shown on
Cuvette Box.
=115microns
  
```




Note: Pathlengths ranging from **90 microns** to **132 microns** can be entered.

4. To change the interval, press  and enter a new value.
5. Check that the new pathlength is correctly displayed, then press . The "Calibration" menu is again displayed:
6. Press  to return to the previous menu, if desired.

Re-Calibration

If the AVOXimeter 1000E results are out of range with either blood samples or controls and troubleshooting does not resolve the issue, re-calibration may be necessary. Please contact Technical Support for more information on re-calibration.

Shutdown

1. Press  (see page 8).
2. Press  followed by . The instrument will shut down.

Or:

1. Simultaneously press  and . The instrument will shut down.

Hemodynamic Calculations

The AVOXimeter 1000E can calculate ten hemodynamic variables (such as body surface area, estimated oxygen consumption rate, and cardiac output using the Fick principle) from data that is entered by the operator or measured by the instrument. It can also calculate differences in oxygen saturation between adjacent anatomical sites (Saturation Step-Ups) from which blood samples were taken, to aid in diagnosing intracardiac and great vessel shunts.

Entering the Patient Age, Height, Weight, and Sex

The patient's age, height, weight, and sex are used to calculate the patient's body surface area and rate of oxygen consumption.

1. Display page 1 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **1** followed by **Enter/On** to display the screen for entering the patient's age:

```
Patient's Age
Age = █ Years
```

Note: If an age is already displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

3. Enter the age using the number keys, then press **Enter/On**. A confirmation prompt is displayed:

```
Patient's Age
Age = 73 Years
OK? [Y/N] █
```

4. Check that the age is correctly displayed, then press **Yes** **+** to confirm. The screen for selecting the units for patient's height is displayed:

```
1.Height:Inches
2.Height:Centimeters
```


5. Select **1** (for **inches**) or **2** (for **centimeters**), then press **Enter/On**. The screen for entering the patient's height is displayed:

```
Patient Height
= █ inches
```


Note: If a height is already displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

6. Enter the height using the number keys, then press **Enter/On**. A confirmation prompt is displayed:


```
Patient Height
= 72 inches
OK? [Y/N] █
```


7. Check that the height is correctly displayed, then press  to confirm. The screen for selecting the units for patient's weight is displayed:

```
1.Weight:Pounds
2.Weight:Kg
```


8. Select **1** (for **pounds**) or **2** (for **kg**), then press . The screen for entering the patient's weight is displayed:

```
Patient Weight
= █ Pounds
```


Note: If a weight is already displayed, delete numerals from the right by pressing  once for each numeral to be deleted.

9. Enter the weight using the number keys, then press . A confirmation prompt is displayed:


```
Patient Weight
= 237 Pounds
OK? [Y/N] █
```

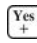
10. Check that the weight is correctly displayed, then press  to confirm. The screen for selecting the units for patient's sex is displayed:

```
1.Male
2.Female
```

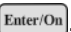11. Select **1** (for **male**) or **2** (for **female**), then press . A confirmation prompt is displayed:

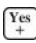


```
Confirm: Male? █ [Y/N]
```


12. Press  to confirm. The "Hemodynamics" menu is again displayed. The Oxygen Uptake Rate and Body Surface Area for the patient have been automatically calculated. This data can be accessed through the "Hemodynamics" Menus (see pages 43-44).

13. Press  to return to the previous menu, if desired.


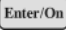
Oxygen Uptake Rate


The patient's rate of oxygen consumption (oxygen uptake rate) is calculated from the patient's age, height, weight, and sex (see page 41). Alternately, a measured value can be entered.

1. Display page 1 of the "Hemodynamics" menu (a submenu of , see page 8).
2. Press  followed by  to display the patient's oxygen uptake rate:





```
Oxygen Uptake Rate
= 337 ml/min
```

3. To enter a measured value, delete numerals from the right by pressing  once for each numeral to be deleted, then enter the measured value using the number keys.
4. Check that the correct value is displayed, then press . A confirmation prompt is displayed:

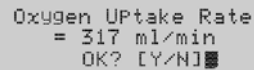


```
Oxygen Uptake Rate
= 317 ml/min
OK? [Y/N]
```

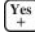5. Press  to confirm. The "Hemodynamics" menu is again displayed.
6. Press  to return to the previous menu, if desired.

Body Surface Area


The patient's body surface area is calculated from the patient's age, height, weight, and sex (see page 41). Alternately, a measured value can be entered.

1. Display page 1 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **3** followed by **Enter/On** to display the patient's body surface area:

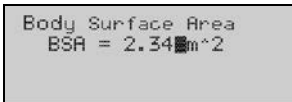

Body Surface Area
BSA = 2.34 m²

3. To enter a measured value, delete numerals from the right by pressing **←** once for each numeral to be deleted, then enter the measured value using the number keys.
4. Check that the correct value is displayed, then press **Enter/On**. A confirmation prompt is displayed:

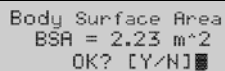

Body Surface Area
BSA = 2.23 m²
OK? [Y/N]

5. Press **Yes** to confirm. The "Hemodynamics" menu is again displayed.
6. Press **Cancel** to return to the previous menu, if desired.

Saturation Step-Ups

Saturation step-ups are the differences in oxyhemoglobin saturation (%HbO₂) between adjacent anatomical sites. These values are used as an aid in diagnosing intracardiac and great vessel shunts.

Important: In order to calculate saturation step-ups for a patient, the AVOXimeter 1000E results for that patient must be labeled with the Patient ID (see page 27) and the anatomical site from which the sample was obtained (see page 23).

The AVOXimeter 1000E uses all measurements stored in the system for a patient when calculating oxyhemoglobin saturation step-ups. The system searches the database for all results for each main oximetry site for that patient, averages the results for each site, and calculates the oxyhemoglobin saturation step-ups from the average result for each main site.

Note: Use of average readings rather than individual values improves the precision of Saturation Step-Up calculations.

1. If needed, enter the Patient ID for which Saturation Step-Ups are to be calculated (see page 27).
2. Display page 1 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
3. Press **4** followed by **Enter/On**. The Patient ID for which oxyhemoglobin saturation step-ups will be calculated is displayed:

```
Press ENTER to
View Step-Ups for
Pt. ID: 004679988
```

4. Verify that the Patient ID is correct, then press **Enter/On**. The first page of oxyhemoglobin saturation step-ups for the patient is displayed:

```
RA-SVC= 7.1
RV-RA= 2.1
PA-PV= 0.9
ENTER For More...
```

Note: The following abbreviations are used:

RA	Right Atrium	PA	Pulmonary Artery
RV	Right Ventricle	PV	Pulmonary Vein
LA	Left Atrium	AO	Aorta
LV	Left Ventricle	SVC	Superior Vena Cava

5. Press **Enter/On** to display the second page of oxyhemoglobin saturation step-ups for the patient:

```
PV-LA= 1.1
LA-LV= 2.6
LV-AO= 0.6
ENTER When Done...
```

6. Press **Enter/On** when done. The "Hemodynamics" menu is again displayed.
7. Press **Cancel** to return to the previous menu, if desired.

Flow Calculations

Flow calculations are used to calculate cardiac output, pulmonary flow, and other blood flow values.

Calculating Cardiac Output:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **5** followed by **Enter/On**. A menu of flow calculations is displayed:

```
1.Systemic Flow
2.Pulmonary Flow
3.Flow Ratio
4.Cardiac Index
```

3. Press **1** followed by **Enter/On**. The screen for entering oxygen uptake (if not already calculated), venous oxygen content, and arterial oxygen content is displayed:

```
**Cardiac Output**
O2 Uptake=    ml/min
Venous[O2]=   ml/dl
Art.[O2]=     ml/dl
```

4. Enter the oxygen uptake (if not already calculated), venous oxygen content, and arterial oxygen content, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If oxygen uptake was already calculated, the value will be displayed.
If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

5. After all values have been entered, press **Yes** to confirm. The calculated cardiac output is displayed:

```
Cardiac Output =
4825 ml/min
ENTER to Continue
```

6. Press **Enter/On** when done. The menu of flow calculations is again displayed.
7. Press **Cancel** to return to the previous menu, if desired.

Calculating Pulmonary Flow:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **5** followed by **Enter/On**. A menu of flow calculations is displayed:

```
1.Systemic Flow
2.Pulmonary Flow
3.Flow Ratio
4.Cardiac Index
```

3. Press **2** followed by **Enter/On**. The screen for entering oxygen uptake, pulmonary venous oxygen content, and pulmonary arterial oxygen content is displayed:

```
**Pulmonary Flow**
O2 Uptake=    ml/min
Pulm.VenO2=   ml/dl
Pulm.Arto2=   ml/dl
```

4. Enter the oxygen uptake, pulmonary venous oxygen content, and pulmonary arterial oxygen content, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

5. After all values have been entered, press **Yes** to confirm. The calculated pulmonary flow is displayed:

```
Pulmonary Flow =
5200 ml/min
ENTER to Continue
```

6. Press **Enter/On** when done. The menu of flow calculations is again displayed.
7. Press **Cancel** to return to the previous menu, if desired.

Calculating Flow Ratios from Saturations:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **5** followed by **Enter/On**. A menu of flow calculations is displayed:

```
1.Systemic Flow
2.Pulmonary Flow
3.Flow Ratio
4.Cardiac Index
```

3. Press **3** followed by **Enter/On**. A menu of flow ratio calculations is displayed:

```
1.From Saturations
2.From Flows
```

4. Press **1** followed by **Enter/On**. The screen for entering arterial and venous oxygen contents is displayed:

```
1.Arterial%HbO2= █
2.MixedVen%HbO2= %
3.Pulm.Ven%HbO2= %
4.Pulm.Art%HbO2= %
```

5. Enter the arterial oxygen uptake, mixed venous oxygen content, pulmonary venous oxygen content, and pulmonary arterial oxygen content, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

6. After all values have been entered, press **Yes** to confirm. The calculated flow ratio is displayed:

```
Flow Ratio
Pulm/Syst = 0.93
ENTER to Continue
```

7. Press **Enter/On** when done. The menu of flow calculations is again displayed.
8. Press **Cancel** to return to the previous menu, if desired.

Calculating Flow Ratios from Flows:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **5** followed by **Enter/On**. A menu of flow calculations is displayed:

```
1.Systemic Flow
2.Pulmonary Flow
3.Flow Ratio
4.Cardiac Index
```

3. Press **3** followed by **Enter/On**. A menu of flow ratio calculations is displayed:

```
1.From Saturations
2.From Flows
```

4. Press **2** followed by **Enter/On**. The screen for entering pulmonary flow and systemic flow is displayed:

```
**Flow Ratio**
PulmFlow= █l/min
SystFlow=  ml/min
```

5. If needed, enter the pulmonary flow and systemic flow, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If pulmonary flow and systemic flow were already calculated, the values will be displayed.

If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

6. After all values have been entered, press **Yes +** to confirm. The calculated flow ratio is displayed:

```
Flow Ratio
Pulm/Syst = 0.92
ENTER to Continue
```

7. Press **Enter/On** when done. The menu of flow calculations is again displayed.
8. Press **Cancel** to return to the previous menu, if desired.

Calculating Cardiac Index:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8)
2. Press **5** followed by **Enter/On**. A menu of flow calculations is displayed:

```
1.Systemic Flow
2.Pulmonary Flow
3.Flow Ratio
4.Cardiac Index
```


3. Press **4** followed by **Enter/On**. The screen for entering cardiac output and body surface area is displayed:

```
**Cardiac Index**
Cardiac Output=
      5200ml/min
Body Area = 2.17_m^2
```


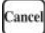
4. If needed, enter the cardiac output and body surface area, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If cardiac output and body surface area were already calculated, the values will be displayed.

If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

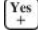- After all values have been entered, press  to confirm. The calculated cardiac index is displayed:




```
Cardiac Index =
  2.396 L/min/m^2
ENTER to Continue
```

- Press  when done. The menu of flow calculations is again displayed.
- Press  to return to the previous menu, if desired.



Systemic and Pulmonary Resistances

Cardiac output and blood pressure readings are used to calculate systemic resistance and pulmonary resistance.


Calculating Systemic Resistance:


- Display page 2 of the "Hemodynamics" menu (a submenu of , see page 8).
- Press  followed by . A menu of resistance calculations is displayed:


```
1.Systemic Resist.
2.Pulmonary Resist.
```

- Press  followed by . A screen for entering cardiac output (if not already calculated), arterial pressure, and right atrial pressure is displayed:



```
*Systemic Resistance
Card.Out=      L/min
Art.Press.=    mmHg
R.AtrialPres=  mmHg
```

- Enter the cardiac output, arterial pressure, and right atrial pressure, using the number keys. After each value is entered, check that the value is correct and press  to move the cursor to the next line.

Note: If cardiac output was already calculated, the value will be displayed.
If an incorrect value is displayed, delete numerals from the right by pressing  once for each numeral to be deleted.

- After all values have been entered, press  to confirm. The calculated systemic resistance is displayed:

```
Systemic Resistance=
0.010 mmHg·min/ml
769.2 dyne·sec·cm-5
ENTER to Continue
```

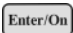- Press  when done. The menu of resistance calculations is again displayed.
- Press  to return to the previous menu, if desired.

Calculating Pulmonary Resistance:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **6** followed by **Enter/On**. A menu of resistance calculations is displayed:

```
1.Systemic Resist.
2.Pulmonary Resist.
```

3. Press **2** followed by **Enter/On**. A screen for entering cardiac output (if not already calculated), pulmonary artery pressure, and left atrial pressure is displayed:

```
Pulmonary Resistance
Card.Out=      l/min
Pulm.Art.Prs=  mmHg
LeftAtrialPrs= mmHg
```

4. Enter the cardiac output, pulmonary artery pressure, and left atrial pressure, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If cardiac output was already calculated, the value will be displayed.

If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

5. After all values have been entered, press **Yes** to confirm. The calculated pulmonary resistance is displayed:

```
Pulmonary Resistance
0.002 mmHg·min/ml
153.8 dyne·sec·cm-5
ENTER to Continue
```

6. Press **Enter/On** when done. The menu of resistance calculations is again displayed.
7. Press **Cancel** to return to the previous menu, if desired.

Stroke Volume and Stroke Index

Cardiac output and heart rate measurements are used to calculate the **stroke volume** (the volume of blood ejected from a ventricle at each beat of the heart). **Stroke Index** is the stroke volume normalized for body mass.

Calculating Stroke Volume:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8).
2. Press **7** followed by **Enter/On**. A menu of stroke calculations is displayed:

```
1.Stroke Volume
2.Stroke Index
```

3. Press **1** followed by **Enter/On**. A screen for entering cardiac output (if not already calculated) and heart rate is displayed:

```
**Stroke Volume**
Cardiac Output=
Heart Rate=
```

4. Enter the cardiac output and heart rate, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If cardiac output was already calculated, the value will be displayed.
If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.

5. After all values have been entered, press **Yes** to confirm. The calculated stroke volume is displayed:

```
Stroke Volume =
80 ml/beat
ENTER to Continue
```

6. Press **Enter/On** when done. The menu of stroke calculations is again displayed.
7. Press **Cancel** to return to the previous menu, if desired.

Calculating Stroke Index:

1. Display page 2 of the "Hemodynamics" menu (a submenu of **DATA**, see page 8)
2. Press **7** followed by **Enter/On**. A menu of stroke calculations is displayed:

```
1.Stroke Volume
2.Stroke Index
```

3. Press **2** followed by **Enter/On**. A screen for stroke volume and body surface area (if not already calculated) is displayed:

```
**Stroke Inde**
Stroke Volume =
      ml/beat
Body Area =    m^2
```

4. If needed, enter the stroke volume and body surface area, using the number keys. After each value is entered, check that the value is correct and press **Enter/On** to move the cursor to the next line.

Note: If stroke volume and body surface area were already calculated, the values will be displayed.

If an incorrect value is displayed, delete numerals from the right by pressing **←** once for each numeral to be deleted.




5. After all values have been entered, press **Yes** to confirm. The calculated stroke index is displayed:


```
Stroke Index =
  34 ml/beat/m^2
ENTER to Continue
```

6. Press **Enter/On** when done. The menu of stroke calculations is again displayed.
7. Press **Cancel** to return to the previous menu, if desired.


Printing Hemodynamic Values

The values that were entered for each calculation and the calculated values can be printed.

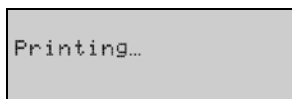



1. (If needed) Turn on the printer.
2. Display page 2 of the "Hemodynamics" menu (a submenu of , see page 8).
3. Press  followed by . A message is displayed while the values are printing:





Printing...

4. When printing is completed, page 2 of the "Hemodynamics" menu is again displayed.
5. Press  to return to the previous menu, if desired.

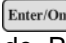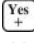Troubleshooting

Symptom	Possible Cause	Action Required
The instrument does not turn on.	1. The AC Adapter is not connected to an AC outlet and/or the instrument, and the battery is discharged.	1. Connect the AC Adapter to an AC outlet and the AVOXimeter 1000E.
The instrument does not respond to keystrokes or cuvette insertion.	1. The microprocessor is locked up.	1. Press the  and  keys simultaneously to turn the instrument off, then turn the instrument back on. 2. If this fails, contact Technical Support (see page ii).
Self-Test fails on power-up. The following error message is displayed: "Cuvette Inserted???"	1. A cuvette is inserted in the instrument. If a cuvette is not inserted in the instrument, see #2 below: 2. Intensities are out of range. 3. The cable from the detector to the circuit board is disconnected. 4. There is no power to the LEDs. The LED cable may be loose, disconnected or improperly placed on the connector at one or both ends. 5. One or more LEDs is defective.	1. Remove cuvette, turn unit off and restart. 2. At "Cuvette Inserted???" screen, press  twice for Diagnostic Mode. Press any key and then press  to re-adjust light sources. When program finishes, turn the instrument off and then on. 3. Open the case and check that the cables are properly connected. 4. Open the case and check that the cables are properly connected. 5. Contact Technical Support (see page ii).
The external printer is not printing.	1. There is no power to the printer. 2. The printer is not connected to the instrument. 3. The wrong print mode, baud rate, or parity is selected. 4. The microprocessor is locked up.	1. Connect the printer to an AC outlet and turn on the printer. 2. Connect the printer to the AVOXimeter 1000E (see page 10). 3. Specify the proper values for the printer (see page 30). 4. See above.

Symptom	Possible Cause	Action Required
Measurements of %HbO ₂ are not accurate	<ol style="list-style-type: none"> 1. The optical detector may be contaminated by blood or foreign material. 2. The instrument may require calibration. 	<ol style="list-style-type: none"> 1. Run optical quality control. If needed, clean the optical detector (see page 59). 2. Re-calibrate the instrument (see page 39).
Measurements of THb are not accurate	<ol style="list-style-type: none"> 1. The wrong cuvette pathlength is entered into the instrument. 2. The optical detector may be contaminated by blood or foreign material. 3. The instrument may require calibration. 	<ol style="list-style-type: none"> 1. Obtain the correct cuvette pathlength value from the cuvette carton and enter the value (see page 39). 2. Clean the optical detector (see page 59). 3. Re-calibrate the instrument (see page 39).
Calculated Pulmonary-to-Systemic Flow Ratio is a negative number or otherwise not believable.	<ol style="list-style-type: none"> 1. Equal values were entered for Pulm. Venous and Pulm. Arterial %HbO₂. 2. An incorrect entry was made for one of the entered quantities. 	<ol style="list-style-type: none"> 1. Obtain correct values for Pulm. Venous and/or Pulm. Arterial %HbO₂. 2. Review and, if necessary, re-enter correct values.
One of the following ERROR: messages is displayed: “%HbO ₂ <> XX.X%” or “%HbCO<> XX.X%” or “%HbMet<> XX.X%” or “%RHb<> XX.X%”	<ol style="list-style-type: none"> 1. A cuvette without a sample was inserted. 2. The cuvette was not properly filled. 3. The optical detector may be contaminated by blood or foreign material. 4. LED intensities are too low. 5. The sample is lipemic. 	<ol style="list-style-type: none"> 1. Test a new cuvette that contains the sample. 2. Ensure that the sample reaches the vent patch when filling and that there are no air bubbles in the optical window. 3. Run optical quality control. If needed, clean the optical detector (see page 59). 4. Remove cuvette, turn unit off, and restart. Verify that the self test passes. 5. If possible, obtain a replacement sample that is not lipemic.
The following ERROR: message is displayed: “THb < 4.0 g/dL”	<ol style="list-style-type: none"> 1. THb may actually be low. In this case, values of THb and %HbO₂ not likely to be accurate. 	<ol style="list-style-type: none"> 1. Contact Technical Support (see page ii).
The following ERROR: message is displayed: “THb > 25.0 g/dL”	<ol style="list-style-type: none"> 1. THb may actually be high. In this case, values of THb and %HbO₂ not likely to be accurate. 	<ol style="list-style-type: none"> 1. Contact Technical Support (see page ii).

Symptom	Possible Cause	Action Required
THb and/or %HbO ₂ readings for yellow or orange filter are out of range.	<ol style="list-style-type: none">1. The optical detector may be contaminated by blood or foreign material.2. The instrument may require calibration.	<ol style="list-style-type: none">1. Clean the optical detector (see page 59).2. Re-calibrate the instrument (see page 39).
The following ERROR: message is displayed: “Uninitialized Vector Service Required”	<ol style="list-style-type: none">1. Microprocessor error.	<ol style="list-style-type: none">1. Press the  and  keys simultaneously to turn the instrument off, then turn the instrument back on.2. If the problem persists, contact Technical Support (see page ii).

This page was left intentionally blank.

5 Maintenance

Periodic maintenance procedures must be performed on the *AVOXimeter 1000E* in order to assure consistent efficient performance or to repair/replace user-serviceable components.

Inspect and clean the exterior of the instrument as required. Remove residual dried blood or other foreign matter using a water-dampened cloth.

Only qualified personnel should perform maintenance as described in this manual.




CAUTION: Maintenance procedures involving opening the instrument are only to be conducted after assuring that the AC adapter has been disconnected from both the instrument and a source of AC power.

Cleaning the Optical Detector


- Materials Needed:**
1. # 0 Phillips screwdriver
 2. 5 mm nutdriver
 3. 1/4 inch nutdriver

1. Disconnect the *AVOXimeter 1000E* from the AC Adapter.
2. Remove the four screws from the bottom of the instrument, using the #0 Phillips screwdriver.
3. Holding the upper and lower covers together, place the instrument in the upright position, with the keypad to your right.
4. Slowly and carefully lift the upper cover, keeping it parallel with the lower part of the instrument, until the upper cover is free of the rear panel.
5. Locate the cable that runs from the battery pack to the main circuit board (see below). Disconnect the cable from the **J302** connector on the main circuit board.

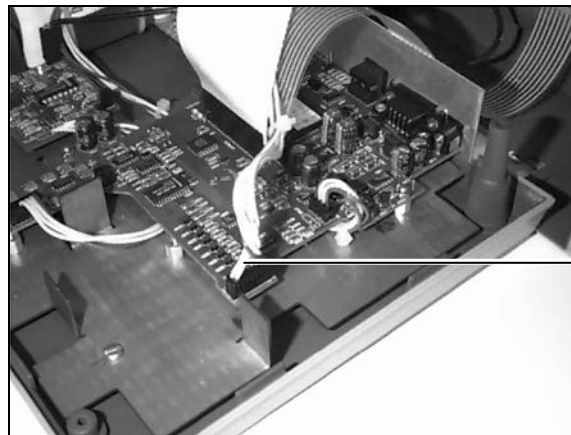

Battery Cable

6. Tilt the upper cover backward to fully expose the main circuit board and small circuit board on the lower part of the instrument.
7. Locate the flat keypad cable that runs from the keypad to the main circuit board. Disconnect the cable from the **Keyboard (J101)** connector on the main circuit board.

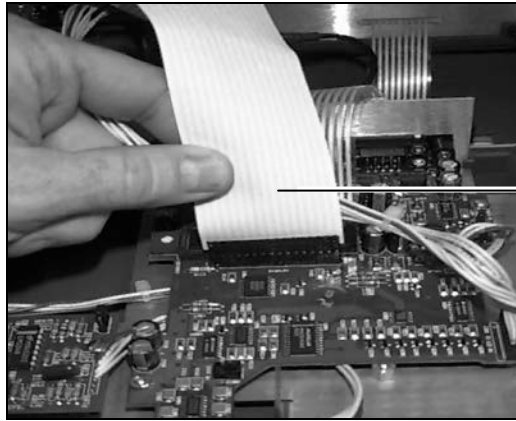


Keypad Cable

8. Locate the flat LCD cable that runs from the LCD to the main circuit board. Disconnect the cable from the **Display (J102)** connector on the main circuit board.


Note: Grasp only the connector at the end of the cable. Gently rock the connector in an upward direction to remove it.

9. Locate the flat cable that runs from the black optical unit to the main circuit board. Disconnect the cable from the **J203** connector on the main circuit board.
10. Locate the coaxial cable that runs from the black optical unit to the small circuit board. Disconnect the cable from the **J1XX** connector on the small circuit board.
11. The upper cover containing the black optical unit can now be rested on a table with the front panel facing down.
12. Using the 5mm nutdriver, remove the four nuts (and washers, if applicable) that secure the black optical unit to the front panel. Then remove the black optical unit from the instrument.

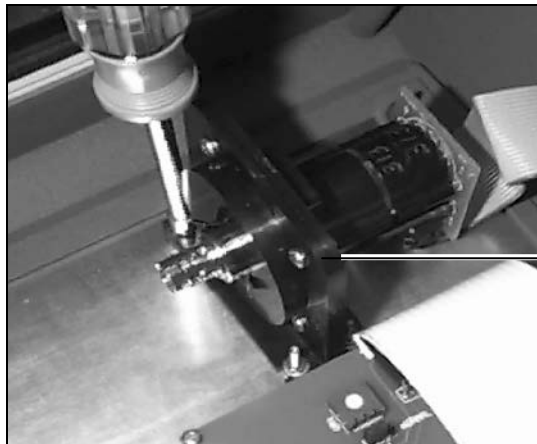


Black Optical Unit

13. Using the 1/4" nutdriver, remove the four screws and four nuts that secure the two halves of the black optical unit. Take care when separating the two halves of the optical unit. Do not lose the small torsion spring or the shutter door. Set them aside in a safe place for reassembly.

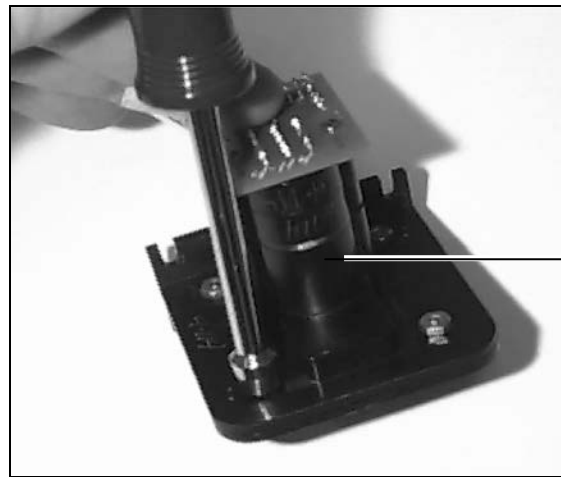

Black Optical Unit


14. Clean the exposed detector with gauze pads dampened with detergent. Do not use abrasives. Then dry off the detector surface, making sure that it is clean. Remove any other debris inside the optical unit.
15. Reassemble the two halves of the optical unit, making sure that the **Shutter Door (AV6400)** and **Torsion Spring (AV6600)** are installed correctly (the two door ribs will face outward). One leg of the spring fits into a hole in the door slot, the other leg rests behind the shutter door. Bolt the optical unit back together.

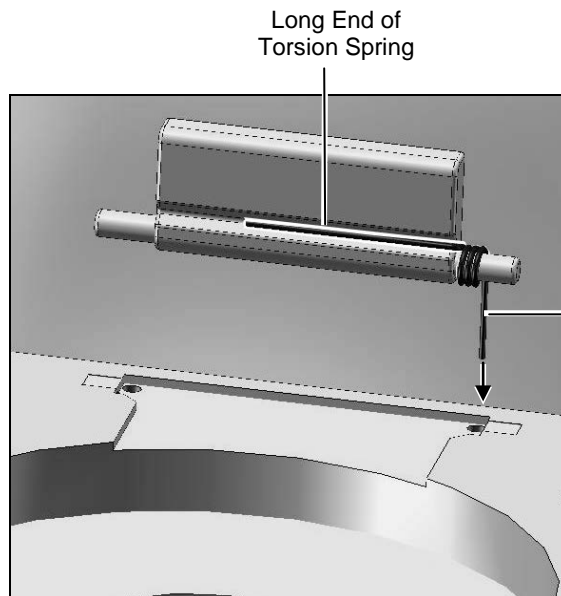

Long End of
Torsion Spring

Short End of
Torsion Spring

16. Place the optical unit back on the front panel. Using the 5mm nutdriver, secure the black optical unit to the front panel with the four nuts (and washers, if applicable).

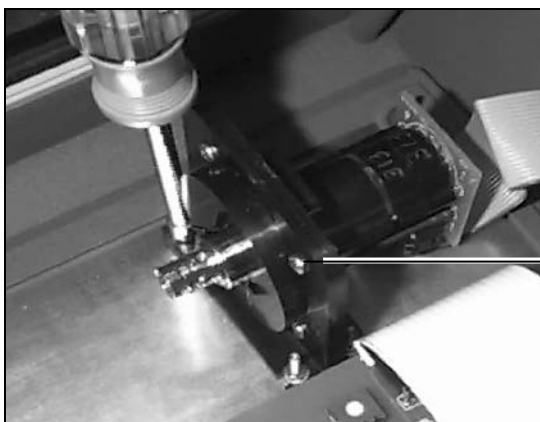

Optical Unit

17. Position the upper cover (containing the black optical unit) in an upright position over the lower part of the instrument, with the keypad to the right.
18. Connect the coaxial cable from the optical unit to the **J1XX** connector on the small circuit board.
19. Connect the flat cable from the optical unit to the **J203** connector on the main circuit board.
20. Connect the flat cable from the LCD to the **Display (J102)** connector on the main circuit board.
21. Connect the flat cable from the keypad to the **Keyboard (J101)** connector on the main circuit board.
22. Connect the cable from the battery pack to the **J302** connector on the main circuit board.
23. Slowly and carefully lower the upper cover onto the lower part of the instrument, ensuring that the metal rear panel slides into the center slot at the rear of the upper cover.

Note: *Ensure that the cables are not snagged or crimped between components when lowering the upper cover onto the lower part of the instrument.*

24. Using the #0 Phillips screwdriver, secure the bottom of the instrument to the upper cover with the four screws.
25. Connect the AC adapter and turn on the instrument. The self-test should run and the **"Ready – Insert Cuvette"** screen should be displayed.

Note: *If the self-test fails or an error message is displayed, the most likely cause is improper reassembly or incorrect connections. Should this occur, disconnect the AC adapter and confirm that all connections have been made properly. Should the problem persist, contact Technical Support.*

26. Reset the time and date (see page 16).

Replacing the Battery

Materials Needed:


1. # 0 Phillips screwdriver
2. Diagonal cutting pliers
3. Replacement battery with cable (E4-BATP-2)
4. Securing cording (packaged with replacement battery)

1. **Steps 1 through 11 of the *Cleaning the Optical Detector* procedure on page 59 Must be performed before moving to step 2 of “Replacing the Battery”, below.**


Note: *The battery is housed in the lower cover.*

2. Using the diagonal cutting pliers, cut the plastic ties that secure the battery to the frame.

Note: *Be sure to cut only the white plastic ties that are wrapped around the battery and secure it to the lower cover. Do not cut the blue and white power cables that are connected to the top of the battery.*

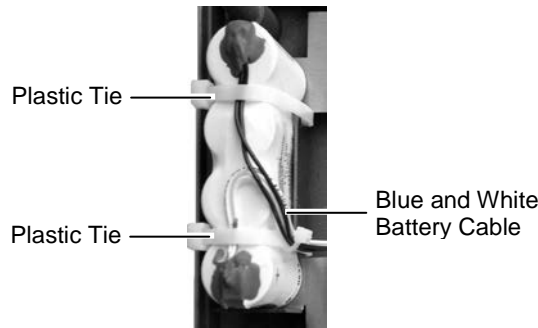

3. Remove the battery, being careful not to snag any components when pulling the cable out from underneath the main circuit board. If the battery well contains tape, remove the tape.
4. Remove the liner from the tape on the bottom of the new battery, and place the new battery in the battery well with the tape side down, ensuring that the blue and white cables are facing up, then route the new plastic ties (packaged with the replacement battery) underneath the battery well.

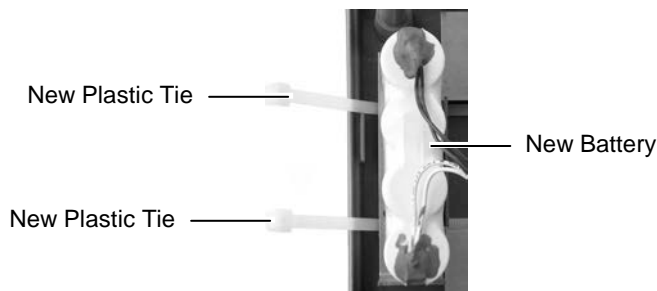

5. Secure the new battery in the battery well, using the new white plastic ties.

Note: *When securing the battery to the battery well, be sure that the plastic ties are routed underneath the battery well. Cut the excess length from the plastic ties using the diagonal cutting pliers.*

6. Thread the blue and white battery cable of the new battery under the main circuit board.
7. Re-connect the cables, replace the upper cover of the AVOXimeter 1000E, connect the AC adapter, and turn on the instrument as outlined in Steps 17 through 25 of the *Cleaning the Optical Detector* procedure on page 59. The self-test should run and the “**Ready – Insert Cuvette**” screen should be displayed.

Note: *Be sure to connect the blue and white battery cable last.*
The new battery is not charged. To ensure adequate charge, leave the instrument connected to the AC Adapter for a minimum of eight hours. Tests can be run while the instrument is charging.

8. Reset the time and date (see page 16).

6 Quality Control Logs

Quality control logs for the yellow and orange filters and liquid control levels 1, 2, and 3 are shown on the following pages.

[illegible]

Orange Filter Values

AVOXimeter 1000E

18.4

18.0

17.6

17.2

16.8

16.4

16.0

THb
(g/dl)

42.6

42.2

41.4

41.0

40.6

398

39.4

39.0

38.2

37.8

0.70	57.4
------	------

36.6

36.2

35.9

35.0

%HbO2

MM/DD/YY

7 Warranty

Certification, Warranty and Service Warranty, and Service

Accriva, declares to the original purchaser that each instrument manufactured and sold by Accriva, or sold by an authorized dealer, shall be free from defects in material and workmanship and, under normal and proper use conditions, warrants it for a period of one year from installation and no more than 13 months from the shipping date, except as otherwise provided in writing.

Accriva's obligation is limited to repairing, replacing, or modifying (at Accriva's undisputed judgment) at Accriva's factory, or elsewhere, the material whose defects have been verified, on condition that the purchaser has informed Accriva of any defects found within 15 days from receipt. Damages caused by or connected to transport are excluded. Transport to and from our facility will be at purchaser's charge and risk, and shall also be prepaid for reshipment, except as otherwise provided in writing. These replacements, repairs, or alterations will in no case determine extension to the above specified warranty period.

The warranty does not cover those parts that deteriorate, or which are in any case considered consumables, or those parts or "items", which by their nature are normally required to be replaced periodically consistent with normal maintenance. It is also understood that, following the purchase and delivery of the instrument, the purchaser shall be deemed liable for any losses, damages, or complaints concerning persons or things incurred by the use, or misuse of the instrument, on behalf of the purchaser, its employees, co-operators, or others. Accriva does not assume any obligation or warranty engagement concerning precision and/or accuracy of the measurements, as well as for any damage to the instrument, directly or indirectly resulting from the use of reagents and/or consumables different from those produced by Accriva specifically for its own instruments, and for the same properly tested.

Warranty will not apply to those defective instruments or materials showing defects or damage arising from the following causes:

1. Insufficient or negligent care by the purchaser.
2. Insufficient or negligent maintenance by the purchaser in relation to the instructions contained in the manuals prepared by Accriva for this purpose; tampering or alterations of the instruments, or in any case interventions or repairs made by any person not duly authorized by Accriva.
3. Misuse due to carelessness, negligence, or inexperience.
4. Employment of materials under heavier conditions than those for which they have been designed and manufactured, and use of the same in combination with incompatible or dangerous products.
5. Non-observance of the regulations relevant to installation, power supply, and operation of the instruments (with particular regard to the regulations for accident prevention).

THIS WARRANTY IS GIVEN EXPRESSLY AND IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. PURCHASER AGREES THAT THERE IS NO WARRANTY OR MERCHANTABILITY AND THAT THERE ARE NO OTHER REMEDIES OR WARRANTIES, EXPRESS OR IMPLIED, WHICH EXTEND BEYOND THE CONTENTS OF THIS AGREEMENT.

No agent or employee of Accriva is authorized to extend any other warranty or to assume for Accriva any liability except as above set forth.

Please contact Technical Support if any malfunction is discovered.

8 Safety Standards

The AVOXimeter 1000E instrument complies with the following safety standard requirements and directives:

CAN/CSA C22.2 No. 61010-1:2012 CAN/CSA C22.2 No. 61010-2-101:2009 CAN/CSA C22.2 No. 61010-2-010:2009	
UL 61010-1:2012	
IEC/EN 61010-1:2010 IEC/EN 61010-2-101:2002 IEC/EN 61010-2-010:2003	
EN 55011:2009 A1:2010 EN 61326-1:2006	Group 1 Class A Class A limit

This instrument has been designed and tested to CISPR 11 Class A. In a domestic environment it may cause radio interference, in which case, you may need to take measures to mitigate the interference.

It is Accriva's responsibility to provide electromagnetic compatibility information to the customer or users. It is the user's responsibility to insure that a compatible electromagnetic environment for the instrument can be maintained in the order that the device will perform as intended.

EN 61326-1:2006

Table 2 limits



CAUTION: Do not use this instrument in close proximity to sources of strong electromagnetic radiation (e.g. unshielded intentional RF sources), as these may interfere with proper operation.

Protection against ingress of liquids: Ordinary (no protection as defined by IEC 60529)

Product cleaning and disinfection: Only according to recommendations of the manufacturer's accompanying documentation

Mode of operation of equipment: Continuous

Degree of safety of application in the presence of flammable anesthetic mixture with air, oxygen or nitrous oxide: Not Suitable.

Note:

As defined in the above standards, the classification of "Not Suitable" DOES NOT MEAN that the instrument is not suitable for use in an Operating Room (OR) environment. Rather, it is intended to indicate that the instrument is not suitable for use in the direct presence of a flammable anesthetic mixture with air, oxygen or nitrous oxide.

All relevant documentation is kept on file at Accriva Diagnostics, Inc., San Diego, CA.

Guidance and Manufacturer's Declaration – Electromagnetic Emissions

The AVOXimeter 1000E is intended for use in the electromagnetic environment specified below. The customer or the user of the AVOXimeter 1000E should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF Emissions CISPR 11	Group 1	The AVOXimeter 1000E uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	The AVOXimeter 1000E is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Complies	

Guidance and Manufacturer's Declaration – Electromagnetic Immunity

The AVOXimeter 1000E is intended for use in the electromagnetic environment specified below. The customer or the user of the AVOXimeter System should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient / Burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical domestic, commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical domestic, commercial or hospital environment.
Voltage dips, short interruptions, and voltage variations on power supply input lines IEC 61000-4-11	$<5\% U_T$ ($>95\%$ dip in U_T for 0.5 cycle) $40\% U_T$ (60% dip in U_T for 5 cycles) $70\% U_T$ (30% dip in U_T for 25 cycles) $<5\% U_T$ ($>95\%$ dip in U_T for 5 sec)	$<5\% U_T$ ($>95\%$ dip in U_T for 0.5 cycle) $40\% U_T$ (60% dip in U_T for 5 cycles) $70\% U_T$ (30% dip in U_T for 25 cycles) $<5\% U_T$ ($>95\%$ dip in U_T for 5 sec)	Mains power quality should be that of a typical domestic, commercial or hospital environment. If the user of the AVOXimeter 1000E requires continued operations during power mains interruptions, it is recommended that the AVOXimeter 1000E be powered from an uninterruptible power supply or battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical domestic, commercial or hospital environment.

Index

backlighting	15	limitations	5
batteries		liquid controls	35
charging	14	entering lot numbers	36
checking	14	tagging a test with lot number	37
calculations	1, 2, 41	lockout	10
calibration	12, 39	menus	8
changing		optical detector	
date and time	16	cleaning	59
units for total hemoglobin	15	optical quality control	34
connections	10	oximetry sites	
cuvettes	9	specifying	23
handling	29	patient ID	
pathlength	39	entering	27
data management	1, 31	requiring	18, 22
display panel	6	precautions	5
handling		printer	10, 30
cuvettes	9	connection	30
hemodynamic calculations	41	printing	
body surface area	44	aborting	33
flow	46	any sample	32
oxygen uptake rate	43	hemodynamic values	54
pulmonary resistance	51	last sample	32
stroke index	53	QC data	31
stroke volume	52	stored data	31
systemic resistance	50	purging test records	33
Hüfner's number	12, 25	QA user ID	
instrument		changing	21
calibration	12, 39	QC lockout	38
connections	10	quality control logs	65
customization	1	running a test	29
description	6	safety standards	70
front panel	6	sample preparation	1
keypad	7	saturation step-ups	3, 45
limitations	5	setting	
lockout	10	patient ID required	18
maintenance	59	standby delay	17
menus	8	user ID required	18
precautions and warnings	5	shutdown	10, 40
safety standards	70	specifications	11
setup	15	standby	10
shutdown	10, 40	summary and explanation	1
specifications	11	symbols	vi
standby	10	test cuvettes	9
startup	27	tests	
unpacking and inspection	13	running	29
warranty	68	troubleshooting	55
intended use	1	unpacking and inspection	13
keypad	7	user ID	
labels and symbols	vi	adding	19

deleting.....	19	requiring	18
entering	27	warnings	5
printing list	20	warranty.....	68

This page was left intentionally blank.

This page was left intentionally blank.

This page was left intentionally blank.

Accriva
d i a g n o s t i c s



Accriva Diagnostics, Inc.
6260 Sequence Drive
San Diego, CA 92121 USA
Tel: 858.263.2300 • Fax: 858.314.6700
www.accriva.com



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



AP1001EN 0515