

# Neuro Technology, Inc.

Instruction Manual for **DigiStim 3 Plus** Peripheral Nerve Stimulator  
Part Number 8-1053-02

## TABLE OF CONTENTS

1. INTRODUCTION
2. SETUP
3. FAMILIARIZATION WITH CONTROLS
4. INDICATORS
5. OUTPUT JACKS
6. BATTERY REPLACEMENT
7. APPLICATION NOTES
8. SPECIAL CONSIDERATIONS AND CONTRAINDICATIONS
9. SPECIFICATIONS

### 1.0 INTRODUCTION

This manual is provided to aid the Anesthesiologists/CRNA in the operation of a *Peripheral Nerve Stimulator (PNS)*.

1.1 The **DIGISTIM 3 PLUS** Peripheral Nerve Stimulator is a constant voltage stimulator for monitoring neuromuscular block and evaluating muscle relaxant dosage during surgery and recovery. The instrument can also be used as an aid in accurately locating nerves when performing nerve block procedures. A digital display provides an accurate readout of the current delivered to the patient.

### 2.0 SETUP

2.1 Location of the instrument should be within six to twelve feet of the patient (depending on leadwire length) and approximately five feet above the floor to provide easy viewing of the touch switches and indicators. The stimulator may be placed on an instrument shelf or attached to an IV pole or anesthesia machine using an optional mounting bracket.

2.2 The leadwires should be attached to the instrument while the numbered control knob is in the *OFF* position. See Section 7.1 for instructions on the use of gelled electrodes.

### 3.0 FAMILIARIZATION WITH CONTROLS

3.1 Power is applied to the instrument by rotating the *control knob* located on the left side of the case in a clockwise direction until an audible click is heard. When the unit is first turned *ON*, a random number will appear on the digital display. The instrument is now in the standby mode and no pulses will be generated. When stimulus pulses are generated, the digital display will indicate the amplitude of the current in *milliamperes (mA)*. The numbered *control knob* is used to adjust the amplitude of the output current. The numbers on the knob serve as a convenient reference for remembering the approximate settings of the output amplitude.

3.2 The eight membrane touch-switches (buttons) located on the front panel provide a convenient means of controlling the functions of the instrument. They are operated by using a finger to depress the desired function designated under each of the eight buttons. Each time a pulse is generated, the *Pulse LED* will *flash* and an *audible beep* will be heard.

3.3 The *STANDBY* button puts the instrument in standby mode in which no stimulus pulses are generated.

3.4 The *DBS* button will automatically generate a single set of *Double Burst Pulses (DBS 3,3)*. This set of *DBS* pulses will be repeated every ten seconds until the *STANDBY* button is depressed or another function is selected. The *DBS* pulses can be repeated as often as one wishes without waiting for ten seconds by depressing the *STANDBY* button and then the *DBS* button again. This cycle can be repeated as often as needed.

3.5 The *1 SECOND* button when depressed will generate one pulse every second continuously. The 1 second pulses can be stopped by depressing the *STANDBY* button or selecting another function.

3.6 The *2 HZ* button, which is used primarily for nerve location, will cause a pulse to be generated at ½ second intervals continuously. The 2 Hz pulses can be stopped by depressing the *STANDBY* button or selecting another function.

3.7 The *100 HZ TETANUS* button is momentary and will deliver 100 pulses per second as long as the button is depressed.

3.8 The *50 HZ TETANUS* button is momentary and will deliver 50 pulses per second as long as the button is depressed.

3.9 The *TOF* (Train-Of-Four) button when depressed will automatically generate four pulses in a period of two seconds. The train-of-four pulses will be repeated every ten seconds until the *STANDBY* button is depressed or another function is selected. The train-of-four pulses can be repeated as often as one wishes without waiting for ten seconds. Immediately after the train-of-four pulses, depress the *STANDBY* button and then the *TOF* button again. This cycle can be repeated as often as needed.

3.10 The *BATTERY* button when depressed will display the voltage of the battery on the digital display. This enables the user to check the condition of the battery before using the instrument. The instrument will function properly when the battery voltage is in the range of 7.0 to 9.5 volts.

#### 4.0 INDICATORS

4.1 The *Pulse LED* will *flash RED* each time an output stimulus pulse is generated. It is located to the right of the digital display and is labeled as *Pulse*.

4.2 An *audible beep* will be heard each time an output stimulus pulse is generated. This enables the user to verify that the unit is stimulating without observing the *Pulse LED*. The loudness of the *beep* can be increased or decreased by using a small screwdriver to adjust the *volume level control* located on the right side of the instrument.

4.3 The *Battery Voltage* will be indicated on the digital display when the *Battery* button is depressed. See Section 6.0 for battery replacement.

4.4 The digital display is a three digit liquid crystal display (*LCD*) that displays the current delivered to the patient in *milliamperes (mA)*. Knowing the precise current setting is important in applications such as nerve locating or detecting fault conditions. Since the digital display measures the actual current delivered to the patient, a number of *fault conditions* can be detected by observing the *LCD*. One fault condition is a *broken leadwire*. Since current cannot flow through an open circuit, any leadwire which breaks connection with the patient will cause the digital display to indicate all zeroes (i.e. no current flowing to the patient). This is an important feature because a total lack of twitch of the hand due to a broken leadwire could mislead the anesthesiologist to believe that the patient is 100% blocked. A quick glance at the digital display can confirm what is happening and significantly add to the anesthesiologist's confidence that a proper connection is made with the patient. A second fault condition is *shorted leadwires*. A large increase in the current displayed could indicate that the leadwires are shorted together, possibly due to the alligator clips touching or the gell from the gelled pads flowing together. A shorted leadwire causes the current to bypass the patient and even though the current displayed is higher, the patient may not be receiving sufficient stimulus. The end result of shorted leadwires is the same as broken leadwires which is no or greatly reduced twitch response.

4.5 The **DIGISTIM 3 PLUS** has a unique feature called *Automatic Power Shutdown*. If the instrument is not used (i.e. no stimulus pulses generated) in a twenty minute period, it automatically turns itself *OFF*. Prior to turning *OFF*, eight groups of four rapid *beeps* will be heard. This feature can be reset by depressing one of the function switches to generate a stimulus pulse(s) or turning the instrument *OFF* and then back *ON* during the twenty minute period. *Automatic Power Shutdown* saves battery life by preventing the user from inadvertently leaving the unit *ON* after a surgical procedure is completed.

#### 5.0 OUTPUT JACKS

5.1 There are three output jacks located on the rear panel of the instrument case. Their colors are *RED (HI)*, *BLACK (COM)* and *YELLOW (LO)*.

5.2 The common output jack is *BLACK (COM)*. This is the negative output and should always be connected to the leadwire with the *BLACK* plug. It is labeled as *COM*.

5.3 The *RED (HI)* jack is the high positive output. The leadwire with the *RED* plug should be connected to the *RED (HI)* jack for use with gelled electrodes. This output can deliver (with a fresh battery) approximately 70 mA into a 2000 Ohm load. It is labeled as *HI*. *The HI output should never be used with a needle electrode.*

5.4 The *YELLOW (LO)* jack is the low positive output and is used with a needle electrode when performing nerve block procedures. The low output can deliver up to approximately 6 mA into a 2000 Ohm load. An important feature of the low output is that one is able to select precise current settings in 0.1 mA increments. It is labeled as *LO*.

#### 6.0 BATTERY REPLACEMENT

The battery is easily replaced by turning the instrument over and removing the battery cover, unsnapping the old battery and snapping in a fresh one. The battery should always be replaced with a 9 volt *alkaline* battery. Battery voltage can be monitored by depressing the *Battery* button on the front panel and observing the voltage indicated on

the digital display. Although the instrument will operate satisfactorily down to approximately 6.5 volts, the stimulus output current decreases rapidly below this voltage. Therefore, the battery should be replaced at a voltage level no lower than 6.5 volts.

## **7.0 APPLICATION NOTES (monitoring of neuromuscular function)**

### **7.1 LEADWIRE ATTACHMENT WHEN USING GELLED ELECTRODES**

The leadwire with the *BLACK* plugs should be plugged into the *BLACK (COM)* jack located on the rear panel of the instrument and the leadwire with the *RED* plugs should be plugged into the *RED (HI)* jack located adjacent to the *BLACK (COM)* jack. Gelled electrodes should be placed in line with and over the ulnar nerve. The distal (negative) electrode should be placed at the level of the proximal flexor crease of the wrist. The electrode pair should be placed directly over and parallel to the flexor carpi ulnaris tendon.

### **7.2 BASELINE ADJUSTMENT**

Before any muscle relaxant is administered, the instrument should be adjusted to provide *Supramaximal Stimulation (SMS)*. *SMS* is defined as the level at which additional stimulation current does not increase twitch response. This setting can be approximated by adjusting the control knob to the level where any further increase in stimulus current would not increase the level of the twitch response. Note the reading on the digital display and maintain this baseline setting throughout the entire procedure.

### **7.3 TWITCH RESPONSE**

The simplest test provided by the *PNS* is the twitch response where individual stimuli are generated at intervals of one to ten seconds. Shortly after administering the muscle relaxant, the twitch response will start to become depressed. At this point more than 70 percent of the receptors should be blocked. When the twitch is completely eliminated, greater than 90 percent of the receptors are occupied by the relaxant. The twitch can be used as a quantitative monitor by adjusting the muscle relaxant administration to maintain a faint, but perceptible muscular contraction (twitch) in response to the *PNS*. This assures adequate operating conditions while avoiding excessive relaxant administration. In the event the twitch has been abolished by an inadvertent relaxant overdose, if one waits until the twitch reappears before administering subsequent relaxant, the incidence of failure of reversal can be minimized.

### **7.4 TETANIC STIMULI**

When the single twitch response has returned to normal, approximately 20 percent of the receptor pool is free. Fortunately, the diaphragm needs fewer receptors available to respond normally than do peripheral muscles. This can be observed clinically in that spontaneous respiration may be detected before an indirectly stimulated response. However, a patient with 80 percent receptor block may still need to be carefully monitored. Thus, it is important to have a means of assessing when recovery has proceeded to a more adequate level. It was to this end that the tetanic stimuli evolved. This is the administration of 50 or 100 Hz stimuli for a period of approximately 5 seconds duration. The higher frequency puts a greater demand on the neuromuscular synapse because as each successive stimulus arrives at the nerve ending, it depletes the local store of transmitters so that the amount of acetylcholine available for release by each succeeding stimulus falls. When the fraction of free receptors is also decreased, the tetanic response does not maintain its initial intensity; it fades. The higher the rate of tetanic stimulation, the more sensitive the test. Fade can be detected at 100 Hz when as few as 50 percent of the receptors are occupied and at 50 Hz when as few as 70 percent of the receptors are occupied. Unfortunately, tetanic stimuli are painful and are therefore of limited value in detecting subtle neuromuscular blockade in the unanesthetized patient.

### **7.5 TRAIN-OF-FOUR**

In the *Train-Of-Four (TOF)* test, the ulnar nerve is stimulated with four supramaximal stimuli 0.5 second apart and the ratio of the fourth twitch to the first twitch is used to determine the degree of neuromuscular block. The primary advantage of *TOF* is that the first response provides a built-in control for the fourth response. Built-in control is a great convenience in the clinical setting in which factors such as patient movement can change the initial tension of the muscle and hence the amplitude of the twitch response. A good rule of thumb is that the degree of block may be estimated by counting the number of twitches seen following the four stimulus pulses. When only one twitch is present, there is greater than 90 percent block. All four twitches appear when the single twitch is depressed by 75 percent. Recovery from the block occurs when all four twitches in the train are the same height. At this time, about 25 percent of the receptor pool is free. Thus, *TOF* is a slightly more sensitive test than the twitch.

### **7.6 DOUBLE BURST STIMULATION (DBS 3,3)**

This pattern of stimulation consists of two short tetanic bursts (50 Hz for 60 msec) separated by .75 second. It is easier to see fade when using *DBS* than when using *TOF* since *DBS* is more sensitive than *TOF* in the normal detection of residual neuromuscular blockade. During the postoperative period and recovery, evaluation of the response to *DBS* has been shown to be superior to evaluation of the response to *TOF*.

## 7.7 NERVE BLOCK PROCEDURES

The *PNS* can be used as an aid in locating any nerve that has a motor component, but is not a substitute for knowledge of basic anatomy and block technique. The basic technique of locating nerves using a *PNS* is outlined in the following paragraphs. It is important to observe the digital display of the output current since the accuracy of nerve location depends on how precisely the current can be set. The lower the current setting, the closer the needle must be to the nerve to obtain a motor response.

**CAUTION:** Do not use the *RED (HI)* output jack for nerve location. Stimulation of nerves using needles requires very low current. Use of the *RED (HI)* output may result in possible needle burns. The leadwire with the *BLACK* plug should be attached to the *BLACK (COM)* jack on the rear panel of the instrument and the leadwire with the *RED* plug should be attached to the *YELLOW (LO)* jack adjacent to the *BLACK (COM)* jack. Short the leadwires together by attaching the alligator clips to each other. Turn the instrument *ON* and depress the 2 HZ button. Adjust the *amplitude control knob* on the side of the instrument until the digital display reads 3.0 mA. This confirms that the instrument is operating properly and also sets the current control knob to approximately the correct position. Disconnect the alligator clips from each other and attach the negative leadwire (*BLACK* alligator clip) to the block needle (*BD STIMEX<sup>tm</sup>* insulated needle or equivalent). Connect the positive leadwire (*RED* alligator clip) to a gelled electrode placed on the trunk of the body some distance from the block site to minimize direct stimulation of local muscles. Following local anesthetic infiltration of the skin, the needle should be inserted in the normal manner for the particular block being performed. Begin advancing the needle, and if necessary, readjust the current to within the 2 to 3 mA range. Continue advancing the needle until a motor response is obtained. Reduce the current until the motor response is barely perceptible and then advance or withdraw the needle to maximize the response. Continue this technique until a motor response is obtained at 1 mA or less. It should be possible to obtain a motor response at 0.5 mA.

## 8.0 SPECIAL CONSIDERATIONS AND CONTRAINDICATIONS

### 8.1 EXPLOSIVE ATMOSPHERES

This device is a possible explosion hazard if used in the presence of flammable anesthetic gases.

### 8.2 MICROSHOCK HAZARD

This device may be hazardous to patients with pacing catheters. If used on such patients, exercise extreme caution to prevent the nerve stimulator output leadwires from contacting the pacing catheter or catheter leadwires. Patients with an implanted electronic device (for example a cardiac pacemaker) or cardiac abnormalities should not be subjected to stimulation unless a specialist's medical opinion has first been obtained.

### 8.3 NEEDLE ELECTRODES

Because of the small surface area and low current values required when using percutaneous (needle) electrodes and the potential for high current density and possible needle burns, it is recommended that only the *YELLOW (LO)* and *BLACK (COM)* output jacks be used when locating nerves. *The HI output should never be used with a needle electrode.*

### 8.4 USE WITH H.F. SURGICAL EQUIPMENT

*Simultaneous* connection of a patient to h.f. surgical equipment (i.e. electrocautery/electrosurgical units) may result in the electronic gating mechanisms in the nerve stimulator being overridden by the cautery pulses causing stimulus pulses to be generated by the stimulator. Simultaneous connection may also result in burns at the site of the stimulator electrodes and possible damage to the stimulator.

### 8.5 USE WITH SHORTWAVE OR MICROWAVE THERAPY EQUIPMENT

Operation in *close proximity* to a shortwave or microwave therapy unit may produce instability in the stimulator output.

### 8.6 SKIN BURNS

Use of *tetanic* stimulation for prolonged periods of time may result in skin burns. The stimulus current should be gradually increased until supramaximal stimulation is achieved. *Use of current levels higher than required for supramaximal stimulation increases the risk of skin burns.*

### 8.7 TETANIC STIMULATION

*Tetanic* stimulation may be uncomfortable for fully conscious patients. Therefore, it is recommended that twitch or train-of-four be used which are better tolerated in awake patients.

## 9.0 SPECIFICATIONS

### 9.1 BUTTON FUNCTION FREQUENCY

Standby.....No stimulus pulses are generated.

Double Burst (DBS 3,3).....50 Hz for 60 msec separated by .75 second. Repeats every 10 seconds.

1 Sec.....1 pulse per second  
2 HZ.....2 pulses per second  
50 Hz Tetanus.....50 pulses per second  
100 Hz Tetanus.....100 pulses per second  
Train-Of-Four.....4 pulses per 2 seconds. Repeats every 10 seconds.  
Battery.....Displays Battery Voltage

## 9.2 PULSE CHARACTERISTICS

Pulse Width.....200 Microseconds  
Pulse Type.....Square Wave Monophasic

## 9.3 OUTPUT CURRENT

RED (HI) output.....0 - 70 mA  
YELLOW (LO) output.....0 - 6 mA

## 9.4 DISPLAY/LED/AUDIBLE INDICATORS

Digital Display: Displays amount of current in mA delivered to the patient.  
Resolution of the display is 0.1 mA / 0.1 Volt. Also, displays battery voltage when the battery button is depressed.  
Pulse LED: *Flashes RED* each time a pulse is generated.  
Stimulus Pulse: An *audible beep* will be heard each time a stimulus pulse is generated. Auto Power Shutdown *Beeps*: Instrument turns itself *OFF* after 20 minutes of nonuse. Eight groups of 4 rapid *beeps* indicates unit is turning itself *OFF*.  
9.5 **POWER**: One nine volt alkaline battery.  
9.6 **POWER CONSUMPTION**: Approximately 9.6 mA  
9.7 **CASE**: High impact ABS plastic  
9.8 **SIZE**: 2.09"H x 3.60"W x 5.75"D  
9.9 **WEIGHT**: 11 oz. Including battery.

10.0 **STERILITY**: Non-Sterile

11.0 **SERVICE LIFE**: The service life of this device is 3 years from date of purchase.

## Limited Warranty

Neuro Technology, Inc. products are guaranteed against defects in materials and/or workmanship when used in normal service for One Year from date of delivery to the original purchaser. Adjustment and/or repair without charge will be made during the one year warranty period only if the instrument in question has not been abused, tampered with, or subjected to unauthorized repair. Damage due to fire, lightning, negligence, water and other liquids, excessive pressure or misuse are not covered under this warranty. Warranty inquiries and requests for warranty repair should be made directly to:

## Mainline Medical Inc.

3250-J Peachtree Corners Circle  
Norcross, Georgia 30092-4301

Tel: (800) 366-2084 • (770) 409-2800  
Fax: (800) 261-3066 • (770) 409-1414

Website: [mainlinemedical.com](http://mainlinemedical.com)  
Email: [service@mainlinemedical.com](mailto:service@mainlinemedical.com)