



Certificate of  
ETL

MIGUN  
MIGUN

# Migun Thermassage Energy Product HY-7000UM-W Owner's Manual

More than a massage.

*It's beautiful health.*



(828) 684-1800  
WWW.MIGUNWORLD.COM

 **MIGUN**  
*Migun means beautiful health.*


Please read this manual carefully for safe and effective use of your Migun bed.  
Please keep this manual safe and refer to it when necessary.

# Contents

## To Migun Family,

Thank you for purchasing our HY-7000UM Thermassage Energy Product.  
Please read this manual carefully for safe and effective use of your Migun.

Please keep this manual safe and refer to it when necessary.

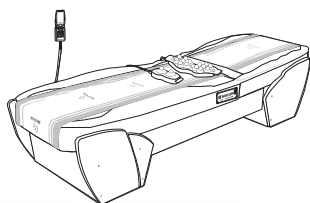

Intended Use .....	3
Contraindications .....	4
Important Safeguards .....	5
Danger .....	6
Important Safety Instructions .....	7
General Warnings .....	8
Caution .....	10
Features of HY-7000UM .....	12
Package Contents .....	13
Designations and Functional Characteristics - Parts .....	14
Designations and Functions - Control Panel .....	16
Remote Control .....	18
Operation .....	20
Operational Modes .....	23
Frame Assembly .....	30
Specifications .....	32
Preparing to Use .....	33
To confirm normal setting .....	35
To confirm normal operation .....	36
Operational Instruction .....	37
Proper Storage and Care .....	50
Error Message/Emergency .....	51
Troubleshooting .....	52
Expandable Items .....	53
Replacing Replacement Items .....	54
Disposing Used Product / Manufacturer's Label .....	55
Warranty Registration .....	56
Memo .....	59

## INTENDED USE OF THE HY-7000UM PRODUCT

**Intended purpose :** Help to muscle relaxation therapy,  
temporary relief of minor and joint pain, stiffness.

## GENERAL RISKS/ FIRST TIME USERS

Whenever a new activity is introduced to the body there is a period of adjustment. This holds true with the HY-7000UM TherMassage Energy Product. First time users will go through a period of adjustment during the initial few weeks. of usage Every person will react differently to the TherMassage Energy Product because of varying skin types, medical conditions, general health and other factors.

The TherMassage Energy Product is capable of heating up to 60°C Celsius or 140°F Fahrenheit. Always wear clothing when using this product. Never expose bare skin to the heating devices of the TherMassage Energy Product. The manufacturer recommends that first time users start at low temperatures and utilize the automatic mode of the device. You can slowly build up tolerance to the heat, and gradually increase temperatures over a several week period.

Individuals with reactive skin disorders such as heat rash, photo allergenic dermatitis, prickly heat, or excessive allergy should consult their physician before commencing sessions, as the heating elements may exaggerate trigger these conditions. Use of a natural skin lotion after a massage may help alleviate skin irritations. Moreover, the roller systems in the TherMassage Energy Product may seem uncomfortable for first-time users. Again, this will be rectified over time as the body gets used to the device. Some people have found that elevating the spine slightly (using shoulder and stomach muscles) will aid comfort during the initial uses of the product.

**IF YOU EXPERIENCE ANY OF THE ABOVE SYMPTOMS WHILE USING THE THERMASSAGE ENERGY PRODUCT, IT IS RECOMMENDED THAT YOU IMMEDIATELY DISCONTINUE USE AND CONSULT YOUR PHYSICIAN.**

## CONTRAINDICATIONS

### DO NOT USE THE HY-7000UM IF ANY OF THE FOLLOWING CONDITIONS EXIST:

1. Phlebitis
2. Individuals with fused discs or implanted scoliosis rods
3. Individuals with diagnosed Osteoporosis
4. Fractures or suspected fractures
5. Individuals with any type of metallic implants
6. Pacemaker or ICD "Internal Cardiac Device"

Individuals with reactive skin disorders such as heat rash, photo allergic dermatitis, prickly heat or allergies should use this product with caution, and consult their physician, prior to use.

**WOMEN: IF YOU ARE PREGNANT, DO NOT USE THIS PRODUCT.**

This is household appliance



## IMPORTANT SAFEGUARDS

Please read this section carefully before using the product. Follow these safeguards as they are for your own safety.

### SAFETY SYMBOLS

The symbols and boxes shown below explain what each heading means. Read and follow all of the messages found throughout the manual.



#### Warning

Be alert to the possibility of serious injury or death if the instructions are not followed. Be sure to read and carefully follow all of the messages.



#### Caution

They indicate a potentially hazardous situation which, if not avoided, may result in minor injury, damage to the equipment, or property damage.



#### Warning


This product should only be serviced by a qualified technician.

Do not disassemble, repair or modify the product.

Procedures other than those described in this manual **must** be performed only by a qualified technician



Call for  
After service

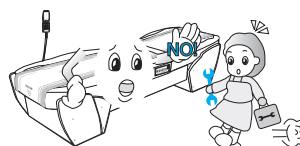

SAVE THESE INSTRUCTIONS

The contents of this manual may change without notice

## DANGER

### To reduce the risk of electric shock:

1) Always unplug this appliance from the electrical outlet immediately after using and before cleaning.

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other).

This plug will fit in a polarized outlet only one way.

If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet.

Do not change the plug in any way.

## IMPORTANT SAFETY INSTRUCTIONS

Read all instructions before using

**WARNING** To reduce the risk of burns, fire, electric shock, or injury to persons:

- 1) An appliance should never be left unattended when plugged in.  
Unplug from outlet when not in use, and before putting on or taking off parts.
- 2) Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- 3) Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- 4) Use this appliance only for its intended use as described in this manual.  
Do not use attachments not recommended by the manufacturer.
- 5) Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.
- 6) Keep the cord away from heated surfaces.
- 7) Never operate the appliance with the air openings blocked.  
Keep the air openings free of lint, hair, and the like.
- 8) Never drop or insert any object into any opening.
- 9) Do not use outdoors.
- 10) Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- 11) To disconnect, turn all controls to the off position, then remove plug from outlet.

SAVE THESE INSTRUCTIONS

- The warnings and cautions are provided for users in order to operate our products safely. Please follow the instructions to avoid any potential injury.
- The symbols specify a particular condition or operation which may cause possible injury, property damage, or damage to the product.



## GENERAL WARNINGS

### Power Connection and Installation

- The product has an electrical rating of 120V AC 60Hz.
- Make sure to connect the product to a proper outlet; otherwise, electrical shock or a short circuit may occur.
- Do not unplug the product by pulling on the power cord.
- Keep the power cord plug clean.
- Do not use the product if the power cord is frayed or damaged. A damaged cord may cause electrical shock, or a short circuit.
- Do not attempt to replace a damaged power cord yourself. Contact your local Migun dealer for replacement.
- If an extension cord is used, be sure it has the proper electrical rating (120V AC, 15A).
- To avoid electrical shock, unplug the power cord before cleaning the product.  
Do not plug or unplug the product if your hands are wet.
- Use only the accessories provided to ensure that the HY-7000UM works properly.



## GENERAL WARNINGS

### Operation and Location

- If any problems occur during use of the product, stop using it immediately and contact your Migun Dealer.
- The HY-7000UM TherMassage Energy Product has a weight limit of 382 lbs.
- Do not use the HY-7000UM TherMassage Energy Product in an environment of extreme cold or heat ( $< 104^{\circ}\text{F}$  or  $> 140^{\circ}\text{F}$  ).
- Avoid direct sunlight.
- Do not attempt to service or repair the product by yourself.
- Do not use the product if a connection cable is damaged.
- Do not remove any bulbs from the thermomechanical probes and projectors. Contact your Migun Dealer if you need a replacement.
- Do not expose the product to water or wet conditions.
- Do not remove the mattress from the frame during operation. To avoid injury, keep your hands away from internal parts.
- Always wear clothing when using this product. Never expose bare skin to the heating devices of the HY-7000UM TherMassage Energy Product.
- It is important that you do not doze off, or sleep while you are using this product. The heat may result in a burn if not properly monitored.
- Lower the temperature setting if the heat becomes uncomfortable for you.
- Keep children away from the product at all times. Do not let them play on or around the product, as they may become injured.
- Please restrict use of the Thermassage Energy Product to areas where electromagnetic waves are generated, if possible.
- Please wrap the external jade massage head with a towel or a cloth to stay away from burning the skin.
- If the cover is damaged, please call for after-sales service.



## Caution

### Operation and Location

- This equipment is not suitable for use in the presence of FLAMMABLE ANAESTHETIC MIXTURE WITH AIR, OR WITH OXYGEN OR NOTROUS OXIDE; Not category AP or Category APG EQUIPMENT.
- Keep sharp objects away from the product.  
Sharp objects may cut the upholstery and cause damage.
- Do not place anything on the product, when it is not being used.
- Do not eat or drink while using the product.  
Keep liquids away from the product to avoid spillage,  
which may cause damage to the product.
- Do not use the product near a heating device or heat vent.





## Caution

### Before using this product

#### DO NOT USE THE HY-7000UM IF ANY OF THE FOLLOWING CONDITIONS EXIST:

1. Phlebitis
  2. Individuals with fused discs or implanted scoliosis rods.
  3. Fractures or suspected fractures.
  4. Individuals with any type of metallic implants.
  5. Pacemaker or ICD "Internal Cardiac Device".
- 
- Follow the recommendations of your physician for use of this product.
  - Consult with your physician before use if you have:
    - reactive skin disorders such as prickly heat
    - photo allergic dermatitis
    - prickly heat or excessive allergies
    - malignant tumors
    - high fever
    - visual problems
    - artificial organs or skeletons

#### IF YOU ARE PREGNANT, DO NOT USE THIS PRODUCT.

- Migun recommends that you use this product no more than 4 times a day.
- If you use this product more than once per day, allow 4 hours between each use.
- Do not use the external probes near or around your eyes or in the facial area.
- Do not use the external probes for more than 2 minutes on one specific area of the body.
- Be sure to properly position yourself on the bed during use, or some discomfort may occur.  
Your head should be approximately 3" to 3 1/2" from the top edge of the mattress, when using the product.




## Features of HY-7000UM

### Features of HY-7000UM


- The treatment area of the main body has been designed to cover from the base of the skull to below the tail bone.
- The treatment area of the lower body has been designed to cover from the thigh to the ankle, thereby treating the entire body.
- Users can choose from eight operational modes.  
(Refer to operational modes on pages 18 through 27.)
- The external projectors can be applied on desired areas.
- Depending on the selected operational mode, moving speed of the internal thermomechanical probes change between low, moderate, and high settings.
- A simple click of the remote control makes the operation easy and quick.
- The specialized functions can be operated with a handheld remote control, with the operational status confirmed through an LCD window.



## Package Contents

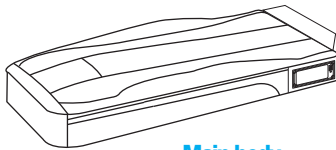


**Main body**

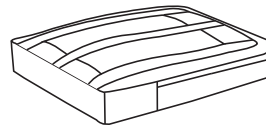

**Lower Body**



**2-Way Jade  
Thermomechanical Projector**

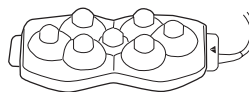


**7-Way Jade  
Thermomechanical Projector**  
(Sold Separately)



**15-Way Jade  
Thermomechanical Projector**



**Remote  
Control**

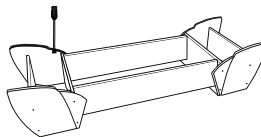

**Supporting Frame for  
Main and lower bodies**




**Power Cord**




**Connecting Cable  
for Main and  
Lower Bodies**



**Cloth Cover**

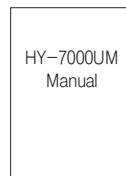

**Owner's Manual**

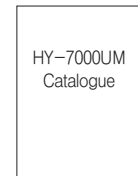

**Product  
Catalogue**



## Designations and Functional Characteristics - Parts

### 1 Remote Control



It controls temperature settings and movements of all thermomechanical probes and projectors. It can be easily stored in the supporting frame.

### 2 Cloth Cover

It protects the internal thermomechanical probes and can be detached for cleaning.


3

### Supporting Frame for Main and lower bodies


### 4 External Jade Thermomechanical Projectors



They provide a thermal treatment to any desired area of the body.

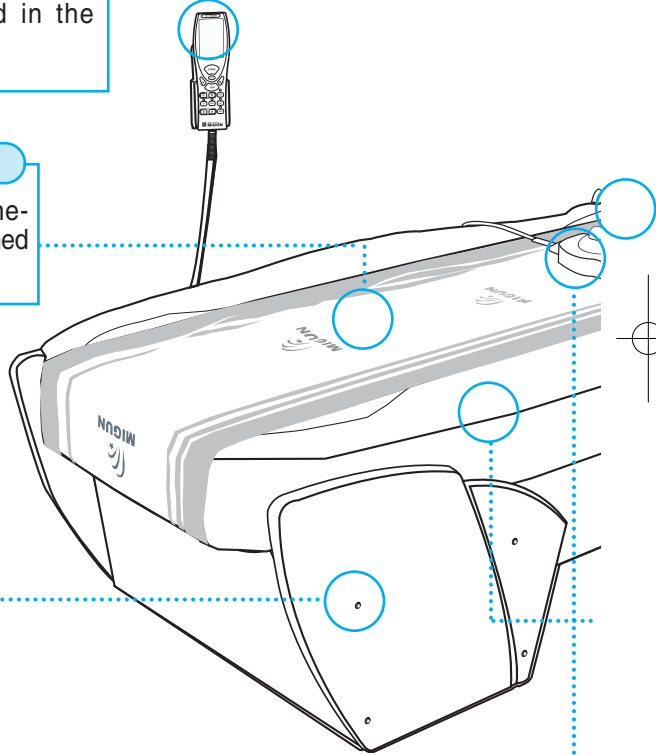

## 5 Connection Terminals

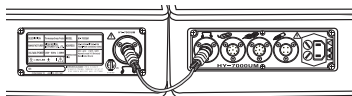

It contains terminals for the lower body, power supply, external thermomechanical projectors, and remote control.

## 6 Lower Body

Two internal thermomechanical probes provide a thermal and massage treatment to your lower body based on the chosen operational mode.

## 7 Main Body

Two internal thermomechanical probes provide a thermal and massage treatment to your upper body according to the operational mode.

## 8 Main Power Panel




Main power switch is located here.




## Designations and Functions - Control Panel

### Main body Label Panel

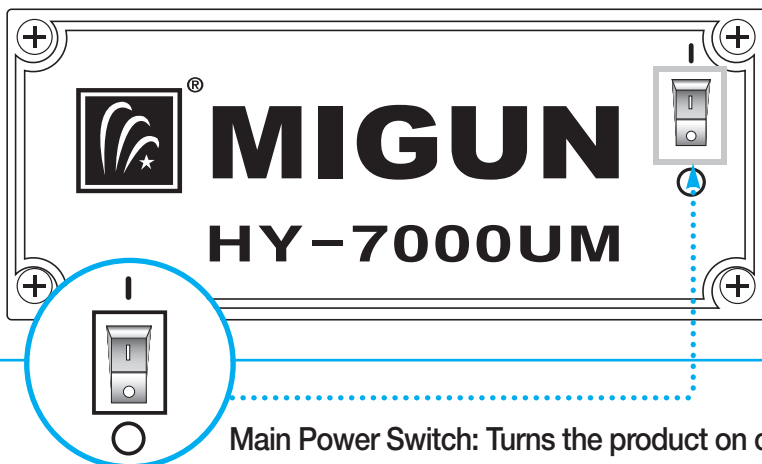


**Main Power Switch:** Turns the product on or off.

### Lower Body Label Panel

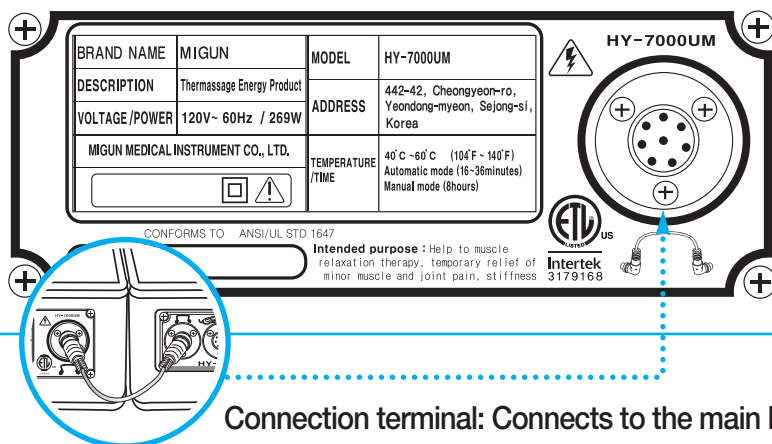

**Connection terminal:** Connects to the main body



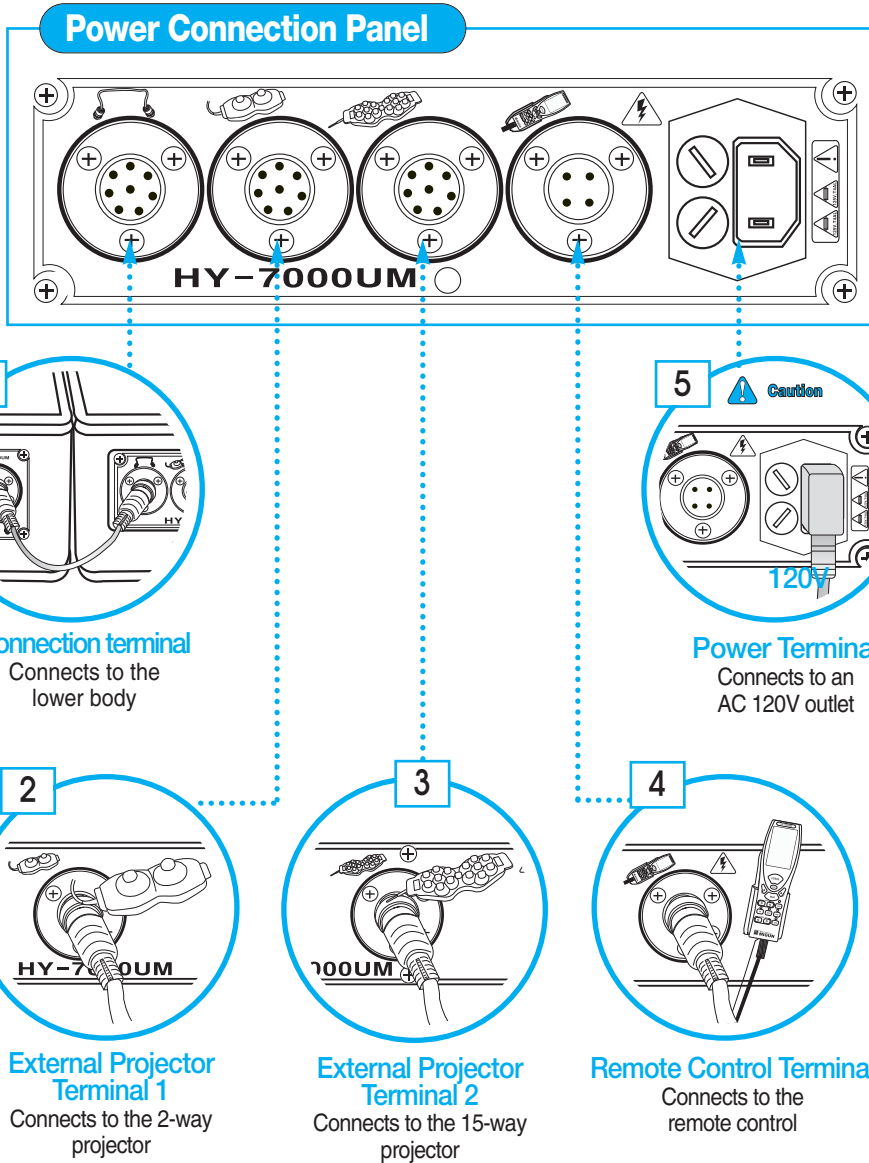



## Remote Control

### LCD window

- ▶ Displays the operating condition and operational mode, setup and current temperature, plus system's Error Code

### POWER button

- ▶ ON/OFF switch

### MODE button

- ▶ For selecting the mode (P1~P6). Upper main probes operate this automatically.

### MOVE button


- ▶ For moving the internal thermo-mechanical probes of the main body in the manual mode.

### PAUSE button


- ▶ Pauses movement of the internal thermo-mechanical probes.

### EXTERNAL Jade button

- ▶ For setting up the temperature of the internal thermo-mechanical probes.

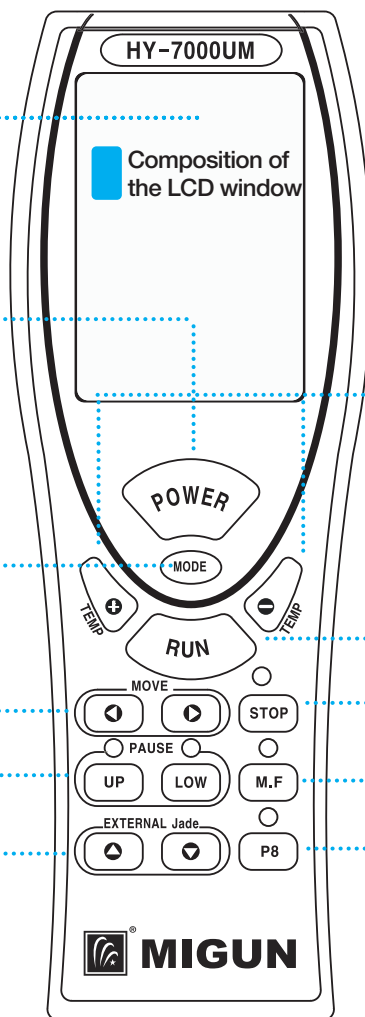

## Composition of the LCD window

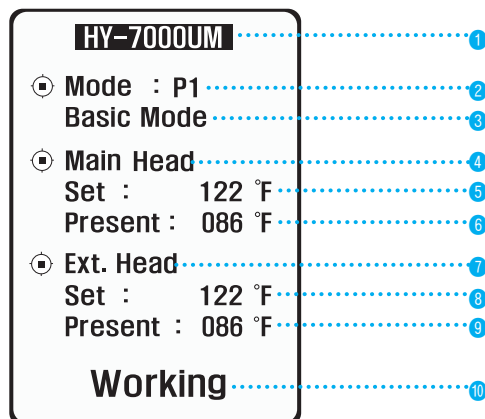

### TEMP control button

► For setting up the temperature of the internal thermo-mechanical probes.

### RUN button

► Starts the system.

### STOP button

► Stops the system.

### M.F (Manual function) button

► For selecting manual operation of the internal thermo-mechanical probes in the main body.

### P8 button

► For selecting immediately the exclusive program (HY-3000) of massage and acupressure therapy.

- ❶ Model name
- ❷ Display the kind of mode
- ❸ Display the kind of selected therapy classified by mode
- ❹ Display on the main Head
- ❺ Displays the set temperature on the upper/lower part of the main Head
- ❻ Displays the present temperature on the upper/lower part of the main Head
- ❼ Displays the external Head (2 way and 15 way)
- ❽ Displays the set temperature of the external Head (2 way and 15 way)
- ❾ Displays the present temperature of the external Head (2 way and 15 way)
- ❿ Indicates operational status and remaining time



## HY-7000UM Operation

The HY-7000UM Thermassage Energy Product provides both heat and massage therapy.

You can choose from 7 automatic operational modes, or you can use the product in the manual mode.

The automatic modes will deliver a combination of therapies to specific areas of the body, for a predetermined amount of time. The manual mode allows you to position the thermomechanical probes manually, and apply therapy to the area selected.

A complete description of the therapies, operational modes, and treatment areas is provided below.

### Heat Therapy

Heat therapy is provided using infrared helium lamps. The lamps are mounted to jade thermomechanical projectors, which are under the pads of the main body and lower body of the table. The lamps have a temperature range of 40°C to 60°C Degrees Celsius or (104°F to 140°F).

They may be adjusted in 5 increments using the remote control.

If a specific temperature is not selected, the system automatically defaults to 50°C Degrees Celsius or (122°F)

### Massage Therapy

There are three different types of massage therapy:

Full Massage Therapy, Sectional Massage Therapy, and Acupressure Therapy.

#### **Full Massage**

During full massage therapy, the thermomechanical probes move slowly from the starting point of the massage location to the end point of the massage location, without stopping.

Throughout the cycle, light massage pressure is applied to the area of the body in one continuous motion, for a specific period of time.

The helium infrared lamps on the thermomechanical probes also apply heat therapy during this type of treatment.

## Massage Therapy

### Sectional Massage Mode

During sectional massage therapy, thermomechanical probes move from the starting point of the massage location to a specific area of the body-and then stop. They then move back and forth slightly for a few seconds before moving to the next stopping point in the cycle. When the next stopping point is reached, they will again move back and forth slightly for a few seconds, then move to the next stopping point.

This will continue, until the entire massage session has been completed. As with the full massage, the infrared lamps on the thermomechanical probes apply heat therapy during the treatment.

### Acupressure Therapy

Acupressure therapy is when the thermomechanical probes are stopped and not moving.

In this position, the weight of your body on the stationary thermomechanical probes result in natural pressure to the area of the body where the thermomechanical probes are located.

This is somewhat similar to a physical therapist using the thumbs or palms of the hands, to apply pressure to a specific part of the body.

Heat therapy is provided by the helium infrared lamps during this type of treatment as well.

## Treatment Areas

Treatment areas are the parts. of the body in which the operational modes apply the treatment.

While the exact location of the treatment varies somewhat depending on your height, the area of treatment is essentially the same for most people.

The product has four different treatment areas in the automatic operational modes. These are, Full Upper Body, Upper Back, Lower Back, and Abdomen.

The lower body has one treatment area which is Lower Body. The chart below describes the location of each treatment area .



## HY-7000UM Operation

### Treatment Areas, Main Body Section of the Table

Description	Treatment Area
Full Upper Body	Base of the Skull to the 2-3" Below the Tail Bone
Upper Back	Base of the Skull to the Middle of the Back
Lower Back	Middle of the Chest to 2 -3" Below the Tail Bone
Abdomen	Middle of the Back to 2-3" Below the Tail Bone
Lower Body	From the Mid-thigh Area to Mid-ankle Area

Also note that the treatment area of the lower body section of the HY-7000UM may be adjusted to cover a larger area of the lower body to accommodate taller users.

This adjustment must be made only by qualified Migun personnel.

Contact your Migun Dealer if you wish to have this larger treatment area.



## Operational Modes

Operational modes provide a combination of heat therapy, massage therapy, and acupressure for a predetermined periods of time.

The operational modes focus on different treatment areas of the body as described below.

### 1. Operational Mode P1-Full Massage

In this mode, the main body section of the table automatically applies a combination of heat therapy, full body massage, sectional massage, and acupressure treatment on the entire upper body.

At the same time, the lower body section provides heat, full massage, and acupressure to the lower body area.

The sequence of the session for the main body is:

1. Sectional massage of the upper body
2. Full body massage of the upper body
3. Acupressure therapy of the upper body
4. Repeat of the Steps 1 through 3
5. Full body massage of the upper body

The sequence of this session for the lower body section is:

1. Full massage of the lower body
2. Heat and acupressure of the lower body

The thermomechanical probes in the main body will stop at 34 different locations during the session, when applying acupressure and sectional massage therapy.

The thermomechanical probes in the lower body will stop at 14 locations.

The lower body massage sequence will repeat itself continuously, until the operation in the main body section stops.



## HY-7000UM Operation

### 2. Operational Mode P2 - Upper Back Mode

In this mode, the main body section of the table automatically applies a combination of heat therapy, full body massage, sectional massage, and acupressure treatment focused on the upper back area.

At the same time, the lower body section provides full massage to the entire lower body. The sequence of the session for the main body is:

1. Full massage of the upper body
2. Full massage of the upper back, only
3. Full massage of the upper body
4. Acupressure therapy of the upper body
5. Repeat of step 4.
6. Full massage of the upper body
7. Sectional massage of the upper body

The lower body will continue to provide full massage until the operation in the main body stops.

The main body thermomechanical probes will stop at 24 various locations during this session, when applying acupressure therapy.

The session lasts approximately 31 minutes.

In mode P2, you may also pause the lower body thermomechanical probes anytime during the session to deliver a sectional massage to that area.

When the pause button is pushed, the lower thermomechanical probes will pause in the area selected, and move back and forth applying sectional massage. Note that this feature is available only in mode P2.

The thermomechanical probes will remain in this mode until the pause button is pressed a second time.

### 3. Operational Mode P3- Lower Back Mode

In this mode, the main body section of the table automatically applies a combination of heat therapy, full body massage, sectional massage, and acupressure treatment focused on the lower back area.

At the same time, the lower body section applies the same treatment therapy as it does in Mode P1.

The sequence of the session for the main body is:

1. Full massage of the upper body
2. Full massage of the lower back, only
3. Full massage of the upper body
4. Acupressure therapy of the upper body
5. Repeat of step 4
6. Full massage of the upper body
7. Sectional massage of the upper body
8. Full massage of upper body

The thermomechanical probes will stop at 24 different locations during this session, when applying acupressure therapy.

The session lasts approximately 30 minutes.

### 4. Operational Mode P4 - Acupressure Therapy

In this mode, the HY-7000UM automatically applies acupressure and full massage treatment to the upper body. At the same time, the lower body section applies the same treatment therapy as it does in Mode P1.

The main body sequence includes three successive acupressure therapy sessions of the upper body, followed by a full massage of the upper body.

The thermomechanical probes in the main body will stop 24 times during the entire session.

The session lasts approximately 24 minutes.



## HY-7000UM Operation

### 5. Operational Mode P5- Heat and Massage Therapy

In this mode, the HY-7000UM automatically applies heat and full body massage to the upper body, but does not apply sectional massage or acupressure.

At the same time, the lower body section applies the same treatment therapy as it does in Mode P1.

The main body session starts with low speed massage, followed by a moderate speed massage and ends with a high speed massage. The session lasts approximately 15 minutes.

The thermomechanical probes do not stop during the entire session.

### 6. Operational Mode P6- Abdomen Massage

In this mode, the HY-7000UM automatically applies heat, full massage, sectional massage, and acupressure to the abdomen area at the same time, while you are laying face down on the table, if the same time, If the lower body returns toward the starting point (on the foot), then the lamp on the probes switches off after reaching the set temperature.

**Caution:** Make sure to position yourself on the bed to prevent the internal thermomechanical probes to be applied to your ribs.

The sequence of this session is:

1. Full massage of the abdomen area only
2. Sectional massage of the abdomen area only
3. Acupressure therapy of the abdomen area only
4. Sectional massage of the abdomen area only
5. Acupressure therapy of the abdomen area only

The thermomechanical probes will stop 12 times at various locations during this session, when applying acupressure therapy.

The session lasts approximately 15 minutes.

## 7. Operational Mode P7-Manual Mode, Main Body of the Table

In this mode you may manually position the thermomechanical probes anywhere in the upper body area, and pause them to deliver heat therapy and acupressure to that area.

Please note that as a safety feature, internal thermomechanical probes should be turned off if it is stopped at an area for more than two minutes.

Once you have positioned them in the desired location, they will remain there for two minutes, before turning themselves off.

The treatment session can be resumed by pressing the temperature button on the remote control.

The lower body section applies the same treatment therapy as it does in Mode P1 and cannot be manually adjusted.

## 8. Operational mode P8, the massage and acupressure of the main body

In this mode, the HY-7000UM automatically gives full massage and acupressure therapy focused on the entire upper body.


At the same time, the lower body section provides the same treatment as given in mode P1. The main body provides a full massage to the upper body at least three times, and continuously provides acupressure therapy over 19 times.

The session lasts approximately 24 minutes.



## HY-7000UM Operation

For easy reference, the different operational modes are graphically illustrated on the back of the remote control and are defined below:

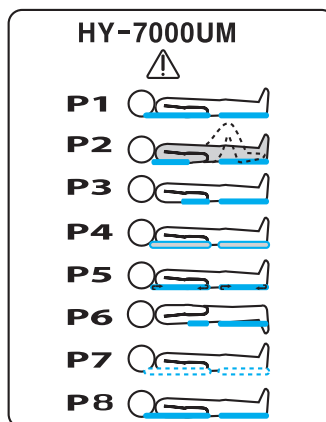







<p><b>Operational Mode 1 (P1)</b> <b>Full Massage</b></p>	<p><b>Full Upper Body Therapy, Main Body</b> Applies heat, sectional massage, full massage, and acupressure to the entire upper body</p> <p><b>Full Lower Body Therapy, Lower Body</b> Applies heat, full massage, and acupressure to the entire lower body</p>
<p><b>Operational Mode 2 (P2)</b> <b>Upper Back</b></p>	<p><b>Upper Back Therapy, Main Body</b> Applies heat, sectional massage, full massage, and acupressure therapy focused on the upper back area.</p> <p><b>Full Lower Body Massage</b> Applies full massage to the entire lower body, or may be paused to apply sectional massage.</p>
<p><b>Operational Mode 3 (P3)</b> <b>Lower Back</b></p>	<p><b>Lower Back Therapy, Main Body</b> Applies heat, sectional massage, full massage, and acupressure therapy focused on the lower back area.</p> <p><b>Full Lower Body Therapy, Lower Body</b> Defaults to Mode P1</p>



<b>Operational Mode 4 (P4)</b> <b>Finger Pressure</b>	<b>Finger Pressure Therapy, Main Body</b> Applies heat, acupressure, and full massage therapy to the entire upper body <b>Full Lower Body Therapy, Lower Body</b> Defaults to Mode P1
<b>Operational Mode 5 (P5)</b> <b>Heat and Massage</b>	<b>Heat and Massage Therapy, Main Body</b> Applies heat and full massage therapy to the entire upper body <b>Full Lower Body Therapy, Lower Body</b> Defaults to Mode P1
<b>Operational Mode 6 (P6)</b> <b>Abdomen</b>	<b>Abdomen Massage Therapy, Main Body</b> Applies heat, sectional massage, full massage, and acupressure to the abdomen area, while lying face down on the table. If the lower body returns toward the starting point (on the foot), then the lamp on the probes switches off after reaching the set temperature.
<b>Operational Mode 7 (P7)</b> <b>Manual</b>	<b>Manual Mode, Main Body</b> In this mode, you may place the main thermomechanical probes in any location and pause them for heat and acupressure therapy to the specific area. <b>Full Lower Body Therapy, Lower Body</b> Defaults to Mode P1
<b>Operational Mode 8 (P8)</b> <b>Massage, Finger Pressure</b>	<b>Full Massage and Acupressure Therapy by the Main Body</b> The main body section provides heat, a full massage and acupressure therapy to the upper body area. <b>Full Lower Body Therapy, Lower Body</b> Applies heat, full massage, and acupressure to the entire lower body

## Assembling diagram for supporting frame

Note - When you insert the bolts, make sure that you do not force the required torque at the beginning but force the temporary weak torque until the final secure tightening.  
- Assemble the sticker part of side panel to face up.

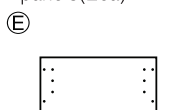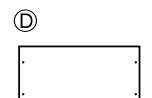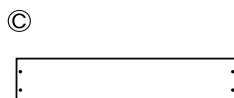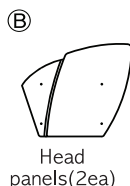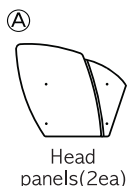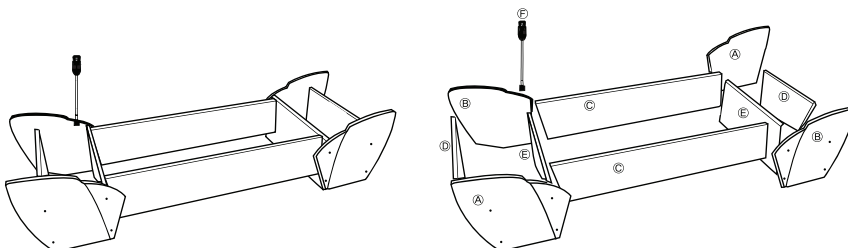

Nuts(24ea)



Bolts(24ea)



Round-head bolt



Hex nut




Flat washer



Spring washer


Tool : +Screwdriver, Monkey wrench (Not included)

### Step 1

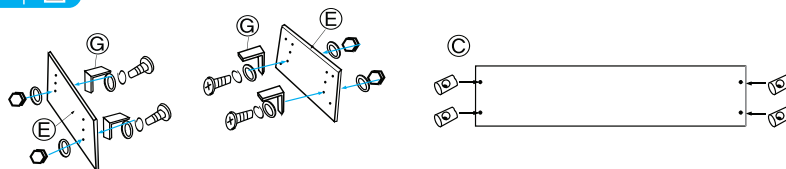

- Assemble the following order into the side panel (E)  
(Round-head bolt → Spring washer → Flat washer → G → E → Flat washer → Nut)
- Insert the nuts into the side panel (C).  
※ Insert the nuts to show the one groove so that the groove should be parallel.


## Step 2

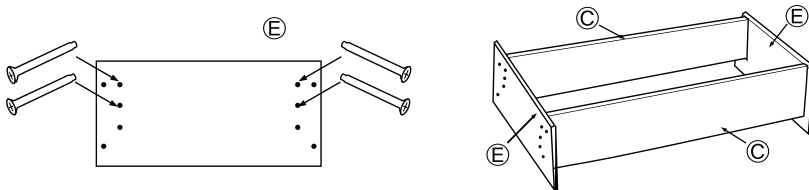

Assemble the side panels (C) to the side panels (E) using bolts.


※ Insert the bolts into the hole and fix it upon fixing the side panels as shown in figure.

## Step 3

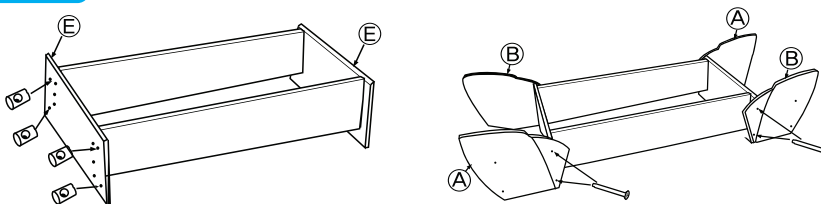


Insert the nuts into the side panels (E) and then assemble head panels (A) and (B) through bolts.

## Step 4

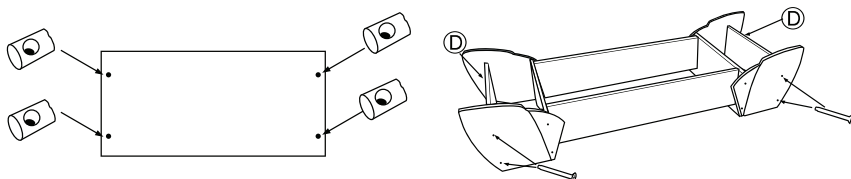

Insert the nuts into the side panels (D) and then assemble head panels (A) and (B) using bolts as shown in the figure.

## Step 5

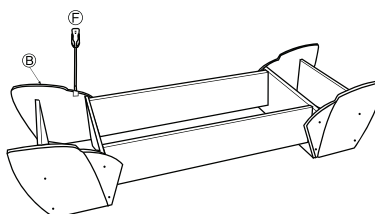

Insert supporting frame for the remote controller into side panel (B) and fix it by turning handle for fixing.



## Specifications

Manufactured by: Migun Medical Instruments Co., LTD. Chungnam, Korea

Model: HY-7000UM 120V -4.0A,

Frequency: 60Hz

Fuse Type: 4A, 125V

Class II Equipment

Use of Equipment: Continuous

Dimensions: 79" x 27" x 17" (LxWxH) or 2007 mm x 686 mm x 432 mm

Weight: Total Weight 165 lbs or 75 kg

Safe Working Load: 382 lbs or 172 kg

Not intended to be used with Flammable Gases

Service : MIGUN MEDICAL INSTRUMENT CO.,LTD.

442-42, CHEONGYEON-RO, YEONDONG-MYEON,

SEJONG-SI, KOREA

Phone: 82-44-864-1551 Fax: 82-44-865-6102

Device should not be used if you have a pacemaker or ICD "internal cardiac device".

Intended Use: relaxing heat and massage for relaxation and general health maintenance.

Other restrictions may apply; consult your physician before initial session.

Method of Cleaning: Use a mild soap and water solution or a vinyl cleaner.

### Environmental Conditions:

Storage & Transport: -25°C ~ 70°C (-77°F ~ 158°F)

Altitude shipping and storage limitations: 500 hPa to 1060 hPa

Relative humidity : Below 80%

Power source: 120V ~ 60Hz ± 10%

Dispose of this product according to local laws and ordinances.

### Warnings and Symbols:

- I. Protection against electrical shock: Class II equipment.
- II. Protection against type of applied part: Type BF applied part.
- III. Mode of Operation: Continuous operation.
- IV. Protection against Water: Ordinary protection (IPXO).
- V. Equipment not suitable for use in presence of FLAMMABLE AESTHETIC MIXTURE WITH AIR OR WITH OXYGEN OR NITROUS OXIDE; Not Category AP or Category AP Equipment.




## PREPARING TO USE


### Notes

1. The HY-7000UM works the best when placed on a flat and level surface.
2. Avoid placing the product in direct sunlight.
3. Use only a proper 120V AC outlet. If an extension cord is used, be sure it has the proper electrical ratings.
4. Room temperature should be between 50 and 77°F (10°C to 25°C), and humidity should be between 35-80%.


- a Once the frame has been assembled, secure the upper and lower bodies to each other with the Velcro straps on the mattresses.

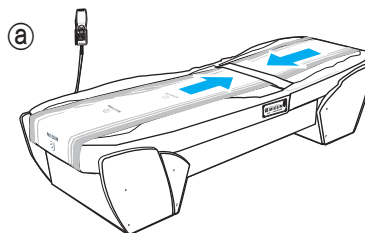

- b Plug the connecting cable for the main and lower bodies to the connection terminal and secure the locking cap by turning it clockwise.

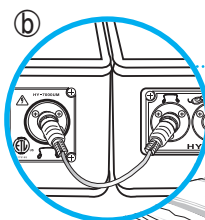

- c Plug the remote control cable to the remote control terminal and secure the locking cap by turning it clockwise.

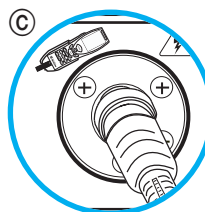




## PREPARING TO USE

- Plug the connecting cables for the external thermomechanical projectors 1 and 2 to the external projectors terminal 1 and 2, and secure the locking caps by turning them clockwise.


\* Note: 7-way thermomechanical projector can be plugged into external projector at terminal 2.

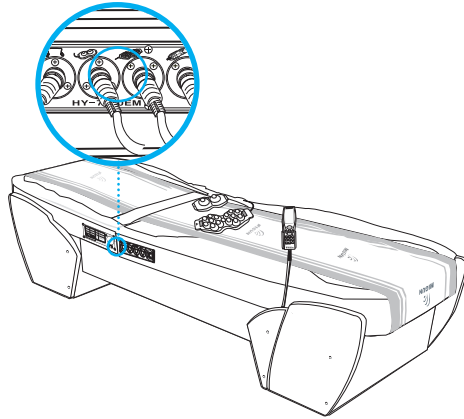

- Connect the power cord to the power terminal on the power connection panel.
- Make sure the supplied voltage is AC 120V/ 60Hz.
- Plug the power plug to the outlet.

### Caution

Do not plug or unplug the power plug when your hands are wet.

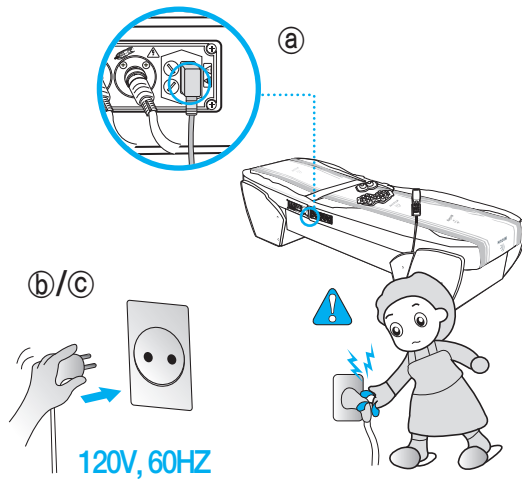




## To confirm normal setting

1


- Turn on the 'main power' switch on the label panel of the main body.

- If the main probes stop at an abnormal position, it stops after moving towards the starting point (on the head).

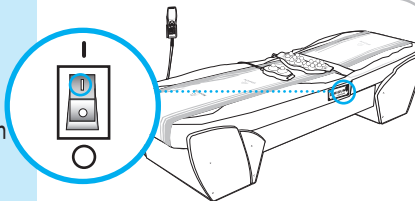

2

- Press POWER button of the remote control.
- The MIGUN mark and logo will appear on the LCD panel.
- Blue LED lights up on the upper part of the LCD window.
- STOP LED lights up in a green color.


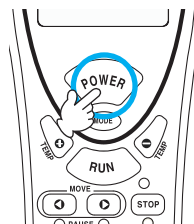




## To confirm normal setting



3

- Press RUN on the remote control.
- Unless another button is pressed on the remote control, when STOP LED is turned off automatically, the product operates in the basic setting mode (P1).
- If the LCD window shows the figure as shown, and also Working and Remaining time alternate, the product is ready for normal installation.

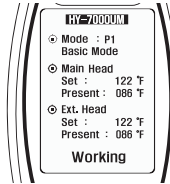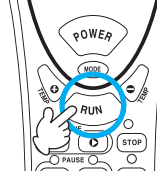

## To confirm normal operation

- If the List of Errors Message is not shown on the LCD window of the remote control, the product has been installed correctly and can be operated normally.

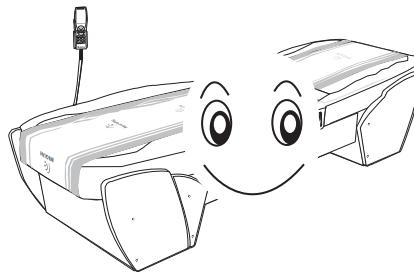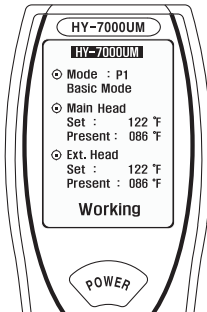





## Operational Instruction



### **WARNINGS AND CAUTIONS WHILE OPERATING THE PRODUCT**

- Do not set the temperature setting too high, but rather to one that is comfortable for you.
- Migun recommends that the first time user start from a low temperature and gradually increase to a higher temperature as your body becomes more adjusted to the heat lamps.
- Lower the temperature setting if the heat becomes uncomfortable for you.
- Do not apply thermal treatment to one area for an excessive period of time. When using the external projectors, do not leave them on one specific area for longer than 2 minutes at a time.
- Do not use the external projectors near or around your eyes, or in the facial area.
- Always wear clothing when using this product. Never expose bare skin to the heating devices of the HY 7000UM TherMassage Energy Product.
- It is important that you do not doze off or sleep while you are using this product. The heat may result in a burn if not properly monitored.
- Keep children away from the product at all times. Do not let them play on or around the product as they may be injured.
- DO NOT USE this product more than four times a day as it may cause undesirable side effects.
- If you use this product more than once a day, allow 4 hours to elapse between each use.
- DO NOT USE the external probes for more than 2 minutes on one specific area of the body.



# Operational Instruction


## Automatic Operation

### In Default Operation

**NOTE:** Main Body Defaults to P1 Full Upper Body.  
Lower Body Defaults to P1 Full Lower Body.


1

- Turn on the main power switch on the label panel of the main body.
- If the main probes are not at the starting point, they will move to the starting point (on the head) before stopping.

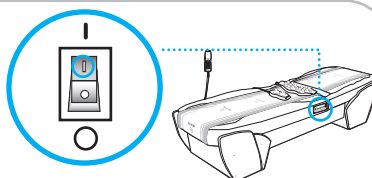

2


- Lay down comfortably on the thermassage energy product. Please allow approximately 3" to 3.5" space from the edge of the bed to the top of your head.

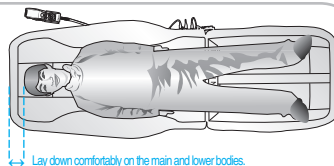

Lay down comfortably on the main and lower bodies.

3

- Press POWER button on the remote control.
- The MIGUN mark and logo will appear on the LCD panel.
- Blue LED lights up on the upper part of the LCD window.
- STOP LED lights up in a green color.

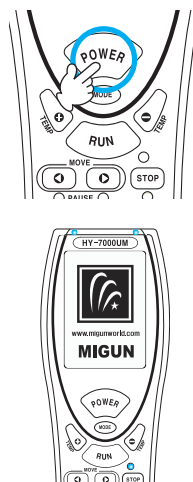

## In Default Operation

### 4

- Press RUN button on the remote control.
- Unless pressing another button on the remote control, while STOP LED is turned off automatically, the product operates in its default setting (P1).

On the LCD window


Displays operational mode (P1)

Displays 122°F as the basic set temperature of the main probes, and the present temperature.

Displays 122°F as the basic set temperature of the external probes, and the present temperature.


Working and Time Left-35 are indicated alternately on the lower part of the LCD window.

- Blue LED in the top of the LCD window turns on and off while the product is operating.

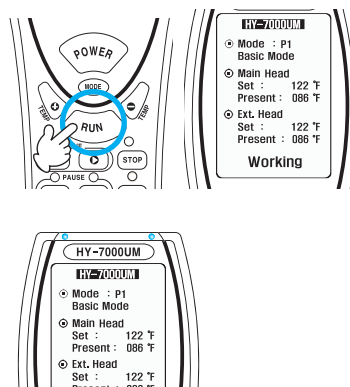

### 5

- To pause the movement of the internal thermo-mechanical probes on the main body, press the PAUSE UP button.
- The PAUSE UP LED will light up in green.
- To resume the movement, press the same button again.
- The internal thermo-mechanical probes will start moving after the PAUSE UP LED turns off.

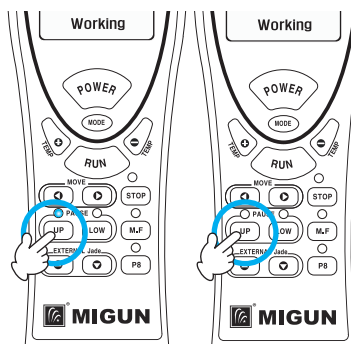

#### Safety Feature:

If the internal jade thermomechanical massage projectors paused for more than 2 minutes, the internal thermomechanical probes will be turned off for safety purposes. To turn on the internal thermomechanical probes again, just press any internal temperature set button.




# Operational Instruction

## In Default Operation


6

- To pause the movement of the internal thermo-mechanical probes in the lower body, press the PAUSE LOW button.
- The PAUSE LOW LED will light up in green.
- To resume the movement, press the same button again.
- The internal thermo-mechanical probes will start moving after the PAUSE LOW LED is pressed

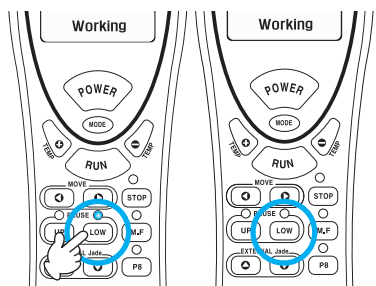

7


- The product displays the MIGUN mark and the STOP LED lights up in a green color after displaying completion on the LCD window, and the alarm will sound, has completed the automatic operation mode automatically.
- If the power is not turned off, the selected operational mode will be saved for the next session.
- If the power is turned off, the operational Mode 1 (P1) will be saved as a default for the next session.
- When the RUN button is pressed, the saved operational mode will resume.
- Press STOP button whenever you want to stop during operation. The main probes stop after the upper part and lower part are returned to the starting point.

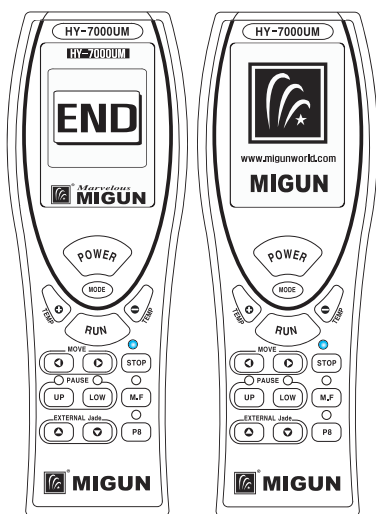

## To Select Operational Mode (Mode1 - Mode6,P8)


### 1

- Press down MODE button on the remote control.
- LCD window displays previously selected mode.
- Keep pressing the MODE button until the mode you want (P1 to P6) appears.

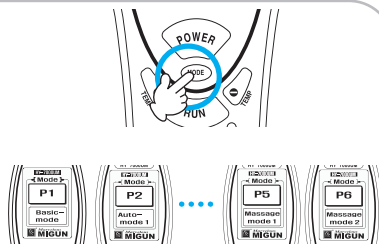


### 2

- Please press the RUN button on the remote control.
- The product will operate at the selected operational mode.
- (If a different operational mode is selected during operation, it will start the selected operational mode from the beginning. The "pause" buttons will not work when the internal thermo-mechanical probes in the main body are relocating to the starting point (head area) in order to start a new operational mode.

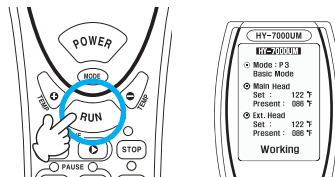

### 3

- Press the P8 button on the remote control if you want to select the P8 operational mode.
- While the P8 LED shows a green light, the product starts operating automatically.
- You may use the mode (P8) without pressing any special button.
- If you want to stop the operational mode P8 during operation, press the same button again. P8 LED will turn off.
- The product is transferred to the previously selected mode and starts to operate.

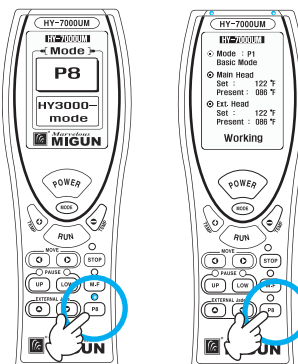


## In Default Operation

!Y To pause the movement of the internal thermo-mechanical probes in the lower body, press the PAUSE LOW button.

!Y The PAUSE LOW LED will light up in green.

!Y To resume the movement, press the same button again.

!Y The internal thermo-mechanical probes will start moving after the PAUSE LOW LED is pressed

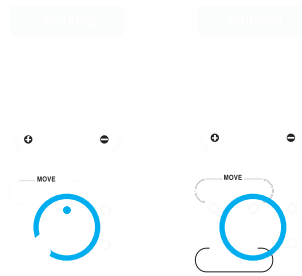


!Y The product displays the MIGUN mark and the STOP LED lights up in a green color after displaying completion on the LCD window, and the alarm will sound, has completed the automatic operation mode automatically

!Y If the power is not turned off, the selected operational mode will be saved for the next session.

!Y If the power is turned off, the operational Mode 1 (P1) will be saved as a default for the next session.

!Y When the RUN button is pressed, the saved operational mode will resume.



!Y Press STOP button whenever you want to stop during operation. The main probes stop after the upper part and lower part are returned to the starting point.

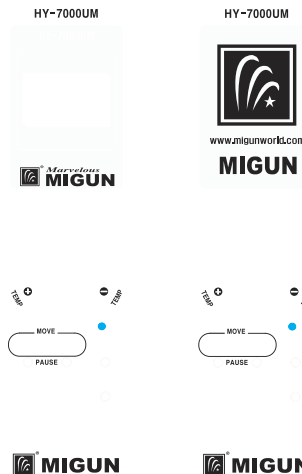

## To use Main Body Manually (P7)



### 3

- If you press the coming down(▶) button of MOVE button, the upper main probes will move toward the waist.
- In manual operation, the lower mechanisms gives heat, massage and acupressure therapy to the lower part of the body.
- If the upper main probes stop for two minutes or more, the upper main probes will turn off automatically as a safety feature. If you want to re-connect the power supply of the main probes press any part of the moving button.
- In manual operation, the temperature of the external thermo-mechanical probes will keep the temperature set during the previous operation.

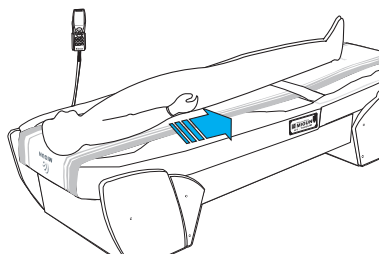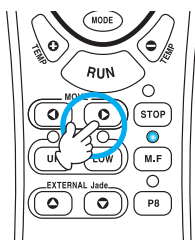

### 4

- Press the M.F button if you want to stop the manual operational.
- The LED will go out and the product will then start from the automatic operational mode selected beforehand.

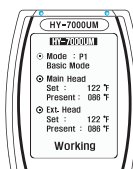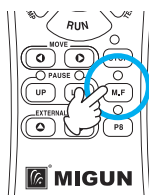




## Operational Instruction

**NOTE:** When using mode P2, you may set the jade thermomechanical probes in the lower body to deliver sectional massage to a desired area, using the Pause button.

### To Set Massage Area for Lower Body for Operational Mode 2(P2)

1

- Press RUN button on the remote control.
- Keep pressing the MOVE button until the operational mode 2 (P2) appears.
- Operational Mode 2 (P2) massages the entire lower body by default.

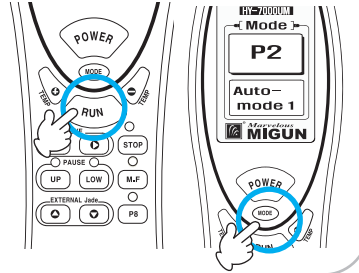

2

- To apply sectional massage, press the PAUSE LOW button when it reaches the desired position. The PAUSE LED will light up in green.
- After applying sectional massage to the lower body pause for a short time, then it will start on the set area.





## To Change Temperature of Internal Jade Thermomechanical Probes



1

- Press RUN button on the remote control.



2

- Select desired temperature by pressing either the up(+) or down(-) button of the internal temperature set buttons.

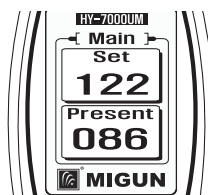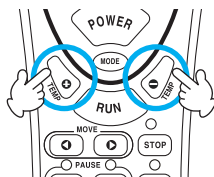




## Operational Instruction

### To Change Temperature of Internal Jade Thermomechanical Probes


3

The range of temperature is between 104°F~140°F (40°C~60°C) with an increment of 9°F (5°C). The window will be changed to the default window if you do not press the button after setting the temperature.

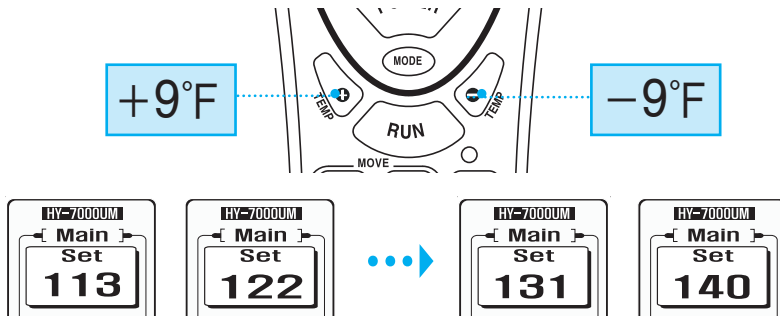

4


- The internal temperature set button simultaneously sets and controls the temperature of the main and lower body.
- Temperature may be changed at any time during the operation without affecting the program.

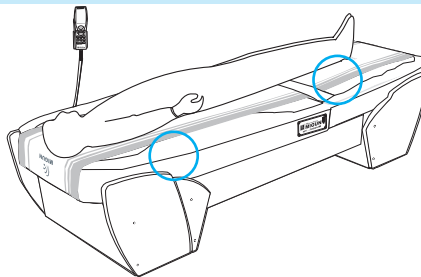

## How To Use External Jade Thermomechanical Projectors


# 1

Turn on the main power switch located on the main body label panel.

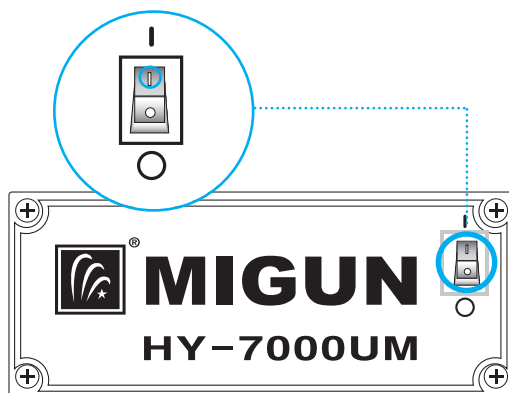

# 2

- Turn on the external thermo-mechanical projectors.
- The power switches are located on the side.

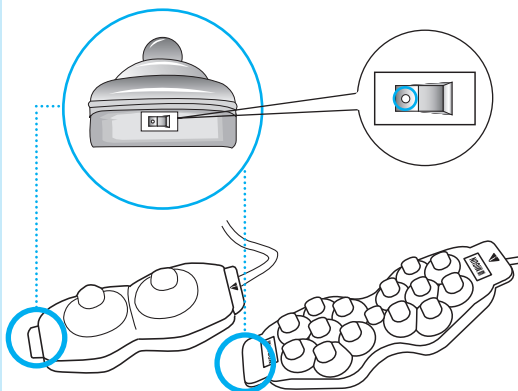




## Operational Instruction

### How To Use External Jade Thermomechanical Projectors


# 3

- Press POWER button on the remote control.
- The external probe will light up.

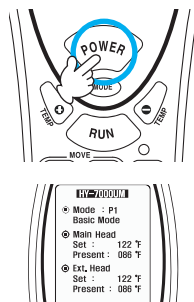

# 4

- Select the desired temperature by pressing either the up(▲) or down(▼) button of the EXTERNAL Jade.
- The range of temperature is between 104°F ~ 140°F (40°C ~ 60°C) with an increment of 9°F (5°C).
- This feature sets and controls simultaneously both the 2 way jade projectors and the 15 way jade projectors on the external thermal device.

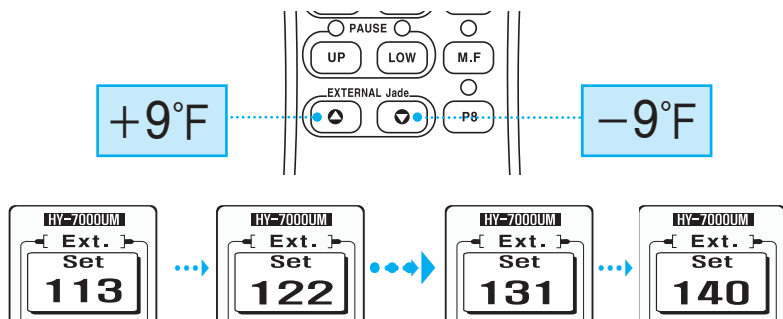


## How To Use External Jade Thermomechanical Projectors

### 5

Apply the external thermo-mechanical projectors on the desired areas of your body.

#### \* Warning

DO NOT leave the external thermo-mechanical projectors on one specific area of the body for longer than 2 minutes at a time.

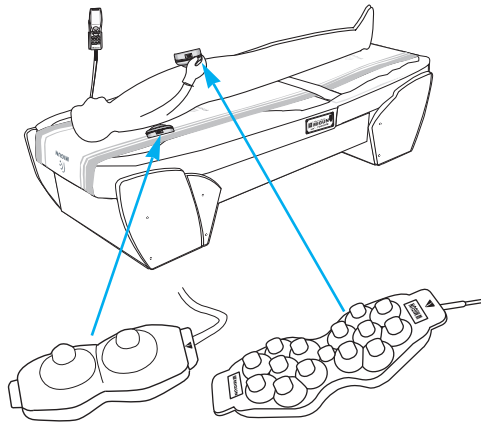


### 6

When a treatment session is finished, turn off the external thermo-mechanical projectors.

#### \* Warning

You can change the temperature at any time during the operation.

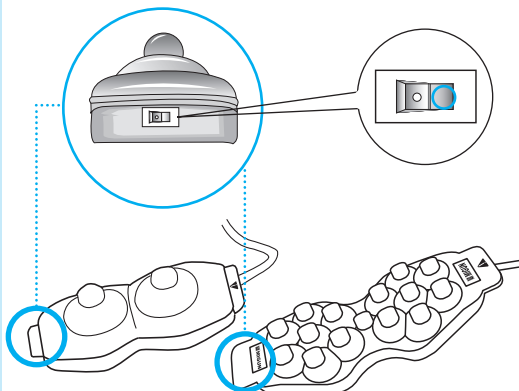



## Proper Storage and Care

### After Using The Product

1. If an external jade thermomechanical projector was used, turn off the power switch located on the side of the projector.
2. Turn off the main power switch, located on the right side of the product from the user's point of view.
3. Unplug the power plug.

### Storage and Care for the Product

1. Store the product in an area that is well-ventilated and is between -25°C~70°C (-77°F~158°F)
  2. Do not store or use the product in the environment with high humidity and temperature.
  3. Avoid placing the product in direct sunlight.
  4. Always place accessories that are not in use in their appropriate storage areas.
  5. Keep the product clean.
  6. Always disconnect power when cleaning.
  7. Do not place the product on any raised surface or platform (more than 1m or 3.28 feet). Dropping the product may cause a malfunction.
- ※ Cleaning instructions for extended use:
- 1) Remove dust from the product on a regular basis.
  - 2) Clean the top of the thermomechanical probes occasionally.  
Make sure that the light is always visible when it is in use.

## How to Clean the Product



### Caution

Unplug the power plug before cleaning.

#### • Mattress Cover (Synthetic Leather)

Use a leather cleaner or a dry cloth to clean.

#### • Mattress Cover (Cloth)


Remove dust using a vacuum cleaner or a dust remover.

#### • Frame

Use a dry cloth with a small amount of water or furniture cleaner.

#### • Cloth Cover

Detach for cleaning.  
Do not apply excessive force when cleaning.

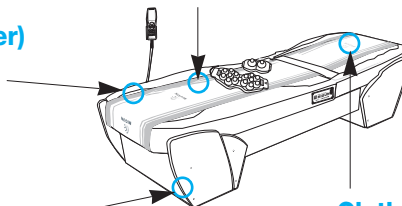




## Error Message/ Emergency

### List of Error Messages

If the following error messages appear on the display monitor, stop using the product and contact the Migun dealer where you purchased the product.

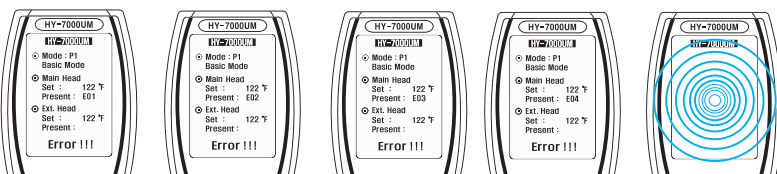

When an alarm continuously beeps

### In case of an Emergency

- Unplug the power cord when there is a lightning or thunderstorm.
- If any health problem occurs, stop using the product immediately and consult a doctor.
- When a power failure or malfunctioning occurs, turn the power switch off and stop using the product immediately.
- If water is spilled on the product, turn the power switch off and unplug the power plug immediately. Contact the Migun dealer where you purchased the product.
- When you smell smoke or if there's smoke coming out of the product, stop using the product immediately and contact the Migun dealer where you purchased the product.



## Troubleshooting

Please check the following lists of problems before contacting Migun dealers.

Symptom	Check list	Do the following
LED on the remote control doesn't turn on.	Is the remote control cable disconnected?	Plug the cable in the remote control terminal properly.
	Is the power plug in the outlet?	Plug the power cord into a correct 120V AC outlet.
	Is the main power switch turned off?	Turn the main power switch on.
The internal thermomechanical probes do not move.	Is the power outlet 120V AC?	Plug the power cord into a correct 120V AC outlet.
	Is the "UP" or "LOW" LED below above the pause button on?	The movement is paused. Press a proper pause button once to resume the movement.
The external thermomechanical projectors do not turn on.	Is the manual operation LED on?	The product is on the manual mode. Press a move (⏮, ⏭) button to move the internal thermomechanical probes.
	Are the connecting cables for the external thermomechanical projectors disconnected?	Plug the cables into the proper external unit terminals.
	Are the power switches of the external thermomechanical projectors off?	Turn the power switches on.
The temperature setting of the external thermomechanical projectors cannot be set.	Is the power plug in the outlet?	Plug the power plug into a correct 120V AC outlet.
		Turn the main power switch on.

※ It is not malfunctioning even though the internal thermomechanical probes in the lower body move when the operation is stopped.

They are moving to the starting point for the next operation.

※ If you still have a problem even after checking all of the above, contact Migun dealers.





## Expendable Items

### Replacement Items


Item \ period		1 Month	2 Mos.	3 Mos.	4 Mos.	1 Year
Mesh Netting	Consumers				Replace	
	Demonstration Centers			Replace		
Cloth Covers	Consumers	Wash				Replace
	Demonstration Centers	Wash			Replace	

- ※ For sanitary purposes, cloth covers should be washed more frequently in summer.
- ※ The replacement items used in demonstration centers would wear out faster because of their frequent uses.
- ※ When replacement items are not properly changed or managed according to the schedule, a noise may occur during an operation. Even though it is not a malfunction, if the items are not properly changed, a severe damage may occur to the product.
- ※ When mesh netting is placed upside down, operation might be noisy.

### Replacement Items List



• Cloth Covers

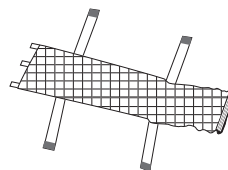

• Mesh Netting




## Replacing Expendable Items

### Replacing Expendable Items


#### Mesh Netting

1. Unplug the power plug.
2. Disconnect the connecting cable for the main and lower bodies.
3. Detach the cloth cover. (Velcro under the product makes it easy to detach.)
4. Open up the mattress cover by unzipping it.
5. Release the rubber band, located on the middle of mesh netting.
6. Unclip the mesh netting clip located at both ends of mesh netting (the main cover and bearing block for the main body) to replace mesh netting. Clip netting clips back in.
7. Put the rubber band back to tighten it up.
8. Put the mattress cover back on and close the zipper. Attach the cloth covers.
9. Connect the main and lower bodies together using the connecting cable.

#### Cloth Covers

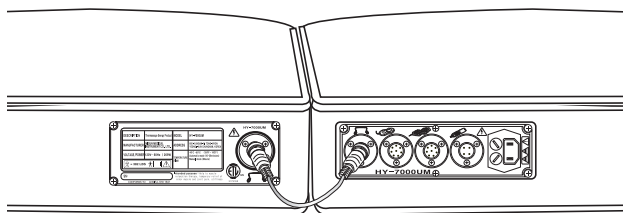

1. Unplug the power plug.
2. Disconnect the cable connecting the main and lower bodies.
3. Detach the cloth cover.

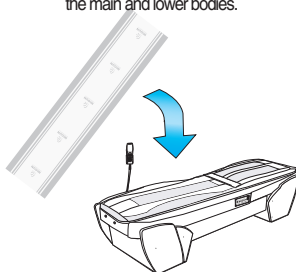

4. After replacing the covers, connect the main and lower bodies together using the connecting cable.



## Disposing Used Product / Manufacturer's Label

### Disposing of Used Product


#### When replacing a product with a new one

Migun will dispose of the old product for free when replaced by a new one.

#### When disposing of a used product

Follow the disposal requirements in your local area for disposal of appliances.

### Manufacturer's Label

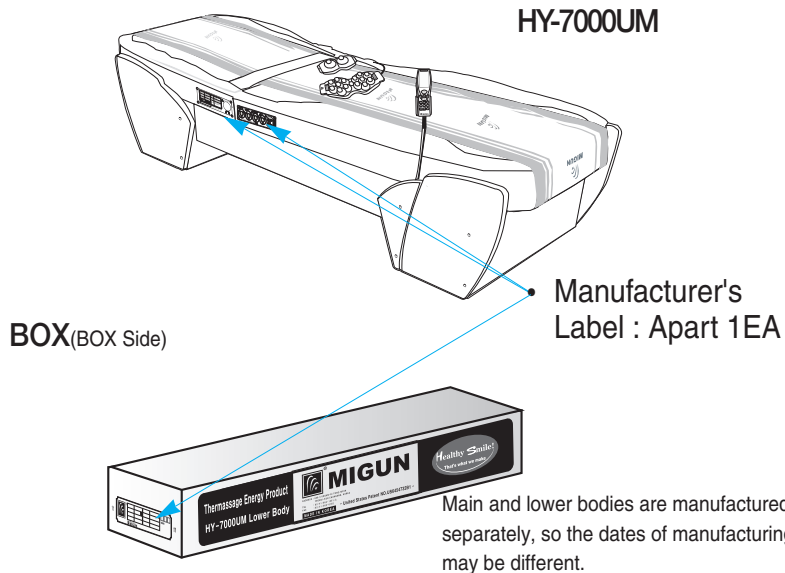

# Warranty Registration

## Written Quality Guarantee of MIGUN Thermassage Energy Product

Product Name	Thermassage Energy Product			Model Name	HY-7000UM
Grace Period	1 year from the date of purchase			Product Number	
Purchase Date	Month	Date	Year	Place of Purchase	
Customer Information	Name				
	Address				

## Service Contract

Service and Coverage : To arrange service for products covered by this Service Contract, please call the store where you purchased the product. Your initial call to MIGUN (Company) will include a fault diagnosis to clarify a problem and is required prior to commencement of repair services.

DO NOT SHIP OR OTHERWISE DELIVER THE PRODUCT TO THE COMPANY UNLESS YOU FIRST CHECK YOUR OWNER'S MANUAL AND TROUBLESHOOTING GUIDE AND UNLESS YOU OBTAIN SERVICE AN AUTHORIZATION NUMBER FROM A CUSTOMER SERVICE REPRESENTATIVE.

- Repairs will be performed at our discretion by a Company service center or by a third-party service contractor.
- The term of this Contract commences on the original product purchase date and expires one year thereafter.

THIS CONTRACT IS MADE INSTEAD OF A MANUFACTURER'S WARRANTY. MIGUN DOES NOT MAKE ANY WRITTEN WARRANTIES AND HEREBY EXEMPTS, TO THE MAXIMUM EXTENT ALLOWABLE BY LAW, ANY IMPLIED WARRANTIES INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND OF FITNESS FOR A PARTICULAR PURPOSE. THE PRODUCT COVERED BY THIS CONTRACT IS SOLD TO BUYER AS-IS, TO THE EXTENT ALLOWABLE BY LAW, AND THE BUYER AGREES HE/SHE HAS NOT RELIED ON ANY PROMOTIONAL MATERIAL, ORAL REPRESENTATIONS OR PROMISES, OR ANY RECOMMENDATIONS MADE BY COMPANY OR ANY OF ITS REPRESENTATIVES IN RELATION TO THE PURCHASE OF THE PRODUCT.

- Products covered by this Contract may be repaired or replaced with a comparable product at this sole discretion of Company.
- In the event that the Product or components of Product must be shipped to the Company for repair or replacement, the Company at its sole discretion may direct Buyer to ship the goods to its headquarters, a Company service center, or third-party service center. All shipping, handling, shipping insurance costs, and risk of loss for shipments to and from Company shall be borne solely by Buyer.
- This Contract covers parts and service charges only for repair and/or replacement of Product or components of Products resulting from manufacturing defects.



## Warranty Registration

### Service Contract

**Dispute Resolution:** Any claims or disputes arising under this Contract shall be resolved in the following manner and sequence: (1) First by telephone consultation with a service advisor handling the Customer Service telephone number listed on this Contract, (2) Secondly by non-binding mediation conducted in the District of Columbia by a third-party mediator selected by the Company, and if non-binding mediation is not successful, (2) Finally by binding arbitration under the rules of the American Arbitration Association. Buyer and Company hereby selects the District of Columbia as the forum state for arbitration of any and all claims or disputes. Buyer hereby waives all right to a jury trial and to settlement of any claims or disputes in state or federal court. Each party is responsible for bearing its own attorney's fees, costs and expenses and for one-half of any fees and costs charged by a third party mediator or arbitrator.

**Purchaser Records:** To obtain services under this Contract, you must have an original, fully-executed Contract by you and the Company and all original receipts.

**Exclusions:** This Contract does not cover repairs or replacements caused by accidental or intentional physical damage, condensation, spilled liquids, misuse, abuse, altered or missing serial numbers, damage caused by non-authorized repair personnel or by Buyer, damage caused by connection of Product in an improper electrical power source or by electrical malfunctions. Also not covered are repair or replacement costs for lost or consumable parts, normal wear and tear, age of Product, cosmetic damage and problems due to improper and/or non-factory authorized installation or repairs. This Contract is not transferable. This Contract also excludes the following circumstances:

- Products that are used for commercial purposes, public rental or communal use in multi-family housing. Use of a product for these purposes will void this Contract.
- Buyer hereby waives any consequential or incidental damages, including but not limited to, loss of use.
- This Contract does not cover any fees related to third party contracts.
- Buyer hereby agrees that this Contract is not represented as an insurance policy or a statement of warranty of any kind.
- This Contract shall be rendered null and void if the covered Product is used or moved outside of the continental United States.
- Buyer is solely responsible for all shipping and handling costs, including insurance, and bears all risk of loss for Product or components of Product shipped to or from Company.
- Certain consumable parts are available for purchase from the Company and are not covered by this Contract. Buyers interested in purchasing consumable replacement parts may request a current product and price list from the Company.
- Receipt of a Product or components of Product from Buyer shall not be construed as the Company's acceptance of such goods for coverage under this Contract. Upon receipt of such goods, Company shall perform an evaluation and provide Buyer a written work order identifying both covered and non-covered parts and services. Buyer is solely responsible for payment of Company's then current price for repair and/or replacement of non-covered parts and problems.

## Warranty Registration

### Service Contract

Consumable Parts Available for Purchase by Buyer:

The following parts are NOT COVERED by this Contract but may be purchased by Buyer, at Company's then current retail price, at Buyer's sole expense. Buyer is also responsible for all expenses related to shipping, handling, and insurance.

- Mesh netting
- Cloth cover
- Leather cover (must be installed by an authorized Company service center)
- Supporting frame for remote control

Parts Covered by this Contract:

The following parts are COVERED by this Contract and will be provided by the Company free of charge, provided that Buyer pays for all expenses related to shipping, handling, and insurance.


- Fuse
- Printed Circuit Board
- Limit Switch
- Remote Control
- Upper and Lower body connector

Covered Product: Only the Product identified herein shall be covered by this Contract.

Any peripheral components or accessories not listed below are excluded from coverage under this Contract:



# M E M O

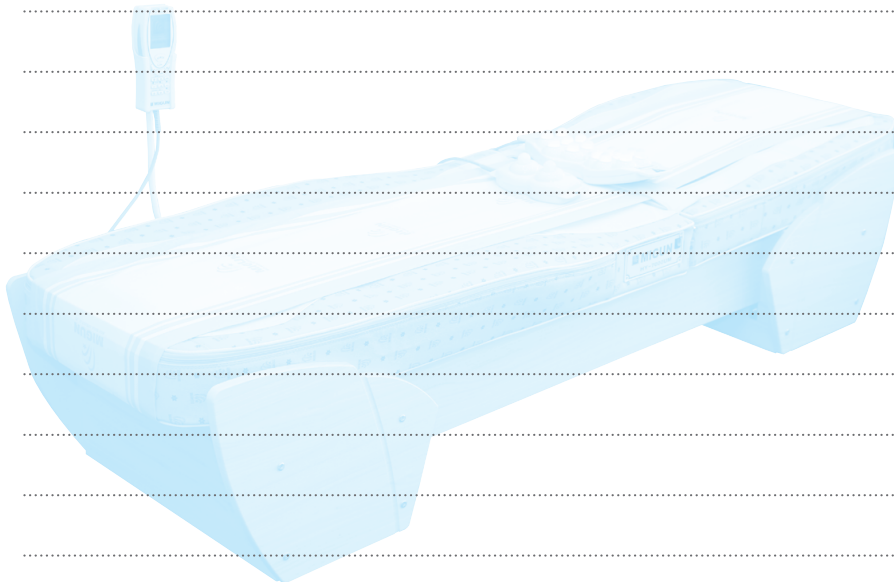



*Healify Smile*

*Get Energized!  
Get Revitalized!  
Get Migunized!*



 **MIGUN**  
*Migun means beautiful health.*

63 Fletcher Commercial Drive  
Fletcher, NC 28732

**(828)- 684 -1800**

**[www.migunworld.com](http://www.migunworld.com)**

**VI17402(Rev.2)**