
# **tobii** dynavox

TD Phone

# Getting Started Guide

Connection Guide and Training Cards



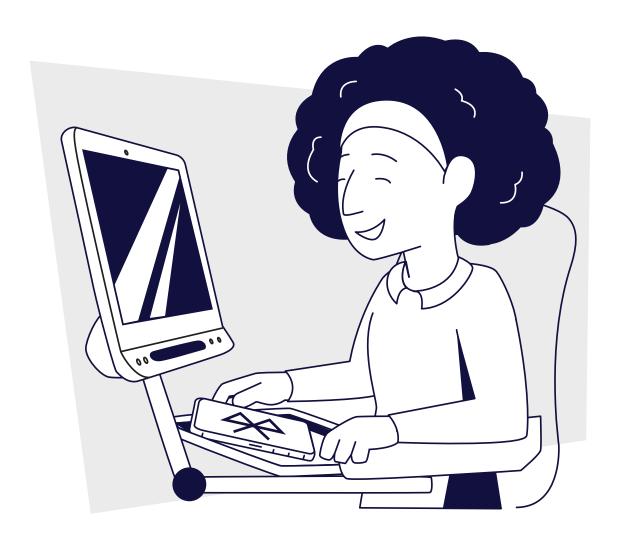

# TD Phone is a way to text and interact on the phone with your eyes.

# Contents

- iPhone Connection Instructions
- **Android Connection Instructions**
- **Tour of TD Phone**
- **Using TD Phone**
- 13 FAQs
- 15 Additional Resources

# Before Connecting

- Identify an assistant to keep the mobile phone unlocked and to make selections, if needed.
- Use correct instructions for your mobile phone.
  - ⊳ iPhone page 1
  - ⊳ Android page 5




## **Essential for Successful Connection**




- Your mobile phone <u>must</u> always remain unlocked throughout the connection process.
- Settings pages pictured in this guide <u>must</u> be visible on your mobile phone.

# iPhone Connection Instructions

Step	Where to Do Step	What to Do
1	TD Phone	Open TD Phone and accept Terms.
2	TD Phone	Select <b>Device Settings</b> .
3	TD Phone	Confirm that your I-Series Bluetooth is ON in this panel then continue. If Bluetooth is off, use instructions in FAQs at the end of this document.   **Bluetooth ON Moke sure your phone's Bluetooth setting is turned on.  **Scan Devices** Scan for devices to add to your devices list.  **Scan
4	Your iPhone	Change Auto-Lock on your iPhone to Never.  Select Settings > Display & Brightness > Auto-Lock > Never.  This keeps the screen unlocked as required by the connection process. We will remind you to return the setting to your preferred time at the end of the connection process.  Auto-Lock Never > Never > Night Shift Off > Never > Night Shift Off > Never > Never > Night Shift Off > Never > Never > Night Shift Off > Never > Never > Never > Never > Night Shift Off > Never
5	Your iPhone	Open the Bluetooth Settings page on your mobile phone.  Settings > Bluetooth

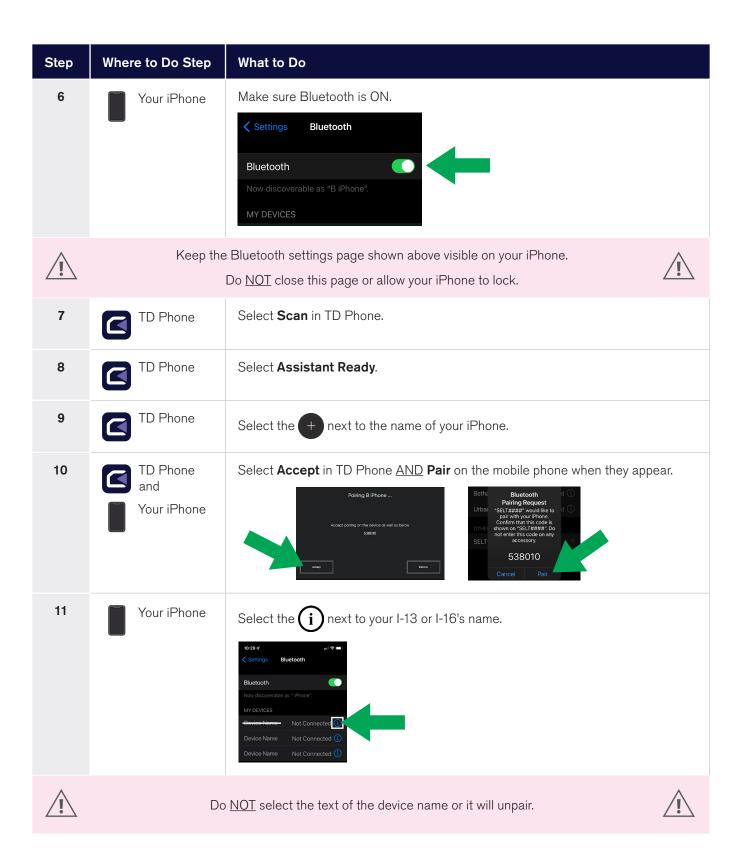


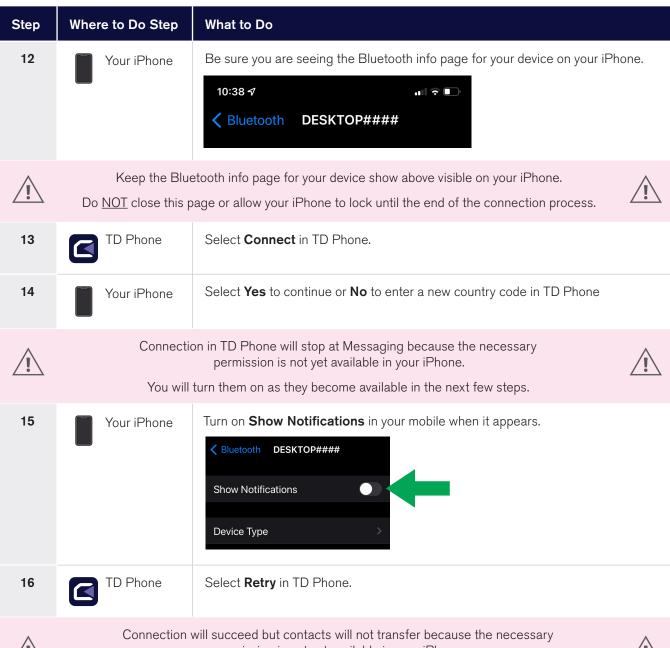

permission is not yet available in your iPhone.



You will turn them on as they become available in the next few steps.

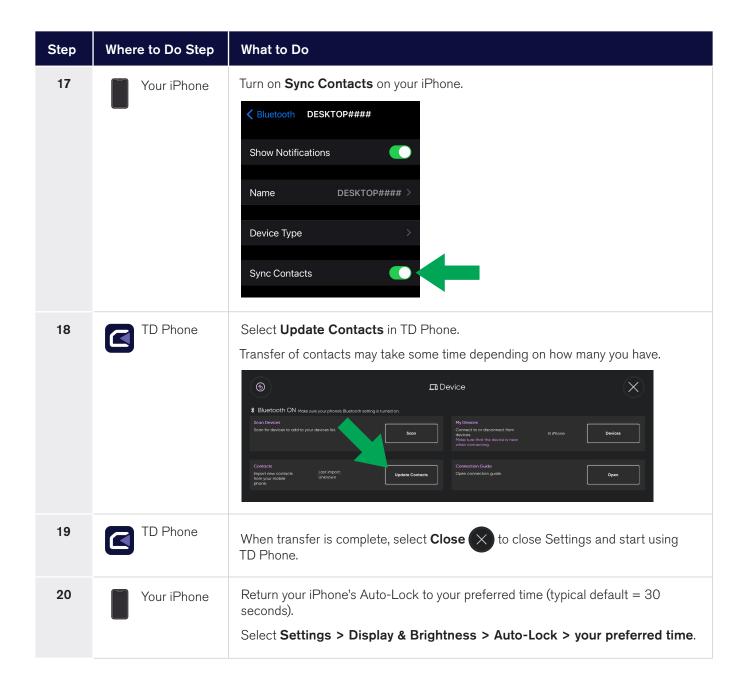

# Before Connecting

- Identify an assistant to keep the mobile phone unlocked and to make selections, if needed.
- Use correct instructions for your mobile phone.
  - ⊳ iPhone page 1
  - ⊳ Android page 5




## **Essential for Successful Connection**




- Your mobile phone <u>must</u> always remain unlocked throughout the connection process.
- Settings pages pictured in this guide <u>must</u> be visible on your mobile phone.


# Android Connection Instructions

Step	Where to Do Step	What to Do
1	TD Phone	Open TD Phone and accept Terms.
2	TD Phone	Select <b>Device Settings</b> .
3	TD Phone	Confirm that your I-Series Bluetooth is ON in this panel then continue. If Bluetooth is off, use instructions in FAQs at the end of this document.   **Bluetooth ON Make sure your phones Bluetooth setting is turned on.  Scan Devices Scan for devices to add to your devices list.
4	Your Andriod Phone	Change Auto-Lock on your mobile phone to the longest possible time.  Select Settings > Security > Automatically Lock > longest possible time.  Automatically lock 5 minutes after sleep  Power button instantly locks  This keeps the screen unlocked as required by the connection process. We will remind you to return the setting to your preferred time at the end of the connection process.
5	Your Andriod Phone	Open the Bluetooth Settings page on your mobile phone.  Open Settings > press and hold Bluetooth

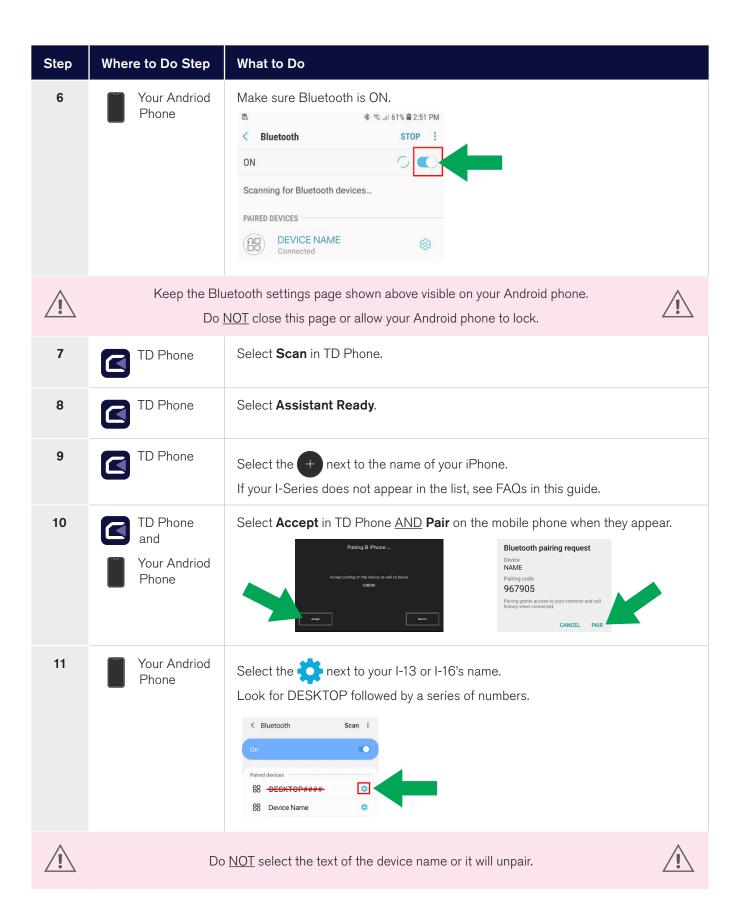


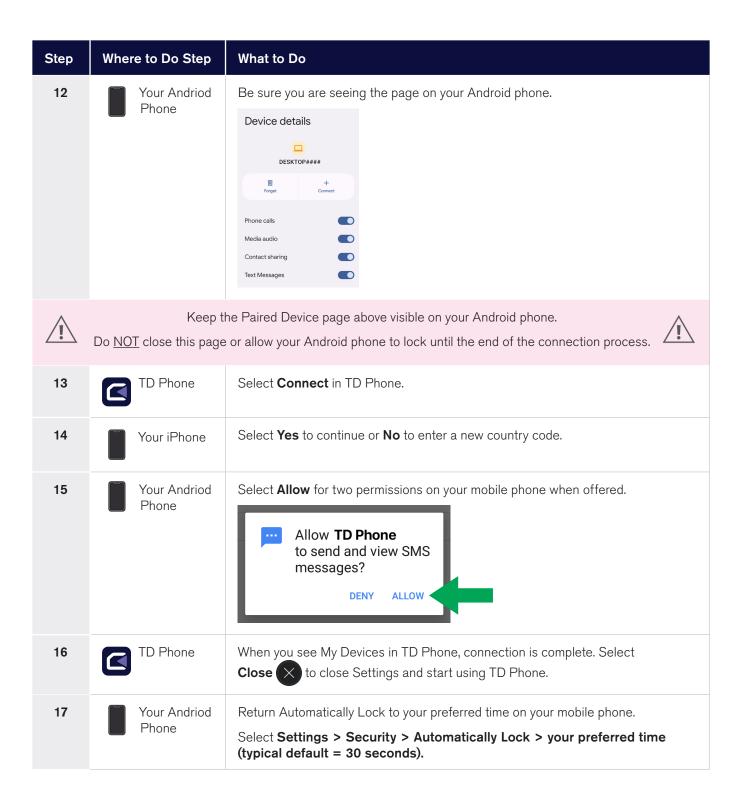

# Tour of TD Phone

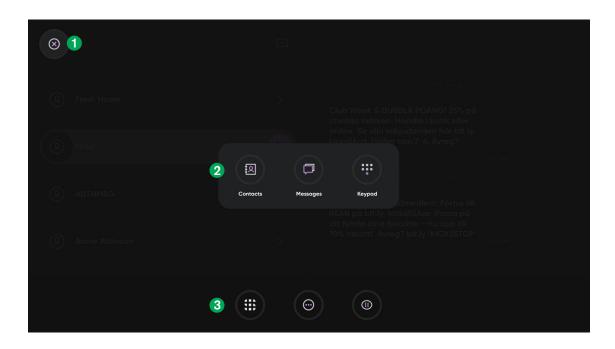

- 1 Open/Close Main Menu.
- 2 Main Menu including Contacts, Messages received and sent, Keypad for texting or calling.
- 3 Off Screen Menu appears when you look below the tracker contains (left to right) Switcher which allows you to move between apps, Dashboard where you will find Settings, and Pause/Resume eye tracking.
- 4 Exit TD Phone.
- **5** Settings where you will connect your mobile phone to TD Phone.
- 6 Close Dashboard to return to the Main Menu.

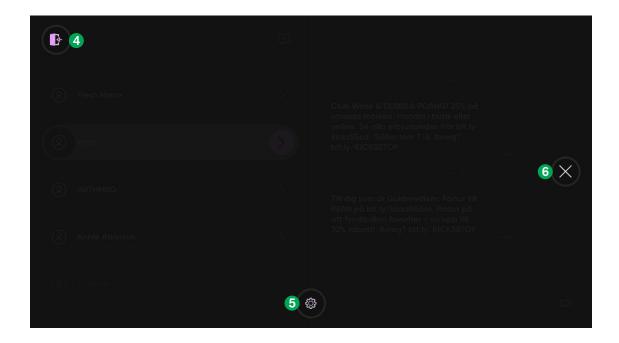

# Using TD Phone



#### **Essential for Use**



- ■TD Phone requires that your mobile phone remain on. It does not need to remain unlocked.
- Your mobile phone should remain within 3-5 meters or 10-16 feet of your computer depending on the mobile phone.

# Pause/Resume Eye Tracking


- Pause eye tracking by looking below your eye tracker. The Off Screen Menu will appear. Select **Pause** on the right.
- Resume eye tracking by looking below the eye tracker and select **Resume**.



# Send a Text Message


### Option 1

Open **Messages** to continue previous conversations. Select the **Keyboard** button to start a new text.

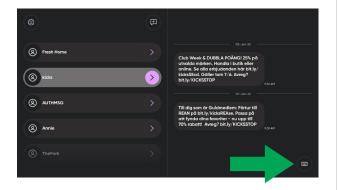


## Option 2

Select a contact in Contacts and select the **Message** icon.

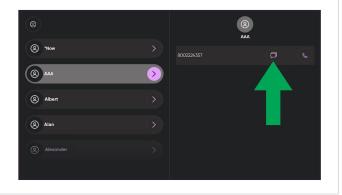

## Option 3

Open **Keypad** to enter a phone number then select the **Message** icon.

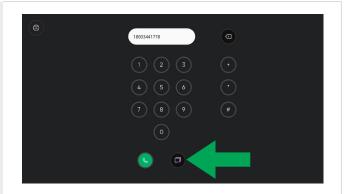

# Receive a Text

- Received texts will be indicated with a number notification on the Messages icon in the Main Menu.
- Review them when you wish.



# Make a Phone Call

## Option 1

Select a contact in Contacts and select the **Phone** icon.



## Option 2

Open **Keypad** to enter a phone number then select the **Phone** icon.



# Pause/Resume Eye Tracking


- A call window will appear over your screen.
- Select **Answer** or **Decline**.



# Send a Text Message


Switcher gives you access to Tobii Dynavox apps, apps you have pinned, and open apps.

- Look below the tracker to pull up the Off Screen Menu.
- Select **Switcher**.

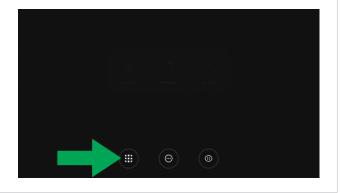


# **Settings**

 Access Settings by opening the Off Screen Menu (look below the eye tracker) and selecting Dashboard.

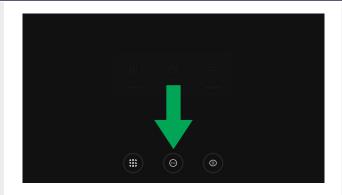


■ Select **Settings**.

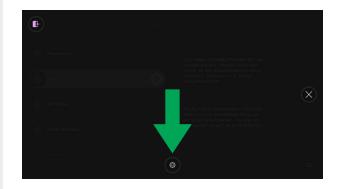

# **Exit TD Phone**

Exit TD Phone only if you no longer need eye tracking. Use Switcher (see above) to continue use of your device with eye tracking.

■ Open the **Off Screen Menu** by looking below your eye tracker. Select **Dashboard**.

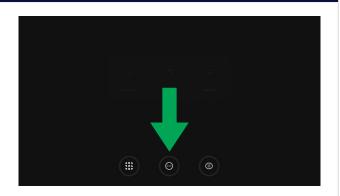


■ Select Exit.



# **FAQs**

#### How do I turn Bluetooth ON in my I-Series (not mobile phone)?

TD Phone tells you whether your I-Series Bluetooth is on or off. See steps 2-4 of the connection instructions for iPhone or Android. If Bluetooth is OFF in your I-Series. Use the instructions below to turn it on.

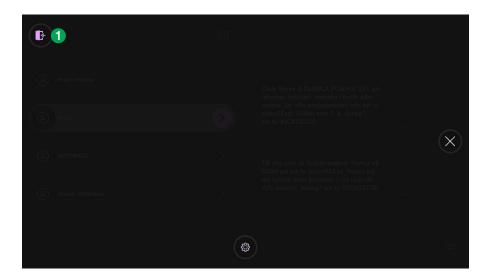

- 1 Exit TD Phone (Open the Off Screen Menu > select Dashboard, then select Exit).
- 2 Turn I-Series Bluetooth ON (Notifications > Bluetooth). It may be in a different location in your notifications.
- 3 Restart connection instructions for your mobile phone.






#### Note

The image depicts how Bluetooth looks when ON. Yours may or may not list any Connections.

#### What if my device does not appear in Bluetooth of my Android phone?

You may need to select **Scan or Find/pair new devices** in the Bluetooth menu.

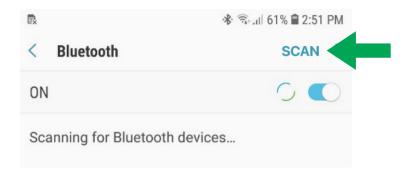

#### How do I recalibrate or adjust dwell times for TD Phone?

Currently, you will need to open Communicator 5, TD Control, TD Browse, or TD Snap to recalibrate and change dwell times.

#### How do I switch to another app on my device?

Switcher gives you access to the other Tobii Dynavox apps, your pinned apps, and apps you have open. Look below your eye tracker to open the Off Screen Menu. Switcher is on the left.



#### Why do I see only the last five of my messages from my iPhone in Messages?

Apple supports importing of only the last five messages. As you use TD Phone, your message history will build. It will be shared between both TD Phone and your iPhone.

#### How can I adjust the voice of TD Phone?

Voice options are located in TD Phone's Settings.

Off-Screen Menu > Dashboard > Settings > Open on Voice Panel

#### How can I increase the volume of a phone call?

Mobile phones may have separate volume controls for internal speakers versus Bluetooth devices such as ear buds and the I-Series with TD Phone. You can increase the volume only during an active phone call. Call someone or ask them to call you then adjust volume using the volume buttons on your mobile phone.

# Additional Resources

Scan the QR codes or use the links.



Tobii Dynavox

tobiidynavox.com



# myTobiiDynavox

mytobiidynavox.com



## **TD Facebook Community**

qrco.de/TDFB



## Tobii Dynavox Learning Hub

learn.tobiidynavox.com



#### North America Technical Support

1-800-344-1778 ext. 1



## **UK Technical Support**

0114 481 0011

support.uk@tobiidynavox.com