



GlobalMed[®] Quick Start Guide: Transportable Exam Station[™] (Enterprise)

Transportable Exam Station (TES) Enterprise Callouts

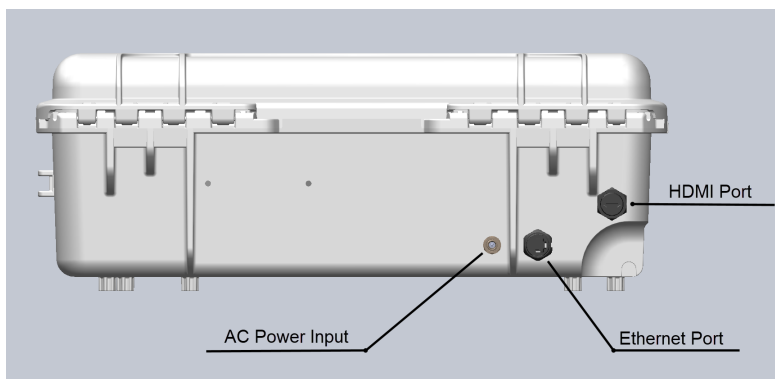

Contact Support: call 1-800-886-3692
option 2, email us at help@globalmed.com,
or log a ticket at globalmed.desk.com.

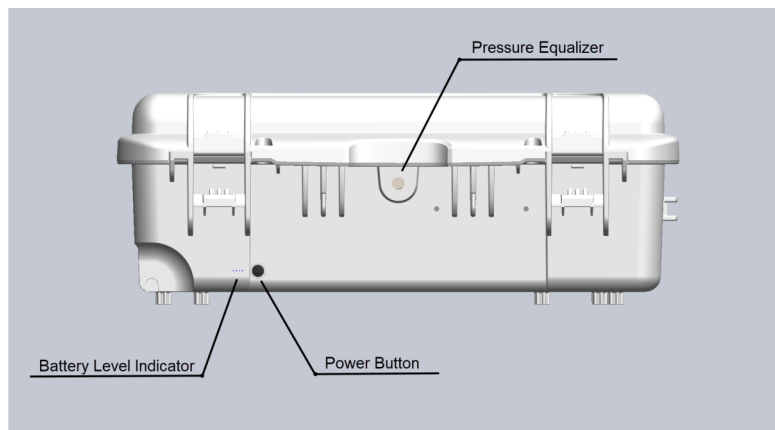


MAN-601070
© 2002-2018 GlobalMedia Group,
LLC.



GlobalMed[®] Quick Start Guide:

Transportable Exam Station[™] (Enterprise)

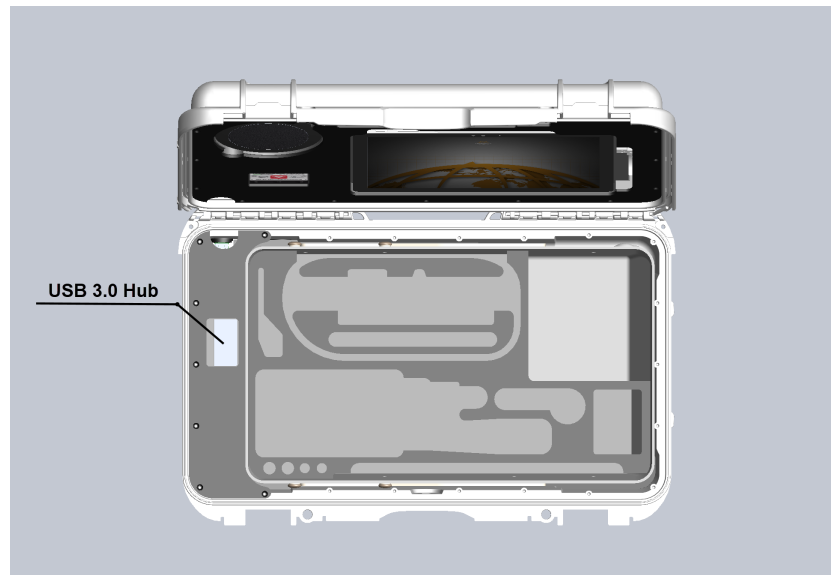

WiFi Hotspot

The TES Extreme is equipped with a Jetpack[™] MiFi 6620L that can be used to set up a WiFi hotspot. For more information on how to do this please refer to the Verizon's [Jetpack User Guide](#).

Note: WiFi is not enabled as a default; you must register your own SIM card and set up your WiFi capability.

Contact Support: call 1-800-886-3692
option 2, email us at help@globalmed.com,
or log a ticket at globalmed.desk.com.



MAN-601070
© 2002-2018 GlobalMedia Group,
LLC.



GlobalMed[®] Quick Start Guide:

Transportable Exam Station[™] (Enterprise)

Power

The TES Extreme must be plugged into a wall outlet overnight before first use. The adapter is located in the bottom tray.

Tip: When charging the TES Extreme make sure to fully insert the charging adapter into the AC power input. The battery level gauge flashes during charging.

Battery

It takes 6-8 hours to fully charge the TES Extreme. The Getac charges while the TES Extreme is charging. The TES Extreme operates on its internal batteries for up to 8 hours, depending on device usage.

Tip: The TES Extreme may be used while charging the battery, AFTER the initial overnight charge has been completed, as long as one light is lit on the battery level gauge.


One LED Light = 0-25% charge

Two LED lights = 25 - 50% charge

Three LED lights = 50 - 75% charge

Four LED lights = 75 - 100% charge

Flashing: TES is about to shut down, which shuts all devices down

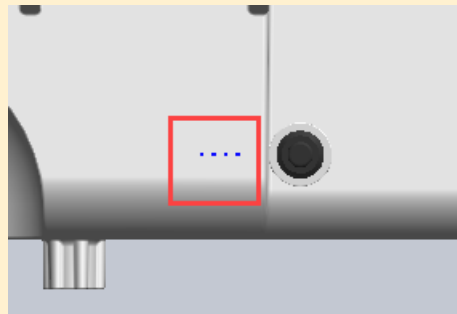

NOTE: Getac must be powered on for speaker, LED light, and USB hub to work.

Contact Support: call 1-800-886-3692
option 2, email us at help@globalmed.com,
or log a ticket at globalmed.desk.com.




MAN-601070
© 2002-2018 GlobalMedia Group,
LLC.




GlobalMed® Quick Start Guide: Transportable Exam Station™ (Enterprise)

Surface Pro 4 Callouts

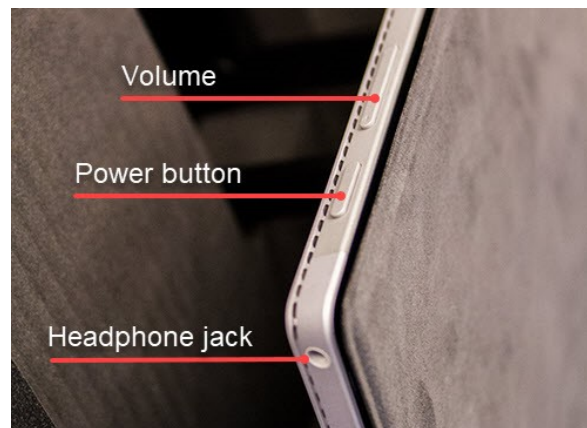

Speaker Callouts

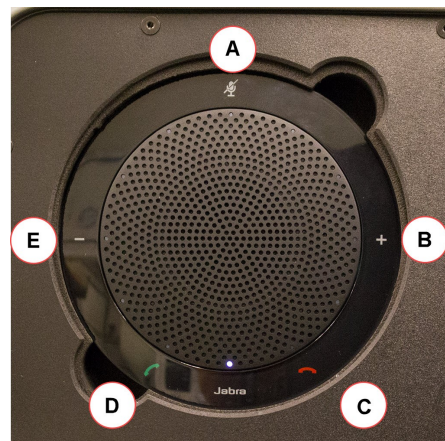

- A Mute
- B Volume up
- C End call
- D Answer call
- E Volume down

Contact Support: call 1-800-886-3692
option 2, email us at help@globalmed.com,
or log a ticket at globalmed.desk.com.



MAN-601070
© 2002-2018 GlobalMedia Group,
LLC.

