

cobas c 513 analyzer

Guide to User Assistance

Publication version 2.0

UDI (01)07613336187722(8012)02-05



Publication information

Publication version	Revision date	Change description
1.0	September 2015	Version 1.0
1.1	January 2016	Information added about how to set up the User Assistance.
2.0	November 2021	<ul style="list-style-type: none"> • Guide revised to match new user interface of User Assistance. • Access information updated. • Front matter updated.

☰ Revision history

Edition notice

This publication is intended for users of the **cobas c 513** analyzer.

Every effort has been made to ensure that all the information contained in this publication is correct at the time of publishing. However, the manufacturer of this product may need to update the publication information as output of product surveillance activities, leading to a new version of this publication.

Images

The images in this publication have been added exclusively for illustration purposes. Configurable and variable data in screenshots, such as tests, results, or path names visible therein must not be used for laboratory purposes.

Copyright

© 2015–2021, Roche Diagnostics GmbH. All rights reserved.

Trademarks

The following trademarks are acknowledged:

COBAS, COBAS C, and LIFE NEEDS ANSWERS are trademarks of Roche.

All other trademarks are the property of their respective owners.

Feedback

Every effort has been made to ensure that this publication fulfills the intended use. All feedback on any aspect of this publication is welcome and is considered during updates. Contact your Roche representative, should you have any such feedback.

Contact addresses

Inside the European Union and EFTA member states



Manufacturer of the instrument
Hitachi High-Tech Corporation
1-17-1 Toranomom, Minato-ku,
Tokyo, 105-6409
Japan



Authorized representative and importer
Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim
Germany

Outside the European Union and EFTA member states

Manufactured by: Hitachi High-Tech Corporation

Manufactured for: Roche Diagnostics GmbH
Sandhofer Strasse 116
68305 Mannheim
Germany

Distributed in USA by: Roche Diagnostics
9115 Hague Road
Indianapolis, Indiana
USA

Roche affiliates

A list of all Roche affiliates can be found at the following link:

www.roche.com/about/business/roche_worldwide.htm

eLabDoc

Electronic user documentation can be downloaded under eLabDoc on the Roche DiaLog website:

www.dialog.roche.com

For more information, contact your local affiliate or Roche Service representative.

Table of contents

About the User Assistance for the cobas c 513 analyzer	5
Selecting a publication from the library	6
The User Assistance window	8
Using the full-text search	10
Browsing the table of contents.....	11
Revisiting recently viewed topics.....	12
Using favorites	12

About the User Assistance for the cobas c 513 analyzer

To search and view information about the **cobas c** 513 analyzer, use the User Assistance.

The User Assistance for the **cobas c** 513 analyzer includes the content of the Operator's Manual, the Safety Manual, and the Basic Analytical Principles.

Accessing the User Assistance

To access the User Assistance, use a browser on a PC or tablet. Supported browsers include Google Chrome and Apple Safari (on the iOS tablet iPad).

Enter the following link in the address bar of the browser:

<https://userassistance.roche.com>

To request a password for the User Assistance, contact your Roche Service representative.

Selecting a publication from the library



Before viewing information about the **cobas c 513** analyzer, you must select the respective publication from the User Assistance library.



- You have a password for the User Assistance


► To select a publication from the library

- 1 Log in to the User Assistance.
- 2 Choose the **Library** tab.
- 3 From the **Library** tab, choose the **Manage library** button.
→ The **Publication search** window is displayed.

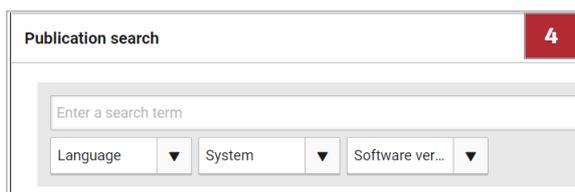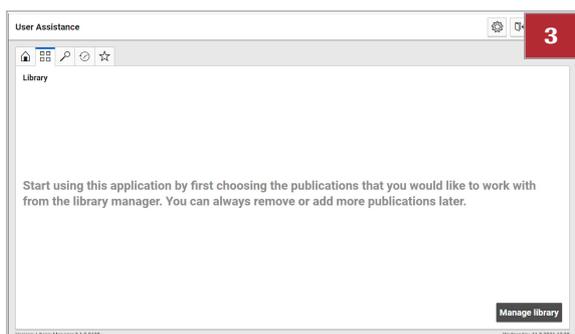

- 4 To filter the listed publications, enter a search term or select options from any of the following drop-down lists:

- **Language**
- **System**
- **Software version**

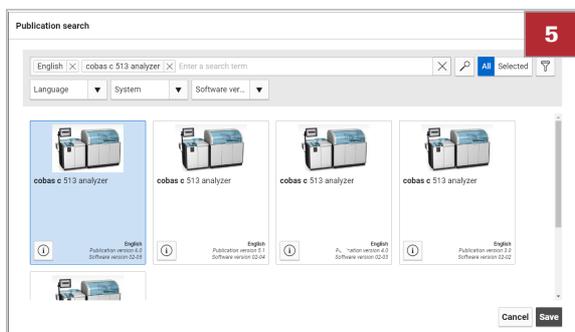

- 5 Select one or more publications.
→ The selected publications are displayed blue.
- 6 Choose the **Save** button.
→ The selected publications are displayed on the **Library** tab.




- 7 From the **Library** tab, select a publication to open it.

The User Assistance window

The User Assistance window enables you to search and view the topics of a publication. You can define favorite topics and go back to recently viewed topics. You can print topics or download the publication as a PDF file.

The following figure displays the main areas of the User Assistance window:

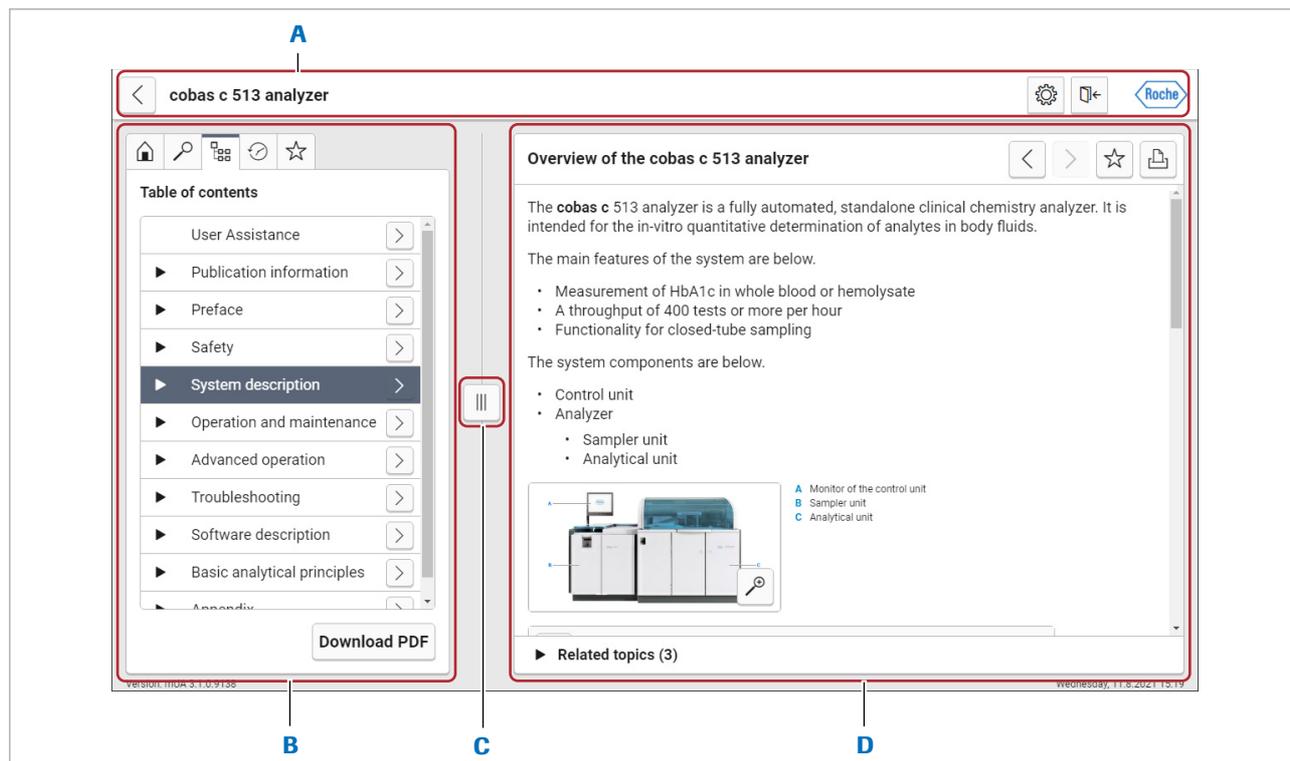

A Title bar

B Function tabs

C Resize slider

D Topics area

Title bar

The buttons on the title bar provide the following functions:



Back

Closes the publication and goes back to the **Library** tab.



Settings

Choose the User Assistance interface language.



Logoff

Exits the User Assistance.

Function tabs

The function tabs provide the following functions:

**Home**

Explains the function tabs.

**Find in this publication**

Displays the full-text search.

**Table of contents**

Displays the table of contents. Select any topic to display its contents in the topics area. Choose the **Download PDF** button to download the publication as a PDF file.

**Recently viewed**

Displays the list of recently viewed topics.

**Favorites**

Displays a list of topics that you added as favorites.

Resize slider**Resize**

Changes the size of the function tabs and topics area by pulling the slider to the left or right.

Topics area

The topics area displays the current topic, e.g., the topic you selected from the table of contents.

Links to other topics are displayed as underlined text. To view a linked topic, choose the link.

The buttons in the topics area provide the following functions:

**Back**

Navigates backward through the recently viewed topics.

**Forward**

Navigates forward through the recently viewed topics. This button is only active after first navigating backward.

**Favorite**


Choosing the light gray **Favorite** button adds the current topic to the list of favorites. The button turns dark gray.



Choosing the dark gray **Favorite** button removes the current topic from the list of favorites. The button turns light gray.

**Print**

Prints the current topic.

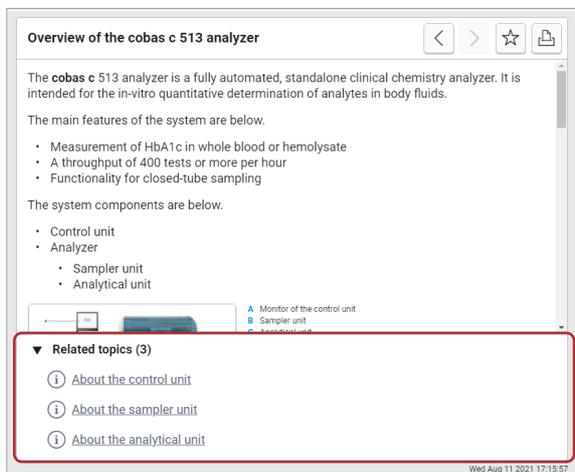


For some topics, in the bottom part of the topics area, a list of related topics is displayed. To view a related topic, choose the respective link.

Using the full-text search


To perform a full-text search, use the **Find in this publication** tab.

► To use the full-text search

- 1 From the User Assistance window, choose the **Find in this publication** tab.
- 2 In the search field, enter a search term.
→ A list of suggested search terms is displayed.
- 3 Perform one of the following actions:
 - From the list of suggested search terms, choose a topic.
 - To display all topics that contain your search term, choose the **List Topics** button.
 → The topics that contain your search term are displayed.

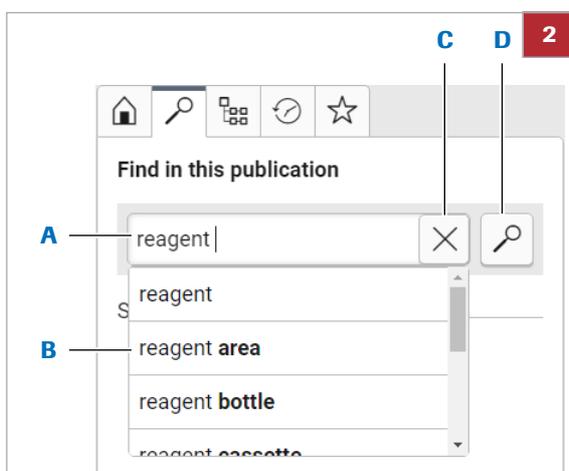

- A** Search field **C** Delete button
B Suggested search terms **D** List Topics button

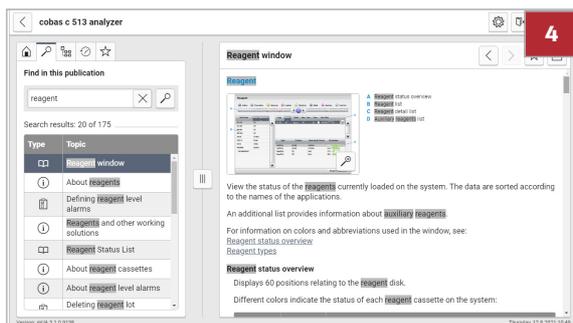

- 4 To view a topic, choose a topic title from the search result list.

Browsing the table of contents

To browse the table of contents, use the **Table of contents** tab.

► To browse the table of contents



- 1 From the User Assistance window, choose the **Table of contents** tab.
→ The table of contents is displayed.
- 2 To view a topic, choose a topic title.
 - ❶ To display subtopic titles in the table of contents, choose the right arrow icon ►.

Revisiting recently viewed topics

To revisit recently viewed topics, use the **Recently viewed** tab.

 Alternatively, you may use the **Back** and **Forward** buttons in the topics area.

► To revisit recently viewed topics



- 1 From the User Assistance window, choose the **Recently viewed** tab.
→ A list of recently viewed topics is displayed.
- 2 To view a topic, choose a topic title.
 - ❶ The most recent topic is displayed at the top of the list.

Using favorites

To add a favorite topic, use the **Favorite** button from the topics area. To view or remove a favorite topic, use the **Favorites** tab.

► To add a favorite topic

- 1 Display the topic in the topics area.
- 2 From the topics area, choose the **Favorite** button (displayed in light gray).
 - The **Favorite** button turns dark gray.
 - The topic is now available in the list of favorites on the **Favorites** tab.



► To view a favorite topic

- 1 From the function tabs, choose the **Favorites** tab.
 - The list of favorites is displayed.
- 2 To view a topic, choose a topic title.



► To remove a favorite topic

- 1 From the function tabs, choose the **Favorites** tab.
- 2 Select the check box next to the topic that you want to remove from the list of favorites.
- 3 Choose the **Remove from favorites** button.
 - ❶ Alternatively, if the topic is displayed in the topics area, choose the **Favorite** button (dark gray) to remove the topic from the list of favorites.

