

A Shared Vision. A Shared Future

develoct displayed columns, click Tools > Options > Data List

Activiare.

-Melesoft ... / Di Actiwatch Informat... (2) Insite - The Reson ... (2) Local Disk (Ci)

Instruction Manual

Actiwatch Communication and Sleep Analysis Software

Actiware[®] 5.0 and Actiware-CT 5.1

Actiwatch-16, -64, -Score, -L

Actireader

Actiwatch[®]-16 / Actiwatch[®]-64 Actiwatch[®]-L / Actiwatch[®]-Score Actiware[®] 5.0 / Actiware[®]-CT 5.1

Actiwatch Communication and Sleep Analysis Software

Instruction Manual



A Shared Vision. A Shared Future.

Respironics, Mini-Mitter, Actiwatch, and Actiware are registered trademarks of Respironics, Inc. and its affiliates. Windows is a registered trademark of Microsoft Corporation. Pentium is a registered trademark of Intel.

ActiReader is covered by the following US patent numbers: 6,129,663. Other patents pending.

© 2006 Respironics, Inc. and its affiliates. All rights reserved.

Mini Mitter, a Respironics Company

WARRANTY POLICY

- 1. **WARRANTY**: Mini Mitter, a Respironics Company, warrants that upon delivery the goods sold hereunder will be free of defects in materials and workmanship, and such goods will substantially conform to the specifications furnished by the Seller, and to any drawings or specifications furnished to the Buyer by the Manufacturer (Mini Mitter, a Respironics Company) if approved by the Manufacturer.
 - a) **Commencement of Warranty** The warranty period begins on the date of delivery.
 - b) **Duration of Warranty** The goods sold hereunder are warranted for a period of twelve months, except PDT-4000 and PDT-4000HR, which are warranted for two years.
 - c) Third Party Product Warranty Third party products purchased by Buyer through the Manufacturer are covered by the third party's warranty and ARE NOT COVERED BY THE MANUFACTURER WARRANTY.
 - d) Place of Repair or Replacement. Buyer must return the defective or nonconforming goods upon request to Mini Mitter, a Respironics Company, at 20300 Empire Avenue, Building B-3; Bend, OR 97701, 800-685-2999 (US or Canada only) or 1-541-598-3800, not later than 15 days after the Manufacturer's receipt of notice of the alleged defect or nonconformance. Buyer shall prepay transportation charges. The Manufacturer shall pay for the return of the goods to Buyer. No goods are to be returned to the Manufacturer without prior authorization and a valid Return Authorization (RA) number.
 - e) Limitation of Warranty The foregoing warranty shall not apply to defects resulting from (a) improper or inadequate maintenance by Buyer; (b) unauthorized modification of the goods; (c) operation of the goods outside of the environmental specifications for the goods; (d) neglect, misuse or abuse of the goods or (e) modification or integration with other goods not covered by a Mini Mitter, a Respironics Company, warranty when such modification or integration increases the likelihood of damage to the goods.
 - f) Technical assistance The warranty set forth above shall not be enlarged, diminished or affected by and no obligation or liability shall arise from the Manufacturer's rendering of technical advice, assistance or services in connection with Buyer's order of the goods furnished hereunder. The Buyer is not relying on the Manufacturer's skill or judgment to select or furnish suitable goods.

THE WARRANTY SET FORTH ABOVE IS EXCLUSIVE AND THE MANUFACTURER GIVES NO OTHER WARRANTY WRITTEN OR VERBAL, EXPRESS OR IMPLIED. MINI MITTER, A RESPIRONICS COMPANY, SPECIFICALLY DISCLAIMS THE IMPLIED WARRANTIES OF MERCHANT-ABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE MANUFACTURER'S STANDARD WARRANTY POLICY COVERING ANY OF THE MANUFACTURER'S PRODUCTS SHALL BE NULL, VOID, AND OF NO EFFECT IF THE BUYER OR ANY THIRD PARTY USES THAT PRODUCT FOR ANY USE OTHER THAN THE INTENDED USE OF THAT PRODUCT.

2. LIMITATION OF REMEDIES AND LIABILITY. The remedies provided herein are Buyer's sole and exclusive remedies. The Manufacturer shall have no responsibility other than, at the Manufacturer's option, to repair or replace defective or nonconforming goods or to refund the purchase price of defective or nonconforming goods in the event of breach of any warranty. IN NO EVENT SHALL THE MANUFACTURER BE LIABLE FOR ANY DIRECT, INDIRECT, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES (INCLUDING LOSS OF PROFITS) WHETHER BASED ON CONTRACT, TORT OR ANY OTHER LEGAL THEORY. In the event that any limitation of warranty or disclaimer of liability is found to be unlawful or inapplicable, or to have failed of its essential purpose, the Manufacturer's liability shall be limited to the amount paid by the Buyer for the specific goods that caused such liability.

MINI MITTER, a Respironics Company

SOFTWARE LICENSE AGREEMENT

NOTICE TO USER: BY CLICKING THE "I ACCEPT THE TERMS IN THE LICENSE AGREEMENT" BUTTON, THE USER ("YOU") AGREE(S) TO BE BOUND BY THE TERMS OF THIS SOFTWARE LICENSE AGREEMENT ("AGREEMENT"). BY CLICKING YES YOU ACCEPT ALL THE TERMS AND CONDITIONS OF THIS AGREEMENT, INCLUDING WITHOUT LIMITATION, THE RESTRICTIONS ON: USE CONTAINED IN SECTION 2; TRANSFERABILITY IN SECTION 4; WARRANTY IN SECTION 6; AND LIABILITY IN SECTION 7. YOU AGREE THAT THIS AGREEMENT IS ENFORCEABLE LIKE ANY WRITTEN NEGOTIATED AGREEMENT SIGNED BY YOU. IF YOU DO NOT AGREE TO BE BOUND TO THE TERMS AND CONDITIONS OF THIS AGREEMENT, YOU MAY OBTAIN A REFUND OF ANY AMOUNT YOU ORIGINALLY PAID IF YOU: (A) DO NOT USE THE SOFTWARE AND (B) RETURN THE SOFTWARE, WITH PROOF OF PAYMENT, TO THE LOCATION FROM WHICH IT WAS OBTAINED WITHIN THIRTY (30) DAYS OF THE PURCHASE DATE.

1. <u>Definitions</u>. "Software" means (a) all of the contents of the files, disk(s), CD-ROM(s) or other media with which this Agreement is provided, including but not limited to (i) MINI MITTER or third party computer information or software; (ii) related explanatory written materials or files ("<u>Documentation</u>"); and (b) upgrades, modified versions, updates, additions, and copies of the Software, if any, licensed to you by MINI MITTER (collectively, "<u>Updates</u>"). "<u>Use</u>" or "<u>Using</u>" means to access, install, download, copy or otherwise benefit from using the functionality of the Software in accordance with the Documentation. "<u>Permitted Number</u>" means one (1) unless otherwise indicated under a valid license (e.g. volume license) granted by MINI MITTER. "<u>Computer</u>" means an electronic device that accepts information in digital or similar form and manipulates it for a specific result based on a sequence of instructions. "<u>MINI MITTER</u>" means MINI MITTER, a Respironics Company, 20300 Empire Ave., Bldg B-3, Bend, Oregon 97701.

2. Software License.

2.1 <u>Grant</u>. MINI MITTER owns all intellectual property in the Software. MINI MITTER hereby grants you a limited, non-exclusive license to Use the Software only in accordance with the terms of this Agreement and for the purposes described in the Documentation. Use of some third party materials included in the Software may be subject to other terms and conditions found in a separate license agreement.

2.2 <u>General Use</u>. You may install and Use a copy of the Software on your compatible Computer, up to the Permitted Number of Computers.

2.3 <u>Backup Copy</u>. You may make one backup copy of the Software, provided your backup copy is not installed or used on any Computer. You may not transfer the rights to a backup copy unless you transfer all rights in the Software as provided under Section 4.

2.4 <u>No Modification</u>. You may not alter or modify the Software or create a new installer for the Software. The Software is licensed and distributed by MINI MITTER for Use with its data collection hardware products.

3. Intellectual Property Ownership and Copyright Protection. The Software and any authorized copies that you make are the intellectual property of and are owned by MINI MITTER. The structure, organization and code of the Software are the valuable trade secrets and confidential information of MINI MITTER. The Software is protected by law, including without limitation, the copyright laws of the United States and other countries, and by international treaty provisions. Except as expressly stated herein, this Agreement does not grant you any intellectual property rights in the Software and all rights not expressly granted are reserved by MINI MITTER. You hereby recognize that MINI MITTER retains all intellectual property rights in the Software, Documentation and all MINI MITTER confidential information, including without limitation, all corrections, modifications and other derivative works to the Software. You hereby assign to MINI MITTER all intellectual property rights you may now or hereafter possess in the Software, Documentation and MINI MITTER confidential information, and all derivative works thereof and you agree to execute all documents, and take all actions, that may be necessary to confirm such rights. You agree to observe complete confidentiality with respect to the MINI MITTER confidential information; not to disclose, or permit any third party or entity access to, the MINI MITTER confidential information (or any portion thereof) without prior written permission of MINI MITTER, and to insure that any employees, or any third parties who receive access to the MINI MITTER confidential information, are advised of the confidential and proprietary nature thereof and are prohibited from copying, utilizing or otherwise revealing the MINI MITTER confidential information. You agree to employ with regard to the MINI MITTER confidential information commercially reasonable procedures that are no less restrictive than the strictest procedures used by it to protect its own confidential and proprietary information.

4. <u>Restrictions</u>.

4.1 <u>Notices</u>. You shall not copy the Software except as set forth in Section 2. Any copy of the Software that you make must contain the same copyright and other proprietary notices that appear on or in the Software, indicating that the Software are owned by MINI MITTER.

4.2 <u>No Modifications</u>. You shall not modify, adapt or translate the Software. You shall not reverse engineer, decompile, disassemble or otherwise attempt to discover the source code of the Software.

4.3 <u>Transfer</u>. You may not, rent, lease, sublicense, assign or transfer your rights in the Software, or authorize all or any portion of the Software to be copied onto another user's Computer except as may be expressly permitted herein. You may, however, transfer all your rights to Use the Software to another person or legal entity provided that: (a) you also transfer (i) this Agreement, and (ii) the Software and all other software bundled or pre-installed with the Software, including all copies, Updates and prior versions, to such person or entity; (b) you retain no copies, including backups and copies stored on a Computer; and (c) the receiving party accepts the terms and conditions of this Agreement and any other terms and conditions upon which you legally purchased a license to the Software. Notwithstanding the foregoing, you may not transfer education, pre-release, or not for resale copies of the Software.

5. Updates. If the Software is an Update to a previous version of the Software, you must possess a valid license to such previous version in order to Use such Update. All Updates are provided to you on a license exchange basis. You agree that by Using an Update you voluntarily terminate your right to Use any previous version of the Software. As an exception, you may continue to Use previous versions of the Software on your Computer after you Use the Update but only to assist you in the transition to the Update, provided that: (a) the Update and the previous versions are installed on the same computer; (b) the previous versions or copies thereof are not transferred to another party or Computer unless all copies of the Update are also transferred to such party or Computer; and (c) you acknowledge that any obligation MINI MITTER may have to support the previous versions of the Software may be ended upon availability of the Update. All Updates will be provided to you in the sole discretion of MINI MITTER.

6. LIMITED WARRANTY.

6.1 <u>Limited Warranty</u>. MINI MITTER warrants for a period of twelve (12) months after the date of delivery to you (the "<u>Warranty</u> <u>Period</u>"), the following: (a) that the Software will perform in substantial compliance with the Documentation, provided that it is used with the data collection hardware, Computer hardware and with the operating system for which it was designed, (b) that the media on which the Software is distributed is substantially free from significant defects in materials and workmanship. MINI MITTER shall have no obligation to provide warranty services if the defect in the Software is caused by malfunction of non-MINI MITTER hardware or software, by modification of the Software not made by MINI MITTER, by operator error, or by use of the Software that is not in accordance with the operating instructions and Documentation for the Software.

6.2 <u>Disclaimer</u>. EXCEPT AS PROVIDED IN SECTION 6.1, MINI MITTER AND ITS SUPPLIERS MAKE NO WARRANTIES OR REPRESENTATIONS, WHETHER EXPRESS OR IMPLIED WHETHER BY STATUTE, COMMON LAW, CUSTOM, USAGE OR OTHERWISE, AS TO ANY MATTER INCLUDING WITHOUT LIMITATION NONINFRINGEMENT OF THIRD PARTY RIGHTS, MERCHANTABILITY, INTEGRATION, SATISFACTORY QUALITY, OR FITNESS FOR ANY PARTICULAR PURPOSE. The provisions of Section 6 and Section 7 shall survive the termination of this Agreement, howsoever caused, but this shall not imply or create any continued right to Use the Software after termination of this Agreement.

6.3 <u>Warranty Claims</u>. Warranty claims must be made in writing during the Warranty Period and within seven (7) days of the observation of the defect, accompanied by evidence satisfactory to MINI MITTER. All Software should be returned to MINI MITTER or the MINI MITTER distributor from which it was purchased and shall be shipped by the returning party with freight and insurance paid.

- 7. LIMITATION OF LIABILITY. IN NO EVENT WILL MINI MITTER OR ITS SUPPLIERS BE LIABLE TO YOU FOR ANY DAMAGES, CLAIMS OR COSTS WHATSOEVER OR ANY CONSEQUENTIAL, INDIRECT, INCIDENTAL DAMAGES, OR ANY LOST PROFITS OR LOST SAVINGS, EVEN IF A MINI MITTER REPRESENTATIVE HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH LOSS, DAMAGES, CLAIMS OR COSTS OR FOR ANY CLAIM BY ANY THIRD PARTY. THE FOREGOING LIMITATIONS AND EXCLUSIONS APPLY TO THE EXTENT PERMITTED BY APPLICABLE LAW IN YOUR JURISDICTION. MINI MITTER'S AGGREGATE LIABILITY AND THAT OF ITS SUPPLIERS UNDER OR IN CONNECTION WITH THIS AGREEMENT SHALL BE LIMITED TO THE AMOUNT PAID FOR THE SOFTWARE, IF ANY. MINI MITTER is acting on behalf of its suppliers for the purpose of disclaiming, excluding and/or limiting obligations, warranties and liability as provided in this Agreement, but in no other respects and for no other purpose.
- 8. Export Rules. You agree that the Software will not be shipped, transferred or exported into any country or used in any manner prohibited by the United States Export Administration Act or any other export laws, restrictions or regulations (collectively the "Export Laws"). In addition, if the Software is identified as export controlled items under the Export Laws, you represent and warrant that you are not a citizen, or otherwise located within, an embargoed nation (including without limitation Iran, Iraq, Syria, Sudan, Libya, Cuba, North Korea, and Serbia) and that you are not otherwise prohibited under the Export Laws from receiving the Software. All rights to Use the Software are granted on condition that such rights are forfeited if you fail to comply with the terms of this Agreement.
- 9. <u>Governing Law</u>. This Agreement will be governed by and construed in accordance with the substantive laws in force in the State of Oregon, USA, without regards to its conflicts of laws principles.
- 10. Term. The term of this Agreement shall begin on your acceptance and continue until the Agreement is terminated.

10.1 <u>Termination for Cause</u>. In the event of any material breach of this Agreement, the non-breaching party may terminate this Agreement by giving thirty (30) days' prior written notice to the other party; provided, however, that this Agreement shall not terminate if the other party has cured the breach prior to the expiration of such thirty (30) day period.

10.2 <u>Termination for Insolvency</u>. Either party may terminate this Agreement, without notice, (i) upon the institution by or against the other party of insolvency, receivership or bankruptcy proceedings, (ii) upon the other party's making an assignment for the benefit of creditors, or (iii) upon the other party's dissolution or ceasing to do business.

10.3 <u>Return of Materials</u>. Upon termination of this Agreement for any reason, you shall return to MINI MITTER all copies of the Software, confidential information, Documentation and all other tangible materials related to the Software, including without limitation, all modifications and derivative works thereof.

- 11. <u>General Provisions</u>. If any part of this Agreement is found void and unenforceable, it will not affect the validity of the balance of this Agreement, which shall remain valid and enforceable according to its terms. This Agreement shall not prejudice the statutory rights of any party dealing as a consumer. This Agreement may only be modified by a writing signed by an authorized officer of MINI MITTER. Updates may be licensed to you by MINI MITTER with additional or different terms. This is the entire Agreement between MINI MITTER and you relating to the Software and it supersedes any prior representations, discussions, undertakings, communications or advertising relating to the Software.
- 12. Notice to U.S. Government End Users. The Software and Documentation are "Commercial Items," as that term is defined at 48 C.F.R. §2.101, consisting of "Commercial Computer Software" and "Commercial Computer Software Documentation," as such terms are used in 48 C.F.R. §12.212 or 48 C.F.R. §227.7202, as applicable. Consistent with 48 C.F.R. §12.212 or 48 C.F.R. §§227.7202-1 through 227.7202-4, as applicable, the Commercial Computer Software and Commercial Computer Software Documentation are being licensed to U.S. Government end users (a) only as Commercial Items and (b) with only those rights as are granted to all other end users pursuant to the terms and conditions herein. Unpublished-rights reserved under the copyright laws of the United States. Manufacturer is MINI MITTER, a Respironics. Company, 20300 Empire Ave., Bldg B-3, Bend, Oregon 97701, USA. For U.S. Government End Users, MINI MITTER agrees to comply with all applicable equal opportunity laws including, if appropriate, the provisions of Executive Order 11246, as amended, Section 402 of the Vietnam Era Veterans Readjustment Assistance Act of 1974 (38 USC 4212), and Section 503 of the Rehabilitation Act of 1973, as amended, and the regulations at 41 CFR Parts 60-1 through 60-60, 60-250, and 60-741.
- 13. <u>Compliance with Licenses</u>. You agree that upon request from MINI MITTER or MINI MITTER's authorized representative, you will within thirty (30) days fully document and certify that Use of any and all Software at the time of the request is in conformity with your valid license from MINI MITTER.

14. Specific Exceptions.

14.1 Pre-release Product Additional Terms. If the product you have received with this license is pre-commercial release or beta Software ("Pre-release Software"), then the following Section applies. To the extent that any provision in this Section is in conflict with any other term or condition in this Agreement, this Section shall supersede such other term(s) and condition(s) with respect to the Pre-release Software, but only to the extent necessary to resolve the conflict. You acknowledge that the Software is a prerelease version, does not represent final product from MINI MITTER, and may contain bugs, errors and other problems that could cause system or other failures and data loss. Consequently, the Pre-release Software is provided to you "AS-IS", and MINI MITTER disclaims any warranty or liability obligations to you of any kind. WHERE LIABILITY CANNOT BE EXCLUDED FOR PRE-RELEASE SOFTWARE, BUT IT MAY BE LIMITED, MINI MITTER'S LIABILITY AND THAT OF ITS SUPPLIERS SHALL BE LIMITED TO THE SUM OF FIFTY DOLLARS (U.S. \$50) IN TOTAL. You acknowledge that MINI MITTER has not promised or guaranteed to you that Pre-release Software will be announced or made available to anyone in the future, MINI MITTER has no express or implied obligation to you to announce or introduce the Pre-release Software and that MINI MITTER may not introduce a product similar to or compatible with the Pre-release Software. Accordingly, you acknowledge that any research or development that you perform regarding the Prerelease Software or any product associated with the Pre-release Software is done entirely at your own risk. During the term of this Agreement, if requested by MINI MITTER, you will provide feedback to MINI MITTER regarding testing and use of the Pre-release Software, including error or bug reports. If you have been provided the Pre-release Software pursuant to a separate written agreement, your use of the Software is also governed by such agreement. You agree that you may not and certify that you will not sublicense, lease, loan, rent, assign or transfer the Pre-release Software. Upon receipt of a later unreleased version of the Prerelease Software or release by MINI MITTER of a publicly released commercial version of the Software, whether as a stand-alone product or as part of a larger product, you agree to return or destroy all earlier Pre-release Software received from MINI MITTER and to abide by the terms of the license agreement for any such later versions of the Pre-release Software. Notwithstanding anything in this Section to the contrary, if you are located outside the United States of America, you agree that you will return or destroy all unreleased versions of the Pre-release Software within thirty (30) days of the completion of your testing of the Software when such date is earlier than the date for MINI MITTER's first commercial shipment of the publicly released (commercial) Software.

If you have any questions regarding this Agreement or if you wish to request any information from MINI MITTER please use the address and contact information included with this product to contact us. **Thank You!** ... for purchasing products from Mini Mitter, a Respironics Company. If you need assistance with Actiwatch; remember, support continues after the purchase. If you have any problems or questions, please call our Technical Support staff of technicians, engineers, and scientists. We are available by telephone, fax, e-mail, or web site.

Contacting Technical Support

Mailing and Shipping Address	Mini Mitter, a Respironics Company 20300 Empire Avenue, Bldg. B-3 Bend, OR 97701 USA
Telephone	(800) 685-2999 (US and Canada only) (541) 598-3800
Fax	(541) 322-7277
E-Mail	mm@respironics.com

Web site www.minimitter.com www.respironics.com

Table of Contents

Contacting Technical Support	Software License Agreement	iv
INTRODUCTION AND SETUP 1.1 Indications for Use 1.2 Intended Use 1.2 Warnings and Cautions 1.3 Symbols 1.5 Installation Prorequisites 1.5 Installation Procedures 1.5 Installation Procedures 1.1 Uninstallation Procedures 1.1 Activeatch Reader 1.1 CatReader Battery Replacement Procedures 1.2 Actigram 2.2 Actogram 2.2 Actogram 2.2 Betting Intervals 1.2 Viewing Statistics 2.1 Exporting Data 1.1 Exporting Data 1.1 Retrieve Data from Actiwatch 2.2 Deleting/Hiding Subjects 2.22 Deleting/Hiding Subjects 2.22 Deleting/Hiding Subjects 2.23 COMMUNICATIONS 3.1 Retrieve Data from Actiwatch 3.1 Retriev	Contacting Technical Support	vii
INTRODUCTION AND SETUP Intended Use 1-1 Intended Use 1-2 Intended Use 1-2 Warnings and Cautions 1-3 Symbols 1-5 Installation Prorequisites 1-5 Installation Procedures 1-6 Installation Procedures 1-7 Inistallation 1-7	INTRODUCTION AND SETUD	
Inductions for USE	Indications for Los	
International of the second se	Indications for use	
Valmigs and Catudons	Wornings and Cautions	
Symbolics 1-3 Installation Prerequisites 1-4 Installation Procedures 1-1 ActiReader Setup 1-14 ActiReader Setup 1-14 ActiReader Battery Replacement Procedures 1-20 ActiReader Battery Replacement Procedures 1-21 GETTING STARTED 2-4 Main Window 2-2 Actogram 2-8 Setting Intervals 2-10 Viewing Statistics 2-17 Exporting Data 2-19 Print Report 2-22 Deleting/Hiding Subjects 2-223 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-1 ADVANCED USAGE 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Tota File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Data List 5-1 <td>Symbole</td> <td></td>	Symbole	
Installing Torequisites	Syllibuls	
Installation Procedures	Installation Frenequisites	
Officialization Procedures 1-14 ActiReader Setup 1-14 ActiReader Setup, Replacement Procedures 1-20 ActiReader Battery Replacement Procedures 1-21 Main Window. 2-2 Actorgam 2-8 Setting Intervals 2-10 Viewing Statistics 2-11 Print Report 2-22 COMMUNICATIONS 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-14 Program Options 4-2 Analysis Parameter Definitions 4-2 Data List 4-10 Text File Batch Export 4-14 Import AWD Files 5-3 Introduction to Electronic Signatures 5-4 Database Validation and Back-up 5-7 SLEEPWAKE ANALYSIS 4-1 Actiwatch Hardbware C-1 Actiwatch Hardbware C-4 Actiwatch Reader Fore C-1 Statistic DeFINITIONS 8-1 Activatch Battery Replacement Procedure C-3	Installing from a CD	
Activatch Reader	AntiDondor Sotup	
Activation Reader Sattery Replacement Procedures	Activisteb Baseder	
Activeduel Battery Replacement Procedures -21 Main Window -22 Main Window -22 Actogram -28 Setting Intervals -210 Viewing Statistics -210 Viewing Statistics -210 Print Report -22 Deleting/Hiding Subjects -222 Deleting/Hiding Subjects -223 COMMUNICATIONS -31 Configure Actiwatch -37 ADVANCED USAGE -41 Program Options -42 Analysis Parameter Definitions -44 Data List -410 Import AWD Files -51 Audit Trail -53 Introduction to Electronic Signatures -54 Database Validation and Back-up -57 Export Files -57 Statistic DEFINITIONS -61 ACTIWATCH HARDWARE -61 ACTIWATCH HARDWARE -61 ACTIWATCH SCORE -62 Actiwatch Battery Replacement Procedure -61 SPECIFICATION D-1 FWIC INFORMATION -61	ActiPooder Pottery Poologoment Broosdurge	
GETTING STARTED 2-1 Main Window 2-2 Actogram 2-8 Setting Intervals 2-10 Viewing Statistics 2-17 Exporting Data 2-19 Print Report 2-21 New Subject 2-22 Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-1 Program Options 4-1 ADVANCED USAGE 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-4 Activatch HARDWARE 5-7 SLEEP/WAKE ANALYSIS A-1 Actiwatch HarDWARE C-1 ACTIWATCH + SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1	Actikeader Battery Replacement Procedures	
Main Window	GETTING STARTED	
Actogram 2-8 Setting Intervals. 2-10 Viewing Statistics 2-17 Exporting Data 2-19 Print Report 2-21 New Subject 2-22 Deleting/Hiding Subjects 2-22 Deleting/Hiding Subjects 2-23 COMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-14 Graph 4-14 Graph 4-14 Graph 4-14 Graph 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Startistic Definitions 8-1 Actiwatch HarDware C-1 Actiwatch HarDware C-4 Actiwatch Cleaning Procedure C-3 Actiwatch Cleaning Procedure C-13 <t< td=""><td>Main Window</td><td></td></t<>	Main Window	
Setting Intervals 2-10 Viewing Statistics 2-17 Exporting Data 2-19 Print Report 2-21 New Subject 2-22 Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-1 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 STATISTIC DEFINITIONS 8-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-SCORE C-5 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1	Actogram	
Viewing Statistics 2-17 Exporting Data 2-19 Print Report 2-22 Deleting/Hiding Subjects 2-22 Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH HARDWARE C-6 ACTIWATCH HARDWARE C-1 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1	Setting Intervals	
Exporting Data 2-19 Print Report 2-21 New Subject 2-22 Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 SLEEP/WAKE ANALYSIS A-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-L C-5 ACTIWATCH HARDWARE C-13 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1	Viewing Statistics	
Print Report. 2-21 New Subject 2-22 Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-2 Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 SLEEP/Wake ANALYSIS 5-7 SLEEP/Wake ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH HARDWARE C-5 ACTIWATCH-L C-5 ACTIWATCH SCORE C-8 Actiwatch Cleaning Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION	Exporting Data	
New Subject 2-22 Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1	Print Report	
Deleting/Hiding Subjects 2-23 COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-2 Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 SLEEP/WAKE ANALYSIS A-1 StatISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	New Subject	
COMMUNICATIONS 3-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4 Data List 4-40 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 Actiwatch Cleaning Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION D-1	Deleting/Hiding Subjects	
COMMUNICATIONS 31-1 Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-4-1 CLINICAL TRIALS 5-3 Introduction to Electronic Signatures 5-3 Introduction to Electronic Signatures 5-3 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS 4-1 STATISTIC DEFINITIONS 4-1 STATISTIC DEFINITIONS 4-1 STATISTIC DEFINITIONS 4-1 ACTIWATCH HARDWARE 4-1 ACTIWATCH HARDWARE 4-1 ACTIWATCH SCORE 4-1 ACTIWATCH SCORE 4-1 ACTIWATCH SCORE 4-1 ACTIWATCH Cleaning Procedure 4-1 C-13 Actiwatch Cleaning Procedure 4-1 C-17 SPECIFICATION 5-1 FLO		
Configure Actiwatch 3-1 Retrieve Data from Actiwatch 3-7 ADVANCED USAGE 4-1 Program Options 4-2 Analysis Parameter Definitions 4-2 Analysis Parameter Definitions 4-4 Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-SCORE C-4 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION D-1	COMMUNICATIONS	3-1
Retrieve Data from Actiwatch	Configure Actiwatch	
ADVANCED USAGE	Retrieve Data from Actiwatch	
Program Options 4.2 Analysis Parameter Definitions 4.4 Data List 4.10 Text File Batch Export 4.11 Import AWD Files 4.14 Graph 4.16 CLINICAL TRIALS 5-1 Audit Trail 5-5 Audit Trail 5-6 Database Validation and Back-up 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-4	ADVANCED USAGE	4-1
Analysis Parameter Definitions	Program Options	
Data List 4-10 Text File Batch Export 4-11 Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	Analysis Parameter Definitions	4-4
Text File Batch Export. 4-11 Import AWD Files. 4-14 Graph 4-16 CLINICAL TRIALS. 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS. B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 EMC INFORMATION E-4	Data List	4-10
Import AWD Files 4-14 Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-4	Text File Batch Export	4-11
Graph 4-16 CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-4	Import AWD Files	4-14
CLINICAL TRIALS	Graph	4-16
CLINICAL TRIALS 5-1 Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	1	
Audit Trail 5-3 Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH HARDWARE C-5 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 EMC INFORMATION E-1	CLINICAL TRIALS	5-1
Introduction to Electronic Signatures 5-6 Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	Audit Trail	5-3
Database Validation and Back-up 5-7 Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	Introduction to Electronic Signatures	5-6
Export Files 5-7 SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	Database Validation and Back-up	5-7
SLEEP/WAKE ANALYSIS A-1 STATISTIC DEFINITIONS B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	Export Files	5-7
STATISTIC DEFINITIONS. B-1 ACTIWATCH HARDWARE C-1 ACTIWATCH-L C-5 ACTIWATCH-SCORE C-8 Actiwatch Battery Replacement Procedure C-13 Actiwatch Cleaning Procedure C-17 SPECIFICATION D-1 FMC INFORMATION E-1	SLEEP/WAKE ANALYSIS	A-1
ACTIWATCH HARDWARE	STATISTIC DEFINITIONS	B-1
ACTIWATCH HARDWARE		
ACTIWATCH-LC-5 ACTIWATCH-SCOREC-8 Actiwatch Battery Replacement ProcedureC-13 Actiwatch Cleaning ProcedureC-17 SPECIFICATIOND-1 FMC INFORMATION E-1	ACTIWATCH HARDWARE	C-1
ACTIWATCH-SCORE	ACTIWATCH-L	C-5
Actiwatch Battery Replacement Procedure	ACTIWATCH-SCORE	C-8
Actiwatch Cleaning Procedure	Actiwatch Battery Replacement Procedure	C-13
SPECIFICATION	Actiwatch Cleaning Procedure	C-17
FMC INFORMATION E-1	SPECIFICATION	D-1
	EMC INFORMATION	F_1

х

INTRODUCTION AND SETUP

Actiware / Actiware-CT

Actiware and Actiware-CT are Windows[®] based application software packages that are designed to do the following:

- Manage subject (patient) information
- Configure Actiwatch devices for data collection
- Retrieve clinical data from Actiwatch devices
- View clinical data
- Analyze data for sleep parameters and rhythmicity
- Export raw data or statistics for external analysis or archive

Actiware-CT is also capable of clinical trial tracking and auditing in accordance with 21 CFR Part 11.

This manual describes the functions of the Actiwatch and outlines the data analysis and file management features available in the Actiware and Actiware-CT software applications. Both applications are part of the Actiwatch activity monitoring system. You can view and analyze the rhythmicity of long-term activity-rest patterns.

Intended Use

Actiware / Actiware-CT is for use only in a clinical, research, or home care provider setting.

Actiwatch-16/64

The Actiwatch[®]-16 or -64 is an ultra-compact, lightweight, wrist-worn activity monitor that can be used to analyze circadian rhythms, automatically collect and score data for sleep parameters, and assess activity in any instance where quantifiable analysis of physical motion is desirable. An Event Marker button is available that allows the subject (patient) to enter time-stamped information and events of significance.

Actiwatch-L

The Actiwatch[®]-L is an ultra-compact, lightweight, wrist-worn activity and illuminance monitor that can be used to analyze circadian rhythms, automatically collect and score data for sleep parameters, and assess activity in any instance where quantifiable analysis of physical motion is desirable. The Actiwatch-L does not have an Event Marker button.

Actiwatch-Score

The Actiwatch[®]-Score is a compact, lightweight, wrist-worn activity monitor that can be used to analyze circadian rhythms, automatically collect and analyze data for sleep parameters, and assess activity in any instance where quantifiable analysis of physical motion is desirable. In addition, the Actiwatch-Score has a built-in score pad that allows the subject to subjectively assign and enter a score from 0 to 15. The score pad can be used as a substitute or in addition to the traditional patient diary used in conjunction with activity monitoring.

This system is designed to conform to the following standards:

- IEC 60601-1 Medical Electrical Equipment Part 1: General Requirements for Safety
- IEC 60601-1-1 Medical Electrical Equipment Part 1-1: General Requirements for Safety - Collateral Standard: Safety Requirements for Medical Electrical Systems for Safety
- IEC 60601-1-2 Medical Electrical Equipment Part 1-2: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests
- IEC 60950-1 Information Technology Equipment Part 1: General Requirements for Safety

Important! Actiwatch 16/64, -L, -Score are not intended for use as diagnostic devices; they are intended for use as physiological parameter recorders.

CAUTION! US federal law restricts this device to sale by or on the order of a physician.

WARNING! WARNING:

A warning indicates the possibility of injury to the user or operator.

- The provider should read and understand this entire manual before using this device.
- This manual serves as a reference. The instructions in this manual are not intended to supersede the health care professional's instructions regarding the use of this device.
- Do not dispose of lithium batteries in fire or flame. An explosion may result. Only dispose in accordance with manufacturer's recommendation or local regulations.
 - Operation of the device may be adversely affected by:
 - Electromagnetic fields exceeding the level of 10 V/m in the test conditions of EN 60601-1-2
 - Operation of high frequency (diathermy) equipment
 - Defibrillators or short wave therapy equipment
 - Radiation (e.g., x-ray, CT)
 - Magnetic fields (e.g., MRI)
 - Do not use this device in the presence of a flammable anaesthetic mixture in combination with oxygen or air, or in the presence of nitrous oxide.
 - If you notice any unexplained changes in the performance of this device, if it is making unusual or harsh sounds, if it has been dropped or mishandled, or if the enclosure is broken, discontinue use. Contact Mini Mitter/ Respironics Technical Support and replace any damaged parts before continuing use.
 - ActiReader Only To avoid electrical shock, disconnect the AC power supply before cleaning. DO NOT immerse the AC power supply in any fluids.

CAUTION!

CAUTION:

A caution indicates the possibility of damage to the device.

- Repairs and adjustments must be performed by Mini Mitter/Respironics authorized service personnel only. Unauthorized service could cause injury, invalidate the warranty, or result in costly damage.
- Dispose of these devices in accordance with local regulations.

Note

Places emphasis on an operating characteristic. *Note: Additional Warnings, Cautions, and Notes may appear throughout this manual.*

Symbol	Meaning
\boxed{i}	Attention, consult accompaning documents
	DC Power
$\mathbf{\dot{\pi}}$	Type BF Applied Part
	Canadian/US Safety Certification
IPX2	Protected against vertically falling water drops when enclosure tilts up to 15 degrees.
IPX7	Protected against the effects of temporary immersion in water (Actiwatch 64, Actiwatch 16, Actiwatch-Light)
-(*)	Alignment Indicator (ActiReader)
CE	European Declaration of Conformity (conformance to Radio and Telecommunications Terminal Equipment (RTTE) Directive)
CE 0123	European Declaration of Conformity (conformance to Medical Device Directive (MDD))
	On (Power)
\bigcirc	Off (Power)
9V] Lithium only	Use 9V Lithium battery only

System Requirements	 Microsoft Windows[®] 2000 or XP operating system Windows XP Professional required for Actiware-CT
·	 Computer with 2 GHz or higher process clock speed recommended; 500 MHz minimum requirement
	• 512 MB RAM; 128 MB RAM minimum requirement
	• 300 MB or higher recommended; 50 MB required for installation
	• Super VGA (1280 x 1024) resolution video adapter and display recommended; SVGA (800 x 600) minimum requirement
	• CD-ROM or DVD drive (for installation)
	Keyboard and Microsoft Mouse or compatible pointing device
	• 9-pin or 25-pin RS-232 communications serial port
	• UL 1950, IEC 60950, or EN60950 approved computer
	• Printer (optional)
	• Note: Data for use by Actiware must reside on the local hard drive. It is not recommended to install the program files or data on a network drive.

Note: Actiware / Actiware-CT was created for use with Windows[®]*. Windows is a registered trademark of the Microsoft*[®] *Corporation.*

Note: Before beginning the installation procedure, make sure that no other applications are currently running on the computer. This includes Microsoft Office[®] and any other utilities. These can interfere with proper installation, resulting in software conflicts.

First Time If this is the first time Actiware will be installed, please continue to *Installing Actiware* later in this section.

Upgrading If you are upgrading from a previous version of Actiware (or Sleepwatch or Rhythmwatch) you should remove any previously installed versions.

See Uninstallation Procedures later in this section.

CAUTION! Before proceeding, back up any previously gathered data prior to installation. Store in a secure location, either on a floppy disk, CD-ROM, or within a separate drive space.

Installing Actiware / Actiware-CT

Note: You must be a member of the Administrator's group in order to install this software.

Note: Users of Actiware (Research Version) must be a member of the Power User or Administrator's group in order to run the software. Actiware-CT (version 5.1 or greater) users do not need to be a member of the Power User or Administrative group.

Installing from a CD

- Insert the CD-ROM into the CD/DVD drive and wait a moment for the Installation Menu to appear. If the CD/DVD drive does not accommodate the autorun function, then select Start > My Computer and select the CD/DVD drive from the list and double-click to access the files. Select CDMENU.EXE and double-click.
- 2. Select Install Actiware Version 5 from the menu.

Installation Menu provides several options: Install Actiware, Install Adobe Reader, View Manual, Contact Mini Mitter.



- 3. The InstallShield[™] Wizard will guide you through the installation process.
- 4. Check the radio button for I accept the terms in the license agreement. Click the Next button to continue.

End-User License Agreement

🙀 Actiware Version 5	×
License Agreement Please read the following license agreement carefully.	4
MINI MITTER COMPANY, INC.	^
SOFTWARE LICENSE AGREEMENT	
NOTICE TO USER: PLEASE READ THIS CONTRACT CAREFULLY. BY USING ALL OR ANY PORTION OF THE SOFTWARE ("SOFTWARE") YOU ACCEPT ALL THE TERMS AND CONDITIONS OF THIS AGREEMENT, INCLUDING, IN PARTICULAR THE LIMITATIONS ON: USE CONTAINED IN SECTION 2; TRANSFERABILITY IN SECTION 4; WARRANTY IN SECTION 6; AND LIABILITY IN SECTION 7. YOU AGREE THAT THIS AGREEMENT IS ENFORCEABLE LIKE ANY WRITTEN NEGOTIATED AGREEMENT SIGNED BY YOUL IF YOU DO NOT I accept the terms in the license agreement I do not accept the terms in the license agreement InstallShield	
< Back Next > Cancel	

Important! The Software License Agreement **must** be accepted before installation will occur.

Note: Actiware-CT requires the user to authenticate using their Windows or Network username and password. Please contact your local administrator with any issues or questions.

Registering Actiware

Both versions of Actiware have a "try it before you buy it" feature. You may download or otherwise obtain an evaluation copy of Actiware and use the software prior to purchasing it. An activation key cannot be obtained without first paying for a license; however this does not prevent you from installing and running a full featured copy of Actiware for 30 days or 50 uses. Even if you have purchased Actiware, it is not necessary to immediately register Actiware. Please press the **Continue Unregistered** button to access Actiware software and contact technical support via e-mail (mm@respironics.com) at a convenient time to obtain your activation key.

NOTE: Registration is not needed for Actiware-CT 5.1 or greater.

Actiwa	are Version 5.01.0005 🛛 🔀			
	Actiware [®]			
	Actiwatch Communication and Sleep Analysis Software			
This or af	unregistered version of Actiware will expire in 31 days, ter 49 uses.			
If you have purchased Actiware and have your activation key, press the "Enter Activation Key" button to register this software.				
If you h http://d	have purchased Actiware, but not yet obtained your activation key, visit: www.minimitter.com/Products/Actiwatch/ActiwareReg.html			
For info http://d	ormation on how to purchase Actiware, visit: www.minimitter.com/Products/Actiwatch/BuyActiware.html			
Or you Mini Mit Phone: Email:	can contact us by phone or email: tter, a Respironics Company 1-800-685-2999 or (541) 598-3800 mm@respironics.com			
	Enter Activation Key			

When you purchase Actiware, a *Proof of Purchase* certificate will be shipped with the installation CD. There will be a Proof of Purchase code printed on the certificate. This code will always begin with "POP-xxxx-xxxx..." and cannot be entered into the software and applied as an activation key. To obtain an activation key, you will need to contact Mini Mitter/Respironics and give the Proof of Purchase code to a customer service associate. The best and most efficient method to contact technical support is via e-mail (mm@respironics.com). You may also call technical support at 800.685.2999 during normal business hours of 8 am to 5 pm PST. The associate will then generate an activation key and e-mail it to you. Once the activation key is applied to Actiware, it will never expire and will be valid for all future versions of Actiware 5.x. The key however, will not work in a future version 6.0 or greater.

Note: To deter the misuse of activation keys, the key can only be applied for 7 days after it is generated.

There are two parts to an activation key: the e-mail address of the registered user and the key itself. A key is very long and will look something like the following:

sleepdoctor@sleeplab.com 000016-BEMQ97-ZKNNYC-Z19ZG9-G11W7U-0TWCZT-U8W8C4-Z0WRG4-ZBJXKT-NC20CA

When you receive your activation key, cut and paste the *Registered User's* e-mail and key as shown and press **Enter Key**.

Enter the Activation Key

Enter Activation Key	×
Enter the Registration Name and Activation Key below, exactly as given to you.	
Registration Name:	
sleepdoctor@sleeplab.com	
Activation Key: (press Ctrl-V to paste key in from email)	
000016-BEMQ97-ZKNNYC-Z19ZG9-G11W7U-0TWCZT-U8W8C4-Z0WRG4-ZBJXKT-NC20CA	
Current Key:	
Development	
For assistance, click: http://www.minimitter.com/Products/Actiwatch/ActiwareHelp.html	
Enter Key	Cancel

Registration entitles the registered user access to technical support. When contacting technical support, you may be required to provide your registration information.

CAUTION! Before uninstalling, back up any previously gathered data and store in a secure location, either on a floppy disk, CD-ROM, or within a separate drive space.

Automatically Uninstalling Previous Actiware Versions

- 1. Select Start > Control Panel > Add or Remove Programs.
- 2. Review the list for any Actiware application currently installed (Actiware-Rhythm, Actiware-Sleep, RhythmWatch, SleepWatch). Select the Actiware version from the list. If the Actiware software is not in the list, a manual uninstallation will be required.
- 3. Click the **Remove** button.
- 4. When the confirmation pop-up window appears, select **Yes** to remove Actiware.

This completes the automatic uninstall process. Delete any desktop or toolbar icon shortcuts you may have created.

Manually Uninstalling RhythmWatch 2.82 or earlier

- 1. Using Windows Explorer, review the contents of the RhythmWatch installation folder (commonly C:\RHYTHMW). Copy any .AWD data files to a secure location such as a floppy disk or separate drive space.
- 2. Delete the installation folder (C:\RHYTHMW) and its contents.
- Remove the RhythmWatch program folder from the Start menu. This can be accomplished using the Start > Settings > Taskbar and Start Menu item, or by right-clicking on the Start button and selecting Open. Also, delete any desktop icon shortcuts you may have created.

This completes the uninstall process for RhythmWatch 2.82 or earlier.

Manually Uninstalling SleepWatch 2.82 or earlier

- 1. Using Windows Explorer, review the contents of the SleepWatch installation folder (commonly C:\SLEEPW). Copy any .AWD and .AWS data files to a secure location such as a floppy disk or separate drive space.
- 2. Delete the installation folder (C:\SLEEPW) and its contents.
- 3. Remove the SleepWatch program folder from the Start menu. This can be accomplished using the **Start** > **Settings** > **Taskbar** and **Start Menu** item, or by right-clicking on the **Start** button and selecting **Open**. Also, delete any desktop icon shortcuts you have created.

This completes the uninstall process for SleepWatch 2.82 or earlier.

Manually Uninstalling Actiware-Rhythm or Actiware-Sleep 3.0

- 1. Using Windows Explorer, review the contents of the Actiware installation folder (commonly C:\ACTIWARE for version 3.0). Copy any .AWD and .AWS data files to a secure location such as a floppy disk or separate drive space.
- 2. Delete the installation folder (C:\ACTIWARE) and its contents.
- 3. Remove the Actiware-Sleep or Actiware-Rhythm program folder from the Start menu. This can be accomplished using the **Start** > **Settings** > **Taskbar** and **Start Menu** item, or by right-clicking on the **Start** button and selecting **Open**. In addition, delete any desktop icon shortcuts you have created.

This completes the uninstall process for Actiware 3.0.

Manually Uninstalling Actiware-Rhythm or Actiware-Sleep 3.1 or later

- 1. Using Windows Explorer, review the contents of the Actiware installation folder (commonly C:\Program Files\Actiware). Copy any .AWD, .AWS and .SWS data files to a secure location such as a floppy disk or separate drive space.
- Remove the installed version of Actiware using the Add/Remove Programs icon in Control Panel. An item labeled Actiware – Sleep or Actiware – Rhythm should be listed here. Select it and click the Add/Remove button.
- **3**. Follow the online instructions. Once the uninstall program has completed, all program files and shortcuts will have been removed. Also, delete any desktop icon shortcuts you created.

This completes the uninstall process for Actiware 3.1 or later.

Manually Uninstalling Actiware-Rhythm or Actiware-Sleep 4.0 or later

- 1. Using Windows Explorer, review the contents of the Actiware installation folder (commonly C:\Program Files\Actiware). Copy any .AWD, .AWS and .SWS data files to a secure location such as a floppy disk or separate drive space.
- 2. Remove the installed version of Actiware using the Add/Remove Programs icon in Control Panel. An item labeled Actiware Sleep or Actiware Rhythm should be listed here. Select it and click the Add/Remove button.
- **3**. Follow the online instructions. Once the uninstall program has completed, all program files and shortcuts will have been removed. Also, delete any desktop icon shortcuts you created.

This completes the uninstall process for Actiware 4.0 or later.

Automatically Uninstalling Actiware / Actiware-CT 5.0 or later

- 1. Select Start > Control Panel > Add or Remove Programs.
- 2. Select Actiware Version 5 from the list. Click the **Remove** button.
- **3**. When the confirmation pop-up window appears, select **Yes** to remove Actiware.

This completes the uninstall process for Actiware 5.0 or later. Also, delete any desktop icon shortcuts you created.

Connecting ActiReader

ActiReader or an Actiwatch Reader is necessary to communicate with an Actiwatch. Connection to a desktop computer or a laptop computer is very similar to any other peripheral device. Upon starting Actiware, you will be prompted to set up the COM port to recognize the ActiReader.

Note: Follow the recommendations supplied with your computer when connecting peripheral devices. Most manufacturers suggest powering down the computer while connecting hardware.

Computer with ActiReader



Note: If you have an earlier design of Actiwatch Reader, refer to Actiwatch Reader *later in this manual.*

ActiReader

- 1. Connect the serial communication cable (supplied) to a COM port on your computer. Actiware's Select COM Port Wizard will assist in determining and setting up the appropriate COM port. The COM port can be changed at anytime.
- 2. Connect the other end of the communication cable to ActiReader. ActiReader power must be **ON** and the red LED Power light should be blinking. If the Power light is not blinking, the battery must be replaced (See *Replacing the ActiReader Battery* later in this manual) or the external power supply must be used.
 - External Power Source

ActiReader can be powered from an external power source (supplied). When plugged in, it will disconnect the battery to conserve it.

• Power ON/OFF switch

This switch will disconnect ActiReader from its power source, whether AC or battery.

• POWER (Red LED)

Whether using external power or the internal battery, the red LED will light when the Power switch is **ON**.

ActiReader Features



• ALIGNED (Green LED)

When the Actiwatch is aligned properly and the communication wizard is active, this green LED will light.

• Communication Pad

This slot ensures Actiwatch is placed properly. On the back of the Actiwatch is a green dot in one corner. When the dot is aligned with the green dot on ActiReader, the green LED on the ActiReader will light. This indicates that communication between the two devices is possible.

• Serial Port

This is where the serial cable connects to ActiReader. The other end connects to the COM port on the computer.

Note: The "back" of the Actiwatch is metal. The "front" or "face" of the device is plastic.

Note: The 'ALIGNED' green LED will not light until appropriate portions of the Communications Wizard are in use when the COM port is on (see page 3-1, later in this manual).

Selecting the COM Port

- 1. Open the Actiware software program.
- 2. From the menu toolbar, select **Communications** > **Select COM Port**.

This will start the COM Port Wizard which will assist you in searching for an available serial communication port on the computer (See illustration on next page).

Start the COM Port Wizard



COM Port Wizard Welcome window

Select COM Port V	Nizard - Welcome!	×
Ports Ports COM1: COM2: COM3:	This wizard will search the computer's hardware for an available serial communications COM port, and will test the communications after a selection is made. To test the connection, you must have an Actiwatch and Reader available. This wizard only needs to be run ONCE. Once it has been completed successfully, the results will be saved. Hence, other users of this computer will not need to repeat this wizard. CAUTION: Do not use an Actiwatch that is currently being used to collect data or has valid data in it. Data in the device may be lost or corrupted when testing the serial connection. In addition, the Actiwatch cannot collect data or keep accurate time while on the Reader. The currently selected serial communications port is None	
Help	Cancel < Back <u>N</u> ext > <u>F</u> inish	

3. Click the **Next** button and the Wizard will search for available COM ports When the search is completed, use your mouse to highlight the COM port to which you connected the ActiReader.

Select COM Port Wizard - Automatic Search Results	X
Please select a COM port from the following list:	
COM1	
If a COM port is missing from the list, verify that it is (1) enabled in the computer's hardware setup, and is (2) not in use by another program, such as a Palm or Pocket PC synchronization manager. Click 'Back' to retry the search.	
Click the 'Next >' button to test the COM port, or click 'Skip Test' to finish.	
Help Cancel < Back <u>N</u> ext > <u>S</u> kip Tes	t

Select the COM port from the list provided by the Automatic Search Results

4. Click Next to proceed to the Communications Port Test.

Note: The ActiReader must be functioning correctly and be connected to the computer to complete the COM port selection. The Actiwatch must be metal side up (Serial Number up) to successfully communicate with the ActiReader.

5. Verify that ActiReader is connected correctly using the 3-point checklist. If correct, select the **Next** button. You can also use the **Skip Test** button, but the connection between the computer and ActiReader will not be tested.

If using a white Actiwatch Reader, click the appropriate button in the wizard or see *Actiwatch Reader* later in this section.

3-point check list and Actiwatch Reader differences are embedded in the COM Port Wizard

Select COM Port Wiza	rd - Preparing to Test				
ActiReader ActiReader VEXTER VEXTER ALIONED ALIONED ALIONED MINI MITTER CO., INC. MINI MITTER CO., INC.	 To test the COM port selection, please do the following: 1 Using the included serial cable, connect the Reader to the serial port connector that corresponds to the COM port you selected. 2 If using a black ActiReader, be sure the switch on the back is flipped to 'ON' and the red POWER LED is blinking. (You may need to replace the Reader battery or connect to an external power supply if the red LED will not blink.) 3 Place an Actiwatch on the Reader as shown by the picture on the left, so that: the METAL side is UP. the DOT is in the UPPER LEFT. the GREEN LED is LIT. 				
Click here if you have a white Actiwatch Reader	If the green LED won't light, verify that the Reader is connected to the correct serial port connector. Click '< Back' to select a different COM port. It may also be necessary to replace the batteries in the Actiwatch or Reader. Once the green LED is lit, click the 'Next >' button to proceed.				
Help	Cancel < Back <u>Next ></u> Skip Test				

If the Communications Test fails, a help window will appear to assist with troubleshooting potential causes. Review each possible explanation.

When ActiReader is successfully connected to the computer, the **Finished**! window will appear.

Select COM P	ort Wizard - Select COM Port Completed 🛛 🛛 🔀					
_						
Finishe	Selected Communications Port: COM1					
.411	Communcations Port Testing: Passed					
COM Port So has success been comple	The COM port selection will be saved in the Windows registry for all users. Hence, other users of this computer will not need to repeat this wizard.					
	If you need to change the COM port selection (such as, if your USB to serial adapter changes port numbers, or you connect to a different 9-pin serial connector) use this wizard, accessible from the 'Communications' menu, or adjust the setting using the Tools > Options menu item.					
Click 'Finish' to save the COM port selection, or click 'Cancel' to exit.						
Help	Cancel < Back <u>N</u> ext > <u>Finish</u>					

6. Select **Finish** to exit the wizard.

Upon completing the COM Port Wizard, you can configure an Actiwatch for subject use or retrieve collected data (see *Communications* in Section 3 of this manual).

Actiwatch Reader and ActiReader are very similar in their operation. Installation and setup are the same.

However, please keep the following in mind when using the Actiwatch Reader:

- The Actiwatch Reader is white. The ActiReader is black.
- Instead of a slot for aligning the Actiwatch, place the Actiwatch over the image and slide it up and down until the Ready LED is lit.
- The small dot on Actiwatch must be in the same corner as the small dot on the reader.
- Researchers with laptop computers may switch the Actiwatch Reader to Battery Power for proper function.
- If using battery power, you may need to replace the reader battery periodically (see *Replacing ActiReader Battery* later in this section).
- As with the ActiReader, the COM port selection in the software must be correct, and the communications wizard must be running, for the Ready LED to be lit.

Actiwatch Reader top panel with orientation dot



Actiwatch Reader



CAUTION! The black ActiReader model uses a 9V Lithium battery. The white Actiwatch Reader uses a standard 9V battery. These batteries are not interchangeable, and may result in damage to your device. Use of the improper battery will void the warranty.

CAUTION! It is important that you thoroughly read the following information prior to changing the battery. Failure to follow procedures may result in immediate or subsequent damage to the device.

Battery Replacement

- 1. Disconnect the AC electrical supply and set the Power toggle switch to OFF.
- 2. Turn the ActiReader over so the battery cover is facing up.
- 3. Slide the battery cover in the direction of the arrow.
- 4. Carefully remove the battery from the housing unit and gently disconnect the battery terminals.
- 5. Replace the battery.
- 6. Slide the battery cover back into place on the ActiReader.
- 7. Set the Power toggle switch to the **ON** position.
- 8. Verify the red LED Power light is blinking.

WARNING! Do not dispose of lithium batteries in fire or flame. An explosion may result. Only dispose in accordance with manufacturer's recommendation or local codes.

1-22 Introduction and Setup

GETTING STARTED

WARNING! Actiware / Actiware-CT is for use only in a clinical, research, or home care provider setting.

The workspace of the Actiware program is similar to many software applications. Dockable windows give the user direct access to the Actiwatch or subject data stored in the selected database. They also provide control over analysis viewing and exports.

The Database Viewer and the Properties Viewer are docked at the left side of the main Actiware window by default. The Actogram View toolbar is shown on the right side of the main Actiware window.

Actiware databases are organized in a hierarchical fashion. The top level is the subject level. The second level is the Actiwatch data for that subject. The third level is for saved analyses of the Actiwatch data. You can navigate through this hierarchy by clicking on the + (plus sign) or – (minus sign) buttons to the left of the subject or Actiwatch data icons.

- Subjects are sorted by subject ID.
- Actiwatch Data entries are sorted by data collection start time.
- Analyses are sorted by saved name.

Main Window

Actiware workspace, full-window



The main Actiware window can be broken into three main sections:

- Toolbars
- Database Viewer
- Properties viewer

Toolbars

File	View	Сору	Communications	Analysis	Interval	Tools	Window	Help		
	- 😤	🕆 📥	🖻 🏂 🄃	🗈 🛳		\sim	i 🔁 🔁		i 🖉 🔁	🔛 🗈

The Menu Bar is similar to most software applications. Keyboard shortcuts, actuated by pressing the **ALT** key plus the hotkey letter, (identified by an underscore) can be used to navigate through the toolbar menus.

The Standard toolbar, just below the Menu Bar, includes shortcuts to associated functions within the Actiware software program. Hover your mouse over the icon for a short text description.

Standard Toolbar Icons

*	Create A New Subject	8	Edit Subject, Data, or Analysis
•	Configure My Actiwatch) ,Q	Show Audit Trail
•	Retrieve Data from My Actiwatch		Open the Database Viewer
7	Import an .awd File	B	Open the Properties Viewer

The following four icons determine how the data are shown in the **Analysis** window.

4 11	Display Data as an Actogram		Display the Data List
	Display Statistics within an Interval	Ş	Display Data as a Graph

4	Print an Actiware Report	Ľ	Create a New Analysis File
•	Export a Data File	(J	Open an Analysis File
Ē	Copy Export Text to the Clipboard		Save an Analysis File

The **Graph** toolbar provides access to functions that are available only when viewing a graph.

5	Print a Graph	Ē	Copy Graph to the Clipboard
	Edit the Graph Display Properties		
The Actogram View toolbar is used for adjusting the viewing properties of an Actogram, and is available only when an Actogram is present. Changes made to this toolbar are immediately visible in the currently selected Actogram.

Actogram View Toolbar		
Days Displayed		
v		
Visibility		
# #		
Activity Scale		
_		
Illuminance Scale		
_		
Light/Dark Bar		
🐯 🔲 🗖 All Light		
🖉 🗖 All Dark		
ON 📃		
OFF		
Interval Legend		
Rest (R)		
Custom (C)		
Evoluded (E)		
Forced Wake (W)		
Forced Sleep (S)		
To set an interval:		
Actogram to place		
epoch label.		
 Right-click or use keyboard controls 		
to set intervals.		

- Days Displayed: controls how many days are displayed in the visible part of the Actogram window. Use the scroll bar on the right of the Actogram to see the remaining days. Pull down or type directly into this control to set the number of days.
- Visibility: these buttons hide and show the data items in the Actogram:

Days Displayed:	Auto	
Component Visibility:	Activity Data	
	🔝 Illuminance Data	
	Score Data	
	Sleep/Wake Scoring	g
	🛱 Grid	
Activity Scale:	Auto 💌	
Illuminance Scale:	Auto 💌	

- Activity Scale: adjusts the maximum value of activity counts displayed on the Actogram. Pull down or type directly into this control to set the activity scale.
- Illuminance Scale: adjusts the maximum illuminance value displayed on the Actogram. Pull down or type directly into this control to set the scale.
- Light/Dark Bar: use these controls to show or hide the bar at the top of the Actogram, to set the light on/light off time, or to set the light to all on (24 hours of light) or all off (24 hours of dark).
- Interval Legend: provides a convenient reminder of the color scheme used for displayed intervals. Click the keyboard controls link to display quick reminders on how to set intervals. See Setting Intervals later in this section.

To save your current Actogram View settings as default settings, click View > Save Current Actogram View as My Default View. Or, to restore your default view settings, click View > Restore My Default Actogram View.

Database Viewer

The Database Viewer is used to provide a hierarchical tree view of the data in your currently selected database. Below is an example view from the Demonstration Database that is provided with Actiware.

Database Viewer



Subjects, Actiwatch data, and Analyses are shown as indicated below:

Level		Content
1st Level (Subject) 2nd Level (Actiwatch Data)	Demo Database.AW5 564738291 4/9/1999, 8:45:00 AM A New Analysis Custom Intervals Forced Wake Dest & Surkided Jatamala	Subject Identity (Sorted Alphabetically or Numerically) Local Start Time (Sorted by Date)
(Analysis)	Analysis Name (Sorted Alphabetically)	

Commonly used functions can be accessed by right-clicking on an item in the Database Viewer and using the popup menu.

Actiware automatically loads the last used database or when run for the first time, the Demonstration Database. To create a new database, click **File** > **Database** > **New** and use the Windows file selection to select a filename and location. Other database functions (such as Open, Save As, Backup, and Export) can be accessed from the **File** > **Database** menu.

The Database Viewer can be undocked or docked by clicking and dragging the window title bar. In addition, it can be resized or closed. To show again, use the **View** > **Database Viewer** menu.

Properties Viewer

The Properties Viewer displays information for the database item that is currently selected in the Database Viewer; hence the Database Viewer and Properties Viewer are linked.

Click the + or – next to the Subject, Actiwatch Data, or Analysis to expand or collapse the properties related to that database item. Left-click on an item to see a description of the property in the notes area at the bottom of the window. For instance, the properties related to the *Rest & Excluded Intervals* analysis are desplayed. Note that the Subject and Actiwatch data properties are also displayed.

Properties					
	Jantitu	564720201			
	Full Name	Mrs. Janet C Hanson			
	Gondor	Fomale			
	Dista of Pirth	0/0/1000			
		6767 1333			
	Age (current)				
	Age (start of data collection)	55 years			
	Street Address	3674 w 123id St New Yesh			
	Clip				
	State Deskal Carda	11140			
	Postal Lode	11143			
╘		USA			
╚	Actiwatch Data	A-5-10-10-10-10-1			
	Actiwatch Type	Actiwatch 16/64			
	Data Collection Start Time	4/9/1999, 8:45:00 AM			
	Data Collection End Time	4/14/1999, 5:09:00 PM			
	lime∠one	(GMT-08:00) Pacific Time (US & Canada); Tijuana			
Time Zone Offset -07:00 hours:minutes Epoch Length 1 minute		-07:00 hours:minutes			
		1 minute			
	Number of Data Samples	7704 samples			
	Number of days	5.35 days			
	Battery Installed Date	Not Available			
	Actiwatch Serial Number	V631234			
	Actiwatch Firmware Version	63			
	Activity Calibration Factor	Not Available			
	Illuminance Calibration Factor	Not Available			
	Analysis				
	Analysis Name	New Analysis			
	Wake Threshold Selection	Medium			
	Wake Threshold Value	40.0 activity counts			
	Sleep Interval Detection Algorithm	By minutes scored as immobile			
	Immobile Minutes for Sleep Onset	10 minutes			
	Immobile Minutes for Sleep End	10 minutes			
	Epochs for Sleep Onset	5 epochs			
	Epochs for Sleep End	5 epochs			
	Actogram Start Hour	12:00 PM			
	Illuminance Threshold	1000.0 lux			
	Number of Intervals	0 intervals			

Items in the Properties Viewer can be edited by double-clicking the item.

The Properties Viewer can be undocked or docked by clicking and dragging the window title bar. In addition, it can be resized or closed. To show again, use the **View** > **Properties Viewer** menu.

An Actogram is a graphic view of the rest/activity history of a subject. Data are displayed in stackable 24-hour periods which allow the user to immediately determine sleep/wake patterns.

The Actogram provides the ability to select periods of time, known as intervals, and to provide statistics for these times of interest. (See Setting Intervals later in this section.)



There are multiple ways to open an Actogram. The most direct way is:

- 1. In the Database Viewer, click the + sign next to a Subject to expand.
- 2. Click the + sign next to the Actiwatch Data record to expand.
- 3. Double-click on New Analysis. An Actogram will be displayed.

This process can be repeated for any analysis (such as Custom Intervals) shown in the Database Viewer.

You can also open an Actogram by clicking the **View** > **Actogram** menu, or by clicking the icon on the Standard toolbar at the top of the window.

Use the Actogram View toolbar (displayed on the right side of the window) to adjust the view settings for the Actogram.

Note: The sleep/wake scoring algorithm is applied automatically to the activity in the entire data set. Epochs scored as wake are indicated by a red mark below the X axis, whereas epochs scored as sleep have no mark below the X axis.

Setting Intervals

 Start UNKNOWN: 10:57:00 PM

 AY 1

 4/8/1999

Adding an Interval Selection to an Actogram

An interval is a user-defined selection of time that allows you to graphically select and analyze data for statistical calculations. For calculation of statistics for specific periods of time, it is necessary to select intervals and assign an interval type to them. The Interval feature in Actiware provides a robust data selection, exclusion and quantitative tool. The interval types that are available:

- Rest user defined
- Sleep automatically determined from Rest intervals
- Active automatically determined from Rest intervals
- Excluded user defined
- Forced Wake Score user defined
- Forced Sleep Score user defined

Rest intervals

These are intervals of data that contain periods of time when the subject activity is low and the subject is likely to be at rest. They must be set by the user and are indicated on the Actogram by light blue shading. Typically this will be used for the In Bed Period. By setting a Rest Interval you are directing the program to apply the Sleep Interval Detection algorithm for generating Sleep Periods.

This algorithm will use the first data point in the rest interval as the Bed Time and the last data point as the Get up Time. From these points, the Sleep Onset and Sleep End will be set using the Analysis parameter values that are indicated on the Analysis tab of the **Tools** > **Options** window. Once you have set a Rest Interval, a Sleep Interval will automatically be created within it.

Sleep Intervals

Data in these intervals represent periods of time in which the subject is likely to be asleep. These intervals are created automatically by the program once a Rest Interval has been set. They represent the period of time between Sleep Onset and Sleep End but are not indicated by any shading.

Active Intervals

Data in these intervals represent periods of time in which subject activity indicates that they are alert and engaged in physical activity. These intervals contain all those data that are outside of Rest Intervals. Active Intervals are created automatically when Rest Intervals are set. No shading is used to indicate these intervals.

Excluded Intervals

Data in these intervals are excluded from all analytical calculations. These intervals are designed for use when subjects remove the Actiwatch or for other invalid periods of data. These must be set by the user and are indicated by dark blue shading.

Custom Intervals

These are intervals of data that are of some interest for any reason. Typically these are used for illuminance calculations when using an Actiwatch-L or numerical rating scale values when using the Actiwatch Score. They may also be used for assessing activity during specific periods of the day. They are indicated on the Actogram by a blue bar above the activity data, and must be set by the user.

Forced Wake

These intervals are periods of time for which the Sleep/Wake score has been forced by the user to Wake. This is typically used if the subject is known to be awake, but very inactive, or as an alternative to excluding data. Epochs included in this type of interval are indicated by a magenta mark below the X axis of the Actogram.

Forced Sleep

These intervals are periods of time for which the Sleep/Wake score has been forced by the user to Sleep. This typically is used if the subject is known to be sleeping but there is significant movement. Epochs included in this type of interval are indicated by a gray mark below the X axis of the Actogram.

Note: Custom, Excluded, Forced Wake and Forced Sleep intervals can be overlapped. However, due to the automatic creation of Sleep intervals, Rest intervals **cannot** be overlapped. If you attempt to overlap Rest intervals, they will be combined into one interval. There are three ways to set intervals. The quick method uses the mouse and hotkey combinations:

Quick Method

- 1. Hold down the CTRL key on the keyboard.
- 2. Click the *left* mouse button where you want the start of the interval to be.

Interval Start Time

		-
444	- 11	-

3. While still holding down the **CTRL** key, click the *right* mouse button where you want the end of the interval.

Interval End Time

En	NUN VS	IKNO	WN:	3:28	3:00 AN	1
يات	Ĵ.					

If you set an interval flag in the wrong location and want to move it, repeat the above process. Also, by holding down the **CTRL** key and the mouse button, you can drag the flag to the precise location you want.

4. Using one of the following hotkey combinations, set the type of interval you want:

CTRL + R	Insert REST interval
CTRL + C	Insert CUSTOM interval
CTRL + E	Insert EXCLUDED interval
CTRL + W	Insert Forced WAKE interval
CTRL + S	Insert Forced SLEEP interval

The new interval will be automatically drawn in the Actogram display. See the legend on the right side of the window (on the Actogram View toolbar) to see how the intervals are indicated on the Actogram.

Note: During the interval creation process you can press the keyboard *Escape* key to clear the markers.

Fine Control Method

To set your intervals graphically to the nearest epoch, use the following procedure:

- 1. Click the left mouse button on the Actogram to place the epoch label.
- 2. Use the keyboard arrow keys $(\leftarrow, \rightarrow, \uparrow, \downarrow)$ to move the epoch label to the desired interval start time.

Close-up of Interval Markers on Actogram



3. To place the interval start marker, press one of the following keys based on the interval type you want to create:

R	Set REST interval start
С	Set CUSTOM interval start
Е	Set EXCLUDED interval start
W	Set Forced WAKE interval start
S	Set Forced SLEEP interval start

- 4. Once again, click the *left* mouse button on the Actogram to place the epoch label, and use the arrow keys to place the epoch label at the desired interval end time.
- 5. To place the interval end marker, press one of the following hotkey combinations based on the interval type you're creating:

SHIFT + R	Set REST interval end
SHIFT + C	Set CUSTOM interval end
SHIFT + E	Set EXCLUDED interval end
SHIFT + W	Set Forced WAKE interval end
SHIFT + S	Set Forced SLEEP interval end

If you set an interval flag in the wrong location and want to move it, repeat the above process.

6. As with the Quick Method above, use one of the following hotkey combinations to set the type of interval you want:

CTRL + R	Insert REST interval
CTRL + C	Insert CUSTOM interval
CTRL + E	Insert EXCLUDED interval
CTRL + W	Insert Forced WAKE interval
CTRL + S	Insert Forced SLEEP interval

Note: During the interval creation process you can press the keyboard *Escape* key to clear the markers.

Menu and Sleep Diary Methods

If you prefer to type in the date and time values for the interval start and end times, such as if you have a patient sleep diary, use the following procedure:

1. Click Interval > Add Interval.

This will launch the Add Interval window.

- 2. Select the Interval Type
 - Rest
 - Excluded
 - Custom
 - Forced Sleep
 - Forced Wake
- 3. Set the Interval Start Time (date and time).

You can type in the date or use the interactive calendar to select the date. You can type in the time or use the arrows to scroll to the appropriate time

4. Set the Interval End Time (date and time).

Note: If you want to create this interval for all days, check the Repeat for all days check box. This will create an interval every 24 hours starting with the interval start time, with the duration equal to the configured interval.

5. Click the Add Interval(s) button to add the interval, or Close to cancel.

You can repeat steps 2 through 5 for each interval without closing the Add Interval window.

Note: You can also click the right mouse button in the Actogram, or use other sub-menus under the Interval main menu to set interval markers or insert intervals.

Interval Quick Start Help

The Interval Quick Start Help message provides an overview of the interval functions. To display this message, click the keyboard controls link on the bottom of the Actogram View toolbar, or click Help > Quick Start > Interval Selection.

Interval Quick Start pop-up message. To access, Help > Quick Start > Interval Selection

Interval Quick Start Help 🛛 🛛 🔀			
i	Interval Selection Quick Start Help		
Y	An interval is a period of time select for analysis and statist	of special interest that you ics generation.	
	QUICK METHOD		
	1. Set Interval START:	CTRL + Left-click	
	2. Set Interval END:	CTRL + Right-click	
	3. INSERT Interval:	CTRL + R = REST $CTRL + C = CUSTOM$ $CTRL + E = EXCLUDED$ $CTRL + W = Forced WAKE$ $CTRL + S = Forced SLEEP$	
	FINE CONTROL METHOD		
	1. LEFT-CLICK on Actogram t keys to move epoch label t	to place the epoch label. Use arrow to interval start.	
	2. Set Interval START:	R = REST C = CUSTOM E = EXCLUDED W = Forced WAKE S = Forced SLEEP	
	3. Using mouse clicks or arrow the desired location for int	w keys, move the epoch label to erval end.	
	4. Set Interval END:	$\begin{array}{l} \text{SHIFT} + \text{R} = \text{REST} \\ \text{SHIFT} + \text{C} = \text{CUSTOM} \\ \text{SHIFT} + \text{E} = \text{EXCLUDED} \\ \text{SHIFT} + \text{W} = \text{Forced WAKE} \\ \text{SHIFT} + \text{S} = \text{Forced SLEEP} \end{array}$	
	INSERT Interval using the keystrokes from step 3 above under QUICK METHOD, or use Interval menu items.		
	Right-click on Actogram, or use main menus, too.		
	(For more information about each type of interval, click the 'More Help' button below. Click menu 'Help' > 'Quick Start' or the 'keyboard controls' link at the bottom right to view this message box.)		
		More Help	

Removing Intervals

To remove a single interval simply click the left mouse button in the interval you want to delete, then press the Delete key.

To remove all intervals, right click on the Actogram and select Clear All Intervals (also accessible from the Interval menu)

To remove all intervals of a specific interval type, use the Interval menu, or the corresponding keyboard shortcuts (such as CTRL + ALT + C to clear all custom intervals).

The Statistics Table is one of two main ways to display the calculated sleep/wake output. Interval, activity, mobility, sleep, illuminance, and score statistics are automatically calculated for intervals that are set by the user.

1. View > Statistics Table

Click on the tabs at the top of the window to view calculated statistics for different types of intervals. Use the scrolling tools to move through the list of intervals or to view different columns of statistics.

Statistics Table

Rest	i î	Active	Ĭ	Sleep) – j	Custom	- Y	Daily		Summary
	Charl Date	Charle Time	EndDate	End Time	TabalAC	Ave AC Inits	H AC	Muun Ed A.C.	Marine Ext. Churd	Autoba Tima
D 1	Start Date	1.00:00 DM	End Date	Lnd Time	TOTAL AL	AVg AC/min	Max AL	AINVAIID AL	Alfivalid SW	ZO1 00
Dayi	1/12/2001	1:06:00 PM	1/13/2001	12:00:00 PM	416020	302.78	2131	0.00	0.15	781.00
Jay 2	1/13/2001	12:00:00 PM	1/14/2001	12:00:00 PM	326487	226.73	3378	0.00	0.00	631.00
Day 3	1/14/2001	12:00:00 PM	1/15/2001	12:00:00 PM	409422	284.32	3378	0.00	0.00	746.00
Day 4	1/15/2001	12:00:00 PM	1/16/2001	12:00:00 PM	388019	269.46	1659	0.00	0.00	845.00
Day 5	1/16/2001	12:00:00 PM	1/17/2001	12:00:00 PM	360973	250.68	2404	0.00	0.00	912.00
Day 6	1/17/2001	12:00:00 PM	1/18/2001	12:00:00 PM	341992	237.49	1511	0.00	0.00	937.00
Day 7	1/18/2001	12:00:00 PM	1/19/2001	12:00:00 PM	332075	230.61	1820	0.00	0.00	850.00
Day 8	1/19/2001	12:00:00 PM	1/20/2001	12:00:00 PM	262993	182.63	1559	0.00	0.00	708.00
Day 9	1/20/2001	12:00:00 PM	1/21/2001	12:00:00 PM	294980	204.85	2060	0.00	0.00	699.00
Dav 10	1/21/2001	12:00:00 PM	1/22/2001	12:00:00 PM	445690	309.51	1997	0.00	0.00	843.00
Dav 11	1/22/2001	12:00:00 PM	1/23/2001	12:00:00 PM	409948	284.69	1764	0.00	0.00	875.00
Dav 12	1/23/2001	12:00:00 PM	1/24/2001	12:00:00 PM	387716	269.25	2260	0.00	0.00	947.00
Dav 13	1/24/2001	12:00:00 PM	1/25/2001	10:07:00 AM	326195	Maximum Acti	vity (counts)	0.00	0.15	810.00
n	ж	×	×	ж	13	13	13	13	13	13
Average(n)	ж	ж	ж	ж	361731.52	253.75	2142.85	0.00	0.02	814.15
Std Dev(n-1)	ж	ж	×	ж	53183.26	37.58	609.42	0.00	0.06	97.01

Controlling the Content of the Statistics Table

1. Tools > Options

This will launch the Options pop-up window.

2. Statistics

Select the **Statistics** tab (mouse-click) to view the available statistic properties. Check those statistics that you want to include. Actiware includes **Select All**, **Deselect All**, and **Restore Defaults** buttons for ease of use.

3. Click OK to save your selections or Cancel to unaccept.

Your selections will appear in the Statistics Table and be included in exported and printed data.

Tools > Options > Statistics

Options		X
General Analysis Actogram	Statistics Data List Export	Print Report
Interval Information ▼ Start Date ▼ End Date Start Day End Day ▼ Start Time ▼ End Time □ Duration ▼ End Time ▲ctivity Statistics ▼ Max AC ▼ Total AC ▼ Max AC ▼ Avg AC/min Invalid Time AC ▲ Avg AC/epoch ▼ %Invalid AC Std AC Sleep Time ▼ Wake Time ▼ Sleep Time ▼ Wake Bouts ▲ Avg Sleep Bout Avg Wake Bout ♥ Onset Latency Invalid Time SW ▼ Sleep Efficiency ▼ MASO	Mobility Statistics Immobile Time Mobile Time XImmobile Time XMobile #Immobile Bouts #Mobile Bouts Avg Imm Bout Avg Mob Bout #1min Imm Bouts Fragmentation X1min Imm Bouts Fragmentation X1min Imm Bouts #Scores #1Manual Scores #Std Score Avg Score Std Score Avg Manual Std Manual Avg Scheduled Std Scheduled Illuminance Statistics Invalid Time L Std Illuminance XInvalid L Max Illuminance Rester	Select All Deselect All Click any checkbox to include or not include the item in Actiware outputs.
	ОК	Cancel

See Advanced Usage in Section 4 for statistic definitions.

Note: The selections that you make here will impact the content of exported files and printed reports.

Exporting Data

Once you have added intervals and analyzed your data, you can export the retrieved raw data or statistical results to a text file that can be easily loaded into Microsoft Excel, FAST, or a database program of your choice.

There are three types of export files that can be created using Actiware:

Standard Individual Export File

This is a comma delimited ASCII text file (.csv) that can be opened directly by Microsoft Excel. It contains the data and analysis results for one analysis.

To create this file, click **File** > **Export** menu or click the button on the Standard toolbar. You will be prompted for a file name using the standard Windows dialog box.

Note: The contents of the export file can be customized using the *Tools* > *Options* window. Use the Statistics, Data List and Export tabs to select what content you would like in the export file.

Instead of creating a file, you can copy the file contents directly to the Windows clipboard by clicking **Copy** > **Export Text**. Then paste into your program of choice using the Clipboard paste function (commonly CTRL + V keyboard hotkey).

FASTTM Export File¹

This is a tab delimited ASCII text file (.txt) designed specifically for importing into the Fatigue Avoidance Scheduling ToolTM Software. A short header, sleep/ wake scores, and per epoch time stamps for the currently selected analysis are included in this export file.

To create this file, click **File** > **FAST Export**. You will be prompted for a file name using the standard Windows dialog box.

¹ FASTTM is a trademark of NTI, Inc.

Combined Export File

This is a comma delimited ASCII text file (.csv) that can be opened directly by Microsoft Excel. It contains the data and analysis results for intervals from *multiple* analyses. These analyses can be from any number of subjects or Actiwatch data records in the currently opened database.

Use the Text File Export Batch Wizard, accessible from the **Tools** > **Text File Export Batch** menu to create this file (or many Standard Individual Export Files).

See Batch Export in Advanced Usage, Section 4, for details.

Note: Actiware-CT does not allow *Copy* > *Export Text* or *File* > *FAST Export* because these are unsecured outputs that do not contain an electronic or digital signature.

You can generate a printed report of your Actogram and statistical results for the currently selected analysis.

To generate the report, click **File** > **Print Report**, or click the button on the Standard toolbar. You will be presented with a Windows printer dialog where you can select the printer, number of copies, and paper orientation (such as Portrait or Landscape).

Note: The contents of the printed report can be customized using the **Tools** > **Options** window. Use the Statistics and Print Report tabs to select what content you would like for the printed report.

New Subject

Once a database exists it can be populated with subjects and subject collected data. To add or edit a subject, first open the database. Select File > Database > Open, as described on the previous page.

To add a new subject to a database, do the following:

1. Click File > New Subject.

This will open a New Subject window to allow for the capture of demographic data.

2. Fill in the three required fields: Identity, Date of Birth and Gender

For Identity, we suggest using a blind descriptor or nickname of the subject. Notice that Age is automatically computed.

Important! Because you can import subjects from other databases, Actiware allows more than one subject with the same ID. Therefore, you need to be sure you choose a subject ID that does not already exist in the currently open database.

3. If you desire, fill in the remaining fields.

For HIPAA considerations, these fields are optional.

When USA or Canada is entered into the Country field, a dropdown list for the State/Province area will become available. Additionally, the Phone field will be optimized for the (###) ###-##### format. To enter a non-US or non-Canadian state code or phone number, enter the Country first.

For purposes of data tracking, you cannot delete subjects from a database. You can hide them by checking the **Hide Subject** box. Once hidden, a subject will not appear in the Database Viewer. Hidden subjects may be revealed by right clicking in the Database Viewer and selecting **View Hidden Subjects**.

SECTION

S COMMUNICATIONS

The Actiware software is used to configure the Actiwatch device and retrieve data from the Actiwatch through the use of the ActiReader.

To set up the ActiReader, see Installation and Setup in Section 1 of this manual.

Configure Actiwatch

Overview

This wizard will configure an Actiwatch for data collection. This wizard assumes that the ActiReader or Actiwatch Reader is connected to the computer and the appropriate COM port is selected. Please refer to *Section 1* of this manual for setting up the ActiReader and selecting a COM port.

The **Configure Actiwatch Wizard** will guide you through several windows. As with other Actiware wizards, you can make changes at each window. You can also travel between windows using the **Back** and **Next** buttons, or **Cancel** to close the wizard.

Use the **Show/Hide Actiwatch Details** button to view additional information associated with the watch.

CAUTION! The Configure Actiwatch wizard will delete all setup information and data stored in the Actiwatch. To stop this wizard, select the *Cancel* button. To retrieve data, proceed to *Retrieve Data from Actiwatch* later in this section. Configure Actiwatch Wizard Welcome window

Configure Actiwat	ch Wizard - Welcome!	×			
Actificador Actificador Universitador Actificador Acti	This wizard will configure an Actiwatch for data collection. In the following steps, you will be prompted for subject information and device settings, which will be sent to the Actiwatch device via the Reader. This wizard assumes that you have already connected the ActiReader or Actiwatch Reader to the computer, installed the supplied Windows drivers, and run the Select COM Port Wizard to select the correct serial COM port for communication. Image: CAUTION: This wizard will delete all setup information and data in the Actiwatch! If you want to retrieve the data in an Actiwatch before configuring it, exit this wizard by clicking the 'Cancel' button, then access the 'Retrieve Data' wizard from the Communications menu. Click the 'Nextb' button to proceed.)			
Show Actiwatch De	Show Actiwatch Details ->				
Help	Cancel < Back Next > Finish				

Configure Actiwatch

Configure Actiwatch

1. Communications > Configure Actiwatch

This will launch the Configure Actiwatch Wizard.

2. Remove the wrist/foot bands from the Actiwatch (optional).

The bands may interfere with proper Actiwatch placement.

3. Place the Actiwatch on the reader.

The green dot on the metal side of the Actiwatch must align with the green dot on the ActiReader. Verify that the metal side is facing up, the green dots are adjacent and the green LED 'ALIGNED' light is on.

4. Click Next to continue.

The wizard will check the Actiwatch serial number and battery life. If a **Low Battery Warning** appears, please choose one of the three options from the popup window. If the warning does not appear, proceed to step 6.

- Change Battery—wizard will guide you through the battery replacement process (see *Actiwach Battery Replacement* later in this manual).
- Changed Battery, update—wizard will update the database with the new battery information.
- Ignore-wizard will ignore the Low Battery Warning.
- 5. Click Next to continue.

Edit Subject Information

Select subject from the dropdown list or use the 'New Subject' button to add a new subject to the database.

Configure Actiwatch Wizard - Edit Subject Information				
Please select the subject who will be wearing the Actiwatch device:				
Subject Identity: Select Subject From List				
Subject Name:				
Subject DOB: Age: Gender:				
Edit Subject New Subject				
Each Actiware database can contain many subjects. Select a subject identity from the list above, or click 'New Subject' to create a new subject.				
(This information will be stored onboard the Actiwatch. Consequently if the Actiwatch is later downloaded on another computer, the software will be able to associate the physiological data with a subject.)				
Once you have selected a subject, click the 'Next>' button to proceed.				
Show Actiwatch Details ->				
Help Cancel < Back Mext > Finish				

6. Select a current subject from the dropdown list and then click **Edit Subject** to make updates.

When an existing subject is selected from the dropdown menu, the areas below (Subject Name, Subject DOB, Age, and Gender) will pre-populate with the information stored in the database.

Note: The subject must be in the current data file to be accessible from the dropdown menu.

Click New Subject to configure Actiwatch for a new subject.

7. Click **Next** when you have completed editing/adding subject information.

Data Collection Setup

During the Data Collection Setup, you can configure the Actiwatch setup items, including start time, epoch length, and time zone with a simple mouse click.

Expanded window displaying the Actiwatch Details.

Configure Actiwatc	h Wizard - Data Colle	ection Setup		×
Please adjust the f to reflect the locati	ollowing Actiwatch setup it on where the data will be c	ems. Adjust the Start Ti collected:	me and Time Zone	
Start Time:	Friday, May 20, 2005	•	1 :12 PM 📫	
Epoch Length:	1 minute 💌	Maximum Recording Time (approx):	45.4 days	
Actual Recording Time (approx):	45.4 days ?			
Time Zone: (GMT-05:00) Eastern Time (US & Canada)Edit This is your PC's current time zone, to change click 'Edit'.				
NOTE: The Start 1 collected. To ensu date, time, and tim	Time and Time Zone above re accurate Actiwatch timir e zone.	: should reflect WHERE ng, please also verify yo	the data will be our computer's	
<- Hide Actiwatch De	tails			
Actiwatch Type: Serial Number: Firmware Version Memory Size:	Actiwatch 64BaV636630Ba63Active64 KBIllu	attery Installed Date: 4. attery Life (approx.): 11 stivity Calibration: 11 uminance Calibration: N	/6/2005 36 days 32 A	
Help	Cancel	< Back	• Einish	

Epoch length

The period of time Actiwatch will accumulate activity counts before saving the sample to memory. Select the epoch length by clicking in this field. The choices will appear sequentially.

When selecting epoch length:

- If actual recording time is in red, then the battery may need to be changed.
- Time zone selection should be where the data will be collected, not the current time zone of the computer location.
- When configuring the Actiwatch for sleep analysis, use an epoch length of 2 minutes or less, otherwise sleep analysis statistics will not be calculated.

Epoch Length	Number of Epochs per Day	Maximum Delay in Start Time
15 seconds	5760	5 days
30 seconds	2880	11 days
1 minute	1440	22 days
2 minutes	670	49 days
5 minutes	288	114 days
10 minutes	144	228 days
15 minutes	96	342 days

Note: For sleep analysis it is recommended you select an interval of one minute or less. Data with sampling epochs of greater than two minutes cannot be analyzed using Actiware-Sleep analysis.

Recording Time

Displays the time at which the activity monitor will fill its memory with samples based on the epoch length selected.

Memory	0.25 min	0.5 min	1 min	2 min	5 min	10 min	15 min
AW-16K	2.8	5.6	11.2	22.5	56.4	112.8	169.3
AW-64K	11.3	22.7	45.4	90.8	227	454	681
AW-L	3.7	7.5	15.1	30.2	75.7	151	225
AW-Score	5.6	11.2	22.4	44.9	112	224	336

Recording Time Table (for epochs > 0.25 minutes)

NOTE: AW-L devices manufactured before July 1st, 1999 have 32K memory and shorter recording times.

NOTE: Maximum battery life is 180 days for AW16, AW64, and AW-L, and 90 days for AW-Score.

8. Click Next when you have completed editing/adding data collection information.

If you are configuring an Actiwatch-Score, continue below. If configuring an Actiwatch-16/64 or Actiwatch-L, skip ahead to step 10.

Actiwatch-Score

If you are configuring an Actiwatch-Score, a Score Range window will appear and you will now be required to set the minimum and maximum score ranges.

Set the ranges by typing in the numeric value (0-15) or use the scroll arrows.

9. Click Next to continue.

Actiwatch-Score Alarm

Click the **Enable Alarm Scheduling** box and format the Score Alarms with the following actions.

- Add Manual Alarm
- Add Recurring Alarm
- Delete Alarms (select item in list)
- Clear List
- Print List

After establishing the Actiwatch-Score Alarm schedule, you can print a listing.

Actiwatch-Score Alarm window

To enable Score Alarms schedule.	, check the b	ox, and u	ise the buttons belo	w to update the	alarm
Enable Alarm Schedu	Jiing <u>Score Ala</u>	arm Sche	dule		
Add Manual Alarm		Day	Expected Date	Time	^
	1	Day 1	6/9/2005	7:05 PM	
Add Recurring	2				
Alarms	3				
Data Marca 1	4				
Delete Alarm(s)	5				_
	6				_
Clear List	1				_
1	8				
Print List	3				~
2	All times	are local	to the start time zor	ne, not UTC/GM	17
Click the 'Next>' button Hide Actiwatch Details	to proceed.				
Actiwatch Type: Act	iwatch-Score	Bat	tery Installed Date:	6/1/2005	1
Serial Number: D8	20168	Bat	tery Life (approx.):	82 days	1
Firmware Version: 82		Act	ivity Calibration:	132	1
Memory Size: 32	KB	- Illur	ninance Calibration:	NA	1

CAUTION! Do not remove the Actiwatch from the ActiReader during the Configure Actiwatch Wizard process until instructed to do so.

10. Click Next to continue.

The Configure Actiwatch process is in operation. When prompted, remove the Actiwatch.

11. Click Finish to complete the Configure Actiwatch Wizard.

Overview

This wizard will assist you in retrieving the physiological data from an Actiwatch device. When the retrieval process is complete, the resulting data will be written to your selected database for analysis and export.

This wizard assumes that the ActiReader or Actiwatch Reader is connected to the computer and the appropriate COM port is selected. Please refer to *Installation and Setup* in Section 1 of this manual.

CAUTION! The Actiwatch cannot collect data while on the reader. Consequently, do not put the Actiwatch on the reader until you are ready to stop data collection and retrieve the data from the Actiwatch.

Retrieve Data from Actiwatch Wizard Welcome window



Retrieve Data from Actiwatch

Retrieve Data from Actiwatch

1. Communications > Retrieve Data from Actiwatch.

Start the Retrieve Data from Actiwatch Wizard. Click Next to continue.

2. Remove the wrist/foot bands from the Actiwatch (optional).

The bands may interfere with proper Actiwatch placement.

3. Place the Actiwatch on the reader.

The green dot on the metal side of the Actiwatch must align with the green dot on the ActiReader. Verify that the metal side is facing up, the green dots are adjacent and the green LED 'ALIGNED' light is on.

4. Click **Next** to continue.

Actiware will communicate with the Actiwatch device and retrieve the data. If you receive a **Duplicate data, retrieve anyway** message, you have the option to abort the retrieve process by selecting the **No** button, or create a duplicate record by selecting the **Yes** button and continue retrieving the data.

Duplicate data window

Duplicat	Duplicate data, retrieve anyway? 🛛 🛛 🕅					
2	Are you sure you want to retrieve data from this Actiwatch?					
~	Actiwatch data with similar parameters were found in the currently opened Actiware database.					
Data Collection Start Time: 11/11/2005, 9:27:00 AM Time Zone Offset: -05:00 Actiwatch Serial Number: V636630						
	NOTE: because Actiwatch devices cannot keep accurate time while they on the Reader, we recommend that the device be reconfigured after EACH retrieval.					
	If you continue, you will have duplicate Actiwatch Data records in the database and Database Viewer (you can always hide items in the Viewer by editing Actiwatch Data properties).					
	Do you want to retrieve the Actiwatch data anyway?					
	Click 'Yes' to retrieve the Actiwatch data and create a duplicate record. Click 'No' to abort so you can read a different Actiwatch or exit the wizard to select a different database.					
	Yes No					

- 5. When the **Retrieval Successful!** message appears, click the **OK** button.
- 6. Check the Launch Actogram when the wizard finishes box to view the Actogram data immediately.
- 7. Click the **Finish** button to exit the wizard.

Retrieve Data Complete window

R	etrieve Data fro	m Actiwatch Wiz	ard - Retrieve Data	a Completed 🛛 🛛 🔀		
		Retrieve Data Sum	mary:			
	Finished!					
		Subject Identity:	564738291			
		Start Time (Local):	Friday, November 11,	2005, 9:27:00 AM		
	1	Epoch Length:	30 seconds	Time Zone Details		
	Read Data has successfully been completed!	Number of Data Samples:	2 (0.0 days)			
		The Actiwatch data has been successfully retrieved. The data will be saved to the currently open database once you click the 'Finish' button. To automatically view your data in Actogram form, check the following				
	DOX.					
	Launch Actogram when wizard finishes					
	Click 'Finish' to save the data and exit the wizard.					
	Show Actiwatch Details ->					
	Help	Cancel	< Back	Next > Finish		

Note: Actiware-CT requires the user to 'Sign and Save' the data by using a Windows or Network username and password. Please contact your local administrator with any issues or questions.

Note: Actiwatch-Score may occasionally make a short "chirping" sound during communications. This is normal. DO NOT remove the device from the reader until instructed.

SECTION

ADVANCED USAGE

Actiware provides greater flexibility to researchers and clinicians when viewing and analyzing data. Advanced users can take advantage of the Actiware Toolbars to manipulate how the data is viewed and exported.

Items covered in this section:

- Program Options (Tools > Options)
- Data List
- Batch Export
- Import AWD files
- Graph

Your input is vital to us. Please explore the program fully and give us your feedback. Please feel free to contact us with any questions or comments on this software.

Program Options

Customize the Actiware software and data properties through the program options window. Select **Tools** > **Options** from the main menu bar to access the option tabs.

Tools >	Options					
Tools	Help					
Auc	Audit Trail					
Tex	Text File Export Batch					
Edit	Edit Graph					
Options						

Note: After you have finished making all of your changes in the options window, click OK to accept or Cancel to not accept.

General Tab (Tools > Options > General)

Select the COM port from the dropdown menu. To reset all Help Window reminders, check the **Reset Reminder Notes** button.

Tools >	Options >	General
---------	-----------	---------

ptions						
General	Analysis	Actogram	Statistics	Data List	Export	Print Report
Communica	tions Serial (COM)	Port:				
COM1	-	N				
		6				
			e de la cara.			54
Reset Reminder Notes Click this button to reset the "Don't show this message again" message boxes so that they will reappear when necessary.						
					12	12.5
					OK	Cancel

Analysis Tab (Tools > Options > Analysis)

An *analysis* includes *parameter settings* and *interval settings* as inputs. Use the Analysis tab for setting parameter values (see Section 2 for how to set intervals). Definitions of each analysis parameter are given on the next few pages. You can also see the definitions of each parameter on screen when running the Actiware software by holding your mouse cursor over the parameter names on the left side of the Analysis tab.

Active Analysis refers to the currently selected analysis, so the analysis parameter settings shown in the Active Analysis frame are those of the currently selected analysis.

New Analysis refers to a newly created analysis, so the analysis parameter settings shown in the New Analysis frame are default values. These default values are unique to each Windows logon username and are used each time a New Analysis window is opened.

Click the << button to set Active Analysis parameter settings equal to New Analysis parameter settings.

Click the >> button to set New Analysis parameter settings equal to those of the Active Analysis.

Click the **Restore Defaults** button to reset to the factory released parameter settings.

Options 🛛								
General Analysis Actogram	Statistics	Data List	Export	Print Report				
General Analysis Actogram Parameters Wake Threshold Selection: Wake Threshold Value: Wake Threshold Value: Sleep Interval Detection Algorithm: Immobile Minutes for Sleep Onset: Immobile Minutes for Sleep End: Epochs for Sleep End: Epochs for Sleep End: Actogram Start Hour: Illuminance Threshold: Definitions Hold mouse cursor over analysis parameter nalar and read definitions in this space.	Active Analysis Medium 40.00 Immobile Minutes 10 10 5 5 12:00 PM 1000.0		New Analysis Medium 40.00 Immobile Min 10 5 5 12:00 PM 1000.0					
			бк	Cancel				

Tools > Options > Analysis

Wake Threshold Selection

Selected level or method of setting the Wake Threshold. If *Automatic* is selected, the Wake Threshold will be computed automatically based on activity data.

Wake Threshold Value

The number of activity counts used to define wake. Detail: Level of activity that is compared to a value (x) generated by the Actiware sleep algorithm for each epoch. If (x) is above the Wake Threshold, then the epoch is scored as wake. The higher the Wake Threshold, the fewer the number of epochs scored as wake (See *Appendix A* for further information).

Sleep Interval Detection Algorithm

This determines the basis for calculating sleep onset and sleep end. The first option uses the number of minutes scored as immobile. By selecting this option, the parameter settings for epochs for sleep onset and sleep end can be edited. The second option uses a number of epochs scored as sleep. By selecting this option, the parameter settings for immobile minutes for sleep onset and epochs for sleep end can be edited. Note: This is a means of automatically setting sleep onset and sleep end intervals, not rest intervals.

Immobile Minutes for Sleep Onset

After the start of each rest interval, the start of each sleep interval will be automatically set to the first epoch of the first section of consecutive epochs "THIS" many minutes long with all but one epoch in the section scored as immobile. Note: This parameter is only applied if the Immobile Minutes option is selected as the sleep interval detection algorithm.

Immobile Minutes for Sleep End

Prior to the end of each rest interval, the end of each sleep interval will be automatically set to the last epoch of the last section of consecutive epochs "THIS" many minutes long with all but one epoch in the section scored as immobile. Note: This parameter is only applied if the Immobile Minutes option is selected as the sleep interval detection algorithm.

Epochs for Sleep Onset

After the start of each rest interval, the start of each sleep interval will be automatically set to the first epoch of the first section of consecutive epochs scored as sleep "THIS" many epochs long. Note: This parameter is only applied if the Sleep Epochs option is selected as the sleep interval detection algorithm.

Epochs for Sleep End

Prior to the end of each rest interval, the end of each sleep interval will be automatically set to the last epoch of the last section of consecutive epochs scored as sleep "THIS" many epochs long. Note: This parameter is only applied if the Sleep Epochs option is selected as the sleep interval detection algorithm.

Actogram Start Hour

The hour at which Actogram rows and daily intervals begin. This setting affects daily interval statistics.

Illuminance Threshold

Level of illuminance, in lux, that is compared to the intensity of illumination the Actiwatch-L is exposed to, for the purpose of calculating the Time Above Illuminance Threshold statistic for given time intervals (Actiwatch-L only).

Actogram Tab (Tools > Options > Actogram)

These are the default Actogram viewing properties. Each time an Actogram is opened, it will be displayed with these properties. Adjusting these properties will not affect any open Actograms. To view the changes, close and reopen the Actogram.

Click the **Restore Defaults** button to reset to the factory released settings.

Tools > Options > Actog	lan						
Options							
General Analy	sis Actovan Statistics Da	ata List Export Print Report					
		_					
Days Displayed:	Auto Light/Dark E	Bar: 🕅 Visibility					
Component Visibility:	Activity Data	All Light (24 hours of light on)					
	🔝 Illuminance Data	All Dark (24 hours of light off)					
	Score Data	6:00 AM 💌 Light ON Time					
	Sleep/Wake Scoring	8:00 PM 💌 Light OFF Time					
	# Grid						
Activity Scale:	Auto						
Illuminance Scale:	Auto						
These are the default Actogram viewing properties. Each time an Actogram is opened, it will be displayed with these properties. Adjusting these settings will not affect any open Actograms.							
		OK Cancel					

Tools > Options > Actogram
Statistics Tab (Tools > Options > Statistics)

Select the items of data displayed from the analysis. Changes made here will impact exported and printed data, and the statistics table window contents. Place your cursor over an item to get a brief definition of that item.

Click **Select All** to include all items in your statistical analysis. Click **Deselect All** to remove all items from your statistical analysis. Click the **Restore Defaults** button to reset to the factory released settings.

Options				
General Analy	sis Actogram	Statist Statist	a List Export	Print Report
Interval Information ✓ Start Date Start Day ✓ Start Time Duration Activity Statistics ✓ Total AC ✓ Avg AC/min Avg AC/epoch Std AC Sleep/Wake Statistics ✓ Sleep Time	 ✓ End Date ✓ End Day ✓ End Time ✓ Max AC ✓ Invalid Time AC ✓ %Invalid AC ✓ Wake Time 	Mobility Statistics Immobile Time %Immobile Time #Immobile Bouts Avg Imm Bout #1min Imm Bouts %Imin Imm Bouts Score Statistics #Scores #Scores #Avg Score Avg Score Avg Score	Mobile Time %Mobile Hobile Bouts Avg Mob Bout Fragmentation Index #No Responses #Scheduled Std Score Std Manual	Select All Deselect All Click any checkbox to include or
 ✓ %Sleep #Sleep Bouts Avg Sleep Bout ✓ Onset Latency ✓ Snooze Time ✓ Sleep Efficiency ✓ WASO 	 ✓ ‰Wake ✓ ₩Wake Bouts ✓ Avg Wake Bout ✓ Invalid Time SW ✓ %Invalid SW 	✓ Avg Scheduled Illuminance Statistics ✓ Total Exposure ✓ Avg Illuminance ✓ Std Illuminance ✓ Max Illuminance	Std Scheduled TALT Invalid Time L Show and L Rest	not include the item in Actiware outputs.
			OK	Cancel

Tools > Options > Statistics

See *Appendix B* for a definition of each statistic.

Data List Tab (Tools > Options > Data List)

Select the columns to be displayed in the Data List window. Changes made on this tab will impact the content of exported files and printed reports.

Click **Select All** to include all items in the Data List. Click **Deselect All** to remove all items from the Data List. Click **Restore Defaults** to return the settings to the factory presets.

Changes made in the **Tools** > **Options** > **Data List** window will be immediately displayed in the **View** > **Data List** window.

Tools > Options > Data List

Options				X
General Analysis	Actogram Stati	stics Data List	Export	Print Report
Data List Columns ↓ Line Number ↓ Epoch Number ↓ Day Number ↓ Elapsed Seconds ↓ Date ↓ Time	 ✓ Activity ✓ Event Marker (16/64) ✓ Illuminance (AW-L) ✓ Score Value (AW-S) ✓ Score Type (AW-S) ✓ Sleep/Wake Score Mobility Score ✓ Interval Status ✓ S/W Status 	К Click any checkbox tr not include the colum list outputs.	o include or n in Actiware	Select All Deselect All
			0K	Cancel

Export Tab (Tools > Options > Export)

Select the items you want to include in your export file (also applies to the Copy > Export Text to windows clipboard function in the non-CT version of Actiware). Use the **Statistics** tab to select individual statistics. Use the **Data List** tab to select individual columns of data.

Click **Select All** to select all of the types of data. Click **Deselect All** to uncheck all of the types of data. Click **Restore Defaults** to return the settings to the factory presets.

Tools >	Options	>	Export
---------	---------	---	--------

Options					X
General Analysis	Actogram	Statistics	Data List	Ex,ort	Print Report
 Full Header Statistics Use the Statistics tab individual statistics. Rest Inter Active Inter Sleep Inter Custom Int Daily Inter Summary S 	to select vals vals ervals vals ervals vals itatistics t t t select data.	Click an not incl Export	ny checkbox to inc lude the item in Act files.	lude or iware	Select All Deselect All
				Res	tore Defaults
				ОК	Cancel

Print Report Tab (Tools > Options > Print Report)

Select the items you want to include in your printed report. Click the **Full Header** box to include a first page of header information (recommended). Click the **Actogram** box to include the Actogram graphic in the report. You can select the number of Actogram days to print per page. Click the **Statistics** box (and subboxes) to include statistics for various types of intervals (by default the report will include Summary Statistics only). Use the **Statistics** tab to select individual statistics.

Click **Select All** to select all of the types of data. Click **Deselect All** to uncheck all of the types of data. Click **Restore Defaults** to return the settings to the factory presets.

Options						
General	Analysis	Actogram	Statistics	Data List	Export	Print Report
<u>र</u> र	Full Header Actogram Days per page:	7 📩	Click ar not incl Print Re	ny checkbox to in ude the item in the sport.	clude or Actiware	Select All Deselect All
Ust indi	Statistics e the Statistics tab vidual statistics. Rest Interv. Active Inter Sleep Inter Custom Inte Daily Interv Summary S	to select vals vals rvals rvals als tatistics				
					Res	tore Defaults
					OK	Cancel

Tools > Options > Print Report

1. View > Data List

You have selected to view a listing of the currently active data as a function of time (epoch-by-epoch).

2. Click the **Epoch-by-Epoch List** tab or the **Marker/Score List** tab to view the data list for that type.

Use the scroll bars to move up and down each List and to pan across data columns as necessary. You may want to maximize the window size by clicking the 'Maximize' button in the upper right corner.

	56473	8291 4/9/1	999, 8:45:00 A	M New A	nalysis (Data Lis	t)	l	
ſ		Epoch-b	y-Epoch List		Ĭ	Marker/Score	List	
	Line	Date	Time	Activity	Marker	Sleep/Wake	Interval Sta	itus 🔼
	1	4/9/1999	8:45:00 AM	684	0	NaN	ACTIVE	. 3
	2	4/9/1999	8:46:00 AM	361	ñ	NaN	ACTIVE	1
	3	4/9/1999	8:47:00 AM	229	Ō	1	ACTIVE	Interval S
	4	4/9/1999	8:48:00 AM	398	ō	1	ACTIVE	Interval 3
	5	4/9/1999	8:49:00 AM	288	õ	1	ACTIVE	
	6	4/9/1999	8:50:00 AM	182	ō	1	ACTIVE	
	7	4/9/1999	8:51:00 AM	60	Ō	1	ACTIVE	
	8	4/9/1999	8:52:00 AM	62	0	1	ACTIVE	
	9	4/9/1999	8:53:00 AM	73	0	1	ACTIVE	
	10	4/9/1999	8:54:00 AM	34	0	1	ACTIVE	
	11	4/9/1999	8:55:00 AM	21	0	1	ACTIVE	
	12	4/9/1999	8:56:00 AM	23	0	1	ACTIVE	
	13	4/9/1999	8:57:00 AM	208	0	1	ACTIVE	
	14	4/9/1999	8:58:00 AM	328	0	1	ACTIVE	
	15	4/9/1999	8:59:00 AM	237	0	1	ACTIVE	
	16	4/9/1999	9:00:00 AM	270	0	1	ACTIVE	
	17	4/9/1999	9:01:00 AM	125	0	1	ACTIVE	
	18	4/9/1999	9:02:00 AM	385	0	1	ACTIVE	
	19	4/9/1999	9:03:00 AM	684	0	1	ACTIVE	
	20	4/9/1999	9:04:00 AM	531	0	1	ACTIVE	
	21	4/9/1999	9:05:00 AM	424	0	1	ACTIVE	
	22	4/9/1999	9:06:00 AM	514	0	1	ACTIVE	
	23	4/9/1999	9:07:00 AM	261	0	1	ACTIVE	
	24	4/9/1999	9:08:00 AM	194	0	1	ACTIVE	
	25	4/9/1999	9:09:00 AM	194	0	1	ACTIVE	
	26	4/9/1999	9:10:00 AM	298	0	1	ACTIVE	
	27	4/9/1999	9:11:00 AM	229	0	1	ACTIVE	
	28	4/9/1999	9:12:00 AM	42	0	1	ACTIVE	
	29	4/9/1999	9:13:00 AM	34	0	1	ACTIVE	
	30	4/9/1999	9:14:00 AM	18	0	0	ACTIVE	
	31	4/9/1999	9:15:00 AM	20	0	0	ACTIVE	
	32	4/9/1999	9:16:00 AM	(U	U	ACTIVE	
	33	4/9/1999	9:17:00 AM	27	U	U	ACTIVE	
	34	4/9/1999	9:18:00 AM	1	U	U	ACTIVE	
	35	4/9/1999	9:19:00 AM	5	U	U	ACTIVE	
	35	4/9/1999	9:20:00 AM	3	U	U	ACTIVE	
	37	4/3/1333	3:21:00 AM	1	0	U	ACTIVE	
	20	4/3/1333	3:22:00 AM	1 E0	0	1	ACTIVE	
	40	4/3/1333	3.23.00 AM 9.24-00 AM	60 C	0	0	ACTIVE	× -
	<							>
Т	o select	or deselect dis	played columns,	click Tools	> Options > Data	List.		

View > Data List

Text File Batch Export

This option is used when you want to export data from multiple analyses for the currently open database. You can elect to send these data to individual text files or to the same composite file.

Text File Export Batch Wizard Welcome window



1. Tools > Text File Export Batch

This will launch the Export Batch Wizard. Confirm that the currently active database is the one you want to export from.

2. Click Next.

Select the type of output you require (individual files for each analysis, or a combined file for multiple analyses).

Select destination of the exported data

😢 Text File Export Batch Wizard - Select Destination	×
Select the type of output you would like to create, and the destination folder or filename.	
Individual Export Text Files (.csv)	
Destination Folder:	
C:\Program Files\Mini Mitter\Actiware\Actiware Data\ Browse	
C Combined Export File (.csv) Destination File:	
C:\Program Files\Mini Mitter\Actiware\Actiware Data\Combined Export Browse	
To adjust which items are included in the export files, click 'Options'	
Click 'Next >' to select the data to export.	
Help Cancel < Back Next	

3. Click Browse.

Use this button to determine the destination for any export output using the **Browse** button and the Windows File Dialog.

- 4. Click the **Options** button to make any changes.
- 5. Click Next.
- 6. Select the data that you want to include in the export file by checking the boxes.

Select data to export

💼 Text File Export Batch Wizard - Select Data to Export	
Select the analyses you want to process and export.	
Image: Second system 564738291 Image: Second system A /9/1999 8:45:00 AM Image: Second system A New Analysis Image: Second system A Forced Wake Image: Second system A Forced Wake Image: A Rest & Excluded Intervals Image: A Rest & Excluded Intervals	
Click 'Next' to proceed.	
Help Cancel < Back Next >	Finish

Note: The controls for this window are hierarchical. If you select a subject, or Actiwatch data entry all analyses associated with that subject or Actiwatch data set will be included in the export file.

Note: Hidden subjects are shown in the display.

Hidden data is indicated by the red circle and the diagonal line.



7. Suppress Errors Messages box.

When this box is checked, you will not be prompted if an error occurs during the batch process. If a file alrady exists, an automatic filename will be created.

8. Overwrite Existing Files box.

Indicate if you want to overwrite existing export file(s) without being prompted.

9. Click Next to process your data and create the export.

More options are available when using the Text File Export Batch Wizard

🔁 Text File Export Batch Wizard - More Options 📃 🗖 🔀
Suppress Error Messages <<< If this box is checked and an error occurs, you will not be prompted, and the batch will automatically continue. If a file already exists, an automatic filename will be created.
Overwrite Existing Files <<< If this box is checked, and an export file already exists, it will be automatically overwritten. If not checked, you will be prompted for a filename.
Note: A batch processing log will be created to track errors and results.
Click 'Next >' to begin processing data.
Help Cancel < Back Next> Finish

10. Click OK when the batch process is completed.

If any errors occur, you have the option to view the log file for details. You can also **Restart** the batch process. If you restart the process you may choose to overwrite the existing file or rename the file.

11. Click Finish to complete the Export Batch Wizard.

Batch Completed! pop-up window



Note: .*AWS files cannot be imported. Only .AWD (Actiwatch) files can be imported.*

Import AWD

You can import data files collected with previous versions of Actiware (Actiware-Sleep/Rhythm v3.4 and earlier).

- 1. *Left* mouse click to highlight the subject you want to import the AWD file to. If you do not already have a subject created, use **File** > **New Subject** menu.
- **2**. Click File > Import AWD.

File > Import AWD

🖬 A	ctiwa	re			
File	View	Сору	Communi	cations	Analy
D	atabase	Э			•
N	ew Sub	ject			
E	dit Anal	ysis Pro	perties	Alt+Ent	er
In	nport A	WD		Ctrl+D	N
E: F/	xport AST Exp	oort		Ctrl+T	15
Pi Pi	rint Rep rint Gra	ort ph		Ctrl+P	

3. Use Windows File Dialog to Browse to the location of the AWD files. Select the file and click **OK** to import. When the file is successfully imported a pop-up window will appear.

Note: The identity from the AWD file is not imported. You must select or create the correct subject when importing.

- 4. Click Yes to edit the Actiwatch Data properties or click No to bypass.
- 5. The data is ready for analysis.

Edit Actiware Data

Edit Act	tiwatch Data Item 🛛 🔀
?	The Actiwatch data file has been successfully imported and an item labeled: 4/9/1999, 8:45:00 AM has been added to the Database Viewer. Would you like to edit the Actiwatch Data properties (such as
	Time Zone or Start Time) now? Don't show this message again. <u>Yes</u>

Graph

View View

The Actiware toolbars provide many new features to assist you with the data.

Graph

1. View > Graph \checkmark

This is the Graph window for viewing Actiwatch data along a single axis.

View > Graph



Graph Icons

2. View > Toolbars > Graph



When viewing a graph, the Graph Icons are available in the standard toolbar.

These icons allow you to

- 1. Print the graph.
- 2. Edit Graph Display Properties.
- 3. Copy a picture of the graph to the Windows Clipboard.

```
Graph Help
```

Graph	
٩	This is the Graph window for viewing Actiwatch data along a single axis.
	> If you have activated this window by double clicking in the Actogram window, a 24 hour period of data will be displayed.
	> If you have activated this window from any other area of the program, the entire data record will be displayed.
	> To ZOOM in, place the mouse at the left edge of the area of interest. Click and drag a rectangle to the right.
	> To ZOOM out, double click.
	> Moving the mouse over any data point will display the recorded value in the status bar at the bottom of the window.
	 > Data are displayed as follows: 1. Black = Activity 2. Yellow = Light 3. Red = Marker Button Presses (AW 16/64) or No Response (AW-Score) 4. Magenta = Prompted Score (AW-Score) 5. Green = Manual Score (AW-Score)
	🔲 Don't show this message again.
	<u>Ok</u>

The Graph Help window provides useful information when using the Graph view.

Edit Graph Display Properties

1. Tools > Edit Graph

M

This more advanced feature allows you to edit the Graph GUI, display and printing features, and plotting parameters of the graph.

4-18 Advanced Usage

SECTION

5 CLINICAL TRIALS

Actiware-CT (clinical trial) software is designed especially for customers who are concerned about data security and compliance with 21 CFR Part 11. New features have been included in the software that

- Restrict access to only authorized users.
- Embed electronic signatures in data files.
- Secure data files with digital signatures.
- Detect and report unauthorized access.
- Record all changes to subject and data properties, as well as analysis settings and results.
- Provide an audit trail window to view versions and electronic signatures.

Actiware-CT operates in the same manner as Actiware. Section 5 outlines the security features in CT. Please refer to *Getting Started* in Section 2 for information on using the software.

Note: Actiware-CT requires you to authenticate using your Windows or Network username and password. Please contact your local administrator with any issues or questions.

Note: Actiware-CT requires the Windows 2000 Professional or the Windows XP Professional operating system.

Authenticate When you launch the Actiware-CT program, the Security Log on window will appear. To access the program, you must log on with your User Name and Password that you would normally use to access the Windows operating system.

Security Log on window

Security Log On						
Please type in Actiware-CT so	your username and password to access Itware.					
User name:	jwinegar					
Password:	*****					
Domain:	USEU					
	OK Cancel					

When you are adding a new subject, attempting to save a file, or making any changes to a subject, Actiwatch data, or analysis, Actiware-CT will prompt you for your user name and password.

- 1. Enter User Name and Password.
- 2. Review the signature meaning.
- 3. Add any comments.
- 4. Click the Sign and Save button.

Sign and Save window

Sign and Save Data						
To save this da to sign the data meaning below	ata, please type your username and password a with your electronic signature (see signature).					
User name:	jwinegar					
Password:	*****					
Domain:	USEU					
Signature Meaning:	Subject Creation					
Comments:	New Subject added					
	Sign and Save Cancel					

5. The file now carries your electronic signature and an Audit Trail is attached to the data file. The meaning of the signature is also recorded in the data file.

Audit Trail

The Audit Trail feature of Actiware 5.0 is designed to display all changes made to subjects, Actiwatch data items and analyses. Because the relationship between these items is hierarchical, the Audit Trail window will display all the parents of any selected item.

If an analysis is selected for Audit Trail display, then the Audit Trail will show all changes for that analysis, it's parent Actiwatch data item and the subject that owns both of them.

If an Actiwatch data item is chosen for Audit Trail display, then that Actiwatch data item and it's parent subject will be displayed in the Audit Trail window.

The picture below is the product of right-clicking on a saved analysis and selecting the Audit Trail menu item. Each row in the left side of the window represents either a record creation or a change to a record. In the screen shot below, there are five audit trail records: a subject creation and an edit to that subject, an Actiwatch data record creation and both an analysis creation and an edit to the same analysis. Note that the row that is highlighted is for a subject creation and no value is listed in the right hand display for *Prior Value* as there is no prior record.

🎾 Audit Trail						
Create Date (local time)	Record Type	Audit Trail ID		Subject		
2/10/2005 1:37:08 PM	Subject	<\$50E26784A5C54DA284E8C5B00C6A(Description	Prior Value	New Value
2/10/2005 1:37:25 PM	Subject	<\$B36BB03FB9544FA0BAA6CC3A5754*		Identity		987654321
2/10/2005 1:39:27 PM	Actiwatch Data	<\$B36BB03FB9544FA0BAA6CC3A5754*		Gender		Male
2/15/2005 10:47:11 AM	Analysis	<\$B36BB03FB9544FA0BAA6CC3A5754		Date of Birth		1/1/1965
				Age (current)		40
				Full Name		Mr John A Smith
				Street Address		123 Pine Hill Drive
				City		Omaha
				State		N
				Postal Code		
				Country		
4		,	Co Co	untry untry of subject.		2
Show Actogram Show S	tatistics Show	Data List Show Graph				Close

Results of right-clicking on a saved analysis and selecting the Audit Trail menu item.

When a highlighted row represents an edit, such as the second line below, then the Prior Value column will contain the values before the edit.

Close

Description Prior Value New Value 2/10/2005 1:37:25 PM Subject <\$\$026803F89544FA08A66C Identity 987654321 987654321 2/10/2005 1:37:25 PM Actiwatch Data <\$\$0368B03F89544FA08A66C Gender Male Male Male 2/15/2005 10:47:11 AM Analysis <\$\$0368B03F89544FA08A66C Date of Birth 1/1/1965 1/1/1965 2/15/2005 10:47:11 AM Analysis <\$\$0368B03F89544FA08A66C Date of Birth 1/1/1965 1/1/1965 2/16/2005 10:47:11 AM Analysis <\$\$0368B03F89544FA08A66C Date of Birth 1/1/1965 1/1/1965 2/16/2005 10:47:11 AM Analysis <\$\$0368B03F89544FA08A66C Date of Birth 1/1/1965 1/1/1965 2/16/2005 10:47:11 AM Analysis <\$\$0368B03F89544FA08A66C Date of Birth 1/1/1965 1/1/1965 2/16/2005 10:47:11 AM Analysis <\$0368B03F89544FA08A66C Date of Birth 1/1/1965 1/1/1965 2/17/2005 10:47:11 AM Analysis <\$0368B03F89544FA08A66C Date of Birth 1/21/1965 1/21/1965 2/16/2000 Code<	reate Date (local time) Record Type	Audit Trail ID	Subject		
2/10/2005 1:37:25 PM Subject <\$836BB03FB9544FA0BAAGC Identity 987654321 987654321 2/10/2005 1:39:27 PM Actiwatch Data \$\$836BB03FB9544FA0BAAGC Gender Male Male 2/15/2005 10:47:11 AM Analysis <\$836BB03FB9544FA0BAAGC Date of Birth 1/1/1965 1/1/1965 2/15/2005 10:47:11 AM Analysis <\$836BB03FB9544FA0BAAGC Gender Male Male 2/15/2005 10:47:11 AM Analysis <\$836BB03FB9544FA0BAAGC Date of Birth 1/1/1965 1/1/1965 Age (current) 40 40 Full Name Mr John A Smith N Ne State N NE Postal Code 56432 Country USA State N NE State N NE State N NE State N N State	2/10/2005 1:37:08 PM Subject	<\$50E26784A5C54DA284E8C	Description	Prior Value	New Value
2/10/2005 1:39:27 PM Actiwatch Data <\$8368B03FB9544FA0BAA6C	2/10/2005 1:37:25 PM Subject	<\$B36BB03FB9544FA0BAA6C	Identity	987654321	987654321
2/15/2005 10:47:11 AM Analysis <\$8368B03FB9544FA0BAA6C Date of Birth 1/1/1965 1/1/1965 Age (current) 40 40 Full Name Mr John A Smith Mr John A Smit Street Address 123 Fine Hill Drin 12	2/10/2005 1:39:27 PM Actiwatch Data	<\$B36BB03FB9544FA0BAA6C	Gender	Male	Male
Age (current) 40 40 Full Name Mr John A Smith Mr John A Smith Street Address 123 Pine Hill Dri 123 Pine Hill Dri City Ornaha Ornaha State N NE Postal Code 56432 Country USA	2/15/2005 10:47:11 AM Analysis	<\$B36BB03FB9544FA0BAA6C	Date of Birth	1/1/1965	1/1/1965
Full Name Mr John A Smith Mr John A Smith Street Address 123 Pine Hill Driv 123 Pine Hill Driv City Omaha Omaha State N NE Postal Code 56432 Country USA			Age (current)	40	40
Street Address 123 Pine Hill Driv, 123 Pine Hill City City Omaha State N Postal Code 56432 Country USA			Full Name	Mr John A Smith	Mr John A Smit
City Omaha Omaha State N NE Postal Code 56432 Country USA			Street Address	123 Pine Hill Driv	123 Pine Hill Dr
State N NE Postal Code 56432 Country USA			City	Omaha	Omaha
Postal Code 56432 Country USA			State	N	NE
Country USA			Postal Code		56432
			Country		USA

•

Note that in the two previous pictures the buttons at the bottom of the screen were not enabled. The most powerful feature of the Audit Trail window is the ability to display all previous revisions of records. When an Actiwatch data item is highlighted, the Show Graph button will be enabled. If you highlight an analysis row, then all the buttons will be enabled and you can create an audit version of an Actogram, statistics table or a data list.

F

🏷 Audit Trail							
Signature Date (local time)	Time Zone Offset	Record Type	Sia		Analysis		
2/10/2005, 1:37:08 PM	-08:00	Subject	Ma	F	Description	Prior Value	New Value
2/10/2005, 1:37:25 PM	-08:00	Subject	Ma		Analysis Name		Custom Intervals
2/10/2005, 1:39:27 PM	-08:00	Actiwatch Data	Ma		Wake Threshold Selection		Medium
2/15/2005, 10:47:11 AM	-08:00	Analysis	Ma		Wake Threshold Value		40.0 activity counts
					Sleep Interval Detection Algorithm		By minutes scored as imm
					Immobile Minutes for Sleep Onset		10 minutes
					Immobile Minutes for Sleep End		10 minutes
					Epochs for Sleep Onset		5 epochs
					Epochs for Sleep End		5 epochs
					Actogram Start Hour		12:00 AM
					Number of Intervals		3 intervals
				<)(m)		<u>></u>
Show Actogram Show Stati	istics Show Data	a List Show G) iraph		← Audit Trail k	outtons	Close

Audit Trail window provides the ability to view all previous revisions of the record.

If the analysis has been edited and saved, then multiple rows will be displayed, such as the picture above. Highlighting the last row will reflect the most recent version of the analysis. Pressing the **Show Actogram** button will display the current Actogram in the normal display window except the title will be preceded with the word **Audit** as shown below.



Audit Actogram window

Note: Closing the Audit Trail window will close all Actograms, data lists and statistics tables that were created by the Audit Trail window. To view historical audit items, keep the Audit Trail window open and either move it out of the way or minimize it.

To view a past revision of the analysis, highlight the desired row and click the **Show Actogram** button etc. Only one audit Actogram can exist. If an audit Actogram is open and a new audit Actogram is created, the open Actogram will be closed and the new one will replace it. The same holds true for statistics tables and data lists. Items created from the Audit Trail window cannot be edited.

To use Actiware-CT you will be required to log on. The log on procedure is part of the security Actiware-CT offers to protect the data that is acquired from Actiwatch.

Each time a record is created or revised, the user must sign with their username and password and an electronic signature is created and attached to the record.

The electronic signature in Actiware-CT is considered to be the equivalent of a handwritten signature executed on paper. When the signature process has been completed, the resulting secure record consists of three parts:

- Data
- Public Key
- Digital Signature

Upon signing and saving the data record, a secret, private key is used to encrypt a special set of characters that represent the data. This is called a hash, or digest of the data. The result of this encryption process is the digital signature. The data in the record is saved along with the digital signature and a public key. The public key is used to decrypt the digital signature when the record is read. When the record is read and the signature verified, the result of decrypting the digital signature (the original hash) is compared to a hash computed from the contents read from the record. When the two hash results are found to be identical, this means the signature is verified, and the file has not been altered or corrupted (including the signature elements that identify the signer: user name and full name, as well as the public key and digital signature).

Actiware-CT Provision	Result
Authentication	The identity of the signer can be determined by anyone.
Integrity	Any alteration can be detected.
Non-repudiation	Verifies the origin or delivery of data. Protects the signer because the recipient cannot change the file. Protects the recipient because the signer cannot deny they sent it.

Because integrity is assured and authentication and non-repudiation are provided, the digital signature/public key/user name/full name inserted in every digitally signed output file can be considered the legally binding equivalent of a traditional handwritten signature.

Actiware-CT software does not provide the ability to change the contents of an electronic record that has already been created. Instead, any change to a record (subject, Actiwatch data, or analysis) is recorded as a new revision with a new digital signature. Any attempt to change a data record manually will make the file unusable by the software. Also, each time an electronic record is created, the user must sign the document, and the user's user name and full name, along with date and time of signing are embedded in the record.

Note: Each revision to the data file is preserved and signed/hashed separately.

The database is validated and automatically backed-up when

- Opening a database.
- Retrieving data from an Actiwatch.
- Closing a database and exiting Actiware-CT.

This means that each record's digital signature is verified and each revision history is checked. Hence, any attempt to delete or change data stored in an Actiware database is detected, making the database inaccessible.

Export Files

Export files created by Actiware, both individual and combined export files, are signed with an electronic signature, including a digital signature. We do this to assist our customers in complying with 21 CFR Part 11.

For assistance in verifying this signature for your validation purposes, please contact technical support.

SLEEP/WAKE ANALYSIS

Actiware scores all epochs as either sleep or wake. Whether a particular epoch is scored as wake is determined by comparing activity counts for the epoch in question and those immediately surrounding it, to a threshold value set by the researcher. If the number of counts exceeds the threshold, the epoch is scored as wake. If it falls below, or is equal to, the threshold, the epoch is scored as sleep.

Sleep = Total Activity Counts \leq Wake Threshold Value Wake = Total Activity Counts > Wake Threshold Value

Calculating Total Activity Counts

Actiware calculates total activity counts based on the sampling epoch. Below are four diagrams showing how this scoring calculation is done.





1-minute sampling epochs





For instance, assume a 1-minute sampling epoch and the following activity values on/and surrounding the time 12:00.

Time	Corresponding Activity Data
11:58	100
11:59	42
12:00	20
12:01	13
12:02	67

The total activity value for the 12:00 epoch would be: 100 * (1/25) + 42 * (1/5) + 20 + 13 * (1/5) + 67 * (1/25) = 37.68.

If this value is less than or equal to the wake threshold value (next paragraph) then the epoch would be scored as sleep.

Wake Threshold Values

The total number of activity counts calculated above is compared to the wake threshold value selected by the researcher. These thresholds are listed in the table below.

Wake Threshold Selection	Wake Threshold Value*
Low	20
Medium	40
High	80
Automatic	Computed automatically based on activity data
Custom	User-selectable value

*The Automatic Wake Threshold Value calculation is this:

- 1. Sum the activity counts for all epochs of the data set;
- 2. Count the number of epochs scored as MOBILE for the data set (see below for definition of MOBILE);
- 3. Compute the MOBILE TIME (number of epochs scored as MOBILE from step 2 multiplied by the Epoch Length) in minutes;
- 4. Compute the Auto Wake Threshold = ((sum of activity counts from step 1) divided by (MOBILE TIME from step 3)) multiplied by 0.88888

Definition of MOBILE:

An epoch is scored as MOBILE if the number of activity counts recorded in that epoch is greater than or equal to the epoch length in 15-second intervals. For example, there are four 15-second intervals for a 1-minute epoch length; hence, the activity value in an epoch would have to be greater than or equal to four to be scored as MOBILE.

B STATISTIC DEFINITIONS

Statistic Definitions

(In the order, left to right, in which they appear in the statistics table window when all are selected.)

Start Time

The time at the start of the given Rest, Active, Sleep, Custom, or Daily Interval (the start of the first epoch in the given interval).

End Time

The time at the end of the given Rest, Active, Sleep, Custom, or Daily Interval (the end of the last epoch in the given interval).

Interval Duration

The time elapsed between the Start Time and the End Time of the given interval, in minutes.

Total Activity

The sum of all valid physical activity counts [see Total Invalid Time, below] for all epochs from the Start Time to the End Time of the given interval.

Average Activity Per Minute

The average of all valid physical activity counts for all epochs from the Start Time to the End Time of the given interval divided by the Epoch Length in minutes.

Average Activity Per Epoch

The average of all valid physical activity counts for all epochs from the Start Time to the End Time of the given interval.

Standard Deviation of Activity

The standard deviation of all valid physical activity counts for all epochs from the Start Time to the End Time of the given interval. (The standard deviation is computed with (n - 1) rather than (n) in the denominator of the variance.)

Maximum Activity

The largest of any valid physical activity count for all epochs from the Start Time to the End Time of the given interval.

Total Invalid Time (Activity)

(The total number of epochs between the Start Time and the End Time of the given interval in which the physical activity count was found to exceed the maximum possible value from a properly functioning Actiwatch [i.e., invalid data due to rare hardware error, communication error, or data corruption] plus the total number of epochs with valid physical activity counts manually excluded from the data set by the practitioner using Actiware) software multiplied by the Epoch Length in minutes (so the Total Invalid Time is in minutes).

Percent Invalid (Activity)

a) The percentage of Total Invalid Time (Activity) [see above] to the Interval Duration [see above]. b) (Total Invalid Time (Activity) divided by Interval Duration) multiplied by 100.

Total Invalid Time (Sleep/Wake)

(The total number of epochs between the Start Time and the End Time of the given interval for which the sleep/wake scoring algorithm did not have enough data to determine a SLEEP or WAKE score) multiplied by the Epoch Length in minutes (so the Total Invalid Time is in minutes). Note: The insufficient data condition can be caused by invalid or manually excluded physical activity data at the epoch, and/or immediately before the epoch, and/or immediately after the epoch – how much before and after being a function of the Epoch Length.

Percent Invalid (Sleep/Wake)

a) The percentage of Total Invalid Time (Sleep/Wake) [see above] to the Interval Duration [see above]. b) (Total Invalid Time (Sleep/Wake) divided by Interval Duration) multiplied by 100.

Sleep Onset Latency

a) The time elapsed between the Start Time of a given Rest Interval and the following Sleep Start Time, in minutes. b) The time required for the onset of sleep after first attempting to get to sleep (i.e, from the "lights out" time). Calculated using the analysis setting made on Tools > Options > Analysis.

Snooze Time

a) The time elapsed between Sleep End Time and the End Time of a given Rest Interval, in minutes. b) The time elapsed between the end of sleep and the time lights are switched on or the subject gets out of bed.

Sleep Efficiency

a) The percentage of Scored Total Sleep Time [see below] to (Interval Duration [see above] minus Total Invalid Time (Sleep/Wake)), for the given Rest Interval.
b) (Scored Total Sleep Time divided by (Interval Duration minus Total Invalid Time (Sleep/Wake)) of the given Rest Interval) multiplied by 100.

Wake After Sleep Onset (WASO)

The total number of epochs between the Start Time and the End Time of the given Sleep Interval scored as WAKE by Actiware software (or manually set as WAKE by the practitioner using Actiware software) multiplied by the Epoch Length in minutes (so the Wake After Sleep Onset is in minutes). Note: Wake After Sleep Onset is identical to Scored Total Wake Time [see below] when the given interval is a Sleep Interval.

Scored Total Wake Time

(The total number of epochs between the Start Time and the End Time of the given interval scored as WAKE by Actiware software [or manually set as WAKE by the practitioner using Actiware software]) multiplied by the Epoch Length in minutes (so the Scored Total Wake Time is in minutes). Note: In order to be scoreable as SLEEP or WAKE, an epoch must have a valid physical activity count [see Total Invalid Time (Sleep/Wake), above], and in addition there must be a sufficient number of epochs before and after the epoch being scored that also have valid physical activity counts.

Percent Wake

a) The percentage of Scored Total Wake Time to (Interval Duration minus Total Invalid Time (Sleep/Wake)), for the given interval. b) (Scored Total Wake Time divided by (Interval Duration minus Total Invalid Time (Sleep/Wake)) multiplied by 100.

Number of Wake Bouts

The total number of continuous blocks, one or more epochs in duration, with each epoch of each block scored as WAKE, between the Start Time and the End Time of the given interval.

Average Duration of Wake Bouts

The Scored Total Wake Time [see above] divided by the Number of Wake Bouts [see above], for the given interval.

Scored Total Sleep Time

(The total number of epochs between the Start Time and the End Time of the given interval scored as SLEEP by Actiware software [or manually set as SLEEP by the practitioner using Actiware software]) multiplied by the Epoch Length in minutes (so the Scored Total Sleep Time is in minutes). Note: In order to be scoreable as SLEEP or WAKE, an epoch must have a valid physical activity count [see Total Invalid Time (Sleep/Wake), above], and in addition there must be a sufficient number of epochs before and after the epoch being scored that also have valid physical activity counts.

Percent Sleep

a) The percentage of Scored Total Sleep Time to (Interval Duration minus Total Invalid Time (Sleep/Wake)), for the given interval. b) (Scored Total Sleep Time divided by (Interval Duration minus Total Invalid Time (Sleep/Wake)) multiplied by 100.

Number of Sleep Bouts

The total number of continuous blocks, one or more epochs in duration, with each epoch of each block scored as SLEEP, between the Start Time and the End Time of the given interval.

Average Duration of Sleep Bouts

The Scored Total Sleep Time [see above] divided by the Number of Sleep Bouts [see above], for the given interval.

Scored Total Immobile Time

(The total number of epochs between the Start Time and the End Time of the given interval scored as IMMOBILE by Actiware software) multiplied by the Epoch Length in minutes (so the Scored Total Immobile Time is in minutes) Note: In order to be scoreable as IMMOBILE or MOBILE, an epoch must have a valid physical activity count [see Total Invalid Time (Activity), above].

Percent Immobile

a) The percentage of Scored Total Immobile Time to (Interval Duration minus Total Invalid Time (Activity)), for the given interval. b) (Scored Total Immobile Time divided by (Interval Duration minus Total Invalid Time (Activity)) multiplied by 100.

Number of Immobile Bouts

The total number of continuous blocks, one or more epochs in duration, with each epoch of each block scored as IMMOBILE, between the Start Time and the End Time of the given interval.

Average Duration of Immobile Bouts

The Scored Total Immobile Time [see above] divided by the Number of Immobile Bouts [see above], for the given interval.

Scored Total Mobile Time

(The total number of epochs between the Start Time and the End Time of the given interval scored as MOBILE by Actiware software) multiplied by the Epoch Length in minutes (so the Scored Total Mobile Time is in minutes). Note: In order to be scoreable as IMMOBILE or MOBILE, an epoch must have a valid physical activity count [see Total Invalid Time (Activity), above].

Percent Mobile

a) The percentage of Scored Total Mobile Time to (Interval Duration minus Total Invalid Time (Activity)), for the given interval. b) (Scored Total Mobile Time divided by (Interval Duration minus Total Invalid Time (Activity)) multiplied by 100.

Number of Mobile Bouts

The total number of continuous blocks, one or more epochs in duration, with each epoch of each block scored as MOBILE, between the Start Time and the End Time of the given interval.

Average Duration of Mobile Bouts

The Scored Total Mobile Time [see above] divided by the Number of Mobile Bouts [see above], for the given interval.

Number of Immobile Bouts 1 Minute in Duration

The total number of continuous blocks 4 epochs in duration if Epoch Length = 15 seconds, 2 epochs in duration if Epoch Length = 30 seconds, 1 epoch in duration if Epoch Length = 60 seconds (not applicable if Epoch Length is greater than 60 seconds), with each epoch of each block scored as IMMOBILE, between the Start Time and the End Time of the given interval.

Percentage of Number of Immobile Bouts 1 minute in duration to the Number of Immobile Bouts

a) The percentage of Number of Immobile Bouts 1 Minute in Duration to the Number of Immobile Bouts [see above], for the given interval. b) (Number of Immobile Bouts 1 Minute in Duration divided by Number of Immobile Bouts) multiplied by 100.

Fragmentation Index

The sum of Percent Mobile [see above] and Percent Immobile Bouts Less Than 1-Minute Duration to the Number of Immobile Bouts, for the given interval. Also known as the Index of Restlessness or Movement and Framentation Index.

Total Illuminance Exposure

(The sum of all valid illuminance data, in Lux, from an Actiwatch-L [see Total Invalid Time (Illuminance), below] for all epochs from the Start Time to the End Time of the given interval) multiplied by the Epoch Length in minutes (so the Total Illuminance Exposure is in Lux-minutes).

Average Illuminance

The average of all valid illuminance data for all epochs from the Start Time to the End Time of the given interval.

Standard Deviation of Illuminance

The standard deviation of all valid illuminance data for all epochs from the Start Time to the End Time of the given interval. (The standard deviation is computed with (n - 1) rather than (n) in the denominator of the variance.)

Maximum Illuminance

The datum of highest value (greatest average intensity of illumination during an epoch) from the set of all valid illuminance data for all epochs from the Start Time to the End Time of the given interval, i.e. the peak value in the data set – the highest intensity of illumination the Actiwatch-L was exposed to during the interval.

Time Above Illuminance Threshold

a) (The total number of epochs between the Start Time and the End Time of the given interval with valid illuminance data greater than the given Illuminance Threshold) multiplied by the Epoch Length in minutes (so the Time Above Illuminance Threshold is in minutes). b) The total accumulation of time, in minutes, during which the Actiwatch-L was exposed to an intensity of illumination above the given Illuminance Threshold.

Total Invalid Time (Illuminance)

The total number of epochs between the Start Time and the End Time of the given interval in which the illuminance datum was found to exceed the maximum possible value from a properly functioning Actiwatch-L [i.e., invalid data due to rare hardware fault, communication error, or data corruption] plus the total number of epochs with valid illuminance data manually excluded from the data set by the practitioner using Actiware software) multiplied by the Epoch Length in minutes (so the Total Invalid Time is in minutes).

Percent Invalid (Illuminance)

a) The percentage of Total Invalid Time (Illuminance) [see above] to the Interval Duration [see above]. b) (Total Invalid Time (Illuminance) divided by Interval Duration) multiplied by 100.

Total Number of Scores

The Number of Manual Scores [see below] plus the Number of Scheduled Scores [see below] entered by the subject while wearing Actiwatch-Score, and not manually excluded later by the practitioner using Actiware software, from the Start Time to the End Time of the given interval.

Number of Manual Scores

The number of Manual Scores entered by the subject while wearing Actiwatch-Score, and not manually excluded later by the practitioner using Actiware software, from the Start Time to the End Time of the given interval.

Number of Scheduled Scores

The number of Scheduled Scores entered by the subject while wearing Actiwatch-Score, and not manually excluded later by the practitioner using Actiware software, from the Start Time to the End Time of the given interval.

Number of No Responses

The number of No Responses to Scheduled Scores (that are not manually excluded by the practitioner using Actiware software), from the Start Time to the End Time of the given interval.

Average Score

(The sum of all Manual Score Values [that are not manually excluded] plus the sum of all Scheduled Score Values [that are not manually excluded]) divided by the Total Number of Scores [see above], for all epochs from the Start Time to the End Time of the given interval.

Average Manual Score

The sum of all Manual Score Values [that are not manually excluded] divided by the Number of Manual Scores [see above], for all epochs from the Start Time to the End Time of the given interval.

Average Scheduled Score

The sum of all Scheduled Score Values [that are not manually excluded] divided by the Number of Scheduled Scores [see above], for all epochs from the Start Time to the End Time of the given interval.

Standard Deviation of All Scores

The standard deviation of all Manual Score Values [that are not manually excluded] together with all Scheduled Score Values [that are not manually excluded] for all epochs from the Start Time to the End Time of the given interval. (The standard deviation is computed with (n - 1) rather than (n) in the denominator of the variance.)

Standard Deviation of Manual Scores

The standard deviation of all Manual Score Values [that are not manually excluded] for all epochs from the Start Time to the End Time of the given interval. (The standard deviation is computed with (n - 1) rather than (n) in the denominator of the variance.)

Standard Deviation of Scheduled Scores

The standard deviation of all Scheduled Score Values [that are not manually excluded] for all epochs from the Start Time to the End Time of the given interval. (The standard deviation is computed with (n - 1) rather than (n) in the denominator of the variance.)

ACTIWATCH HARDWARE

All communication with Actiwatch is accomplished using an ActiReader that is connected to a computer via an RS-232 Serial Port. This connection to the computer supplies power and communication instructions. Settings made prior to the start of data collection, as well as all data values, are communicated through this device. The Actiwatch and ActiReader communicate using short range radio telemetry. Readers are not "unit specific," i.e., any Actiwatch may communicate with any ActiReader or Actiwatch Reader.

There are four models of Actiwatch currently available:

Actiwatch®-64

Actiwatch-64 contains 64 KB of on-board memory. Chronobiological studies can be conducted over several weeks because intervals longer than five minutes may be selected. An event marker button allows subjects to mark times at which important events occur.

Actiwatch®-16 Similar to the Actiwatch-64, but Actiwatch-16 has 16 KB of memory.

Actiwatch®-L Actiwatch-L measures activity and ambient light exposure (illuminance). Actiwatch-L does not have an event marker button.

Actiwatch®-Score

Actiwatch-Score measures activity and subjective weighting of specific times or events.

Actiwatch Accelerometer

Details

The shape of the accelerometer makes it most sensitive to motion changes in certain orientations. However, it is an omnidirectional sensor, resulting in sensitivity to motion changes in all directions. The Actiwatch stores an integration of movement (including directional aspects) as activity "counts". The sampling frequency is 32 Hz.

Accelerometer orientation



Sensor Calibration and Reproducibility

Each Actiwatch has been programmed with a calibration coefficient to normalize data between watches. During calibration at the factory, all Actiwatch devices are normalized to remove most, if not all variation between devices. Regardless of this feature, we emphasize that relative levels of activity within a data set, or as measured by the same device, should be used for drawing inference rather than absolute values between units. This is due to the great variation in mounting the device.

Note: To obtain the most repeatable results, it is important to develop a standardized mounting and positioning protocol.

Time Keeping

The computer clock sets the Actiwatch start time. Within the device a crystal oscillator keeps elapsed time. When a sampling epoch is chosen, Actiwatch waits a specific number of oscillations before storing the number of activity counts. This also applies to delayed starting times. The time-keeping circuitry does not initiate until the device has been removed from the reader.

Note: For Actiwatch 16/64 (SN prior to V90xxxx) and Actiwatch-L, the time-keeping function of the device will not start while on the ActiReader. These Actiwatch devices cannot keep time while retrieving data. The Actiwatch Score and Actiwatch 16/64 (SN V90xxxx and later) may keep time while on the ActiReader for short durations. You should still reconfigure the watch after retrieving data.

Telemetric Communications Link

Actiwatch requires no external communications port for configuration or data retrieval. Communication between Actiwatch and ActiReader is established via a wireless link.

Water Proof

Actiwatch-16, 64, and Actiwatch-L devices are waterproof to IEC Standard 60529 IPX7. They are water tight to a depth of 1 meter for 30 minutes. These devices will tolerate normal daily experiences such as shower, spa (hot tub), swimming, skiing, rain, household chores, etc. Actiwatch-Score is drip-tight to IEC Standard 60529 IPX2. It is resistant to dripping and light splashing of liquids.

CAUTION! Prolonged submersion or exposure exceeding the above limits may cause damage to the device.

CAUTION! When replacing the battery, it is very important that the rubber O-ring in the battery compartment also be changed. The back mounting panel should be firmly screwed in place to achieve a waterproof seal. The waterproof seals of all Actiwatch devices are checked before leaving the factory. To preserve this integrity, it is imperative you follow the procedures in the Change Battery Wizard.

Standard Wrist Bands

Standard Actiwatch wrist bands are washable and disposable. Extra bands and batteries are available from Mini Mitter. Bands may also be interconnected if the device is to be worn on locations other than the wrist. Longer bands for use around the ankle are also available from Mini Mitter, and a soft, nylon band with hook and loop fastener. Other means to attach Actiwatch may be used as long as the protocols of mounting and positioning are followed.

Marker Button

Actiwatch-16, Actiwatch-64, and Actiwatch-Score models are equipped with a marker switch on the device's front panel. When depressed this button provides tactic feedback alerting the subject to a successful marking of the date and time. A light sensor in Actiwatch-L replaces the marker button.

Animal Case

Both Actiwatch-16 and Actiwatch-64 can be fitted in the Actiwatch animal case. This case protects Actiwatch from water, soil, shock, and gnawing. It does not affect performance.

Actiwatch-L

The specifications and functions in this manual for Actiwatch-L pertain to those devices manufactured on or after July 1, 1999. These devices are easy to identify as they have a serial number that begins with the letter "P." Actiwatch-L devices manufactured before July 1, 1999 have serial numbers beginning with "L." These previous devices have different functional specifications and are not supported by Actiware version 5 or later. If you have questions, contact Mini Mitter for additional information.

Actiwatch-Score

This device provides the means to subjectively "score" by two methods.

• Auditory/flashing-display from device

An alarm sounds and the LED display flashes alerting the user to subjectively weight the event. The annunciation times are chosen during the device configuration. An example of this use would be a request of the user to judge the level of sleepiness every half-hour during a long task.

• Arbitrary input from user

In this "manual" mode the user inputs a level of subjective weighting commensurate with the event. An example of this use might be a level of recurring headache pain through the day, or perhaps a level of depression.

Actiwatch-Score devices with serial numbers beginning with "N" are not supported by Actiware version 5 or later. If you have questions, contact Mini Mitter for additional information.

Actiwatch-L and Actiware combine hardware and software functions to determine light (illuminance) exposure for subjects monitored with the Actiwatch-L. Actiware will configure and retrieve data from an Actiwatch-L. When light (illuminance) data are available, Actiware will display them in the Actogram, Graph, Data List, and Statistics Table.

Actiwatch-L does not include a marker button. It is replaced with a light sensor. Hence no marker data are available in the analysis windows for Actiwatch-L devices.

Light Sensor Type

Actiwatch-L has a photodiode for sensing incoming light (illuminance). This optical sensor has a spectral sensitivity approximating that of the human eye.

Light Sensor Performance

The light-measuring performance of the AW-L is as follows:

Parameter	Value	Condition or Unit
Peak spectral sensitivity	580	nm (nanometers)
Wavelength window	330 to 720	nm
Minimum illuminance sensitivity	0.1	lux
Maximum illuminance sensitivity	150,000	lux
Linearity	2%	Typical from 0.1 to 150,000 lux
Uncertainty	<10%	Typical from 400 to 800 nm
Resolution	2% of lux value	Typical
Temperature variation of spectral sensitivity	<0.1%	From 400 to 800 nm
Angular response	±50 degrees	to 1/2 power point
Active area	5.16	mm2
Recommended Calibration

We recommend factory calibration of Actiwatch-L every 12 months. Contact Mini Mitter Technical Support for information.

Influences

There are various factors that may influence the light (illuminance) data collected by Actiwatch-L. The most likely to occur are as follows.

Sensor positioning

The angle at which light impacts Actiwatch-L can result in variations in readings. This is true of all light sensing devices. It is important that researchers develop a standardized location on the subject to enable Actiwatch-L to minimize this variable.

Condition of the lens

Light (illuminance) measurement varies greatly with a buildup of materials on the lens. It is important to check for dirt, moisture, and other obscuring debris prior to initiating data collection. Keeping the device clean will help avoid this problem. Clean as you would eyeglasses, with a soft cloth moistened with water.

Inadvertently covering Actiwatch-L

The light sensor may inadvertently be covered by clothing or other items. The unit may be detached from the band and used on a visor or other piece of clothing to minimize this possibility. Analysis of data should always take these variables into account.

Illuminance Data

Illuminance data ("light data") retrieved from Actiwatch-L are stored in the Actiware database along with activity data. Illuminance data are expressed in the SI (System International) unit of lux (see "Definitions for Actiwatch-L" on the following page).

Light Analysis is integrated seamlessly into Actiware. Instead of having a separate window for analyzing illuminance data, the Actogram, Graph, Data List, and Statistics Table can be used for light analysis.

Actiwatch-L Actogram Data



Illuminance data are displayed in the Actogram as an orange line graph superimposed over the activity data. You can use the Actogram View toolbar to adjust the Illuminance Scale, the vertical height of the illuminance data.

Illuminance data are also shown in Graph as an orange line.

Per epoch Illuminance data can be seen in the Data List along with activity data, and can be exported along with activity data and per epoch sleep analysis results. The Illuminance column can be shown or hidden in the Data List or Export files by adjusting settings in the **Tools** > **Options** > **Data List** window.

To select illuminance statistics, use **Tools** > **Options** window, Statistics tab. The selected illuminance statistics will be included for Rest, Active, Daily, and Custom Intervals. This will cause these statistics to be included for Rest, Active, Daily, and Custom Intervals. You can create Custom Intervals for analyzing periods of interest, without using or affecting sleep statistics. See *Advanced Usage*, section 4 for help in creating intervals.

Definitions for Actiwatch-L

The following definitions are specific when analyzing data from an Actiwatch-L.

Illuminance

Intensity of illumination. The luminous flux incident on a given surface per unit area. Measured in lux.

Luminous Flux

Rate of flow of radiant energy as evaluated by the luminous sensation that it produces. The luminous flux is obtained from the radiant flux of the source corrected according to the effect it has on the observer, i.e. according to the spectral sensitivity of the receptor.

Lux

SI (System International) unit of illuminance. One lumen uniformly incident on an area of one square meter.

Light

The agent that causes a visual sensation when it falls on the retina of the eye (a narrow section of the electromagnetic spectrum).

Actiwatch-Score is an activity monitor with an annunciator, an event marker button, and an illuminated numerical display. It allows the application of subjective levels to time stamped events. The front panel of Actiwatch-Score has a marker button, 2-digit, 7-segment direct-read LEDs, and an annunciator.

Actiwatch-Score activity data is identical to other Actiwatch models. Actiwatch-Score is unique in that the marker button allows the user to "score" an event.



Actiwatch-Score main functions

Actiwatch-Score Setup

For all Actiwatch devices, the Actiwatch-Score is configured using the Configure Actiwatch Wizard. This wizard provides the means to program the scoring range (any range from 0 through 15), and schedule audible alarms. Audible alarms are used to remind the patient to enter a subjective score value into the Actiwatch.

Once the data have been downloading into Actiware using the Retrieve Data from Actiwatch wizard, you can analyze your Score Data.

Score Analysis

Score Analysis is integrated seamlessly into Actiware. Instead of having a separate window for analyzing score data, the Actogram, Graph, Data List, and Statistics Table can be used for score analysis.



Actiwatch-Score Actogram Data

There are three types of Score data.

Scheduled Scores

These are score values where the patient responded to the scheduled alarm you programmed. These scores appear as magenta lines on the display.

Manual Scores

These scores were entered manually, without an alarm. They appear as green lines on the display.

No Response

When the patient does not respond to a scheduled alarm, this is represented by a small red triangle.

Only one score can be recorded per epoch.

The score value and score type can be seen by placing your mouse over the triangle at the top of the score value in the Actogram.

Per epoch score data can be seen in the Data List along with activity data, and can be exported along with activity data and per epoch sleep analysis results. In addition, a Marker/Score List is provided that includes only the score data. The Score and Score Type columns can be shown or hidden in the Data List or Export files by adjusting settings in the **Tools** > **Options** > **Data List** window.

To generate statistics based on score data, use the Tools > Options window, Statistics tab to include Score Statistics. This will cause these statistics to be included for Rest, Active, Daily, and Custom Intervals. You can create Custom Intervals for analyzing periods of interest, without using or affecting sleep statistics. See *Getting Started* in section 2 for help in creating custom intervals.

Annunciator The annunciator is a miniature horn that generates a "chirp" form of audible announcement. The annunciator sound exits through the side of the case. The annunicator is water-resistant as well as the case.

Turning off the annunciator

- 1. Hold the marker button down 3 seconds.
- 2. The LED will display OF to verify the annunciator is off.

Turning on the annunciator

- 1. Hold the marker button down for 3 seconds.
- 2. The LED will display **On** to verify the annunciator is on.

Note: After setup, the annunciator is reset to On.

Marker Button The marker button is a switch that is used to respond to an audible event and enter a score to "weight" the event. It can also be used to turn the annunciator on and off.

Scheduled Events

During a scheduled event, if the marker button is pressed the annunciator will silence, and the score range minimum value chosen during setup will replace the two dashes. Each subsequent press of the marker will increment the display until it reaches the score range maximum value chosen during setup. If the marker button is pressed one more time, the display will begin again at the minimum value.

Manual Event

During a "manual" event, i.e., when no alarm is sounded but a marker is entered, the display will show two dashes. If the annunciator is enabled, Actiwatch-Score will beep once per second for 10 seconds, or until the event is scored.

Note: Once the marker button is pressed, released, and pressed again and held, the LED will increment once per second.

LED Display The display is a 2-digit, 7-segment LED. In addition to displaying the score (event weighting), there are other functions as well.

- When an event is to be scored, the annunciator will beep once per second for 10 seconds. If the annunciator is disabled, only the LED will activate.
- The display will show the annunciator status (on or off) after the marker button is pressed and held for 3 seconds (refer to *Annunciator* on preceding page).

Note: AW-Score will not acquire activity data during the brief time the LEDs are lit, such as when setting score values.

- The number of score entries (whether prompted by an alarm or not)
- Time taken to enter a score (when the LEDs are lit) The following table is the battery life in days as a function of the average scores per day, and the average score entry time (approximate):

	y		Time	taken to enter	score
	nts da		10 sec	20 sec	30 sec
	eve by	5	139 days	123 days	110 days
Score	re o red	10	123 days	100 days	84 days
	Sconter	20	100 days	73 days	57 days
	e	40	73 days	47 days	35 days

Nearly all Actiwatch devices use the CR2025 coin-cell battery. The methods used to change the battery are identical for all current devices. If you have an earlier Actiwatch, refer to the Instruction Manual for that device, or read the Battery Change Instructions included with the battery change kits available from Mini Mitter.

CAUTION! The latest Actiwatch-Score models (serial numbers having a D prefix, e.g. Dxxxxx) use a CR 2025 battery. Earlier Actiwatch-Score models (serial numbers having an N prefix, e.g. Nxxxxx), use a CR 2032 battery. These batteries are not interchangeable, and may result in electronic or mechanical destruction. Use of the improper battery will void the warranty.

Battery Change Wizard

Actiware 5.0 contains a **Change Battery Wizard**. The wizard will take you through the process step-by-step, from replacing the battery to testing the Actiwatch and changing the battery fitted date.

CAUTION! It is important that you thoroughly read the following information prior to changing the battery. Failure to follow procedures may result in immediate or subsequent damage to the product.

CAUTION! To retain the integrity of the waterproof seal, it is recommended that you change the O-ring in the battery compartment at the same time you change the battery.

The Actiwatch battery is required for data collection, reading, and writing. Actiwatch has a non-volatile memory, i.e., data stored are not lost after the battery has run down. If you are attempting to read a device with a low battery, the Ready LED will still light on the reader. It is recommended that you keep a log of the battery changes for each Actiwatch.

The Actiwatch battery is a CR2025, 3-Volt, 220-mAmp-hour Lithium Manganese cell. To properly install this battery, specific items may be required. These items are included in the Battery Change Kit from Mini Mitter.

Typical	Batterv	Rep	lacem	ent	Kit
rypiour	Duttery	1 CP	aucon	CIII	i vit

Description	Quantity
O-ring	6
Battery, 3V lithium cell (CR2025)	6
Screwdriver, flat head	1
Screw, machine, 1.6 x 6 mm	24
Cotton swabs	12
Instructions	1 set

Additional materials that may be required: Denatured alcohol

X-Acto® knife blade (flat tip) or similar sharp knife

1. Remove the band from the watch. Use the screwdriver supplied to loosen (approximately five full turns) or remove the screws in the slots in the battery cover of the Actiwatch.



- 2. Carefully remove the battery cover. If you have loosened the screws, press down slightly and rotate the cover clockwise. If you have removed the screws, simply lift the cover free from the frame.
- 3. Remove the battery and discard. See warning below!

WARNING! Dispose of lithium batteries appropriately. Do not incinerate due to risk of explosion.

CAUTION! Use extreme care to avoid scratching the metal surface of the device or battery cover. Scratches can cause the watch to leak.

4. Thoroughly clean the O-ring channel with alcohol.

CAUTION! Prior to assembling the Actiwatch, check carefully for lint, hairs, or other debris which may compromise the seal integrity.

- 5. Place a new O-ring into the channel in the back cover as shown below. Use the following technique:
 - Pre-stretch the O-ring by gently flexing it in several directions.
 - Place a portion of the O-ring in the channel and hold it with your thumb or finger as shown below.



- Use a finger on the other hand to guide the O-ring into the channel.
- Be sure that the O-ring is properly seated in the channel, and is not twisted or deformed.



6. Place a new battery into the Actiwatch case, positive (+) side up as shown below.



- 7. Replace the back cover on Actiwatch and replace the screws. If the screws have not been removed completely, rotate the back cover counterclockwise until the slots in the back are firmly seated around the screws, and the back is square with the case.
- **8**. Tighten all four screws in an "X" pattern until all screws are snug. Take care not to over-tighten to the extent of stripping the threads or twisting off the heads.



CAUTION! DO NOT over tighten the screws. They can be easily stripped.

The Actiwatch and band may be cleaned by using a cloth moistened with a mild detergent and warm water. Do not use bleach, alcohol, cleaning solutions containing alcohol, or any strong household cleaners.

Cleaning the Actiwatch Band

Actiwatch bands can be washed and reused. However, it should be noted that Actiwatch studies involve mounting Actiwatch on the body. With some subjects, this may raise particular sensitivities with respect to reusing the bands, and that should be taken into consideration. Additional bands are inexpensive and are available from the manufacturer. Clean the band as follows.

- 1 Remove the band from the device.
- 2 Hand wash the band in warm water with a mild detergent. Rinse.
- 3 Gently dry with a paper towel or soft cloth, or air dry.

Cleaning the Actiwatch Device

Only when the battery cover is installed and the fully sealed should Actiwatch be cleaned.

CAUTION! Do not clean this device while the battery cover is off.

Clean the Actiwatch device as follows.

- 1 Clean with the battery cover on.
- 2 Wet a corner of a clean cloth with warm water. Add a drop or two of mild detergent to the wet cloth and gently wipe the Actiwatch device. Wet another corner with warm water and wipe away soap residue.
- 3 Gently dry with a paper towel or soft cloth.
- 4 Mount the Actiwatch back on the clean band.

C-18 Actiwatch Hardware

SPECIFICATION

All communication with Actiwatch is accomplished using an ActiReader that is connected to a computer via an RS-232 Serial Port. This connection to the computer supplies power and communication instructions. Settings made prior to the start of data collection, as well as all data values, are communicated through this device. The Actiwatch and ActiReader communicate using short range radio telemetry. Readers are not "unit specific," i.e., any Actiwatch may communicate with any ActiReader or Actiwatch Reader.

There are four models of Actiwatch currently available:

Actiwatch®-64

Actiwatch-64 contains 64 KB of on-board memory. Chronobiological studies can be conducted over several weeks because intervals longer than five minutes may be selected. An event marker button allows subjects to mark times at which important events occur.

Actiwatch®-16 Similar to the Actiwatch-64, but Actiwatch-16 has 16 KB of memory.

Actiwatch®-L Actiwatch-L measures activity and ambient light exposure (illuminance). Actiwatch-L does not have an event marker button.

Actiwatch®-Score

Actiwatch-Score measures activity and subjective weighting of specific times or events.

Specification for Actiwatch-16/64

ENVIRONMENTAL

	Operating	Storage
Temperature	0° C to 40° C (32° F to 104° F)	-10° C to 50° C (14° F to 122° F)
Relative Humidity	0 to 95% RH	0 to 95% RH
Atmospheric Pressure	500 to 1060kPa (5600 feet to sea level)	

PHYSICAL

Dimensions:	29 L x 37 W x 12 H (mm)	
Weight:	16 grams	w/o band
	22 grams	with standard band
Case Material:	Polyurethane/Polyester Alloy	
Frame and Battery Cover:	Titanium	
Standard Wrist Band:	Nylon with buckle	

ELECTRICAL

Battery Type:	CR 2025 Lithium
Battery Life:	6 Months Typical
Protection Against Electric Shock:	Internally powered
Degree of Protection Against Electric Shock:	Type BF Applied Part
Modes of Operation:	Continuous
Electromagnetic Compatibility:	The device meets the requirements of EN 60601-1-2, second edition (2001).

Fuses: Degree of Protection Against Ingress of Water: There are no user-replaceable fuses. Water tight to a depth of 1 meter for 30 minutes IEC Standard IPX7

FUNCTIONAL

Parameter	Value	Note
Accelerometer sensitivity	0.05 g-force	
Accelerometer bandwidth	3 Hz to 11 Hz	
Sampling epochs	15 and 30 seconds, 1, 2, 5, 10, and 15 minutes	Actiware-PLM software configures Actiwatch to sample at 2 second epoch length.
Memory	16 or 64 kB	16 kB in Actiwatch-16 only

STANDARDS COMPLIANCE

Test Standard	Description
IEC60601-1	Medical Electrical Equipment - Part 1: General Requirements for Safety
IEC60601-1-1	Medical Electrical Equipment - Part 1-1: General Requirements for Safety - Collateral Standard: Safety Requirements for Medical Electrical Systems
IEC60601-1-2	Medical Electrical Equipment - Part 1-2: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests
IEC60950-1	Information Technology Equipment - Safety - Part 1: General Requirements

DISPOSAL

When necessary, dispose of the device and accessories in accordance with local regulations.

Specification for Actiwatch-Score

ENVIRONMENTAL

	Operating	Storage
Temperature	0° C to 40° C (32° F to 104° F)	-10° C to 50° C (14° F to 122° F)
Relative Humidity	0 to 95% RH	0 to 95% RH
Atmospheric Pressure	500 to 1060kPa (5600 feet to sea level)	

PHYSICAL

Dimensions:	37 L x 35 W x 12 H (mm)	
Weight:	19 grams	w/o band
	25 grams	with standard band
Case Material:	Glass-filled Nylon	
Frame and Battery Cover:	Titanium	
Standard Wrist Band:	Nylon with buckle	

ELECTRICAL

Battery Type:	CR 2025	
Battery Life:	3 Months Typical	
Protection Against Electric	Internally powered	
Shock:		
Degree of Protection Against	Type BF Applied Part	
Electric Shock:		
Modes of Operation:	Continuous	
Electromagnetic	The device meets the	
Compatibility:	requirements of EN 60601-1-2, second edition (2001).	
Fuses:	There are no user-replaceable	
	fuses.	
Degree of Protection Against Ingress of Water:	Dripping and light splashing of liquids IEC Standard 60529 IPX2	

FUNCTIONAL

Parameter	Value	Note
Accelerometer sensitivity	0.05 g-force	
Accelerometer bandwidth	3 Hz to 11 Hz	
Sampling epochs	15 and 30 seconds, 1, 2, 5, 10, and 15 minutes	
Memory	32 kB	
Scoring levels	0 to 15	User-selectable
Number of scores per epoch	One	

STANDARDS COMPLIANCE

Test Standard	Description
IEC60601-1	Medical Electrical Equipment - Part 1: General Requirements for Safety
IEC60601-1-1	Medical Electrical Equipment - Part 1-1: General Requirements for Safety - Collateral Standard: Safety Requirements for Medical Electrical Systems
IEC60601-1-2	Medical Electrical Equipment - Part 1-2: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests
IEC60950-1	Information Technology Equipment - Safety - Part 1: General Requirements

DISPOSAL

When necessary, dispose of the device and accessories in accordance with local regulations.

Specification for Actiwatch-L

ENVIRONMENTAL

	Operating	Storage
Temperature	0° C to 40° C (32° F to 104° F)	-10° C to 50° C (14° F to 122° F)
Relative Humidity	0 to 95% RH	0 to 95% RH
Atmospheric Pressure	500 to 1060kPa (5600 feet to sea level)	

PHYSICAL

Dimensions:	29 L x 37 W x 11 H (mm)	
Weight:	16 grams	w/o band
	22 grams	with standard band
Case Material:	Glass-filled Nylon	
Frame and Battery Cover:	Titanium	
Standard Wrist Band:	Nylon with buckle	

ELECTRICAL

Battery Type:	CR 2025
Battery Life:	3 Months Typical
Protection Against Electric Shock:	Internally powered
Degree of Protection Against Electric Shock:	Type BF Applied Part
Modes of Operation:	Continuous
Electromagnetic Compatibility:	The device meets the requirements of EN 60601-1-2, second edition (2001).
Fuses:	There are no user-replaceable fuses.
Degree of Protection Against Ingress of Water:	Waterproof / Watertight to a depth of 1 meter for 30 minutes IEC 60529 IPX7

DISPOSAL

When necessary, dispose of the device and accessories in accordance with local regulations.

FUNCTIONAL

Parameter	Value	Note
Accelerometer sensitivity	0.05 g-force	
Accelerometer bandwidth	3 Hz to 11 Hz	
Sampling epochs	15 and 30 seconds, 1, 2, 5, 10, and 15 minutes	
Memory	64 kB	
Sensor Type	Photodiode	On-board sensor
Spectral response	CIE Y-curve	Approximate
Wavelength window	330 to 720	nm
Minimum illuminance sensitivity	0.1	lux
Maximum illuminance sensitivity	150,000	lux
Linearity	2%	Typical from 0.1 to 150,000 lux
Resolution	2% of lux value	Typical

STANDARDS COMPLIANCE

Test Standard	Description
IEC60601-1	Medical Electrical Equipment - Part 1: General Requirements for Safety
IEC60601-1-1	Medical Electrical Equipment - Part 1-1: General Requirements for Safety - Collateral Standard: Safety Requirements for Medical Electrical Systems
IEC60601-1-2	Medical Electrical Equipment - Part 1-2: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests
IEC60950-1	Information Technology Equipment - Safety - Part 1: General Requirements

Logging Capacity versus Amount of Memory **FPOCH LENGTH (minutes)**

	LFUC		9111 (iii	muiesj			
	0.25	0.50	1.00	2.00	5.00	10.0	15.0
Bytes	Loggin	ig Capaci	ty (days)	by Epoch	Length		
Actiwatch-16 (kB)	2.8	5.6	11.2	23	56	113	170
Actiwatch-64 (kB)	11.3	22	45	90	227*	365*	365*
Actiwatch-L (32 kB)	2.8	5.6	11.3	28	56	113	170
Actiwatch-Score (32 kB)	5.5	11.1	22	44	111+	223+	334+
Actiwatch-L (64 kB)	3.7	7.2	15	30	75	151	227*

*Limited by 180 day battery life. +Limited by 90 day battery life.

Specification for ActiReader

ENVIRONMENTAL

	Operating	Storage
Temperature	0° C to 40° C (32° F to 104° F)	-10° C to 50° C (14° F to 122° F)
Relative Humidity	0 to 95% RH	0 to 95% RH
Atmospheric Pressure	500 to 1060kPa (5600 feet to sea level)	

PHYSICAL

Dimensions:	114 L x 72 W x 34 H (mm)
Weight:	158 grams
Case Material:	ABS plastic

ELECTRICAL

Data Port	DB9 Serial Port (a compatible USB to serial port adapter can be used)
Battery Type:	9V Lithium (Eveready™ L522)
Battery Life:	6 Months Typical (Turn off ActiReader when not being used)
AC Voltage Source:	100 to 240 V AC , 50/60 Hz
DC Voltage Source:	9-12V DC (when operated with the external DC power adaptor accessory)
AC Current:	0.4 A maximum
DC Current:	1.5 A maximum
Protection Against Electric	Internally powered
Shock:	
Degree of Protection Against Electric Shock:	Type BF Applied Part
Sterile	Non-Sterile
Modes of Operation:	Continuous
Electromagnetic Compatibility:	The device meets the requirements of EN 60601-1-2, second edition (2001).
Fuses:	There are no user-replaceable fuses.
Power Supply Model Nos:	DSA-0151D-12

STANDARDS COMPLIANCE

Test Standard	Description
IEC60601-1	Medical Electrical Equipment - Part 1: General Requirements for Safety
IEC60601-1-1	Medical Electrical Equipment - Part 1-1: General Requirements for Safety - Collateral Standard: Safety Requirements for Medical Electrical Systems
IEC60601-1-2	Medical Electrical Equipment - Part 1-2: General Requirements for Safety - Collateral Standard: Electromagnetic Compatibility - Requirements and Tests
IEC60950-1	Information Technology Equipment - Safety - Part 1: General Requirements

DISPOSAL

When necessary, dispose of the device and accessories in accordance with local regulations.

Hereby,

Mini Mitter Company, Inc. A Respironics, Inc. Company 20300 Empire Avenue, Building B-3 Bend, OR 97701,

declares that this class 1 radio equipment is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EEC. It complies with the following harmonized standards for radio equipment: EN 300 220-3 V1.1.1 (2000-09), EN 301 489-01 V1.4.1 (2002-08), and EN 301 489-03 V1.4.1 (2002-08).

Da Buns 12-20.05

Dan Burns Manager, Quality Assurance

Date

EMC INFORMATION

Guidance and Manufacturer's Declaration – Electromagnetic Emissions

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment - Guidance	
RF emissions CISPR 11	Group 1	This device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions CISPR 11	Class B	This device is suitable for use in all establishments, including domestic establishments and those directly connected	
Harmonic emissions IEC 61000-3-2	Class A	to the public low-voltage power supply network.	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Complies		

Guidance and Manufacturer's Declaration – Electromagnetic Immunity

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment - Guidance		
Electrostatic Discharge (ESD) IEC 61000-4-2	±6V contact ±8V air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.		
Electrical fast Transient/burst IEC 61000-4-4	±2kV for power supply lines ±1kV for input/ output lines	±2kV for supply mains ±1kV for input/ output lines	Mains power quality should be that of a typical home or hospital environment. It is noted certification to IEC 60601- 1-2 is obtained recognizing that interruption of the download of data to the reader due to EMI is not considered to cause harm. When a download is stopped, the user is prompted that and error has occurred and the user would then reinitiate the download. Interference that would interrupt a download is anticipated to be of short duration such that the subsequent download would be successful.		
Surge IEC 61000-4-5	±1kV differential mode ±2kV common mode	±1kV differential mode ±2kV common mode	Mains power quality should be that of a typical home or hospital environment.		
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4- 11	<5% $U_{\rm T}$ (>95% dip in $U_{\rm T}$) for 0.5 cycle 40% $U_{\rm T}$ (60% dip in $U_{\rm T}$) for 5 cycles 70% $U_{\rm T}$ (30% dip in $U_{\rm T}$) for 25 cycles <5% $U_{\rm T}$ (>95% dip in $U_{\rm T}$) for 5 sec	<5% $U_{\rm T}$ (>95% dip in $U_{\rm T}$) for 0.5 cycle 40% $U_{\rm T}$ (60% dip in $U_{\rm T}$) for 5 cycles 70% $U_{\rm T}$ (30% dip in $U_{\rm T}$) for 25 cycles <5% $U_{\rm T}$ (>95% dip in $U_{\rm T}$) for 5 sec	Mains power quality should be that of a typical home or hospital environment.		
Note: $U_{\rm T}$ is the a.c. mains voltage prior to application of the test level.					

Guidance and Manufacturer's Declaration – Electromagnetic Immunity

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Immunity test	IEC 60601 Test Level	Compliance Level (FDA)	Electromagnetic Environment - Guidance	
Power frequency (50/60 Hz) magnetic field	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical home or hospital environment.	
IEC 61000-4-8				
			Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.	
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	It is noted certification to IEC 60601-1-2 is obtained recognizing that interruption of the download of data to the reader due to EMI is not considered to cause harm. When a download is stopped, the user is prompted that and error has occurred and the user would then reinitiate the download.	
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Interference that would interrupt a download is anticipated to be of short duration such that the subsequent download would be successful.	
			Recommended Separation Distance	
			$d = 1.2\sqrt{P}$	
			$d = 1.2\sqrt{P}$ 80 MHz to 800 MHz	
			$d = 2.3\sqrt{P}$ 800 MHz to 2.5 GHz	
			where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).	
			Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b	
			Interference may occur in the vicinity of equipment marked with the following symbol:	

Note 1: At 80 MHz and 800 MHz the higher frequency range applies.

b

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

Over the frequency range 150 kHz to 80 MHz, the field strengths should be less than 3 V/m.

Recommended Separation Distances between Portable and Mobile RF Communications Equipment and This Device

This device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment.

Rated Maximum Power Output of Transmitter	Separation Distance According to Frequency of Transmitter (m)				
	150 kHz to 80 MHz MHz MHz		800 MHz to 2.5 GHz		
(W)	$d = 1.2\sqrt{P}$	$d = 1.2\sqrt{P}$	$d = 2.3\sqrt{P}$		
0.01	0.12	0.12	0.23		
0.1	0.38	0.38	0.73		
1	1.2	1.2	2.3		
10	3.8	3.8	7.3		
100	12	12	23		

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

MINI MITTER®

Mini Mitter Company, Inc. A Respironics, Inc. Company 20300 Empire Avenue, Building B-3 Bend, Oregon 97701 USA

RESPIRONICS

Respironics Deutschland Gewerbestrasse 17 8221 Herrsching, Germany **CE CE** 0123

910-0025-01 Rev B JW 12/08/2006