


Use sectional matrix system



1-1.5mm distance from cusp peaks

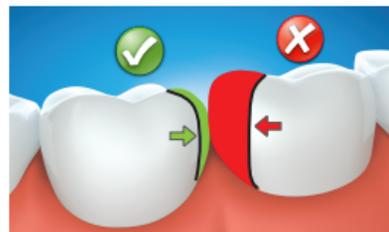


Do not over-contour



1
Recommended:
Apply CAVITY
CONDITIONER (GC) for
10 sec.




2
Rinse and gently dry, do
not desiccate.

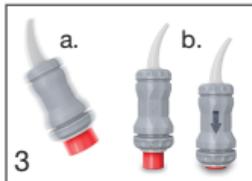

3
a. Shake or tap.
b. Depress plunger
and hold for 2 sec.



4
Mix in capsule mixer for
10 sec. Working time is
1 min. 30 sec.
from start of mix.

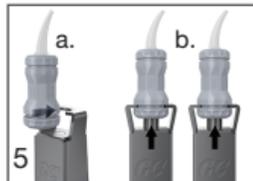

5
a. Insert into Capsule
Applicator.
b. Click twice to prime
capsule.



6
Dispense within
10 sec.



7
Pack and contour.
Avoid moisture
contamination and
desiccation.



8
Final finishing after
2 min. 30 sec.
from start of mix.



9
Finish the restoration
by applying
EQUIA Forte® Coat.
DO NOT AIR BLOW.



10
Light cure for 20 sec.

