INADA CHAIR EMBRACE DELUXE

ENGLISH

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ENGLISH

INADA CHAIR EMBRACE DELUXE

Thank you for purchasing the INADA CHAIR EMBRACE DELUXE.
You can enjoy various types of massage for relieving or soothing fatigue.
We hope the INADA CHAIR EMBRACE DELUXE will be a valued partner in
your daily health preservation for many years to come.

HCP-N333/HCP-N333W

CONTENTS

■ SAFETY PRECAUTIONS	1
■ MAIN FEATURES	4
■ COMPONENTS & DESCRIPTION.....	5
■ BEFORE USE.....	7
■ POSITION ADJUSTMENT	10
■ AFTER USE	13
■ PROGRAMS & SELECTION	14
■ PRESET PROGRAMS.....	15
■ MANUAL SELECTION	17
■ TROUBLESHOOTING	21
■ REGULAR MAINTENANCE FOR SAFETY	22
■ FAQ.....	23
■ CARE AND STORAGE.....	25
■ CUSTOMER SERVICE AND WARRANTY	26
■ SPECIFICATIONS	27

Operating Manual

Warranty card provided separately

- Please read this Operating Manual before use.
- Keep this Operating Manual nearby so that you can refer to it anytime.
- This product is designed for home use.

SAFETY PRECAUTIONS

Types of Indications

	Warning	This action could result in serious injury or death.
	Caution	This action could result in personal injury or damage to property.

Types of Symbols and Description


	Prohibited.		Do not disassemble.
	Do not handle with wet hands.		Do not use in a location where the chair may become wet or do not expose to water.
	Do not use in a bathroom or shower room.		
	Required action.		Unplug the power plug from the wall outlet.

Remember to keep this Operating Manual in an easily accessible place.

Safety Precautions: Installation

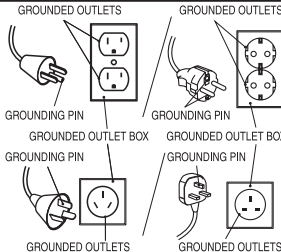 Caution			
	<ul style="list-style-type: none"> ● Do not install the chair in an area of high humidity, such as a bathroom or sauna. → Could result in electric shock or damage to the chair. 		<ul style="list-style-type: none"> ● Never connect the ground wire to gas pipes, water pipes, telephone lines, or lightning rods.
	<ul style="list-style-type: none"> ● Install on a flat, level surface only. → Chair could tip over accidentally if placed on an uneven surface, resulting in an accident. 		

Safety Precautions: Installation

<div> Warning</div>			
<div> Electrical grounding</div>	<div><div>● Space Requirements</div><p>This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electric current in order to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.</p><ul style="list-style-type: none">• Improper connection of the equipment-grounding conductor can result in the risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product if it will not fit the outlet; have a proper outlet installed by a qualified electrician. This product uses on a normal Alternating Current(AC) circuit and has a grounding plug that looks like the plug illustrated in the figure to the right. Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.</div>	<div><div></div></div>	
<div> Required Action</div>	<div><div>● Always unplug this appliance from the electrical outlet immediately after using and before cleaning.</div></div>		
<div> Caution</div>			
<div> Prohibited</div>	<div><div><div>● Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.</div><div>● Use this appliance only for its intended use as described in this Operating Manual. Do not use attachments not recommended by the manufacturer.</div><div>● Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a service center for examination and repair.</div><div>● Do not carry this appliance by supply cord or use cord as a handle.</div><div>● Never operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.</div><div>● Never drop or insert any object into any opening.</div><div>● Do not use outdoors.</div><div>● Do not operate where aerosol (spray) products are being used or where oxygen is being administered.</div><div>● Children should be supervised to ensure that they do not play with the appliance.</div></div></div>		
<div> Required Action</div>	<div><div><div>● An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.</div><div>● Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.</div><div>● Keep the cord away from heated surfaces.</div><div>● To disconnect, turn all controls to the off position, then remove plug from outlet.</div><div>● Keep children away from extended foot support (or other similar parts).</div><div>● Connect this appliance to a properly grounded outlet only. See Grounding Instructions.</div><div>● This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.</div></div></div>		

Safety Precautions: Before Use

Warning	
Prohibited	<ul style="list-style-type: none"> ● The following persons must not use this massage chair: <ul style="list-style-type: none"> ○ Persons who have been instructed by a doctor to avoid massage therapy. Ex. Thrombosis, severe aneurism, acute varicosity, or dermatitis and/or skin infection including inflammation of the typodermis. ○ Persons with suspected acute neck sprain (whiplash injury) ● Do not use the chair with any power source other than AC 220-240 V. <ul style="list-style-type: none"> → May result in fire or electric shock. ● Do not use with any type of transformer device. <ul style="list-style-type: none"> → May result in damage to the chair or electric shock. ● Do not use the chair, if the power cord or plug is damaged, or if the wall outlet connection is loose. <ul style="list-style-type: none"> → May result in electric shock, short circuit, or combustion. ● Before each use, check for holes or tears in the chair covering. Also, check the other parts for tearing. Unplug the chair and contact the vendor for assistance (see back page). Never use the chair with the chair covering removed or with holes or tears in the covering, because doing so is extremely dangerous. <ul style="list-style-type: none"> → Using the chair with holes or tears in the covering could result in injury or electric shock.
Required Action	<ul style="list-style-type: none"> ● Persons with any of the following conditions should seek medical advice before using the chair: <ul style="list-style-type: none"> ○ Serious heart conditions ○ Sensory impairment caused by serious peripheral circulatory disorders due to diabetes and other diseases ○ Osteoporosis, spinal fractures, acute pain and/or symptoms thereof ○ Persons being treated for personal injuries ○ High fever (38°C (100°F) or higher) Ex. Strong acute inflammation illness caused by fatigue, chills, blood pressure fluctuation, or asthenia ○ Persons in the early stages of pregnancy, or immediately after giving birth. ○ Persons who have electronic medical device(s) easily susceptible to electromagnetic interference, e.g. pacemaker ○ Abnormal or curved spine ○ Persons planning to use the chair for treatment or for rehabilitation purposes ○ Persons with malignant tumor(s) ○ Persons under bed rest orders ○ Persons with acute gastro-intestinal complaints or ailments such as gastritis, enteritis, or hepatitis ○ Persons under a doctor's care, or those experiencing unusual physical discomfort ○ Joint dysfunction due to rheumatism, gout, or other diseases ○ Persons with symptoms such as qualm, dizziness, or tinnitus ○ Inflammation due to sprains, muscle strain, or other injuries ○ Persons with thecitis or suspected thecitis ○ Persons with thermoanesthesia ● Elderly people and persons with weak bones should seek medical advice before using the chair, even if they do not have a specific disease. ● If your condition does not improve after use, consult a doctor or specialist.
Caution	
Prohibited	<ul style="list-style-type: none"> ● Do not use the chair in any way not prescribed by the Operating Manual. Do not use the chair together with any other therapeutic device or with an electric blanket. <ul style="list-style-type: none"> → May result in injury or have a negative effect on health. ● Do not move the chair while someone is in it. <ul style="list-style-type: none"> → Accident or injury may result if the chair tips over. ● Do not sit down on the chair with the leg rest raised. <ul style="list-style-type: none"> → Any sudden impact on the leg rest could cause damage to the mechanism. Also, an accident or injury may result if the chair tips over. ● Do not use with anything other than the accessories supplied with this product.
Required Action	<ul style="list-style-type: none"> ● Before plugging the power cord into the wall outlet, make sure that the power switch is set to OFF. <ul style="list-style-type: none"> → May result in damage to the chair or electric shock. ● Make sure that the plugs on the power cord are inserted fully into the cord connector and the wall outlet. <ul style="list-style-type: none"> → May result in electric shock, short circuit, or combustion. ● Before using the chair for the first time, turn on all the switches in order as indicated in this Operating Manual to check that the chair is operating properly. ● Before using, raise the reclining seat to its fully upright position. Then recline the seat back slowly to the most comfortable position, remembering to start off with a lighter massage. <ul style="list-style-type: none"> → Starting with a deep massage may hurt the user. Elderly people and persons with weak bones should be especially careful when selecting massage intensity. ● For safety, sit down in the chair slowly without turning or twisting the body to ensure that the spinal rollers are placed at either side of the spine. ● Do not sit, stand, or lean on the leg rest unless you are sitting in or getting up from the chair. <ul style="list-style-type: none"> → Personal injury or damage to the chair may result if the chair tips over. ● If you use the chair after it has not been used for a long time, read this Operating Manual again thoroughly and check if the chair operates properly.

[SAFETY PRECAUTIONS]

Safety Precautions: During Use

Warning	
Prohibited	<ul style="list-style-type: none"> ● Do not wrap the power cord or the remote controller cord around your body, or place the chair on top of either cord. Do not scratch, tear, treat, unduly twist, stretch or bend the power cord or the remote controller cord. <ul style="list-style-type: none"> → The cords may be damaged, resulting in fire or electric shock. ● Do not spill water and/or other liquids on the chair or remote controller. <ul style="list-style-type: none"> → May result in electric shock, short circuit, or combustion. ● Children and those who are unable to express their intentions should not use the chair. It is advisable to keep children and pets off the chair and away from the surrounding area (especially behind the reclining seat back or under the seat or leg rest). <ul style="list-style-type: none"> → Could result in accident or injury. ● Do not use the chair while more than one person is sitting in it. In particular, do not use the chair while holding a small child (or baby). <ul style="list-style-type: none"> → Could result in accident or injury. ● Do not use the chair while your body is wet, or with wet hands. <ul style="list-style-type: none"> → Could result in electric shock or damage to the chair. ● Do not use the chair if the covering is torn. If it is, unplug the power cord from the wall outlet. <ul style="list-style-type: none"> → Could result in an accident, injury or electric shock. For repairs, contact the vendor for assistance (see back page). ● Do not release hands suddenly while lifting the leg rest by hand.
Required Action	<ul style="list-style-type: none"> ● Make sure no one, especially children or pets, is in the way when lowering or raising the reclining seat back. Be careful that people or objects are not caught behind the seat back or under the leg rest.

SAFETY PRECAUTIONS

Safety Precautions: During Use

⚠ Caution	
Prohibited	<ul style="list-style-type: none"> ● Do not use the chair unclothed. ● Do not use the chair for purposes other than massage, or other than the ordinary uses of a chair. <ul style="list-style-type: none"> → Could result in accident or injury. ● Do not use the chair to massage the head, chest, stomach, and/or joints (knees, elbows). <ul style="list-style-type: none"> → May result in nausea or injury. Place a soft towel behind your neck if you want to avoid intense stimulation of the area. ● Do not place your hands or feet into the roller tracks, into the space between the seat back and the seat or into the moving parts. Do not sit wearing hard objects, such as hair accessories, or a necklace, watch, bracelet or ring. <ul style="list-style-type: none"> → May result in personal injury. ● Do not place anything other than legs and feet on the leg rest. <ul style="list-style-type: none"> → May result in personal injury. ● Do not operate the massage chair or leave it operating when there is no one in it. <ul style="list-style-type: none"> → Could result in accident or injury.
Required Action	<ul style="list-style-type: none"> ● In an emergency, or if you find any abnormal condition, push the EMERGENCY STOP button on the remote controller to stop operation immediately. ● If you get acute pain or other unusual physical discomfort while using the chair, immediately discontinue the use of the chair and seek medical advice. ● If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop operation immediately and contact the vendor for assistance (see back page). <ul style="list-style-type: none"> → May result in injury if you continue use with high stimulation under such circumstances. ● Do not use with the back pad removed. <ul style="list-style-type: none"> → May result in personal injury. ● Do not fall asleep in the chair during a massage session. <ul style="list-style-type: none"> → May result in personal injury. ● Keep massage sessions to no longer than 15 minutes. Do not exceed 5 minutes for an area-specific massage of any particular part of the body. <ul style="list-style-type: none"> → Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. The total amount of time for a daily massage should be limited to about 30 minutes keeping appropriate intervals between each massage session. ● If your leg or foot comes out of the leg rest during the massage, do not try to force it back in. <ul style="list-style-type: none"> → Undue force may result in damage to the mechanism. ● If your arm comes out of the upper arm unit or forearm unit during the massage, do not try to force it back in. <ul style="list-style-type: none"> → Undue force may result in damage to the mechanism.
Unplug the chair	<ul style="list-style-type: none"> ● If there is a power outage or the possibility of a power outage, stop the chair immediately and turn the main POWER switch to OFF. Unplug the power plug from the wall outlet. <ul style="list-style-type: none"> → Accident or injury may result if the chair is still in operating mode when the power resumes. ● If there is a threat of lightning, stop operation immediately and turn the main POWER switch to OFF. Unplug the power plug from the wall outlet. ● If an operation does not start, or if you find any abnormality, stop the chair immediately and turn the main POWER switch to OFF to prevent an accident. Unplug the power plug from the wall outlet. Contact the vendor for assistance (see back page). <ul style="list-style-type: none"> → Electric shock or fire caused by a short circuit may result if the chair is left plugged in under such circumstances.

Safety Precautions: After Use or When Not in Use

⚠ Caution	
Prohibited	<ul style="list-style-type: none"> ● Be sure to turn the Main POWER switch to off after every use and to unplug the power plug from the wall outlet. <ul style="list-style-type: none"> → An accident or injury may result if children or pets play with the chair. ● Do not use the power cord for any other products. ● After using the chair, be sure to return the reclining seat back to its upright position and lower the leg rest to its lowest position. ● Wipe the chair off after every use, and store the chair in a place with low humidity. ● Do not place any object on the chair.
Unplug the chair	<ul style="list-style-type: none"> ● Hold the power plug when unplugging from the wall outlet. Do not pull on the power cord. <ul style="list-style-type: none"> → May result in electric shock or fire caused by short-circuiting. ● When cleaning the chair, make sure to unplug the power plug from the wall outlet first. Do not plug or unplug the power plug with wet hands. ● If the chair is left unused for a while, be sure to unplug it from the wall outlet. <ul style="list-style-type: none"> → Insulation degradation by dirt or moisture may cause electric shock, or fire due to short-circuit.

Other Precautions

⚠ Warning	
Prohibited	<ul style="list-style-type: none"> ● Do not stand on the chair. Do not make a major impact or sit on any part of the chair other than the seat. <ul style="list-style-type: none"> → Personal injury or damage to the chair may result if the chair tips over.
Do not disassemble	<ul style="list-style-type: none"> ● Do not try to repair any malfunction of the chair. Contact the vendor for assistance (see back page). ● For safety, if the power cord is damaged contact the vendor for assistance (see back page). ● Do not modify or remodel the chair. <ul style="list-style-type: none"> → May result in fire, electric shock, or injury.
⚠ Caution	
Required Action	<ul style="list-style-type: none"> ● If you lend, sell, or otherwise transfer the chair to another party, make sure to include this Operating Manual.

Note

- Before disposing of the product or its accessories, consult your local authorities about the proper procedure.

MAIN FEATURES

Main Features

Compact and high-performance

New Hybrid Massage Function

The chair has a compact, high-performance design enabling a roller massage from the shoulders to the lower back, and an air massage of the buttocks and legs, and also from the shoulders to the fingertips. It gives you a more thorough massage.

Optic sensors provide for

Automatic Shiatsu Point Locator Function

Automatically locates shiatsu points which vary from person to person. This allows for an effective massage that tailors itself to your size and shape.

Easy-to-see LED

Stylish Remote Controller

An LED lights up for the selected program, making the buttons easy to understand and use. Also, the remote controller semicircle area displays LEDs for the shoulder position search and operations inside the panel of the remote controller.

Easy to adjust to the most comfortable position

Partially Linked Automatic Reclining Function

The seat back can be adjusted in a range of 115° to 165° from the floor, and the leg rest operates to approximately 45° in conjunction with the movement of the reclining seat back. Additionally, the air foot rest can be adjusted up or down a maximum of 90°.

Air Massage Kneading

Full Body Air Massage Function

The chair performs a full body air massage from the leg rest that massages the feet and calves, seat that massages the thighs, and forearm/upper arm units that massage from the shoulders to the fingertips.

Advanced healthcare programs

Full-Body Stretch Action

The expert kneading of the rollers and the air massage combine with the reclining of the seat to deliver a unique full-body stretch action. This program is recommended after playing sports or to cool down.

Relieve any kind of fatigue

Various Course Functions

Preset programs allow you to select the optimal program according to your purpose, while the manual selection mode allows you to combine functions to suit your particular needs.

Warm on hand and foot

Equipped with a heater function

Combination of the heater and the massage will provide you more relaxation to your entire body.

[MAIN FEATURES]


Benefits

As a therapeutic massage device in the general households, the INADA CHAIR EMBRACE DELUXE provides the following benefits:

- Relaxes muscles
- Relieves minor muscle aches and pains

COMPONENTS & DESCRIPTION

Chair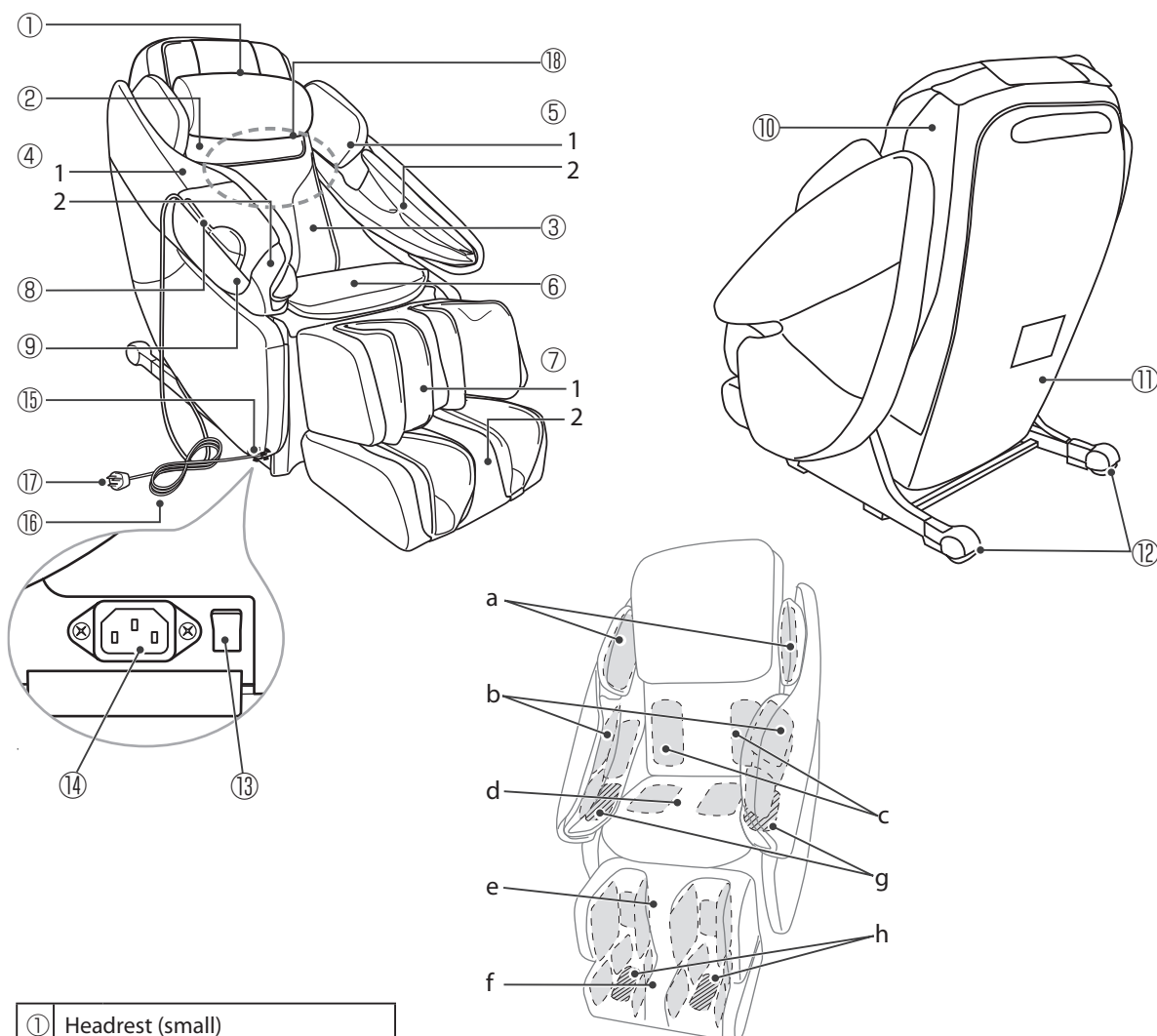


①	Headrest (small)
②	Headrest (large)
③	Back Pad
④	Armrest (right)
· 1	Upper arm unit
· 2	Forearm unit
⑤	Armrest (left)
· 1	Upper arm unit
· 2	Forearm unit
⑥	Seat
⑦	Leg Rest
· 1	Calf Unit
· 2	Foot Unit

⑧	Remote Controller
⑨	Remote Controller Pocket
⑩	Reclining Seat Back
⑪	Back Cover
⑫	Wheels
⑬	Main POWER Switch
⑭	Power Connector
⑮	Cord Connector
⑯	Power Cord
⑰	Plug
⑱	Rollers Normally stored inside the upper part of the seat back.


a	Upper arm air cushion /1 each on left and right
b	Forearm air cushion /4 each on left and right
c	Lower back air cushion /x 2
d	Seat Air Cushion /x 4
e	Calf Air Cushion / x 6
f	Foot Air Cushion /x 10
g	Palm heater /1 each on left and right
h	Foot cover heater /1 each on left and right

Remote Controller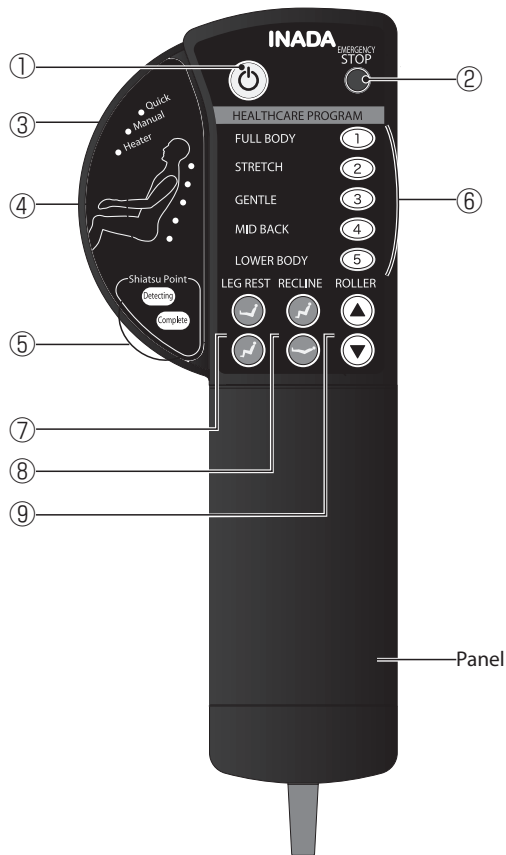

1	Button	Turns the remote controller power ON and OFF. Pushing the POWER button while the chair is activated causes the rollers to return to the starting position, after which the chair turns itself off. The button lights in green while the power to the remote controller is ON, lights in orange while the power is OFF, and flashes in red while the chair is stopped in an emergency.
2	EMERGENCY STOP Button	Use this button to shut the chair off immediately in an emergency or if there is a malfunction during operation of the chair. The rollers will stop immediately.
3	Program Selection Display	When a program is selected inside the remote controller panel, the corresponding program light turns on.
4	Roller Position Display	The position of the roller during the massage is displayed.
5	Shiatsu Point Locator Indicator	A light indicates when the shiatsu point locator is operating and when the operation has finished.
6	Preset Program Buttons 1 (Healthcare Programs)	Selects the corresponding program.
7	Leg Rest Button	Use to adjust the angle of the leg rest.
8	Reclining Button	Use to adjust the angle of the reclining seat back.
9	ROLLER-Position Adjustment Buttons	Use to adjust the shoulder position while the shiatsu point locator is operating, and to adjust the roller position during manual selection mode.

Remote Controller/Inside the Panel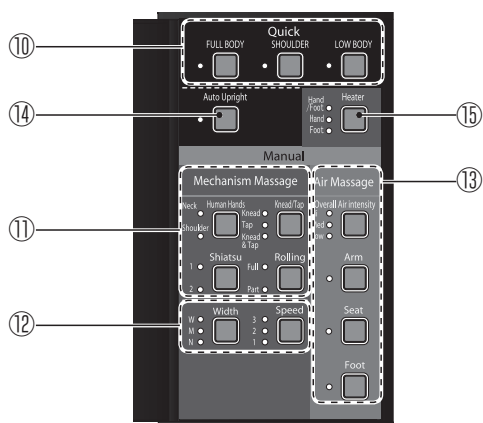

10	Preset Program Buttons 2 (Quick Programs)	Selects the corresponding program.
11	Mechanism Massage Button	Selects among basic massage operations such as Kneading, Tapping, or Rolling.
12	Intensity Buttons	Adjusts the mechanical massage.
13	Air Massage Buttons	Selects among arm, back, or leg air massage. *These 3 air massage types can all be selected at the same time. Also, the air intensity can be adjusted by pressing the Adjust All button.
14	Auto Upright Button	The automatic seat reclining rise operation when the course is completed can be set ON/OFF.
15	Heater Button	The heater can be switched ON/OFF.

Warning

- Do not spill water and/or other liquids on the remote controller.
- Do not handle the remote controller with wet hands.

Caution

- Do not sit on the remote controller. Do not drop or step on the remote controller. Do not pull the remote controller cord forcibly.
- Do not use pointed or hard objects to push the buttons.
→ May result in damage to the remote controller.

BEFORE USE

- This product is shipped with the reclining seat back in its fully reclined position. Be sure to raise the seat back to its fully upright position.
- Do not expose the chair to sudden bursts of heat. If the room where the chair is to be used is cold, turn the heat up slowly. If the chair is moved from a cold place into a heated room, wait one hour before using the chair.
→ Condensation could occur in the moving parts of the chair, which may compromise performance.

Installation


Allow for at least 50 cm (20 inch) clearance behind and in front of the chair when reclining so as to be able to recline the seat back fully without obstruction.

Caution

- Do not install the chair in an area of high humidity, such as a bathroom or sauna.
- Install on a flat, level surface only.
- Do not use in areas of high debris or dust content. Do not use where the ambient temperature is 40°C (104°F) or over.
- Do not install the chair where it can come into contact with direct heat, such as direct sunlight or a radiator.
- Keep fire sources such as cigarettes and ashtrays away from the chair.


- To avoid damaging floor surfaces and to minimize the chair noise, it may be better to install the chair on a carpeted surface or to lay a piece of carpet under the chair.
(Do not use electric carpets.)
- The chair is very heavy, so please be careful when unloading it so as not to damage floor surfaces.
- Install the chair at least 1 m (40 inch) away from TVs, radios, and other audio or video equipment. Otherwise they may make noises affected by the operation of the chair.
- The load to the floor per 1 cm² (0.15 inch²) is approximately 2.3 kg (5.1 lb), with a person weighing 100 kg (220 lb) sitting on the chair.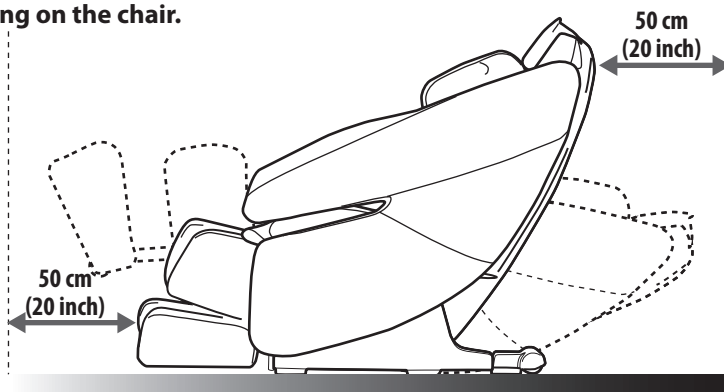

Checking the Supplied Items

Check that the chair and the other items are all in the package.

Chair

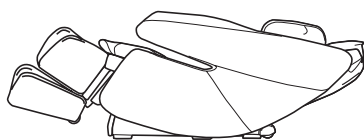Chair

Items Included

Operating Manual
(this document)


AC Cord

Buffer Pad
(thin/thick)


Shiatsu Balls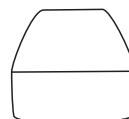

Remote Controller Pocket

How to Transport or Move the Chair

Be sure to turn off the main POWER switch and unplug the power plug from the wall outlet before carrying the chair.

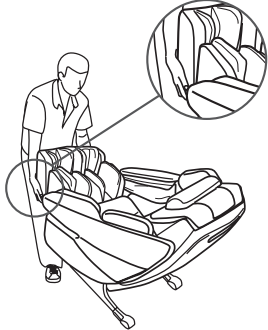To Move with the Wheels	To Carry
Raise the reclining seat back to its upright position, hold the sides of the leg rest, and then move the chair on the wheels by lifting and holding up the front part of the chair.	Fully recline the seat back and carry the chair by holding the handle on the upper part of the back cover and the sides of the leg rest.
	

- The chair is very heavy, so be careful not to hurt your lower back when transporting or moving it.
- The remote controller is attached to the chair, so make sure that it does not fall when moving the chair.


Caution


- Do not move the chair while someone is in it.
- Be sure to disconnect the power cord from the wall outlet before moving the chair.
→ The power cord may be damaged, resulting in fire or electric shock.

Main Power Connection

- ① First, plug the power cord into the cord connector located at the front right of the chair. Then, plug the other end into the wall outlet.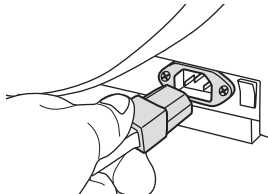

- ② Set the power switch located on the right side of the chair to ON.

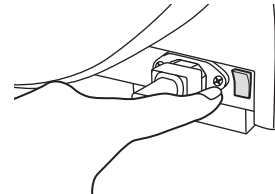*If the main POWER switch is turned ON while the rollers are stopped partway, they will automatically return to the starting position (upper part of the seat back) and then stop.

Warning

- Do not use with any type of transformer device.
→ May result in damage to the chair or electric shock.

Caution

- Before plugging the power cord into the wall outlet, make sure that the power switch is set to OFF.
- Make sure that the plug on the power cord is inserted fully into the wall outlet.
- Before using the chair for the first time, turn on all the switches in order as indicated in this Operating Manual to check that the chair is operating properly.



[BEFORE USE]

BEFORE USE

Checking the Seat Cover

Before using the chair, check the seat cover for tearing with the following procedure.

- (1) Open the 2 fasteners and peel open the velcro on the upper side.
- (2) Fold the back pad forward, and check that the chair fabric is not damaged.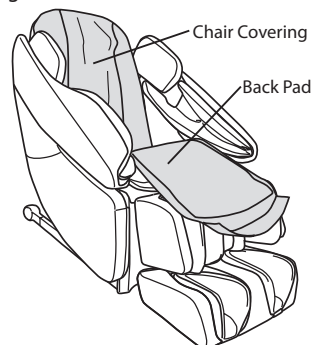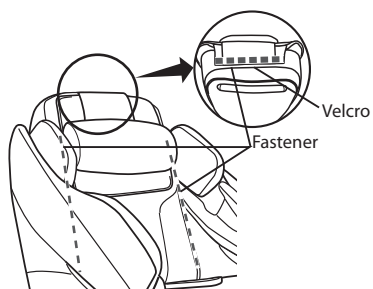

Caution



● Be sure to check for tearing on the chair before every use, as shown in the illustration. Also, check the other parts for tearing. Unplug the chair and contact the vendor for assistance (see back page). Never use the chair with the chair covering removed or with holes or tears in the covering, because doing so is extremely dangerous.

How to Install the Remote Controller Pocket

Attach the supplied remote controller pocket to the arm rest (right) as shown in the diagram below.

Place the remote controller pocket on the arm rest (right).

There is Velcro for installing the remote controller pocket on the inside of the arm rest (right). Wrap the remote controller pocket around the arm rest (right) to install it.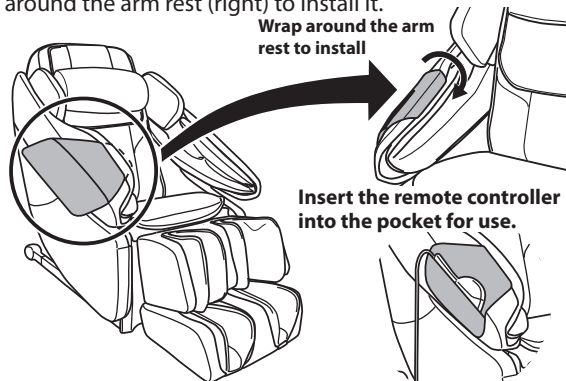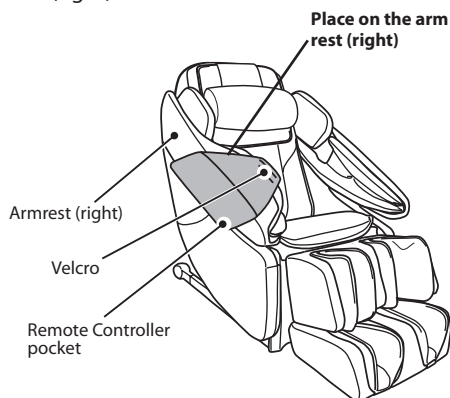

[BEFORE USE]

POSITION ADJUSTMENT

How to Sit

When the chair is off, the rollers are normally drawn back into the left and right sides of the upper part of the seat back (Storage Position). For safety, first check the position of the rollers. If they are in any position other than the storage positions, turn the main POWER switch ON before sitting down to allow the rollers to return to their storage position. When you finish checking, sit down slowly in the center of the seat.



Caution

- **Do not sit down on the chair with the leg rest raised.**
→ Any sudden impact on the leg rest could cause damage to the mechanism. Also, an accident or injury may result if the chair tips over.
- **Before using, raise the reclining seat to its fully upright position. Then recline the seat back slowly to the most comfortable position, remembering to start off with a lighter massage.**


Headrest (Small) Adjustment

You can adjust the position of the headrest (small) to keep your head in a comfortable, natural position during your massage.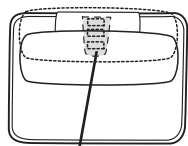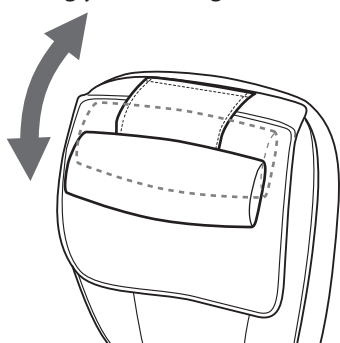


Adjust the height by the position of the velcro

Proper Position of Small Headrest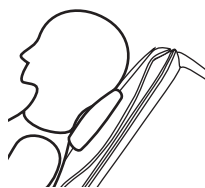

The headrest (small) makes contact with the back of the neck.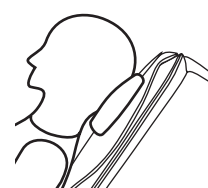

The headrest (small) makes contact with the back of the head (occipital).


- The height of the headrest (small) can be adjusted using the velcro on the back of the headrest. Adjust the headrest (small) to match the height of your head before starting the massage.
- Place the headrest (small) in a proper position. If it is not placed properly, the automatic shiatsu point locator may not function properly.

Removing the Headrest Cover


You can remove the cover of the headrest (small) by following the procedure below.

- ① Pull the cover so that the slit is at the top.
- ② Pull the slit to the side, and remove the headrest through the slit.


*Rear of the headrest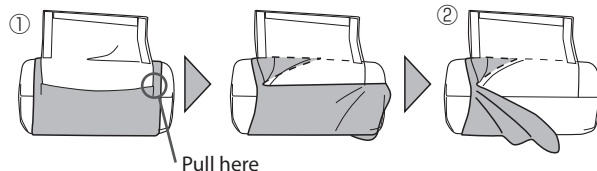

Pull here

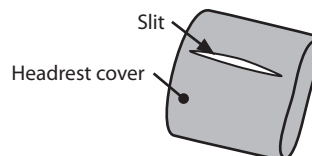* View from the top of the reclining seat back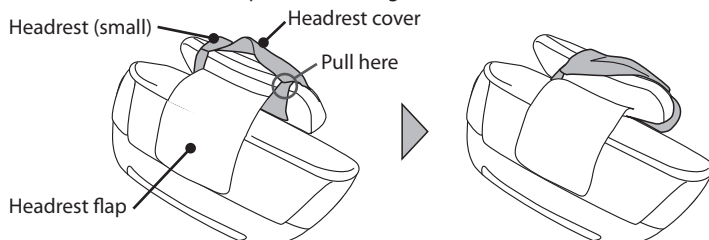

*Be careful not to forcibly roll the headrest cover, as doing so may cause the cover to tear.

*Follow this procedure in reverse order to reattach the cover.

POSITION ADJUSTMENT

How to Use the Buffer Pad


If you feel that the massage is too intense, insert the supplied buffer pad (thin) into the back pad. If you still feel that the massage is too intense, replace the thin buffer pad with the thick one before starting the massage again.

How to Insert a Buffer Pad

Turn over the back pad using the same method as the item in P9 "Checking the Seat Cover."

Peel open the Velcro of the buffer pad pocket on the rear of the back pad, and insert the buffer pad.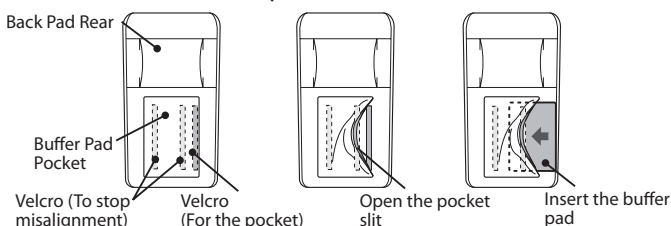

* The inside of the pocket contains Velcro for preventing the buffer pad from becoming misaligned. It may become attached to the pocket, so peel it off from the pocket before inserting the buffer pad.


Caution

- Do not use both buffer pads at the same time.
→ May result in accident or injury.
- Make sure that the buffer pad does not curl up or become lop-sided.
→ May result in accident or injury.

How to Use the Shiatsu Balls

Place the provided Shiatsu balls into the special pocket in the foot unit leg rest for use. There is a heel side and toe side to the special pocket. Insert the Shiatsu balls according to your preference.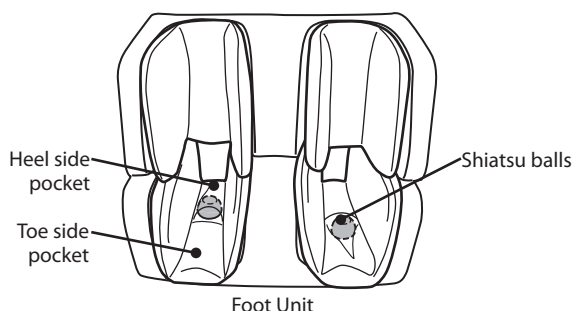


- Remove the Shiatsu balls if you want to make the massage lighter.

How to Use the Armrest

Place your arm as shown in the diagram below so that your elbow is between the forearm unit and upper arm unit.

* Place your elbows between the forearm unit and upper arm unit.

* Insert so that your palm is facing down.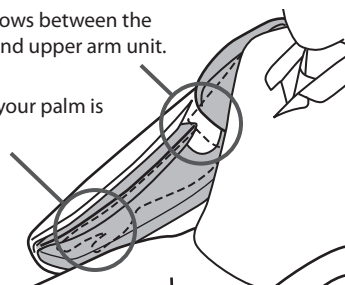

Caution

- Do not use the upper arm unit and forearm unit for anywhere other than your upper arms and forearms respectively.
→ May result in personal injury.
- If your arm comes out of the upper arm unit or forearm unit during the massage, do not try to force it back in.
→ Undue force may result in damage to the mechanism.
- Do not massage your elbows. Also, be sure that your palms are facing down when massaging your arms.
→ May result in personal injury.

How to Use the Leg Rest

Vertical Adjustment

■ To raise the leg rest

Press the top LEG REST button to raise the leg rest.

■ To lower the leg rest

Press the bottom LEG REST button to lower the leg rest.


Movement Range of Leg Rest


The moveable range differs when the seat back is raised compared to when the seat back is reclined.

- When the seat back is raised, movement is possible in the 0° - 45° range.
- When the seat back is lowered, movement is possible in the 45° - 90° range.

When the seat back is raised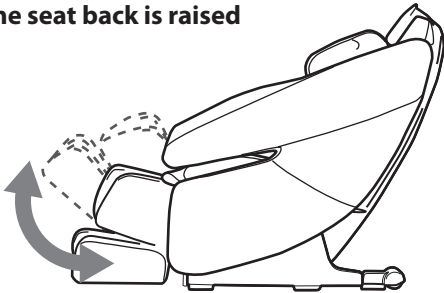

When the seat back is reclined

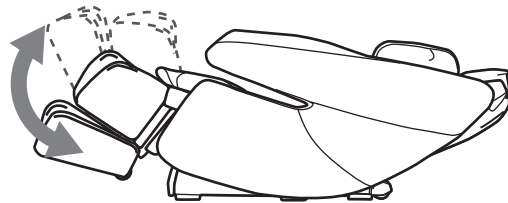
Hold Function

Hold the button for about 2 seconds to raise (or lower) the leg rest automatically for a set amount of time.

Stop the operation to press the button again.

Leg Rest Extension and Retraction

When you sit in the chair and raise the leg rest to the specified height, it naturally retracts.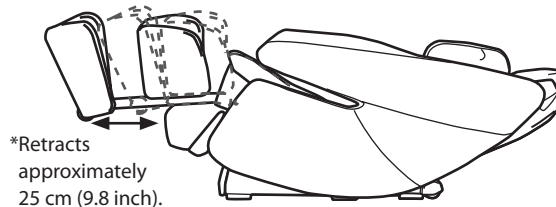

- The leg rest can be adjusted up or down when the main POWER switch located at the front right of the chair is set to ON. Note that it cannot be operated while the chair is stopped in an emergency.

⚠ Warning

- Make sure no one, especially children or pets, is in the way when lowering the leg rest. Be careful that people and/or objects are not caught under the leg rest.
- Do not release hands suddenly while lifting the leg rest by hand.

⚠ Caution

- Do not place anything other than legs and feet on the leg rest.

POSITION ADJUSTMENT

How to Use the Reclining Seat Back

■ To raise the reclining seat back

Press the top RECLINE button to raise the reclining seat back.


■ To lower the reclining seat back

Press the bottom RECLINE button to lower the reclining seat back.

Reclining Movement Range

The chair reclines up and down in conjunction with the movement of the seat back and leg rest.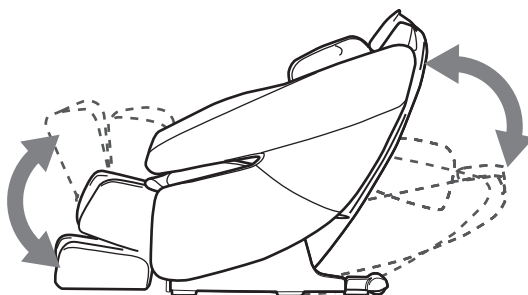

Hold Function

Hold the button for about 2 seconds to raise (or lower) the reclining seat back automatically* for a set period of time.

Stop the operation to press the button again.

*The seat back stops automatically when it reaches its maximum position, and the leg rest stops automatically after operating for a certain period of time.

■ The reclining seat back can be adjusted up or down when the button of the remote controller is ON. Note that it cannot be operated while the chair is stopped in an emergency.



Warning

● Make sure no one, especially children or pets, is in the way when lowering the reclining seat back. Be careful that people and/or objects are not caught behind the reclining seat back.

How To Use the Heater

When the heater button is pressed, the heater lamp in the program selection display lights, and the heater switches on.

Each time the button is pressed, the order can be changed in the order **Hand/Foot** → **Hand** → **Foot** → **Off**.

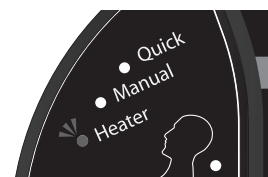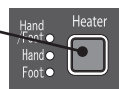AFTER USE

While making sure there is nothing in the surrounding area, return the reclining seat back to its upright position and lower the leg rest to its lowest position.

Turn off the main POWER switch on the front right of the chair!

Unplug the power plug from the wall outlet. This will save energy because no standby power will be consumed.

Caution

- Be sure to turn the Main POWER switch to off after every use and to unplug the power plug from the wall outlet.
→ An accident or injury may result if children or pets play with the chair.
- After using the chair, be sure to return the reclining seat back to its upright position and lower the leg rest to its lowest position.
- Wipe the chair off after every use, and store the chair in a place with low humidity.
- Hold the power plug when unplugging from the wall outlet. Do not pull on the power cord.
- If the chair is left unused for a while, be sure to unplug it from the wall outlet.
- Be careful not to tangle yourself in the cords and trip over.

PROGRAMS & SELECTION

Preset Programs

The preset programs make it easy to select a program that suits your preferences or objectives.

Healthcare Programs

These programs are for recovering from muscle fatigue.

Fullbody

This program is recommended for persons who feel symptoms such as lack of energy or fatigue.

Stretch

This program includes light stretch actions where the entire body is bent backward.

Gentle

Improves blood circulation with a massage that feels like gentle rubbing on your skin.

Mid-Back

Presses your waist with the back air bags, and vibrates your stomach with a tapping operation around your lower back.

Lower Body

This massage is focused on the lower half of the body.

Quick Programs

These programs are intended for refreshing yourself in a short period of time.

Fullbody

This program is recommended for persons who want to refresh themselves in a short time.

Shoulder

This program gives a concentrated massage of the neck and shoulders in a short time.

Low Body

This program gives a concentrated massage of the lower back in a short time.


Manual Selection

You can combine your favorite massage types. You can also adjust the speed, intensity, and roller positions.

PRESET PROGRAMS

Preset Program Messages

- 8 automatic programs provide a variety of different massages to relieve fatigue.
- With the preset program buttons, you can choose your favorite program to give you a massage according to the preset menu.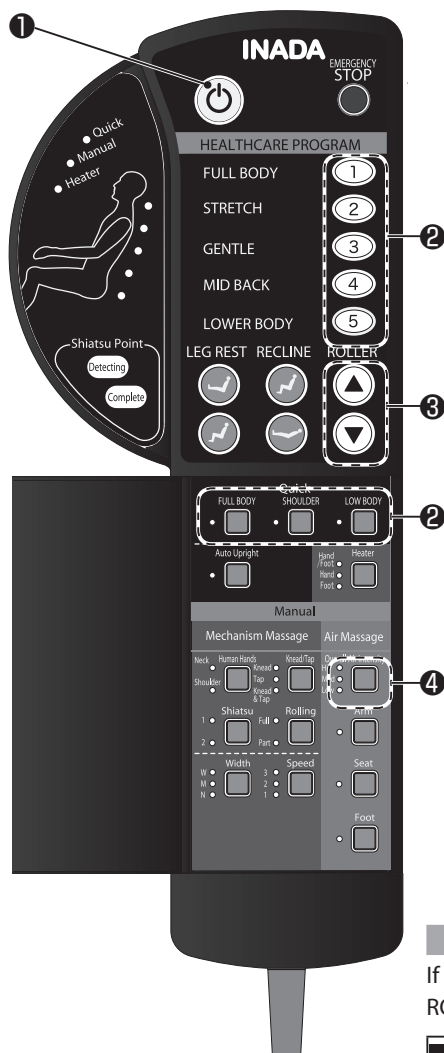


1 Press the button.

If no selection is made within 5 minutes, the power shuts off automatically.

2 Press the desired program button.

The reclining will operate for a short time. Then, the shiatsu point locator indicator turns on and shiatsu point locating is performed.
[Note]

Sit as far back as possible in the chair and put your head against the headrest in a relaxed manner. Shiatsu points cannot be accurately located if your body is raised.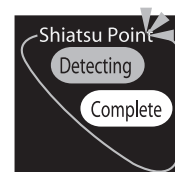


3 The shoulder positions can be adjusted.

After the shiatsu points are located, the ROLLER buttons flash. The shoulder positions can be adjusted while the buttons flash. If no adjustment is required, do not use these buttons.

Shoulder Position Adjustment

If the shoulder position is not correct, adjust the shoulder position while the ROLLER buttons are flashing.


	▲	The rollers move up while this button is pressed.
	▼	The rollers move down while this button is pressed.

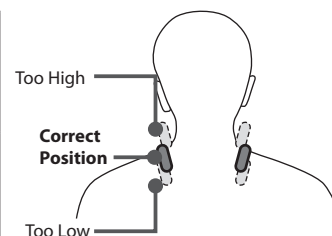4 The program starts.

When shiatsu point locating is completed, the Complete light turns on and the program starts.

During operation, you can adjust the air intensity or cancel the air operation.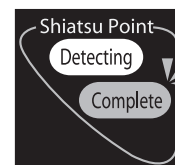

Air Massage Intensity Adjustment and Cancellation	
	The air massage intensity can be adjusted. Each time the Overall Air Intensity button is pressed, the intensity changes in the order of Low, Medium, and High . (The default setting when the program starts is Medium.)
	The arm air massage can be turned ON/OFF. Set the arm air massage operation during the course. Each time the button is pressed, the setting switches between ON/OFF .
	The seat air massage can be turned ON/OFF. Set the seat air massage operation during the course. Each time the button is pressed, the setting switches between ON/OFF .
	The leg air massage can be turned ON/OFF. Set the leg air massage operation during the course. Each time the button is pressed, the setting switches between ON/OFF .

5 Program completed.

The reclining seat back automatically rises, and the rollers return to their storage position (upper part of the seat back). Then, the remote controller power switches off. Readjust the vertical position of the reclining seat back and leg rest after switching the power back on again.

Auto Upright Cancel	
	The function in which the seat back automatically rises when the course is completed can be cancelled. A green lamp lights while it is cancelled. Press the button again to release the cancellation, and the seat back rises when the course is completed.

Changing Courses

Before the course in progress finishes, you can change it to another course. After a total duration of 30 minutes, the course in progress stops even if not completed.

Switching from One Preset Program in Progress to Another

When one preset program button is pressed while another preset program is in progress, the rollers return to their storage positions, then the selected program starts from the automatic shiatsu point location operation. The shoulder positions can be adjusted while the shiatsu points are locating.

Switching from a Preset Program to Manual Selection Mode

When one of the buttons for manual selection mode located inside the panel of the remote controller is pressed while a preset program is in progress, the preset program immediately stops, and the selected operation starts. When selecting the manual programs with the shoulder point locator function, the operation starts after the shiatsu points are located.

(To select an air massage, first select an operation such as kneading or tapping, or press the button to stop the preset program in progress before making the selection.)

Stopping in the Middle of a Massage

To stop mid-massage, press the button. The rollers return to their storage positions and stop.

In an emergency or if there is some malfunction during the operation of the chair, press the EMERGENCY STOP button. The rollers will stop immediately. After confirming safety, press the button. The rollers return to their storage positions and stop.

When a preset program is selected, the reclining seat back may recline automatically. Also, the reclining seat moves automatically during the Stretch Action program. For this reason, before using the chair, make sure that there is no person or obstruction behind it. Also, do not let other people approach the chair during use.

Caution	<ul style="list-style-type: none"> ● If you feel the massage is too strong, reduce the stimulation according to this Operating Manual. If this solution does not work, stop operation immediately and contact the vendor for assistance (see back page). → May result in injury if you continue use with high stimulation under such circumstances. ● Do not place your hands or feet into the roller tracks, into the space between the seat back and the seat or into the moving parts. Do not sit wearing hard objects, such as hair accessories, necklace, watch, bracelet, or ring. ● If your leg or foot comes out of the leg rest during the massage, do not try to force it back in. ● Keep massage sessions to no more than 15 minutes in length. Do not exceed 5 minutes for an area-specific massage of any particular part of the body. → Continuous use for extended periods of time can result in over-stimulation, which could cause the opposite of the desired effect, or even injury. The total amount of time for a daily massage should be limited to about 30 minutes.
----------------	---


MANUAL SELECTION

Description of Manual Selection Operations


You can combine massaging actions of your choice from among menus ① to ③.


- A circle in the adjustment column means that the particular adjustment can be made in the course of that massage action.
- The width cannot be adjusted if an action for which width adjustment is not available is combined with rolling.

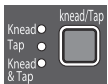① Basic Operations

Button	Operation	Action	Speed Adjustment	Width Adjustment	Up/Down Adjustment
	Kneading	This performs a kneading type of massage.	○	—	○
	Tapping	This performs a tapping type of massage.	○	○	○
	Kneading & Tapping	Kneading and tapping are simultaneously performed.	○	—	○
	Neck Shiatsu	This performs Shiatsu massage at the center of the neck.	—	—	—
	Shoulder Kneading	This performs a kneading massage centered on the shoulders.	—	—	—
	Shiatsu 1	Performs Shiatsu massage.	—	○	○
	Shiatsu 2	Performs rhythmical Shiatsu massage.	—	—	○


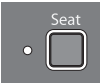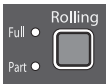
② Rolling

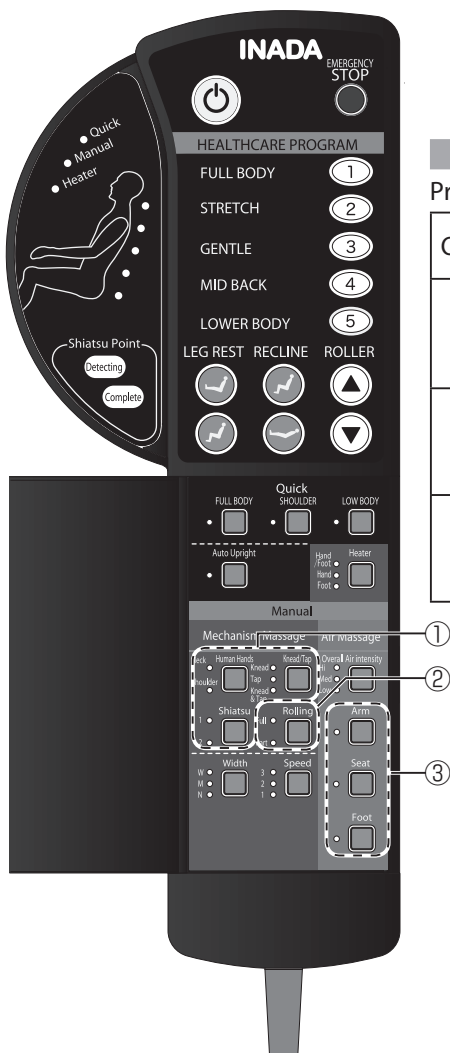Button	Operation	Action	Width Adjustment	Up/Down Adjustment
	Full	This performs full stretching of the spine using up and down movements.	○	○
	Partial	This performs partial stretching of the spine using up and down movements.	○	○


③ Air Massage

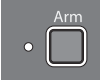Press the button for the area that you want to massage.

Operation	Button	Action	Intensity Adjustment
Arms		This performs an air massage from shoulders to fingers.	
Seat		This performs an air massage from lower back to thighs.	
Foot		This performs an air massage from calves to toes.	

MANUAL SELECTION

Operation of Parts and Position Adjustment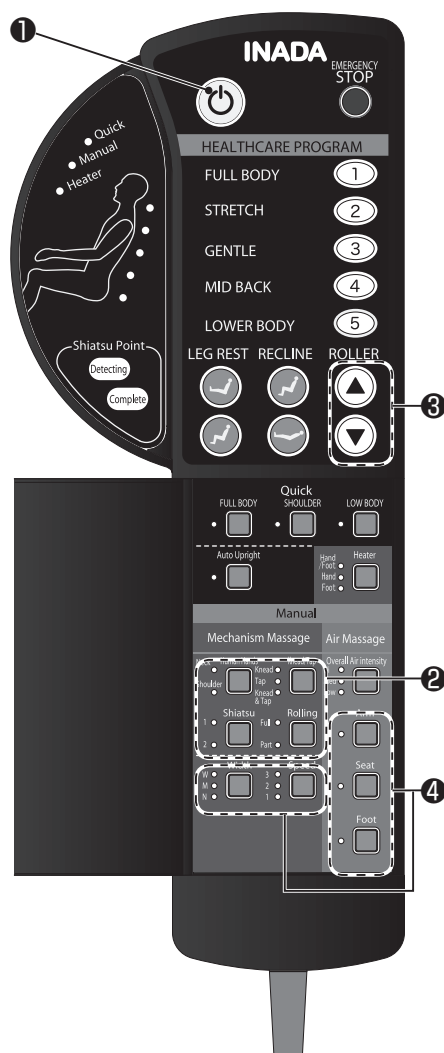

[MANUAL SELECTION]

1 Press the button.

If no selection is made within 5 minutes, the power shuts off automatically.

2 Press the desired button.

Select a combination of your operations of choice from a basic operation + rolling + air massage.

3 The shoulder positions can be adjusted.

(When Rolling or Human Hands is selected)

After the shiatsu points are located, the ROLLER buttons flash.

The shoulder positions can be adjusted while the buttons flash.

If no adjustment is required, do not use these buttons.

4 The program starts.



When shiatsu point locating is completed, the Complete light turns on and the program starts.

A basic operation can be changed and the air can be canceled or its intensity adjusted even while a program is operating.

5 Program completed.

When the massage program finishes, the rollers return to their storage positions (upper part of the seat back). Then, the remote controller power switches off.

Readjust the vertical position of the reclining seat back and leg rest after switching the power back on again.

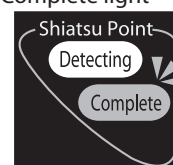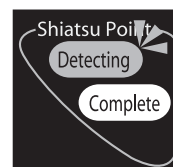Roller Position Adjustment		
	When rolling is not selected: The roller position can be adjusted to the area that you want to massage.	<div>▲ The rollers move up while this button is pressed.</div> <div>▼ The rollers move down while this button is pressed.</div>
	When full rolling is selected: Changes between up and down.	<div>▲ When pressed during a down movement, changes to up.</div> <div>▼ When pressed during an up movement, changes to down.</div>
	When partial rolling is selected: The center position of the range of partial rolling can be adjusted. The position at which you release the button becomes the center of the operation.	<div>▲ The rollers move up while this button is pressed.</div> <div>▼ The rollers move down while this button is pressed.</div>

- When up (or down) is selected if the roller is already at the highest (or lowest) position or in the shoulder position where the shiatsu points were located, a warning alarm sounds.

Caution!

- If neither a basic operation nor rolling has been selected, there will be no operation even if you press a ROLLER button.
- When the Human Hands is selected, adjustment cannot be made even if you press a ROLLER button.

Width Adjustment	
	During a basic operation where width adjustment is possible, the width of the rollers can be adjusted in 3 steps (narrow, medium and wide).

Speed Adjustment	
	During a basic operation where speed adjustment is possible, the speed of action can be adjusted in 3 steps.

Intensity Adjustment of Air Massage	
	The air massage intensity can be adjusted. Each time the Overall Air Intensity button is pressed, the intensity changes in the order of Low, Medium, and High . (The default setting when the program starts is Medium.)

Changing Courses

Before the course in progress finishes, you can change it to another course. After a total duration of 30 minutes, the course in progress stops even if not completed.

Switching from Manual Selection Mode to a Preset Program

When one manual selection button is pressed while another preset program is in progress, the rollers return to their storage positions, and then the selected program starts from the automatic shiatsu point location operation. The shoulder positions can be adjusted while the shiatsu points are locating.

Stopping in the Middle of a Massage



To stop mid-massage, press the  button. The rollers return to their storage positions and stop.

In an emergency or if there is some malfunction during the operation of the chair, press the EMERGENCY STOP button. The rollers will stop immediately. After confirming safety, press the  button. The rollers return to their storage positions and stop.

TROUBLESHOOTING

If there is still a problem after trying the solutions given in this troubleshooting guide, stop using the chair immediately to prevent accident or injury, and unplug the power plug from the wall outlet. Contact the vendor for assistance (see back page).

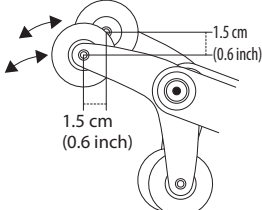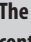Problem	Cause & Solution	
The right and left rollers are out of alignment.	<p>The rollers are designed with an alternating tapping mechanism, so they are sometimes out of alignment. This is part of the design and is not a malfunction.</p> 	This is part of the design and is not a malfunction.
If I try to raise or lower the reclining seat back or leg rest during air massage, the air massage stops.	For safety, the air pressure massage is designed to stop if the reclining seat back is adjusted or the leg rest is raised or lowered.	
The rollers moved up to the top and then stopped.	The auto timer will cause the rollers to move up to the top of the chair when the massage time is finished, so as not to interfere with your movements when getting out of the chair.	
The rollers stop during the massage.	Sometimes when there is too much pressure to the rollers, they can stop moving temporarily. Slightly lift your body or lightly raise the reclining seat back. (Be especially careful when a person who weighs 100 kg (220 lb) or more uses this chair.)	
The  button of the remote controller and the program button flash.	Set the main POWER switch located at the front right of the chair to OFF, and then back ON after about 5 seconds.	If there is still a problem after the check, stop using the product immediately.
The automatic shiatsu point locator function is not working properly. (The height of the rollers does not match the shoulders.)	Sit further back in the chair so that your head rests firmly on the headrest. You can adjust the shoulder position with the ROLLER buttons immediately after the shiatsu points are located.	
Remote controller buttons do not seem to be operating normally. Or the power does not turn on.	Check for the following: ① Is the power plug fully inserted into the wall outlet? ② Is the power switch located on the right side of the chair set to ON? ③ Are you pressing two buttons at the same time?	
The power cord or plug is abnormally hot.	Stop using immediately.	

For details, contact the vendor for assistance.

Anyone other than an authorized serviceperson must not repair or disassemble.

Noise during chair operation

These noises are a normal part of the chair's operation and do not affect its performance in any way.

The following noises are made during operation.

Noises during tapping
Sound of the rollers rubbing against the seat cover during kneading
Sound of raising and lowering the rollers, sound of roller operations
Sound of the air massage cushions
Sound of the air pump
Sound of air discharge
Sound of raising and lowering the reclining seat back

[TROUBLESHOOTING]

REGULAR MAINTENANCE FOR SAFETY

Regular Maintenance Will Preserve the Life of Your Chair

Have you experienced any of these problems?

- The chair does not start with the Main POWER switch set to ON.
- The power cord has been cracked or scratched.
- The power cord or plug becomes abnormally hot.
- The chair produces a burnt smell, abnormal sound or vibration.
- The auto timer function does not work in some cases.
- Other failures.

If you find anything listed on the left, stop using and unplug the power cord from the outlet to prevent failure or an accident. Be sure to contact the vendor for assistance.

To use safely

The chair fabric (roller fabric) is a consumable part, and its service life is 3 years (conditions of use: two 15-minute sessions a day, every day). For safe use, periodic inspection by the vendor or "Repair Service Center" is recommended before the end of the service life.

* Service life varies depending on conditions of use.

* About 5 years after purchase, a professional inspection for the chair is recommended.

For inspection or repair fees, contact the vendor for assistance.

[REGULAR MAINTENANCE FOR SAFETY]

FAQ

Q Can I put the chair onto an electric carpet?

A Do not place the chair onto an electric carpet because it could cause a fire.

The chair is very heavy, so it could damage the heat generating parts of the electric carpet and start a fire.

Q How do I make the massage lighter?

A Perform the following:

- Raise the reclining seat back.
- Insert the thin or thick buffer pad into the rear of the back pad.
- Remove the shiatsu pad and shiatsu sheet.

(When installed)

With the methods above, you can make the massage more gentle.

Q Can I put the chair onto a wooden floor or a heated floor?

A Yes.

However, because the chair could damage the floor surface, we recommend laying a carpet or something similar under the chair. (Same applies to other floor surfaces)

Q Can I use both buffer pads at the same time?

A Do not use both pads at the same time.

Using both buffer pads at the same time may injure your body.

Q How do I make the massage more intense?

A Perform the following:

- Sit back further in the chair so that your weight is against the reclining seat back.
- Lower the reclining seat back.
- Remove the buffer pad from behind the back pad.

(When installed)

With the methods above, you can intensify the massage.

Q Can the reclining seat back raise be stopped when a course is completed?

A Yes. It can.

You can stop the automatic reclining seat back raise operation from operating when the course is completed by pressing the Auto Upright button.

Q Can someone who is under 150 cm (5 feet) or over 180 cm (6 feet) use the chair?

A Yes.

Because the correct shoulder positions may not be found during shiatsu point locating, we recommend using the chair as follows.

- Short person: Raise the reclining seat back.
- Tall person: Recline the reclining seat back.
- After shiatsu point locating is performed, adjust the shoulder positions.

Q I spilled water on the chair accidentally. What should I do?

A There is a danger of an electric shock. Immediately stop using the chair and perform the following.


- Turn off the main POWER switch on the front right of the chair!
- Unplug the power plug from the wall outlet.

After performing the above, contact the vendor for assistance.


Q The remote controller panel has come off. Can I fix it?

A Reattach with the following procedure.

- ① Hold the remote controller as illustrated, and insert the projection on the bottom left of the panel into the hole on the remote controller.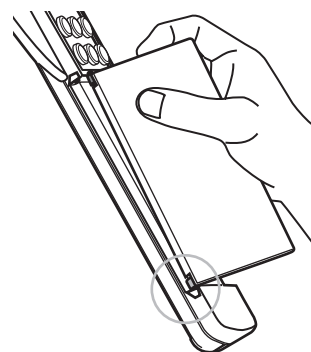

- ② Lightly push the panel in the direction of the arrow and insert the projection on the top left of the panel into the hole on the remote controller.

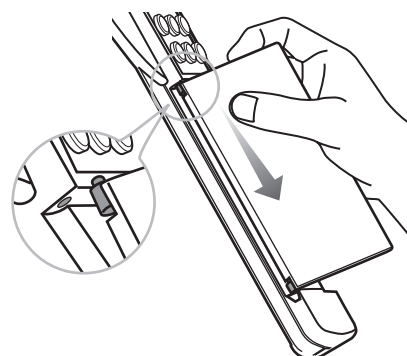[FAQ]

CARE AND STORAGE

Storage

- After wiping off any dust or dirt, store the product in a place that is relatively dry.
- If you are not going to use the product for some time, put a cover on it that will protect it from dust and dirt.
- Pets can sometimes chew on power cords, so try to keep pets away from the product even when it is being stored.

Caution

- Do not install the product where it can come into contact with direct heat, such as direct sunlight or a radiator.

Care and Maintenance

Caution

- When cleaning or caring for the product, make sure to unplug the product from the wall outlet first. Do not plug or unplug the power cord with wet hands.

Caring for the Pad and Covers

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added. Then thoroughly wring it out before cleaning off any spots or grime.

Caution

- The fabric color can change if the product is in contact with vinyl covers or some wall coverings for a prolonged period of time.

Caring for the Remote Controller

Wipe dirt off the remote controller with a dry cloth.

- Never use a moistened or wet cloth. → May result in mechanical failure.

Caring for Other Plastic Parts

Soak a piece of soft cloth in lukewarm water in which a small amount of mild detergent has been added, thoroughly wring it out, and clean off any spots or grime. Then wipe with a dry cloth so that no detergent residue remains.

- Do not use alcohol based or other household cleaning products or abrasives because they can scratch or crack the finish and/or cause color to fade.

CUSTOMER SERVICE AND WARRANTY

1. Warranty

Please read the contents carefully and keep it in a safe place. Please complete and mail in the warranty card that was included with your chair.

If the chair is to be used outside the home (in a commercial setting, for example), fees will be charged for repairs.

2. Repairs & Service

Before calling for repairs or service, use the Troubleshooting guide on pages 21-22 to see if the problem can be solved easily.

If the problem persists, contact the vendor for assistance. Have the following information ready.

- 1) Your name, address, and phone number
- 2) Product name and model number (see the Back Cover)
- 3) Serial No. (see the Serial No. label on the Back Cover)
- 4) Date of purchase
- 5) Trouble status
- 6) Proof of purchase

● If the warranty is valid:

A service charge may apply, subject to the nature of the trouble and/or repair according to the terms of the warranty.

See the warranty for full details.

● If the warranty has expired:

See the back of this booklet and contact our technical service department.

If the product can be repaired, we will give you an estimate of the cost to repair the product.

● Replacement parts stock:

We will stock service parts (except fabric) of this product for at least six years after terminating the production of this model. Service parts refer to those necessary for maintaining the intended performance of the product.

● For more information on service:

Please see the back page of this booklet.

SPECIFICATIONS

Name & Model Number		HCP-N333	HCP-N333W
Power Supply		AC100-240 V~	
Power Consumption		142W (with the heater ON), Power consumption of the heater only: 16 W	
Power Frequency		50-60 Hz	
Rated Time		30 minutes	
Upper Body	Kneading Speed	3 levels, from minimum 5 times/minute to maximum 20 times/minute (approx.)	
	Tapping Speed	3 levels, from minimum 180 times/minute to maximum 500 times/minute (approx.)	
	Roller Width	3 steps (narrow, medium and wide)	
	Up/Down Speed	One full cycle in approx. 28 seconds (approx. 3.6 cm (1.4 inch)/sec)	
	Back Stroke Range	Approx. 69 cm (28 inch) (in partial roller massage: approx. 15 cm (6 inch))	
Preset Programs		5 Healthcare Programs, 3 Quick Programs	
Manual Selection		12 types (automatic shiatsu point locator function included in some programs)	
Air Pressure		Approx. 37 kPa	
Auto Timer		Approx. 15 minutes (preset programs vary in duration)	
Leg Rest Adjustment Range		Approx. 25 cm (9.8 inch)	
Reclining Angle		Approx. 115° - 165° from floor level, continuous motion (Automatic reclining)	
Chair Dimensions The depth and height figures in () represent the dimensions when fully reclined and with the leg rest flat		Approx. 73 × 128 (185) × 103 (75) cm (W/D/H) (Approx. 29 × 50 (72) × 41 (30) inch (W/D/H))	Approx. 80 × 133 (190) × 103 (75) cm (W/D/H) (Approx. 31 × 52 (74) × 41 (30) inch (W/D/H))
Weight		Approx. 70 kg (154 lb)	Approx. 73 kg (160 lb)
Outer Covering		Synthetic leather	
Supplied Items		Power Cord, Buffer Pad (thick)/(thin), Shiatsu Balls x2, Remote Controller Pocket	

Meaning of the Crossed-Out Wheeled Dustbin

This symbol indicates that the waste of electric and electronics equipment must be separated from a collection of household waste. The disposal of electric and electronics equipment in household dump-site is prohibited by the regulation of EU WEEE Directive 2002/96/EC. Please use appropriate collection systems for the discarded equipment. Separated collection contributes to reuse of materials and to protect the environment. For more information on available collection system, please contact the vendor.

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Manufactured by FAMILY CO., LTD.