**ichigo ichie®**

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User Manual  
使用說明書

## Important suggestions!

- Read this user manual before first use of the commode and keep it at hand for the user.
- Keep this user manual at hand for future use and resale.

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# 1. Introduction

In this manual you will find information on safe and proper installation, use, operation, maintenance and care of the commode. Read these operating instructions carefully before using the product. Note in particular the safety instructions and act accordingly.

## 1.1 Intended proper use

The all-in-one commode chair is for homecare and institutional purposes. The commode chair is to be used exclusively for assistance during showering or bathing, using a toilet or for moving of persons inside bathrooms. The commode chair is not designed for long periods of sitting.

# 2. Disposal

Products that can no longer be used are to be disposed of separately from regular household rubbish. This must be done according to the local and national regulations for environmental protection and raw material recycling. Please contact your local disposal company or officials for further information.

# 3. Safety instructions for use

When using the chair, the following safety instructions must be observed:

## 3.1 Danger of slipping

- Before using, check to be sure the seat is securely pressed into the seat frame.
- Always lock the swivel castors before using the chair. This is to prevent the chair from rolling away.
- When sitting down into the chair, the user must not sit on the front edge of the seat or seat cushion. Be sure the user sits all the way back on the seat.

## 3.2 Danger of tipping over

- The chair may be pushed only using the push bow (backrest).
- The footrests must be folded up whenever the patient sits down or gets up. The patient must never attempt to stand on the footrests!
- The chair must not bump into door sills, jambs or on uneven surfaces.
- Use the chair only indoors and on solid, even, non-slippery surfaces.
- The chair may be pushed only at a walking speed or less (with or without the patient).
- Do not lean out of the chair (when the chair is still or in motion). If the swivel castors are swivelled inwards, then the stability of the chair is decreased.

### **3.3 Danger of breakage**

- The chair only is suitable for carrying persons. Be sure to observe the maximum allowable patient body weight.
- Never attempt to carry or lift the chair while a patient is sitting in it.
- Do not grab hold of or pull on attached objects, such as IV Pole, armrest cushions.

### **3.4 Danger of pinching!**

- To prevent body parts from getting pinched, do not reach between the seat and the frame when pressing the seat into the seat frame.
- Be sure other parts do not get jammed when folding and unfolding the footrests.

### **3.5 Danger of falling!**

- Sitting down, standing up and moving the patient should be supervised by a qualified care-worker, as to the health condition of the patient.
- Use of the chair by patients who do not have complete physical control and normal movement must not be done unsupervised.
- Rolling the chair on any sloped surface may be done only with the assistance of a care worker.

## **4. Scope of Delivery**

The product has been carefully inspected in the factory for freedom of defects and for completeness. After having received it, carefully check over the product for possible transport damages and for completeness.

- 1 commode wheel chair
- 1 footrest
- 1 commode pan with lid
- 1 Toilet seat cover
- 1 U-shaped toilet seat
- 1 User manual
- 6 Screw set with nuts

## 5. Set Up

Please read the following instructions before set up.

1. Installing the toilet seat	 <p>1.1 Expand the frame of the commode and place it on an even, steady surface as shown.</p>	 <p>1.2 Insert 4 legs of the U-shape toilet seat into the plastic tubes on the frame as shown. Keep it horizontal and press down as shown.</p>
	 <p>1.3 Screw the legs of the U-shape toilet seat into the plastic tubes on the frame (as shown) in clockwise direction.</p>	 <p>1.4 Screw all 4 legs into the plastic tubes on the frame.</p>
2. Installing the footrest	 <p>2.1 Carefully place the footrest above the wheels and click into the holes on the plastic tube.   Mind the fingers</p>	 <p>2.2 finish the installation of footrest as shown in the picture.</p>
3. Installing the commode pan	 <p>3.1 Locate the track for the commode pan as shown.</p>	 <p>3.2 Slide the commode pan along the track as shown.</p>

## 6. Using the Commode

**When using the chair, the following points must be adhered to:** Before use it must be assured that the chair is in a proper condition for safe use. If necessary, depending on the physical and strength limitations of the user (patient's ability to move and muscle strength), sitting down and getting up should not be done without assistance from a care worker. The chair should not be used for patient transport.

### 6.1 Sitting down

Before sitting down in the chair, be sure it is secured against slipping away (see: 6.4) and tipping over, that the armrests are secured, that the commode seat is properly mounted, and that the footrest is folded up.

The patient must stand as closely as possible near the chair. Then he has to sit down slowly on the whole seat during supporting himself with using both armrests. If this is not possible due to his physical limitations, then a care-worker must assist him. If necessary a patient lifter may have to be used.

**Caution – danger of falling!** To prevent the patient from falling out of the chair, be sure the patient is seated completely on the chair, and not just on the edge. For moving the patient, be sure he is leaned back against the seat back and be sure he uses the footrests.

## 6.2 Getting up

Before getting up, be sure the chair is stable and secured against slipping away (see: 6.4) and tipping over, and the footrest is folded up. The patient must then move his body forward until his feet are flat on the floor.

The patient must then place both hands on the armrests and slowly lift himself up. The patient must not lean too far forwards, in order to avoid falling. The patient should have the care worker assist him when getting up.

**Caution – danger of falling!** The patient must not stand on the footrest when getting up or leaving the chair.

## 6.3 Removing the seat

The seat can be removed from its receptacles for the purpose of cleaning or exchanging simply by pulling it up.

## 6.4 Using the wheel locks

To lock the wheels, press the locking levers down with your foot. The wheels are now locked and not loose. Releasing the wheels is done by lifting up the locking levers with your foot.

## 6.5 Handling the commode pan

Push the commode pan under- the seat from behind the chair. To remove the pan, lift it up slightly and pull it out.

## 7. Care

In general the chair can be cleaned using a mild conventional cleaning and disinfecting agent. Simply wipe the chair with clean water and cloth, and then wipe dry with a clean cloth.

Do not use an abrasive scouring agent or cleaner (read the product instructions!) that contain aggressive solvents (acids, thinners). Do not use stiff cleaning brushes, as these will damage the plastic surfaces or cause deposits to build up on the plastic parts and other materials.

Keep the product clean and clean it in regular intervals. In the sensitive areas is hygiene very important.

## 8. Maintenance & Repair

The cause of accidents or damage may be due to the late detection of wear and/or improper use, as well as irregular or no maintenance.

For safety reasons and for preventing accidents, the chair must undergo a complete maintenance at least once a year. (Refer to the Medical Products Act.)

### Maintenance

- Periodic safety-related controls are needed:
- Check for apparent damage: (cracks, fractures, loose parts).
- All screwed connections for tightness (especially the castor wheels and the frame).
- Function of the armrests
- Function of the footrests (replace defective footrests).
- Rolling ability, check function of the wheels (replace defective wheels).
- Check accessories for defects.

### Repair

- If you have questions concerning maintenance and repairs to the chair, then contact an authorized dealer or medical equipment supply outlet.
- When making repairs, use only original spare parts from the manufacturer. Observe all advisories concerning guarantee and warranty.
- Never make any unauthorised modifications or conversions to the chair. These will only adversely affect the safety and function of the chair. In such cases the manufacturer assumes no liability!
- Follow all the instructions when ordering spare parts, re-use and product lifetime of the chair.
- After a complete maintenance or repair of the chair is done, we recommend a complete cleaning for assuring hygiene. Follow the cleaning instructions.

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wholly owned by Humana Medical Ltd