Add this addendum to the front of your HAMILTON-C1 Operator's Manual.

The HAMILTON-C1 software version 2.1.x introduces some important enhancements and updates to the device software and Operator's Manual.

1. Changes to Operator's Manual

The *HAMILTON-C1 Operator's Manual*, PN 624326/02, has been updated to reflect the new functionality and options, and includes updated safety information. Review the manual before operating the device with the updated software.

Note that some of the features and options described in the updated Operator's Manual *only apply to HAMILTON-C1 ventilators with SN 6000 and higher*. This addendum provides information about which features are and are not supported in devices with SN < 6000 with SW v2.1.x.

2. What's new in software version 2.1.x

The following features and options have been added or updated in version 2.1.x. The page and section references are for the *HAMILTON-C1 Operator's Manual*, PN 624326/02.

Table 1. Changes in software version 2.1.x

Feature/Option	SN ≥ 6000	SN < 6000	See ...
New display languages added: Croatian, Finnish, Romanian	✓	✓	--
High-capacity, longer life Li-ion batteries	✓	✓	pg 2-25; pg A-5
Peak flow increased to 260 l/min	✓	✓	Table A-3 on pg A-4 Table A-12 on pg A-24

Table 1. Changes in software version 2.1.x

Feature/Option	SN ≥ 6000	SN < 6000	See ...
Alarm loudness setting moved to Settings window	✓	✓	pg 4-22
Ability to configure a minimum alarm volume (loudness) for the device	✓	✓	pg 1-6
Alarm lamp now flashes for high- and medium-priority alarms and technical faults	✓	✓	pg 8-3
The following alarms have been added or updated: • Battery low • Loss of external power • Performance limited by altitude • Battery replacement required (new) • High PEEP (new) • High oxygen • Inspiratory volume limitation (now disabled in noninvasive modes)	✓	✓	Table 8-2 on pg 8-10
Replaced the numerical code with text and added troubleshooting information to the Operator's Manual for the following technical alarms: • Battery communication error (244011) • Battery totally discharged (444001) • Blower fault (431001) • Buzzer defective (243004) • Loudspeaker defective (243003) • Options not found (249001) • Self test failed (485002)	✓	✓	Table 8-2 on pg 8-10

Table 1. Changes in software version 2.1.x

Feature/Option	SN ≥ 6000	SN < 6000	See ...
Default values changed for the following parameters: ETS, I:E, Rate, TI, Vt/kg	✓	✓	Table A-5 on pg A-7
P-ramp limitation by TI added	✓	✓	Table A-5 on pg A-7
Trends up to 72 h ¹	✓	✓	pg 6-11
Activate/deactivate IntelliSync with PSIMV+ mode ¹	✓	✓	pg B-22
HPO/LPO selection moved to Utilities window, available in Standby	✓	✓	pg 2-31
TI:I:E breathing timing selection for PCV+ and (S)CMV+ modes	✓	✓	pg I-7
Blower lifetime increased	✓	✓	pg 10-14
Block protocol support	✓	✓	pg H-3
Once activated, CO2 monitoring remains activated until manually deactivated	✓	✓	--
Neonatal option ¹	✓		Chapter 5, Neonatal ventilation
nCPAP and nCPAP-PC option ¹ <i>Requires Neonatal option</i>	✓		Chapter 5, Neonatal ventilation
SpO2 option ¹	✓		Appendix J, Pulse oximetry
Communication board (CO2, SpO2, COM1)	✓		Appendix J, Pulse oximetry
Day/Night key	✓		pg 9-13
Expiratory valve detection (Adult/pediatric vs neonatal)	✓		pg 5-3; Table 8-2 on page 8-10

Table 1. Changes in software version 2.1.x

Feature/Option	SN ≥ 6000	SN < 6000	See ...
Fully compliant with IEC 60601-1, third edition	✓		--
Updated Intended use statement.	✓	See below	

1. Not available in all markets.

2.1. Intended use statements

Devices with SN ≥ 6000

The intended use is provided on page vi of the HAMILTON-C1 Operator's Manual, v2.1.x, PN 624326/02.

Devices with SN < 6000

The HAMILTON-C1 ventilator is intended to provide positive pressure ventilatory support to adults and pediatrics.

Intended areas of use:

- In the intensive care ward or in the recovery room
- During transfer of ventilated patients within the hospital

The HAMILTON-C1 ventilator is a medical device intended for use by qualified, trained personnel under the direction of a physician and within the limits of its stated technical specifications.

CAUTION (USA only) Federal law restricts this device to sale by or on the order of a physician.

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