




QUICK GUIDE

icare
TONOVET

LOADING THE PROBE

ALWAYS USE WRIST STRAP FOR SAFE AND SECURE USE

1. Press the measuring button.
2. When display shows, load the single use probe into the tonometer as follows:
3. Open the probe tube by removing the cap and insert the probe into the probe base. Be careful not to drop the probe from the tonometer.
4. Activate by pressing the measuring button once. The display will show indicating that the tonometer is ready for measurement.

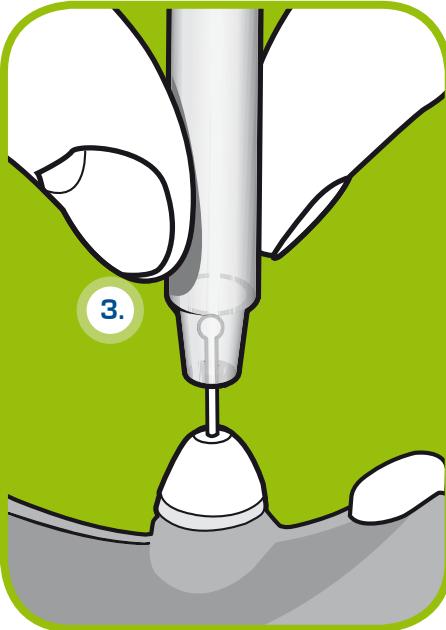

MEASURING THE IOP

1. Bring the tonometer near the patient's eye. Check that the **central groove is in a horizontal position**. It's not necessary to look at the display during the measurement since the device gives sound signals about errors etc.
2. Keep the distance from the tip of the probe to the patient's cornea at **4 to 8 millimeters (1/6 - 1/3 inch)**.
3. Press the measuring button six times. The tip of the **probe should make contact with the central cornea**.
4. Once the six correct measurements are done, the IOP will be shown on display. In case of an erroneous measurement, clear the error message by pressing the measurement button and continue measuring.

Please refer to the manual for additional information.



Press either selector button until the display shows 'End'. Press the measuring button for two seconds - the display will show 'byE' and the tonometer will switch off.

If the tonometer is not used for two minutes, it will automatically switch off.

Changing the settings for different species

1. Press the right or left selector button until the display shows "End".
2. First press the measurement button, **keep it pressed down and immediately press the right selector button**. - The buttons must be pressed down at the same time. The display will show "CAL" [or software version].
3. Press the measurement button.
4. Toggle between settings, using the selector button.
5. Choose the setting and activate by pressing the measurement button.

do = dog / cat (default calibration)
ho = horse



ONE SHORT BEEP —


Indicates a correct single measurement.



DOUBLE BEEP — —

Indicates an incorrect measurement. (See error codes from manual).


LONG BEEP —————

Indicates that six successful measurements are done and IOP result is available at the display.

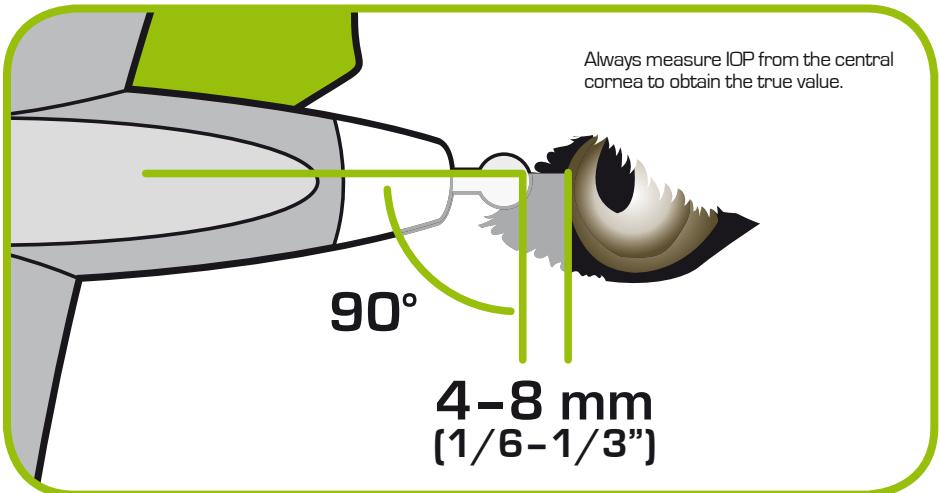

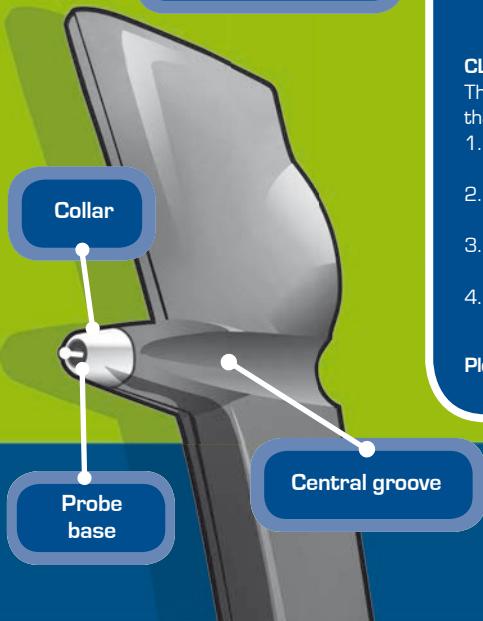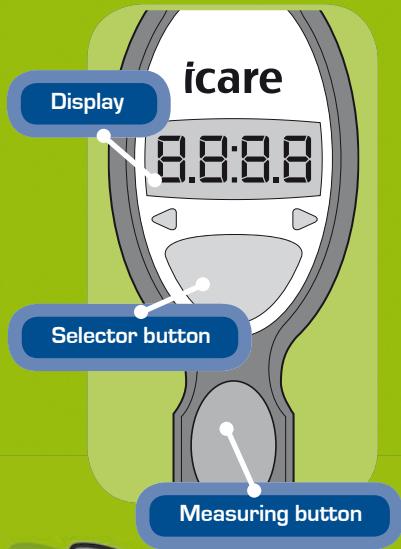

CHANGE/CLEAN THE PROBE BASE REGULARLY TO MAINTAIN THE ACCURACY AND TO AVOID UNNECESSARY SERVICE.

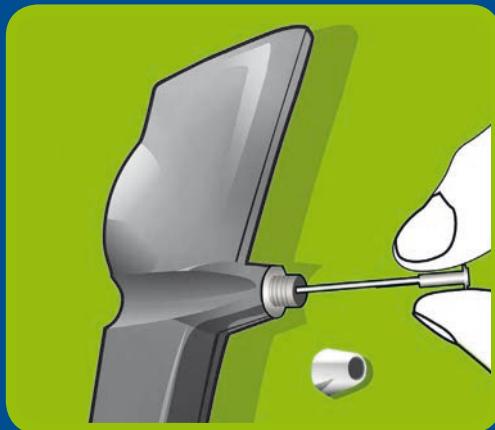

CLEANING THE PROBE BASE

The probe base can be reused after careful cleaning. Make sure that the probe base is completely dry before using it.

1. Unscrew the probe base collar and remove the probe base from the tonometer.
2. Carefully inject an alcohol solution through the inside of the probe base, for instance using a pipette or syringe.
3. Dry the probe base by blowing some clean canned or compressed air into it.
4. Reinsert the completely dry probe base and screw the collar back into the tonometer.

Please refer to the manual for more detailed instructions.

icare

measuring
excellence

www.icaretonometer.com