



puregraft[®] 250 850


QUICK REFERENCE GUIDE

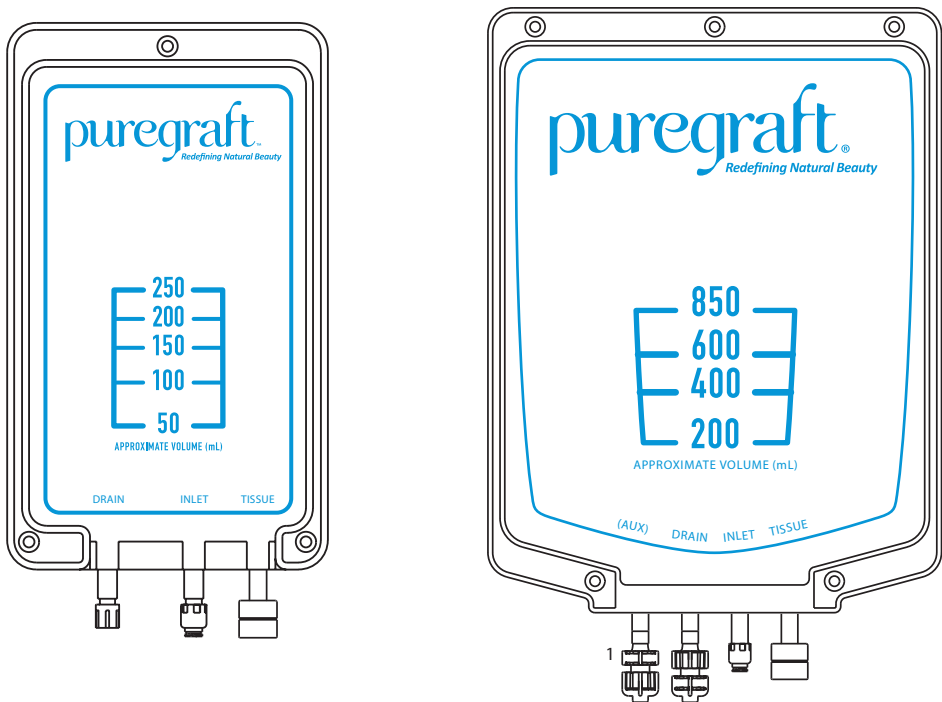

Puregraft System² contents:

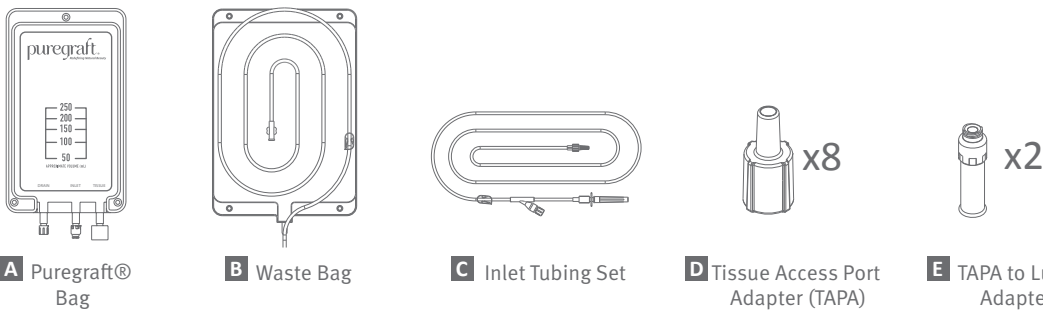


Additional supplies needed:

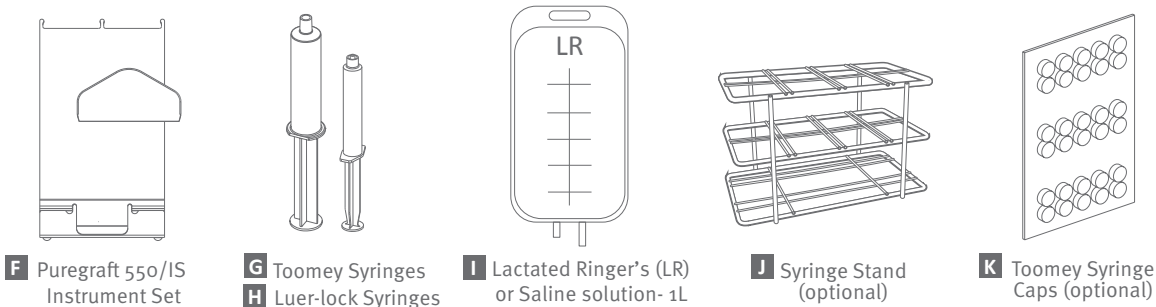


¹AUX port is not used during the Puregraft procedure
²Actual contents may vary based on the Puregraft size

STEP 1. SETUP

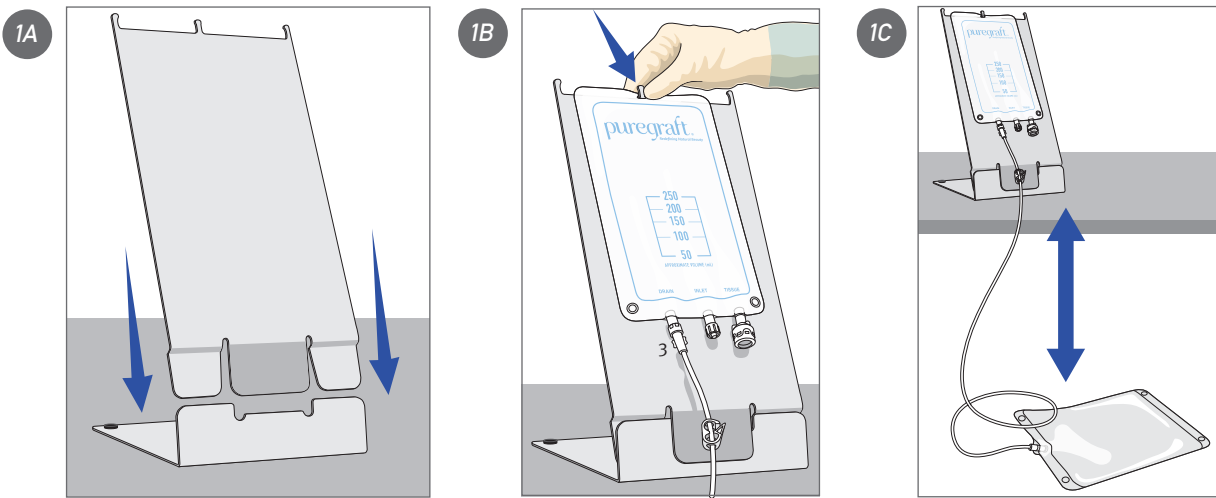


³ For Puregraft 850, waste bag must be connected manually.

STEP 2. TISSUE ADDITION

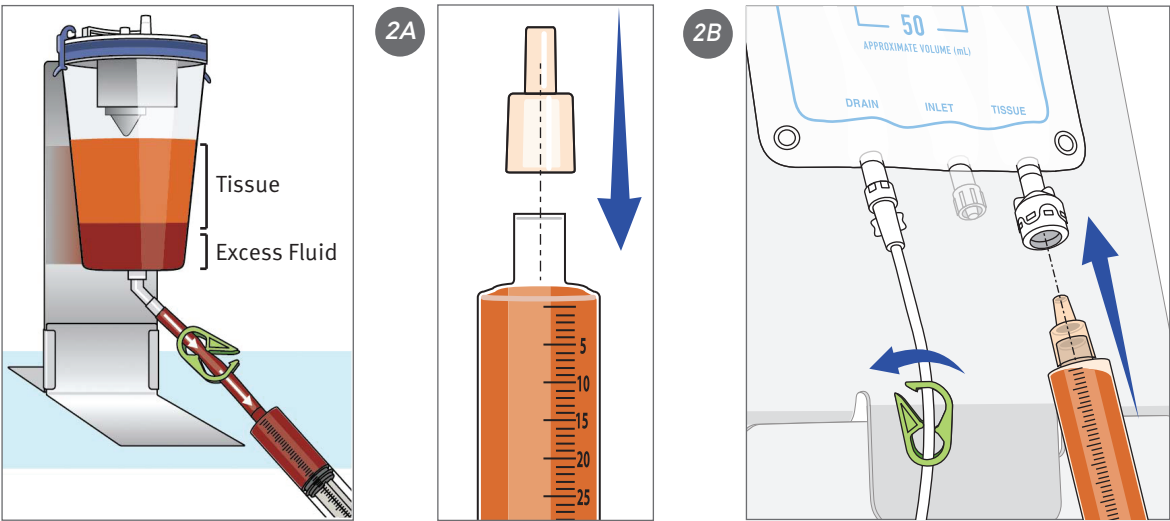

(Only for suction-assisted harvest)

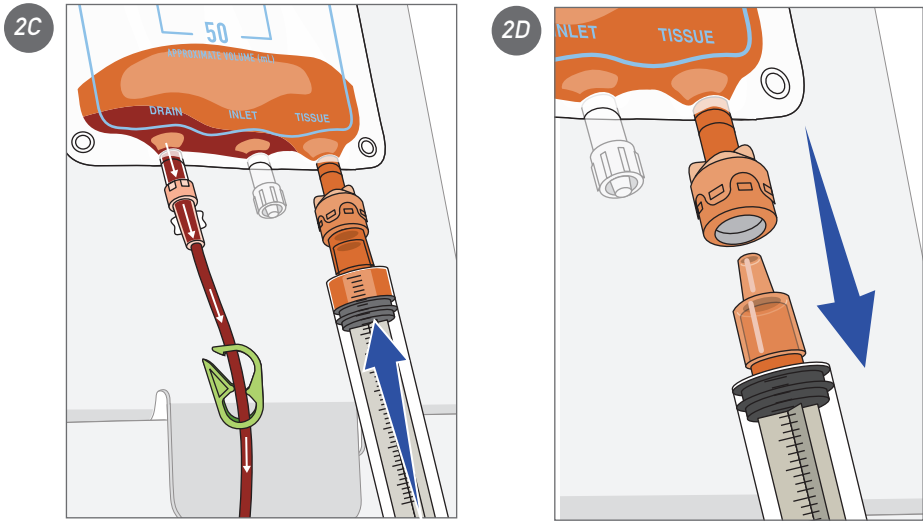


Repeat Steps
2A - 2D
for the entire
tissue volume ⁵

Tissue Volumes ⁵ (Min-Max)	
250	(50 mL - 250 mL)
850	(200 mL - 850 mL)

⁴ Tissue volume does not include excess fluids.
⁵ Add 30-50% more tissue than the desired graft volume.

STEP 3 . TISSUE WASHING

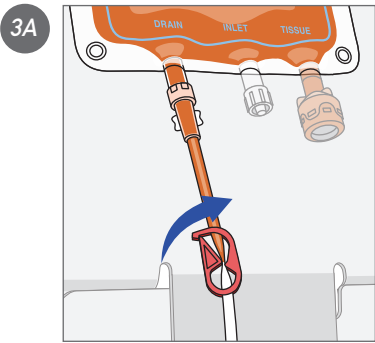


250


Tissue Volume (mL)	Lactated Ringer's (mL)
50	50
100	100
150 - 250	150

850

Tissue Volume (mL)	Lactated Ringer's (mL)
200	100
400	200
600	300
850	425


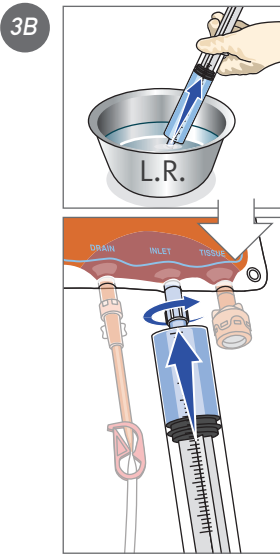


OR

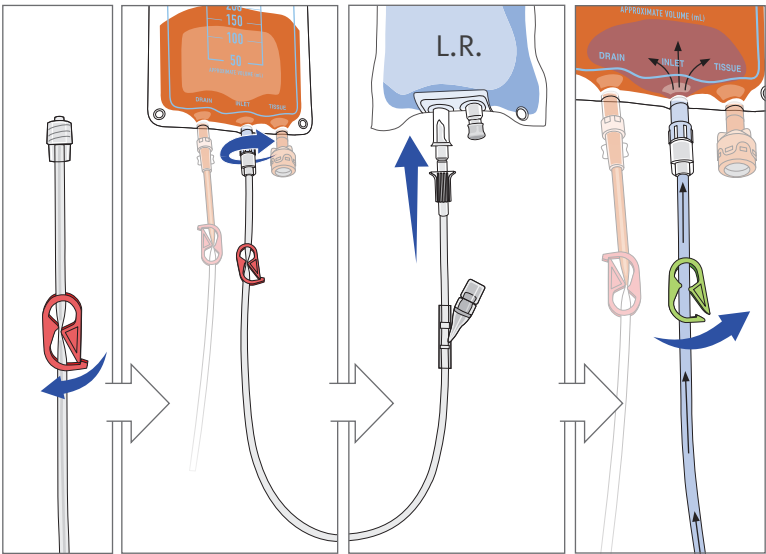

Syringe Method


Inlet Tubing Method

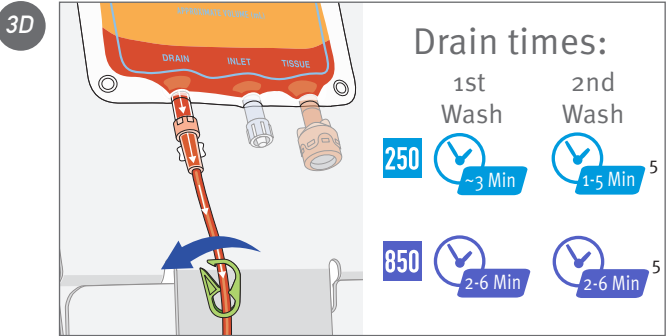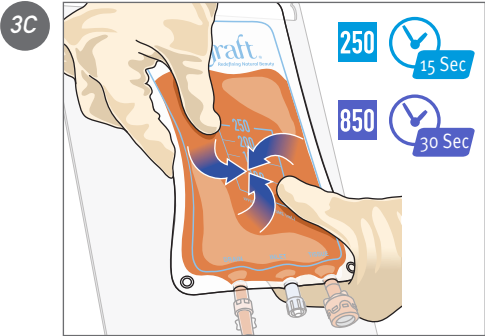

Drain times:

1st Wash 2nd Wash


250	~3 Min	5	850	2-6 Min	5
-----	--------	---	-----	---------	---

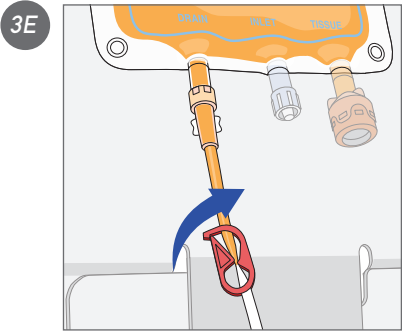


Repeat Steps 3B - 3E for 2nd Wash

⁵ Depending on desired graft hydration level

STEP 4. GRAFT EXTRACTION

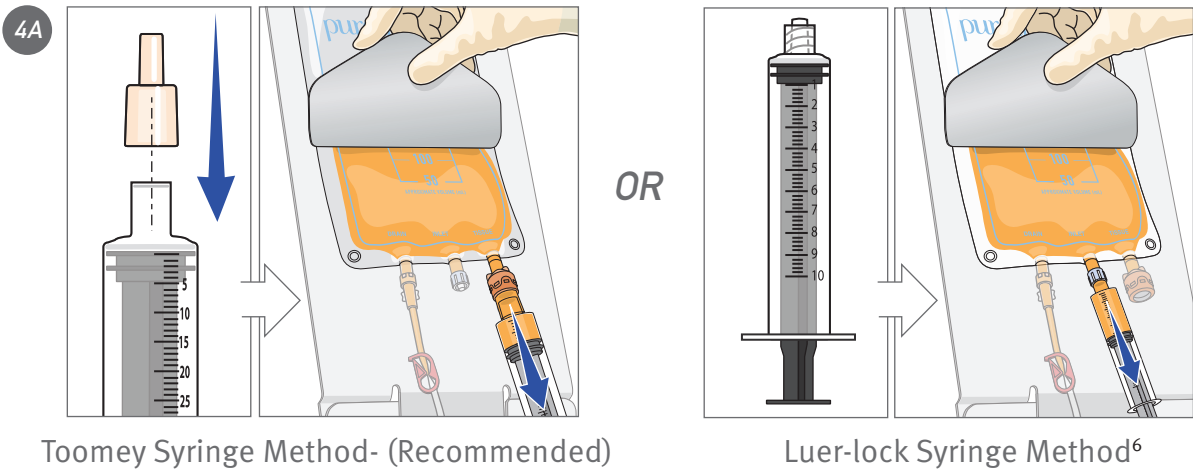

GRAFT TRANSFER

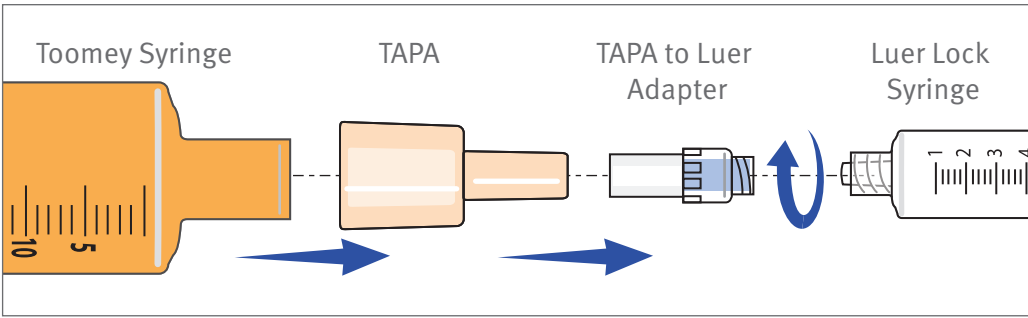

⁶May increase risk of clogging

Helpful tips for Puregraft Users:

1. Puregraft System works best with tissue harvested with 3mm or smaller cannulae.
2. For best results, use HSW Soft-Ject branded luer-lock syringes.
3. Do not fill the Puregraft bag beyond the recommended max tissue volumes.
4. Extract and discard any excess fluid trapped under the “Tissue” port before removing processed graft.
5. For fibrous tissue, a small amount of LR and/or air can be introduced in to the bag to help with tissue extraction and preventing clogging.

Puregraft, LLC.
3020 Callan Road
San Diego, CA 92121, USA

Tel: 1.858.348.8050
Email: info@puregraft.com
www.puregraft.com

14-PG-0031-EN-GBL-A

Puregraft and the Puregraft logo are registered trademarks of Puregraft, LLC. in the United States and other selected countries.

© 2014 Puregraft LLC. All rights reserved.