

Panasonic®

Hearing Instrument Quick Tips for Use


Model: WH-216KZ, WH-212LZ, WH-209MZ

- Wearing the Hearing Instrument
- Removing the Hearing Instrument
- Switching between Memory Programs
- Care and Maintenance
- Inserting/Removing Zinc Air Battery
- Component Descriptions



• The intent of this Quick Tips is to give information on how to handle and maintain the Panasonic Hearing Instrument in a simplified and comprehensible description by using photos. Be sure to carefully read and understand the operating instructions as well before using your hearing instrument. The featured instrument might vary slightly in appearance from your instrument.

Component Descriptions

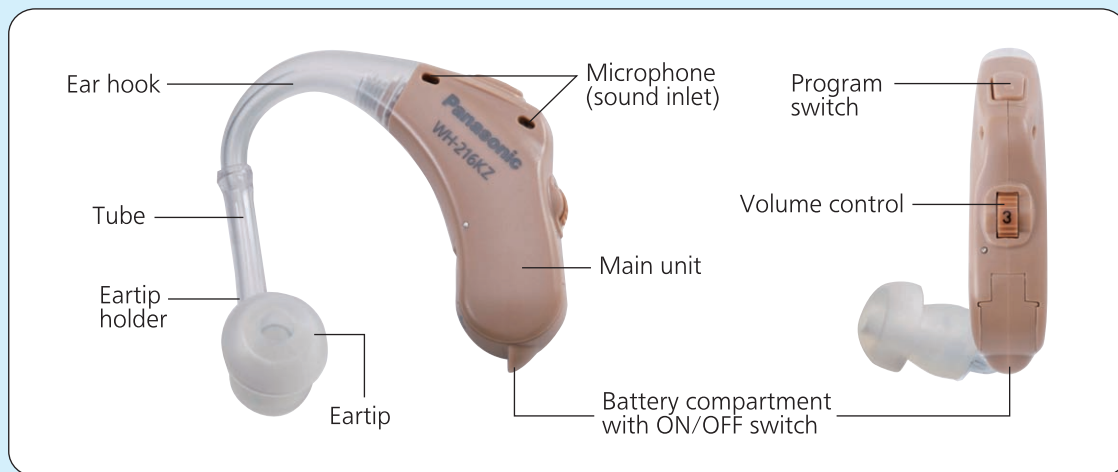

Side

Front

See Page 20 of the Operating Instructions.

Inserting/Removing Zinc Air Battery

1



Replace the battery by opening or closing the battery door.

See Page 21 of the Operating Instructions.

Inserting/Removing Zinc Air Battery

2



Pull the battery door outward with your forefinger.

See Page 21 of the Operating Instructions.

Inserting/Removing Zinc Air Battery

3




Insert a new battery or replace the battery.

See Page 21 of the Operating Instructions.

Inserting/Removing Zinc Air Battery

4

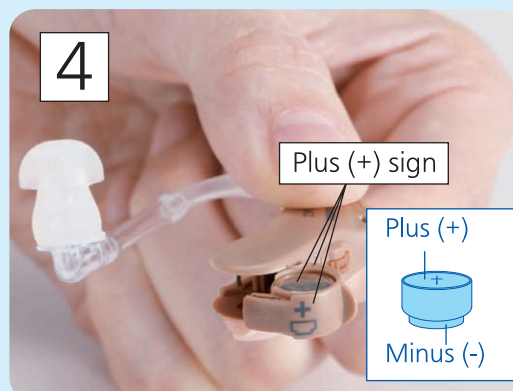

Place the new battery in the compartment with the plus (+) sign facing up. Your hearing aid uses a size 13 battery.

See Page 21 of the Operating Instructions.

Wearing the Hearing Instrument



Hold the main unit as shown in the image.

See Page 22 of the Operating Instructions.


Wearing the Hearing Instrument



Position the hearing instrument behind your ear.

See Page 22 of the Operating Instructions.

Wearing the Hearing Instrument

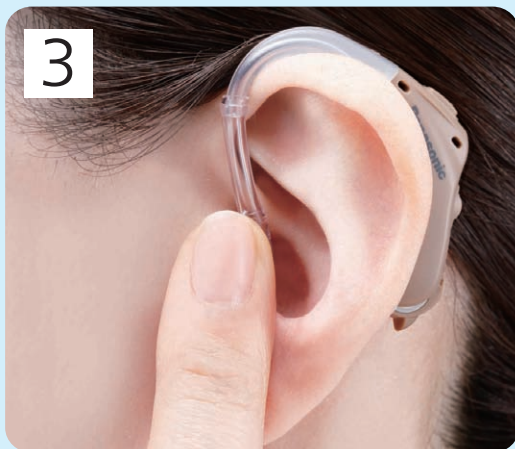

Insert the eartip in your ear.
(After positioning the hearing instrument at the right place, turn on the hearing instrument.)

See Pages 22 and 23 of the Operating Instructions.

Removing the Hearing Instrument



Hold the receiver unit and pull it straight out.
(Make sure that the hearing instrument is powered off.)

See Page 25 of the Operating Instructions.