# GUARDIAN®

## DO NOT REMOVE PACKAGING MATERIAL UNTIL PATIENT USAGE.

Packaging material provides protection from Dust, Dirt and Oxidation.

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## FITTING INSTRUCTIONS

1. Adjust crutches to approximate patient height. Top of crutch should be approximately two inches below the underarm.
2. Ensure detent button is fully engaged.
3. Stand with crutches placed 6" in front and 6" to side of feet. Adjust handle by removing bolt and inserting into hole that allows elbow to be bent approximately 20° to 30°.

INSTRUCTIONS CRUTCHES SHOULD BE EXACTLY FITTED AND GAIT TRAINING GIVEN BY A PROFESSIONAL. PLEASE READ THE IMPORTANT INSTRUCTIONS.

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# GUARDIAN®

## USER INSTRUCTIONS

Keep wrists straight when using crutches.

Carry full weight on hands, not the underarm.

When going up stairs, place uninjured foot up stair first, followed by crutches and injured foot.

When going down stairs, place crutches down on stair first, followed by injured foot, then uninjured foot.

## WARNING

For safety, these precautions must be taken when using this product:

## Before each use make sure:

- You are correctly fitted.
- All detent buttons are fully engaged.
- Tips are in good condition. Replace tips immediately when worn.
- Hand grips are attached securely and do not slip.
- Nothing is loose or uneven and there are no rattles or cracks in the metal, plastic, etc. or any other irregularities.

Do not exceed the maximum weight capacity listed on your crutches.

Use only as instructed by your healthcare provider.

Always use extreme care when using crutches. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

Failure to follow the above instructions may result in serious injury.

## LIFETIME LIMITED WARRANTY

Your Guardian® product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser. This warranty does not cover product failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement.

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