*No. 1 choice
of hospitals
& mothers*

Freestyle™

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
I Freestyle™ Breastpump

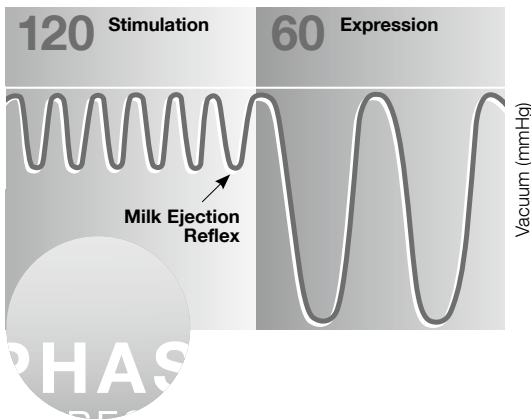Mother's milk is the best nutrition for babies. Freestyle is designed for busy mothers. It allows to double pump and therefore saves time and addresses the needs for more pumping freedom. The attractive breastpump is especially suitable for frequent pumping and encourages milk production in the long term, too.

Freestyle has 2-Phase Expression® the only research based breastpump technology to mimic a baby's natural nursing rhythm.

Stimulation:

Fast pumping rhythm to start your milk flowing.

Expression:

Slower pumping rhythm to express your milk gently and efficiently.

Proven to feel most like your baby and to reduce your pumping time.

II Indications For Use

The Freestyle Breastpump is a powered breastpump:

- To be used by lactating women to express and collect milk from their breast during absences from the baby (e.g. working part- or full-time).
- To initiate and maintain a milk supply if breastfeeding is temporarily interrupted.
- To relieve engorgement.
- For home use and on the go.

III Important Safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

Read all instructions before use.

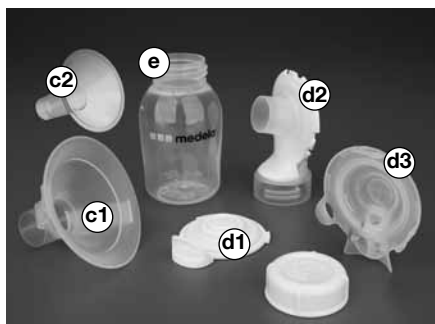
- Use the product only for its intended use as described in this instruction manual.
- Only use the mains adapter or battery that comes with the Freestyle Breastpump.
- Make sure the voltage of the mains adapter is compatible with the power source.
- Plug the mains adapter into Freestyle Breastpump first and then into wall socket.
- Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.
- Keep the mains adapter away from heated surfaces.
- Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.

- Do not use outdoors.
- Keep appliance dry and do not place in direct sunlight.
- Do not get the product wet.
- Do not use while bathing or showering.
- Do not place in or drop motor unit into water or other liquid.
- Do not store or use this product where it can fall or be pulled into a tub or sink.
- Do not spray or pour liquid directly onto the pump.
- Do not reach for any electrical device if it has fallen into water. Unplug immediately.
- **Use Medela original accessories only.**
- Check pump kit components for wear or damage.
- This is a single user product. Use by more than one person may present a health risk and voids the warranty.
- Do not use while sleeping or overly drowsy.
- Supervision is necessary when this product is used in the vicinity of children.
- If problems or pain occur, consult your lactation consultant or physician.
- Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.
- Portable and mobile radio frequency communications equipment can affect the Freestyle Breastpump.
- The Freestyle Breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use.
- There are no user-serviceable parts inside the Freestyle. Do not attempt to repair the Freestyle Breastpump yourself.

SAVE THIS INSTRUCTION MANUAL


IV Assembly Instructions

Assembling the pump kit


Insert pale yellow membrane into breastshield body. If placed properly, ridges will align.

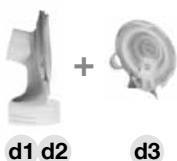IV Assembly Instructions

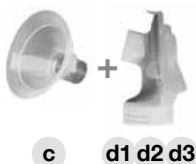
Snap clear back cap onto breastshield body enclosing membrane.



Make sure all three tabs are snapped in place.

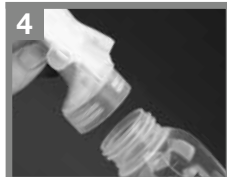Make sure breastshield (PersonalFit or SoftFit Breastshield) and connector are securely attached.


Screw bottle into connector.

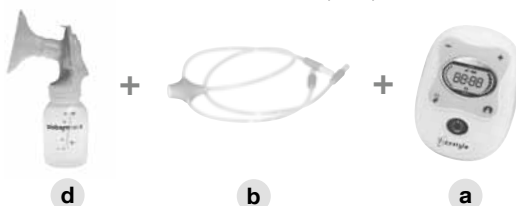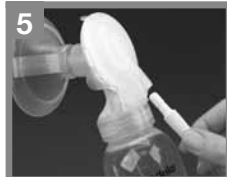Connect tubing to the breastshields and the pump.

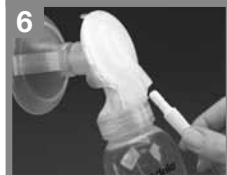Single Pumping

To switch from double pumping to single pumping, simply unplug tube from one side.

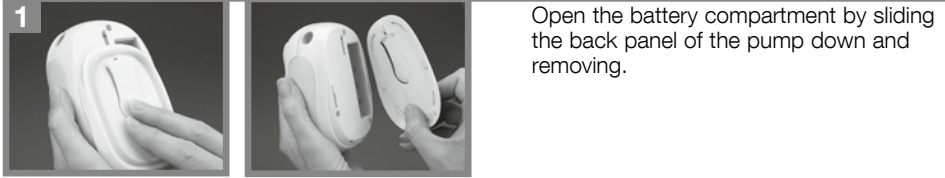
Place unused tube into tubing dock.

V Operation

Setting up the pump

Before using for the first time:

Open the battery compartment by sliding the back panel of the pump down and removing.

Insert the battery into the compartment with the arrows and the note «This Side Down» pointing towards the pump. Do not use force to insert the battery.

Close the battery compartment.

Plug in the mains adapter and charge until the battery power indicator shows four bars and stops flashing a). (this operation can last for up to 20 hours) **When charging the first time or after every removal of the battery, the display shows "BATT" b).** Pump may be used while charging. Do not unplug the mains adapter until the charging is completed c).

Avoid removing battery. Removing the battery will reset the battery indicator. A discharging for up to 20 hours and a new recharging time of 4 hours is required to readjust the battery indicator.

Control buttons and LCD Display

- 1) On / Off button
- 2a) Stimulation Phase
- 2b) Expression Phase
- 3) Vacuum increase button (+)
- 4) Vacuum decrease button (-)
- 5) Let-down button
- 6) Memory button
- 7) Timer
- 8) Memory «on» indicator
- 9) Battery power indicator
- 10) AC power indicator

V Operation

Electrical operation

The pump can be mains adapter operated.

Plug the electrical connector on the end of mains adapter cord into power jack (12V DC).

The mains adapter symbol appears on the display when the pump is plugged in.

Battery operation

The battery will charge every time the pump is plugged in and when it is not in-use.

When the pump is switched on, the battery indicator on the display will show how much battery power is left. If the battery symbol starts to flash showing just one bar, there is only power left for approx. 20 minutes pumping time.

Pumping time (full battery) = approx. 3 hours

Charging time (empty battery) = approx. 4 hours*

* When charging the first time or after every removal of the battery, the display shows "BATT".

This operation last for up to 24 hours.

Avoid removing battery. Removing the battery will reset the battery indicator (the battery symbol starts to flash showing just one bar) and requires a recharging time of up to 24 hours to readjust the battery indicator.

Caution:

The battery used in this device may present a risk of fire or chemical burn if mistreated. Do not disassemble, heat above 60° C or incinerate. Replace battery with Medela part number 200.1512 only. Use of another battery may present a risk of fire or explosion.

When storing the battery for a limited length of time, store within the following temperature range: -4° to 140° F (-20° to 60° C). When storing the units for an extended time, store within the following temperature range: 32° to 77° F (0° to 25° C).

Dispose of used battery promptly. Keep away from children. Do not disassemble and do not dispose of in fire.

Note

If the pump has been running for 30 minutes without any manipulations (e.g. adjustment of the vacuum), the Freestyle will automatically switch off.

VI Pumping

1

Center your breastshield over your nipple.

- Do not tilt or overfill bottles when pumping.
- Take care not to kink tubing while pumping.

2

Switch on the breastpump by pressing the On/Off button. The pump will begin pumping in the **Stimulation Phase**.

3

Adjust the vacuum during pumping to your comfort level by pressing the increase (+) or decrease (-) button. If you keep pressing the button (+ or -), the vacuum level can be adjusted more rapidly.

4

Once your milk starts to flow or you sense milk ejection, press the let-down button to switch to the **Expression Phase**.

If you do not press the let-down button, the pump will automatically switch to the Expression Phase after two minutes. Set your **Maximum Comfort Vacuum™** which is the highest vacuum you can use and still feel comfortable (this is different for every mother). To find your Maximum Comfort Vacuum increase vacuum until pumping feels slightly uncomfortable (not painful), then decrease slightly.

Research has shown that you will pump more efficiently – meaning get more milk in less time – when pumping at Maximum Comfort Vacuum™ in the Expression Phase.

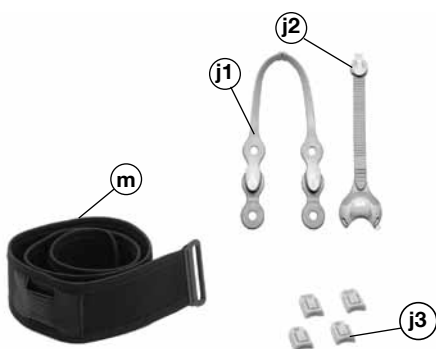
Once finished pumping switch off the breastpump, and disassemble the pump kit items. Close the bottle and store it for later use.

Use bottle stand to prevent tip over.

The displays remains on for a few seconds after turning the pump off.

Some mothers prefer going back and forth between Stimulation and Expression to achieve second let-down throughout a pumping session. You can do this by pressing the let-down button.

VII Hands-free pumping

Freestyle comes with accessories that enable you to be mobile while pumping. The following instructions will explain how you can use the hands-free accessories to achieve the freedom you desire.

Handsfree-Kit

j1 / j2 / j3

Belt with extension

m

VII Hands-free pumping

Setup of Hands-free Kit

Connect hands-free strap (j1) to hands-free adapter (j2).

Snap hands-free adapter (j1/j2) onto back of each breastshield.

Open your bra flap and insert clasp into your bra.

Use bra adapters (j3) if necessary.

Center your breastshield over your nipple.

Attach bottom clips to bra band until secure fit is achieved. Be sure the nursing bra flap is not trapped under the breastshield.

If necessary, adjust the fit by moving the clips to a different strap setting.

Adjust length of top strap by pulling down on the loose end.

Attach tubing.

Now you are ready to pump handsfree.

Use Belt for mobility

Wrap belt around waist, Velcro facing away from the body.
If you want a longer belt, add the extension piece.

Slide clip of pump into the slot in the belt.

VIII Memory

Once you have identified a pumping pattern that is comfortable and effective for you, it is easy to program Freestyle to repeat your personal pattern without having to reset it every time you pump.

Setting the memory allows you to duplicate:

- Length of Stimulation phase
- Vacuum level in Stimulation Phase
- Vacuum level in Expression Phase

Store your preferred setting:

- Start pumping session following usage instructions (VI Pumping).
- Press the memory button while pumping in the Expression Phase for three seconds, until the display flashes and the pump beeps three times.
- Finish pumping session as usual.

Activate your preferred setting:

- Start the pump and press the memory button.

Timer

The timer automatically starts when the pump is turned on.

If for any reason you want to restart the timer during a pump session, press the Let-Down button and hold for 3 seconds. This will restart the timer, but Freestyle still continuous to pump.

IX Breastshield Fitting

Choosing the right size of Breastshield

A correctly fitted breastshield avoids compression of any milk ducts during pumping to ensure optimal breast emptying and maximum milk output.

Your breastshield is the right size if your answer to the following questions is «Yes»:

- Is your nipple moving freely in the tunnel?
- Is minimal or no areolar tissue being pulled into the tunnel of the breastshield?
- Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- Do you feel the breast emptying all over?
- Is your nipple pain-free?

If your answer to any of the above questions is «No», try a larger (or smaller) breastshield.

Medela PersonalFit™ Breastshield: Available in five sizes – because one size does not fit all.

X Cleaning and Hygiene

Follow these cleaning instructions unless you are told otherwise by your healthcare professional.

Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Disassemble all parts prior to cleaning.

Prior to first use and once daily

- Disassemble, wash and rinse all parts of your pump kit that come in contact with the breast and milk (c + d + e + f + g + h).
- Place parts in a saucepan of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- Heat it up and allow to boil for 5 minutes.
- Remove from stove and drain water off.
- Dry parts with a clean towel.

Alternatively, the disassembled parts can be

- cleaned in top rack of dishwasher or
- sanitized with the Medela Quick Clean™ Microwave Bag or
- steam-sterilized according to manufacturer's instructions

Note: Food colorings may discolor components which will not affect performance.

After each use

- Disassemble and handwash all parts of your pump kit that come in contact with the breast and milk (c + d + e + f + g + h) in warm, soapy water.
- Rinse in clear water.
- Dry parts with a clean towel.
- Use a damp (not wet) cloth to wipe the motor unit after each use.

Hints After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid. Alternatively, the parts can be wrapped in a clean paper or cloth towel.

- Hygiene**
- Wash hands thoroughly with soap and water before touching the breast and pump parts.
 - Avoid touching the inside of bottles and lids.
 - Clean the breast and surrounding area with a damp cloth.
 - Do not use soap or alcohol which can dry the skin and cause cracked nipples.

XI Storing / Thawing Breastmilk

Check with your hospital for specific instructions.

Breastmilk Storage (For Healthy Term Babies)

	Room Temperature	Refrigerator	Refrigerator Freezer Compartment	Deep Freezer
Freshly expressed breastmilk	<i>Do not leave at room temperature</i>	<i>3 – 5 days at approx. 4° C</i>	<i>6 months at approx. -16° C</i>	<i>12 months at approx. -18° C</i>
Thawed breastmilk (previously frozen)	<i>Do not store</i>	<i>10 hours</i>	<i>Never refreeze</i>	

- *When freezing do not fill bottles/bags more than 3/4 full to allow space for expansion.*
- *Label bottles/bags with the date of pumping.*
- *Do not store the breastmilk in the door of the refrigerator.*
- *In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle/bag under running warm water (max. 37° C).*
- *Gently shake the bottle/bag to blend any fat that has separated.*
- *Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).*

In the Freestyle Cooler bag

One contoured cooling element is necessary for the Freestyle Cooler bag. As soon as breast-pumping is finished, place the milk bottle or bag in the compartment with the cooling element. Milk will keep cool there until you return home later that day to transfer the bottles/bags to the refrigerator or freezer.

XII Troubleshooting

If there is low or no suction

Make sure

- the breastshield is assembled correctly and the back cap is firmly attached at three connection points.
- the breastshield forms a complete seal around the breast.
- all connections of the kit and breastpump are secure.
- If single pumping, make sure the unused tube is securely connected to the tubing dock.

Take care not to kink tubing.

If the pump gets wet

- Unplug the mains adapter from the wall socket and turn the pump off.
- Make sure not to turn the pump upside down. Display has to face up.
- Store the pump for 24 hours in a dry and warm place.
Note: Do not try to speed up the drying process by applying heat or by using any drying equipment.
Do not expose the pump to direct sunlight.

These indications show defect of the pump:

- «Err» accompanied by 3 beeps indicates a problem with the pump.
- Mains adapter symbol flashing and pump does not turn on indicates a problem with the power supply.
- Battery power indicator flashes and shows four bars indicates a problem with the battery.

There are no user-serviceable parts inside the Freestyle.

Do not attempt to repair the Freestyle Breastpump yourself.

Please contact local Medela distributor or location where you purchased your pump.

XIII Warranty

This is a single user product. Use by more than one person may present a health risk and voids the warranty. 1-year warranty for the unit from the date of purchase, excluding pump kit, freestyle bag and cooler bag. 6 months warranty for rechargeable battery. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons.

XIV Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects in the environment and on public health.

Article numbers

a	Motor unit	200.1591
b	Freestyle™ Tubing	200.1524
c1	SoftFit™ Breastshield	810.0402
c2	PersonalFit™ Breastshield	810.7084
d	Freestyle™ Connector	200.1513
e	Bottle (150 ml)	811.0021
f	Cap	810.0474
g	Lid	810.7062
h	Disc	810.0473
i	Bottle stand	810.0462
j	Handsfree-Kit	200.1514
k	Mains adapter	200.1558
l	Rechargeable Battery	200.1512
m	Belt with extension	200.1548
n	Cooling element	811.7010
o	Cooler bag	200.0383
p	Freestyle™ Bag	200.1547

*Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.
Subject to change.
Content may vary from picture.*

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Electromagnetic Compatibility (EMC, IEC 60601-1-2:2001)

IEC 60601-1-2, Table 201

Electromagnetic Emissions		
The Freestyle is intended for use in the electromagnetic environment specified below. The customer or the user of the Freestyle should assure that it is used in such environment.		
Emission Tests	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Freestyle uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Freestyle is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 60000-3-3	Complies	

Warning: The Freestyle should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Freestyle should be observed to verify normal operation in the configuration in which it will be used.
The Freestyle was not tested for immunity to electromagnetic disturbances.

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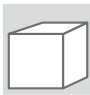
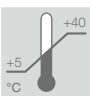
In		Out
100 ... 240 V	50/60 Hz	12 V 
		1 A
		~ alternating current
		 direct current

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Medela AG, Medical Technology
Medical Technology Co., Ltd.